

BeneFusion VP3
BeneFusion VP3 Ex

Infusion Pump

Service Manual

Intellectual Property Statement

Shenzhen Mindray Scientific Co., Ltd. holds the intellectual property rights to this product and this manual. Shenzhen Mindray Scientific Co., Ltd. is a member of Mindray Medical Group.

©2019-2020 Shenzhen Mindray Scientific Co., Ltd. All rights reserved. No parts of this Manual may be reproduced, modified or translated by any individual or organization without the written consent of Shenzhen Mindray Scientific Co., Ltd.



MINDRAY, **BeneFusion**, **mindray** and  are the registered trademarks or trademarks of Mindray. With the consent of Mindray, Shenzhen Mindray Scientific Co., Ltd., is authorized to use the above registered trademarks or trademarks.

Declaration

Shenzhen Mindray Scientific Co., Ltd. has the final right to interpret this service manual.

Shenzhen Mindray Scientific Co., Ltd. is responsible for the safety, reliability and performance of the product only if all of the following requirements are met:

- Assembly operations, expansion, re-adjustment, improvement and repair are performed by professionals recognized by Shenzhen Mindray Scientific Co., Ltd..
- The parts to be replaced in all repairs, associated accessories and consumables are original or approved by Shenzhen Mindray Scientific Co., Ltd..
- The relevant electrical equipment complies with China national standards and the requirements of this service manual.
- Product operations are performed in accordance with this manual.

6.6 Others

6.6.1 Others Parts List

No.	Order Number	Part Description	Qty	Remark
1	009-005810-00	4P-4P DC output wire 100mm	1	cable
2	009-006431-00	4P-6P motor wire 80mm	1	cable
3	115-039032-00	VP3 drip sensor assembly FRU	1	cable
4	115-013821-01	Drop sensor	1	accessory
5	047-016287-00	VP3 mask (EN)	1	/
6	047-016289-00	VP3 EX mask (EN)	1	/
7	047-016293-00	VP3 Vet mask (EN)	1	/
8	047-016544-00	VP3 mask (SP)	1	/
9	047-016545-00	VP3 mask (FR)	1	/
10	047-022203-00	VP3 Ex mask (FR)	1	/
11	047-022128-00	VP3 operation guide	1	/
12	047-016546-00	VP3 mask (RU)	1	/
13	047-024867-00	VP3 mask (RU) Jumedica	1	/
14	047-016547-00	VP3 mask (TR)	1	/
15	047-016548-00	VP3 mask (PR)	1	/
16	047-024874-00	VP3 Ukraine mask (ENG) Heaco	1	/
17	047-016295-01	VP1/VP3 operation guide (EN)	1	/
18	047-024875-00	VP3 Ukraine mask (RU) Heaco	1	/
19	047-016558-01	VP1/VP3 operation guide (SP)	1	/
20	047-016559-01	VP1/VP3 operation guide (FR)	1	/
21	047-016560-01	VP1/VP3 operation guide (RU)	1	/
22	047-016561-01	VP1/VP3 operation guide (TR)	1	/
23	047-016562-00	VP1/VP3 operation guide (PR)	1	/
24	047-018201-01	VP1/VP3 operation guide (DE)	1	/