
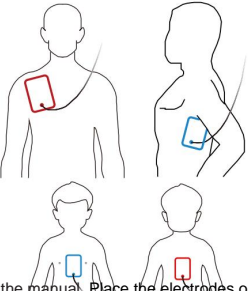
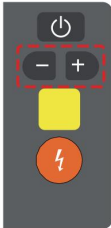
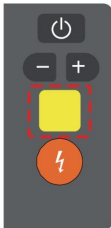
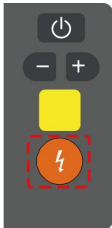



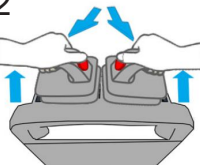
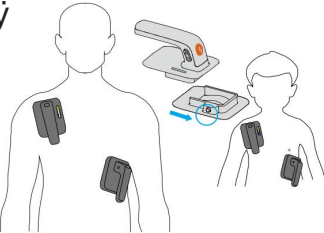
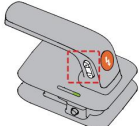
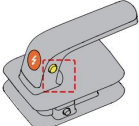
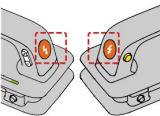
BeneHeart Series Monitor/Defibrillator Quick Reference Guide

(for BeneHeart D60/D50A/D50/D50C/D30/D20A/D20/D20C)







Use electrodes for manual defibrillation

<p>1</p>  <p>Press the power switch and then select the Defibril./Marcap shortcut key. If necessary, press the energy select button in the manual. Place the electrodes on the patient.</p>	<p>2</p>  <p>as shown in the image.</p>	<p>ÿ</p>  <p>front panel.</p>	<p>ÿ</p>  <p>Press the Charge button on the front panel.</p>	<p>5</p>  <p>Press the Download button on the front panel.</p>
---	--	--	---	---

Use external paddles for manual defibrillation

<p>1</p>  <p>Press the power switch, and then select the Defibril./Marcap shortcut key for manual defibrillation procedures.</p>	<p>2</p>  <p>Push both handles. Press the button firmly and lift the external blades. Place the external blades in the external blade power selector.</p>	<p>ÿ</p>  <p>patient as shown in the image.</p>	<p>ÿ</p>  <p>the outer blades.</p>	<p>5</p>  <p>Press the External Blade Charge button.</p>	<p>ÿ</p>  <p>Press the Download button on the external blades simultaneously.</p>
---	--	--	--	---	--

Status indicator

- Solid green:**  A power supply is connected external and the equipment works correctly.
- Only the battery is connected; the equipment It is turned on and working properly.**
- Only the battery is connected; the equipment is switched off and functioning correctly.**
- Flashing green:**  Self-check failure, or an error has been detected in the equipment.
- Red with blinking:**  The battery is connected and its charge level is low (battery symbol) or the battery is faulty (battery  symbol). 
- Only the external power supply is connected; the " **No battery**" option is selected. It is configured as an **active status indicator** on the equipment.**
- Off**  The external power supply and battery are disconnected.

Cleaning and disinfection

Make sure to clean and disinfect the equipment before performing maintenance tasks.

Commonly recommended cleaning products: water and ethanol (70%)


Commonly recommended disinfectants: Wipes for DIAN'ERKANG surfaces, ethanol (70%), sodium hypochlorite (0.5%), JIAN ZHI SU surface disinfectant spray

Maintenance

A test must be performed on each shift (at least once a day) or after each use. For more information, consult your institution or refer to the operator's manual.

The equipment must be turned off and connected to the external power supply and battery after each shift test.

A user test must be performed weekly or whenever the status indicator flashes red. To perform the user test, follow this procedure:

1. Press the power switch to turn on the equipment.
2. Select the shortcut key **Main Menu**  from the **Common** column select **User Test**.
3. If there is an error, fix it by following the instructions shown.
4. Select **User Test**.
5. Perform the operations according to the instructions provided. They are displayed until the device indicates that the user test has been passed.

