



Address: Zibo City, Shandong Province, China
 Tel: 008618560896869 Fax: 00862222678
 www.shanchuangroup.com Email: sales@shanchuangroup.com

DATASHEET OF VAGINAL SPECULUM FOR SINGLE USE

Product:	Sterile Vaginal Speculum For Single Use	- For Single Use
REF. Code	QR10-0101, QR10-0102, QR10-0103	- EO Gas Sterilized
Model 1	S,M,L SIZE	- Non-Toxic
Type	Side Screw/Lateral Screw Type	- Non-Pyrogenic
Materials:	PS	- Latex Free
Sterilization	Ethylene Oxide (E.O)	STERILE EO CE 0123
Shelf Life	5 Years	
Classification	MDD 93/42/EEC, Class IIa	
Certificates	CE,ISO13485	
Manufacturer	Shanchuan Group---Changzhou Chuangjia Medical Appliance Co., Ltd	
Intended Uses	Intended for as a tool for women checking.	
Storage	Should Be Stored in The Relative Humidity Not to Surpass 80%, No Corrosive Gas, Cool, Good Ventilated, Dry and Clean Room.	
Cautions	> Check The Lot No. And Exp. Date Carefully Before Use. > Don't Use If The Package Is Damaged. > Use Immediately After Unsealed. > For Single Use Only, Destroy After Use.	
Instructions	1:Tear The Inner Individual Package. 2: Gently insert the speculum with your right hand: 3:Fully insert the speculum with the screw facing sideways and the blades vertical Rotate 90 degrees during insertion so the screw faces upwards and the blades become horizontal 4: Slowly open the blades and use light to inspect the cervix	

	Item	Components	Material
	1	Upper Blade	PS
	2	Lower Blade	PS
	4	Handle	PS