


### 3.2.2 FH-20HR, FH-20HS, FH-21HR, FH-21HS

#### Control Panel

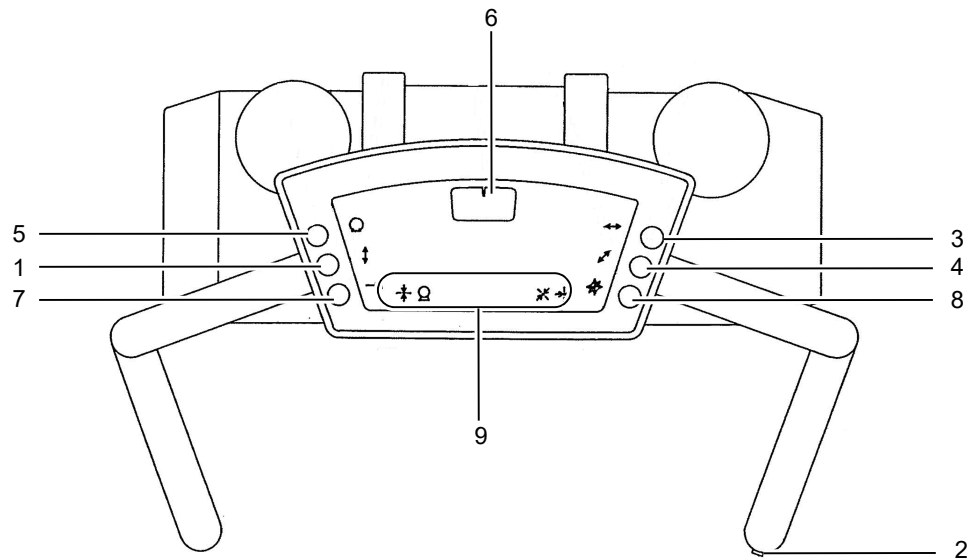

No.	Name	Function
1	Lock release switch for vertical movement	The X-ray tube unit can be moved upward or downward.
2	Lock/release switch for vertical movement (for about 5 seconds)	
3	Lock release switch for longitudinal movement	The X-ray tube unit can be moved along the guide rail (the floor and the ceiling rails).
4	Lock release switch for lateral movement	The X-ray tube unit can be moved forward or backward.
5	Lock release switch for tilt	The X-ray tube unit can be rotated.
6	Tilt indicator of X-ray tube unit	Displays the rotation angle of the X-ray tube unit.
7	Center stop mode switch	Enter the center position stop mode by pressing the center position stop mode switch.
8	Simultaneous lock/release switch for vertical movement, longitudinal movement, and lateral movement	Releases the locks for vertical, longitudinal, and lateral movements simultaneously.
9	Indicator lamp	Indicates system status by blinking or changing color.