# STORZ

KARL STORZ — ENDOSKOPE

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**INSTRUCTION MANUAL**  
IMAGE1 S™ 4U camera head TH 120

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Thank you for placing your trust in the name KARL STORZ. Like all of our other products, this product is the result of years of experience and great care in manufacture. You and your organization have decided in favor of a modern, high quality piece of equipment from KARL STORZ.

This instruction manual, in combination with the IMAGE1 S™ instruction manual (96206286D), is intended to aid you in the proper installation, connection, and operation of this device. All required details and all actions on your part are clearly presented and explained. Please read this manual carefully before proceeding with the equipment. Keep this instruction manual available for ready reference in a convenient location near the equipment.

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**It is recommended to check the suitability of the product for the intended procedure prior to use.**

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## 1 Safety instructions

Safety instructions are measures intended to protect the user and patients from the risks which could arise through the use of the system.

### 1.1 Explanation of warnings and cautions

Please read this manual and follow the instructions carefully. The words **Warning**, **Caution** and **Note** convey special meanings. Wherever they are used in this manual, the accompanying text should be carefully reviewed to ensure the safe and effective operation of the system. To make these words stand out more clearly, they are accompanied by a pictogram.

**WARNING:** A Warning indicates that the personal safety of the patient or physician is at risk.

**CAUTION:** A Caution indicates that particular service procedures or precautions must be followed to avoid possible damage to the device.

**NOTE:** A Note indicates special information about operating the device or clarifies important information.

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**1.2 General safety information**

**WARNING:** Read this instruction manual thoroughly before using the equipment.

**WARNING:** Check that the equipment is functioning properly prior to each surgical intervention. Ensure that the correct video image appears on all video monitors before beginning each intervention. It is recommended that a replacement system is kept available.

**WARNING:** Perceived electric shocks may be experienced when using the camera system in conjunction with electrosurgical units.

**WARNING:** When using high frequency surgical accessories, care must be taken to avert contact between the HF surgical device and the endoscope or endoscopic camera. Inadvertent contact may represent a thermal hazard to the surgeon.

**WARNING:** When energized endoscopes are used with energized endotherapy devices, patient leakage currents may be additive. It should be noted that this is particularly important if a Type CF applied part endoscope is used, in which case a Type CF applied part energized endotherapy device should be used in order to minimize total patient leakage current.

**WARNING:** Always ensure the connector is completely clean and free of debris. If moisture is present, dry thoroughly with sterile towel prior to insertion.

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**WARNING:** All endoscopic devices (e.g., HF surgical probes) used in conjunction with KARL STORZ endoscopic cameras must comply with IEC 60601-2-18, the standard for medical electrical equipment – particular requirements for the safety of endoscopic equipment. The insulation of the endoscopic devices must be designed to withstand the highest HF voltage encountered during surgery. HF-generating equipment is dangerous and cannot be allowed to make contact with the camera head or system via the accessories or endoscopes connected to the camera head or system.

**WARNING:** Risk of blindness. Never look into the open end of a light cable.

**WARNING:** Risk of burns. Contact with the open end of a light cable connected to a light source may cause burns.

**WARNING:** Test all equipment prior to each surgical use. In the event that the image becomes unusable during surgery, the camera may be disengaged from the endoscope and the procedure continued optically. If this is not possible, it is up to the discretion of the surgeon how best to proceed. Availability of a spare system is recommended.

**WARNING:** High energy radiated light through endoscopes may give rise to high temperatures in front of the light outlet and to the tip of the endoscope. To minimize the risk of burns, always adjust the light source to the minimum illumination intensity necessary to achieve optimum illumination of the endoscopic scene when coupled to the video camera.

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## 2 Intended Use Camera Heads

The camera head is used together with the camera control unit (CCU) and is attached to a telescope during endoscopic or microscopic procedures.

### U.S. Indications for Use

The IMAGE1 S™ 4U camera head is intended to be used with the IMAGE1 S™ Camera Control Unit (CCU) and compatible endoscope for visualization, image recording and documentation during general endoscopic and microscopic procedures.

### 2.1 Contraindications

No contraindications relating directly to the medical device are currently known. The responsible physician must decide whether the foreseen application is admissible based on the general condition of the patient.

### 2.2 User qualification

The IMAGE1 S™ 4U camera head may only be used by physicians and medical assistants who have a corresponding specialized qualification and who have been instructed in the use of the IMAGE1 S™ 4U camera head.

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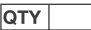
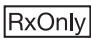
- ① Instrument coupler
- ② Focusing ring/knob
- ③ User control button: for scrolling (downwards) through the menu or activating a camera function
- ④ User control button: for scrolling (upwards) through the menu or activating a camera function
- ⑤ User control button: for accessing the menu/ making a selection

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**4 Symbols employed**

	Fragile, handle with care
	Keep dry
	Temperature limit
	Serial number
	Number of products in the product packaging
	CE marking
	top
	Consult instructions for use
	Manufacturer
	Federal (USA) law restricts this device to sale by or on the order of a physician.
	This equipment is MR Unsafe. Keep the device away from the Magnetic Resonance Imaging (MRI) Scanner Room and mobile MRI scanner devices.

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## 5 Operating instructions for the IMAGE1 S™ 4U camera head

**i** **NOTE:** While operating with surgical lasers intensive laser light may occur and affect image visualization.

### 5.1 Frequency of the IMAGE1 S™ 4U camera head (50/60 Hz)

The IMAGE1 S™ 4U camera head is suitable for operation at either 50 Hz or 60 Hz. The camera head must be selected to suit the local conditions.

**i** **NOTE:** The method for switching frequency is described in the instruction manual IMAGE1 S™ – TC 200, TC 300, TC 301 (96206286D).

### 5.2 Operating the IMAGE1 S™ 4U camera head

#### 5.2.1 Connecting an endoscope and light cable

The IMAGE1 S™ 4U camera head has an integrated instrument coupler and integrated telescope. To connect an endoscope to the instrument coupler, turn the outer ring of the coupler clockwise and insert the endoscope eyepiece. To prevent rotation of the endoscope, the outer ring of the instrument coupler must be turned counterclockwise until tight. Connect the light cable to the endoscope eyepiece by tightening the knurled screw on the light cable through a quarter turn.

**!** **CAUTION:** Heat from the light source can cause disinfectant residue on the light cable or on the light inlet of the endoscope to burn. This can have a detrimental effect on the light conduction and thus the image quality.

**i** **NOTE:** For further information regarding light cables, please refer to the manual 'Fiberoptic light cables' (97000213), which can be downloaded or requested at [www.karlstorz.com](http://www.karlstorz.com).

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### 5.2.2 Adjusting the focus

The focus on the IMAGE1 S™ 4U camera head can be adjusted by turning the focusing ring/knob ② on the camera lens.

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### 6.2 Storage and Operating conditions

Refer to the compatible camera control unit (CCU) manual for relevant storage and operating conditions. The CCU and camera head were evaluated as a system for IEC 60601-1.

The camera should be stored in its sterile packaging at room temperature. It is to be kept in a clean, well-ventilated area that provides protection against dust and moisture. The camera should be stored in a way that will not cause damage to the packaging or the camera. If the integrity of the packaging is compromised (e.g., torn, wet, or punctured), repack and re-sterilize the camera before use. Consult facility guidelines and recommendations on timeframes for sterilized device storage.

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### 6.3 Standard compliance

**According to IEC 60601-1; EC 60601-2-18.**

Provides Type CF protection when used with KARL STORZ Camera Control Units that have CF symbol on connection port.

### 6.4 Directive compliance

**According to the Medical Device Directive (MDD):**

This medical device belongs to Class I

This medical device bears a CE mark in accordance with MDD 93/42/EEC.

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**7 Reprocessing  
(not validated for  
US customers)**

In the following chapter, you will find information on care and cleaning of the IMAGE1 S™ 4U camera head.

**7.1 Care and handling**

To maximize the life of your camera, please follow the guidelines given below for the correct handling and care of the camera head:

DO NOT PLUG THE CAMERA HEAD CABLE INTO THE PROCESSOR RECEPTACLE IF IT IS MOIST OR NOT CLEAN. Always ensure that the connector is completely clean. Remove any moisture present with a sterile cloth before connecting.

DO NOT PICK UP THE CAMERA HEAD BY ITS CABLE. Always pick up the camera head first, and then pick up and support the cable and connector. The cable contains many tiny conductors and must therefore be handled with care.

DO NOT STORE THE CAMERA SYSTEM in direct sunlight or at high temperatures.

ENSURE THAT THE CABLE IS NOT COILED TOO TIGHTLY OR FOLDED UP WHILE THE CAMERA IS IN STORAGE. Always arrange the cable in loose coils with a diameter of 6 inches (15 cm) or more.

DO NOT PULL FORCEFULLY ON THE CABLE WHEN UNCOILING. Undesirable 'kinks' may result. Always allow the cable to uncoil gently and loosely.

HANDLE THE SYSTEM WITH CARE, avoiding any strong impacts. When transporting, use the original packaging to prevent damage to the device.

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- Sterilization method has been validated.
- X** On the grounds of potential damage, this reprocessing step must not be performed.

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**7.2.2 General warnings**

**WARNING:** Risk of infection: Incorrectly reprocessed medical devices expose patients, users and third parties to a risk of infection as well as the risk that the medical device may malfunction. Observe the manual 'Cleaning, Disinfection, Care and Sterilization of KARL STORZ Instruments' and the accompanying documentation.

**WARNING:** Risk of infection: These medical devices are not sterile upon delivery. The use of non-sterile medical devices poses a risk of infection for patients, users and third parties. Inspect medical devices for visible contamination. Visible contamination is an indication that reprocessing has not been carried out or has been carried out incorrectly. Reprocess the medical devices before initial use and before and after every subsequent use using validated procedures.

**WARNING:** When carrying out any work on contaminated medical devices, national regulations regarding personal safety must be observed.

**CAUTION:** When preparing and using solutions, follow the chemical manufacturer's instructions paying close attention to proper concentration, exposure time, and service life. Prolonged immersion and incorrect concentration may result in damage. Bear in mind the microbiological range of action of the chemicals used.

**CAUTION:** Danger of damage to medical devices: Medical devices may be damaged by using chemicals that have not been approved by KARL STORZ.

**CAUTION:** National laws and regulations must be observed.

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**i** **NOTE:** The manual 'Cleaning, Disinfection, Care and Sterilization of KARL STORZ Instruments' can be downloaded or requested by visiting [www.karlstorz.com](http://www.karlstorz.com).

**!** **CAUTION:** Inspect the camera head cable for breaks and cracks before and after reprocessing. Camera heads with damaged cables must NOT be reprocessed. A break or cut in the cable will allow fluids to enter the camera cable and cause damage. Return the damaged camera head to KARL STORZ for repair.

**!** **CAUTION:** Chemical residue on the camera head plug can cause failure of the camera electronics.

**!** **CAUTION:** The camera head must be thoroughly cleaned and disinfected/sterilized using validated infection control procedures before use and before every subsequent application. Any deviations from the recommended parameters for cleaning, disinfection and sterilization should be validated by the user.

**!** **CAUTION:** The use of sterilization/disinfection methods other than those recommended by KARL STORZ may have adverse effects on the materials of the camera and endoscopic adaptor. Any resulting damage will not be covered by the warranty.

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**Reprocessing  
(not validated for  
US customers)****7.3 Accessories**

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- Brush: 27652

**7.4 Preparation for cleaning  
and disinfection**

Heavy soiling, corrosive solutions and pharmaceuticals must be removed from the medical device immediately after use. To this end, preclean the medical device by wiping down and rinsing, for example. As a general rule, KARL STORZ recommends manual precleaning under cold running water.

**CAUTION:** Inspect the camera head cable for breaks or cracks. Camera heads with damaged cables must not be sterilized or disinfected. Following wipe-down disinfection, return camera heads with damaged cables to KARL STORZ for repair.

**7.4.1 Manual wipe-down disinfection**

The exterior surfaces of the medical device must be wiped clean with a disposable cloth moistened with disinfectant or a ready to use soaked disinfectant cloth. Due to their protein-fixating effect and material incompatibility, alcohol-based agents must not be used. The chemical manufacturer's specifications regarding material compatibility must be observed. At the end of the necessary exposure time for the disinfectant as specified by the manufacturer, wipe the surfaces with a dry low lint cloth.

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**7.4.2 Manual precleaning****7.4.2.1 Brushing the surfaces**

Depending on the degree of soiling and the degree to which residues have dried, heavy soiling must be cleaned from the surfaces of instruments with the aid of a brush/sponge under cold running water until any visible contamination has been completely removed.

**7.4.2.2 Auxiliary pretreatment with ultrasound**

For technical reasons, the medical device is not suitable for ultrasound treatment.

**7.4.3 Manual cleaning**

The medical device must be completely immersed in a cleaning solution. It must be ensured that even surfaces with restricted access are covered and no air bubbles are present. After the necessary exposure time as specified by the manufacturer, clean the instrument mechanically with the aid of a brush or sponge. The device must then be rinsed with cold water to ensure neutralization.

**7.4.4 Manual disinfection**

The medical device must be completely immersed in a disinfectant solution. It must be ensured that even surfaces with restricted access are covered and no air bubbles are present. After the necessary exposure time as specified by the manufacturer, the medical device must be rinsed several times in order to remove all chemical residues. To this end, water of the best possible quality must be used, taking into account the national regulations. Finally, all of the surfaces and joints are dried completely with (preferably sterile) compressed air. The cleaning gun with accessories (Art. no. 27660) is ideal for this purpose.

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- Wipe the camera head connector with a cotton tip applicator soaked in 70 % isopropyl alcohol to remove any residual disinfectant. Do not allow exposed glass windows to air dry. Glass surfaces can be cleaned using a cotton tip applicator soaked in 70 % isopropyl alcohol to prevent streaks and spots.
- Afterwards, a functional check must be carried out.

Before sterilization and/or disinfection, coil the camera head cable into loops with a diameter of at least 6 inches (15 cm). In doing so, do not kink or twist the cable.

**i NOTE:** During care procedures, use items from the catalog 'Care, Sterilization and Storage Techniques'.

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### 7.7 Sterilization

The procedures as well as the process-relevant parameters for the individually validated methods are described in detail in the manual ‘Cleaning, Disinfection, Care and Sterilization of KARL STORZ Instruments’. The procedure must be selected taking the respective applicable national requirements into account and in consultation with the device manufacturers.

**The following sterilization methods have been validated and approved by KARL STORZ for the IMAGE1 S™ 4U camera head:**

#### 7.7.1 Hydrogen peroxide (H<sub>2</sub>O<sub>2</sub>) sterilization – ASP STERRAD®

**WARNING:** Remember that there are restrictions concerning the instruments which can be sterilized in the different STERRAD® sterilization systems in respect of dimensions and material.

**CAUTION:** For more detailed information, please consult the user handbook of the respective device.

**CAUTION:** STERRAD® sterilization may cause cosmetic changes to the camera head that do not necessarily impact the camera’s functionality.

**NOTE:** Refer to the ‘STERRAD® Sterility Guide’ to ensure that the medical device concerned can be sterilized using the different STERRAD® devices.

**The following sterilization methods have been validated and approved by KARL STORZ for this medical device:**

- STERRAD® 100S
- STERRAD® NX® ‘standard’ cycle
- STERRAD® 100NX® ‘standard’ cycle
- ‘DUO’ cycle

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Detailed information on the selection of the appropriate cycle for the various device generations is available from the STERIS® manufacturer.

**CAUTION:** V-PRO® sterilization may cause cosmetic changes to the camera head that do not necessarily impact the camera's functionality.

**7.7.3 Ethylene oxide sterilization (EO)**

The ethylene oxide procedure is validated with 100 % ethylene oxide at 55 °C and a hold time of 30 – 45 minutes.

**WARNING:** Medical devices must be dismantled for sterilization.

**NOTE:** Place the camera head in a sterilization tray. Do not place any instruments on top of the camera.

**CAUTION:** For gas sterilization with ethylene oxide, the airing times stipulated by the device manufacturer must be observed due to gas absorption by the materials. The airing times depend on the processing technique of the EO sterilizer (concentration, process control).

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**7.7.4 Chemical low-temperature  
sterilization with peracetic acid  
– STERIS® System 1E®**

More detailed information on the selection of the sterilization parameters and the STERIS® Quick Connect Kit (QCK) required to rinse out the lumina is available from the manufacturer STERIS®.

**7.7.5 High-Level Disinfection**

**WARNING:** High-level disinfection should ONLY be used for instruments which come into contact with intact mucous membranes.

**CAUTION:** Any deviations from the recommended parameters for disinfection must be validated by the user.

**CAUTION:** Before disinfection, the instruments must be thoroughly cleaned, rinsed, and dried.

**CAUTION:** When preparing and using solutions, follow the chemical manufacturer's instructions paying close attention to proper concentration, exposure time, and service life.

**The following high-level disinfection methods  
have been validated and approved by  
KARL STORZ:**

**7.7.6 CIDEX (2.4 % glutaraldehyde  
solution)**

In order to prevent potential damage, do not place camera head in a container with other instruments. Completely immerse the camera head in the undiluted 2.4 % glutaraldehyde solution for a minimum of 45 minutes at 25 °C (77 °F). It must be ensured that the surfaces are covered and that no air bubbles are present.

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To finish, rinse the camera head three times with sterile water. Each rinse should be for a minimum of 1 minute in duration. Discard the water after each rinse, as it will be contaminated with the disinfectant solution. Use fresh sterile water for each rinse.

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**Reprocessing  
(not validated for  
US customers)**

Thorough rinsing of the camera head and the card-edge connector with sterile water is very important for removing any residual disinfectant. Dry the camera head and the card-edge connector with a lint-free, sterile cloth. Ensure that the card-edge connector is completely clean and free of moisture before inserting it into the camera processor. Wipe the camera head connector with isopropyl alcohol to remove any residual disinfectant.

**7.7.7 Resert XL HLD (2.0 % hydrogen  
peroxide solution)**

In order to prevent potential damage, do not place camera head in a container with other instruments. Completely immerse the camera head in the undiluted 2.0 % hydrogen peroxide solution for a minimum of 8 minutes at 20 °C (68 °F). It must be ensured that the surfaces are covered and that no air bubbles are present.

**CAUTION:** Do not leave the camera head in the solution for any longer than 60 minutes.

To finish, rinse the camera head three times with sterile water. Each rinse should be for a minimum of 1 minute in duration. Discard the water after each rinse, as it will be contaminated with the disinfectant solution. Use fresh sterile water for each rinse. Thorough rinsing of the camera head with sterile water is very important for removing any residual disinfectant.

Dry the camera head and the card-edge connector with a lint-free, sterile cloth. Ensure that the card-edge connector is completely clean and free of moisture before inserting it into the camera processor. Wipe the camera head connector with isopropyl alcohol to remove any residual disinfectant.

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- Sterilization method has been validated.
- X On the grounds of potential damage, this reprocessing step must not be performed.

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**8.2.2 General warnings**

**WARNING:** Risk of infection: Incorrectly reprocessed medical devices expose patients, users and third parties to a risk of infection as well as the risk that the medical device may malfunction.

**WARNING:** Risk of infection: These medical devices are not sterile when delivered. The use of non-sterile medical devices poses a risk of infection for patients, users and third parties. Inspect medical devices for visible contamination. Visible contamination is an indication that reprocessing has not been carried out or has been carried out incorrectly. Reprocess the medical devices before initial use and before and after every subsequent use using validated procedures.

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**CAUTION:** When preparing the cleaning solutions, follow the manufacturer's instructions for dilution to proper concentration and temperature.

**CAUTION:** Danger of damage to medical devices: The use of chemicals which have not been approved by KARL STORZ may cause damage to the medical devices.

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**CAUTION:** Inspect the camera head cable for breaks and cracks before and after preparation. Camera heads with damaged cables must NOT be processed. A break or cut in the cable will allow fluids to enter the camera cable and cause damage. Return the damaged camera head to KARL STORZ for repair.

**CAUTION:** Chemical residue on the camera head plug can cause failure of the camera electronics.

**CAUTION:** The camera head must be thoroughly cleaned and sterilized using validated infection control procedures before use and before every subsequent application.

**CAUTION:** The use of sterilization methods other than those recommended by KARL STORZ may have adverse effects on the materials of the camera and endoscopic adaptor. Any resulting damage will not be covered by the warranty.

**CAUTION:** Use only one method of sterilization per reprocessing cycle.

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### 8.3 Accessories

Accessories required for carrying out reprocessing:

- Brush: 27652

### 8.4 Water Quality Requirements

#### Utility water:

Water as it comes from the tap that meets specifications as defined per AAMI TIR34 Water for Reprocessing of Medical Devices. Utility water is acceptable for pre-cleaning and cleaning of the instruments.

#### Critical water:

Water that is extensively treated (usually by a multistep treatment process that could include a carbon bed, softening, DI and RO, or distillation) to ensure that the microorganisms and the inorganic and organic material are removed from the water; a final submicron filtration could also be part of the treatment process. Critical water is recommended for rinsing after high-level disinfection.

### 8.5 Bedside pre-cleaning

1. Immediately after a procedure, at the point of use, detach the camera head from the endoscope and CCU.
2. Wipe the camera head with a soft, lint-free, disposable cloth moistened with water or enzymatic cleaning solution to remove all heavy soiling.
3. Contain and transport the camera head to the decontamination area for cleaning.
4. If immediate cleaning is not feasible, place the camera head in a container and soak in neutral/mild pH enzymatic cleaning solution (e.g. Enzol), diluted with distilled/demineralized water to proper concentration per manufacturer's instructions for no longer than an hour.

**i NOTE:** Complete cleaning of the camera head should be started within two hours of the bedside pre-cleaning.

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### 8.5.1 Manual cleaning

1. Prepare neutral pH enzymatic solution (e.g. Enzol) in a container large enough to fully submerge the camera head. Dilute the solution to proper concentration per manufacturer's instructions.
2. Under running cold tap water:
  - Rinse the camera head to remove all gross debris, paying close attention to the seam beneath the focus ring, for 2 minutes.
  - Brush all camera head surfaces using the soft bristle brush (P/N 27652), paying close attention to the seam beneath the focus ring. Brush for 2 minutes or until there is no visible debris on the brush.
3. Completely immerse the camera head in a neutral pH enzymatic solution. Keep immersed for 5 minutes.
 

**NOTE:** If heavy debris collects on the brush while brushing, rinse the brush until there is no visible debris on the brush.
4. While immersed, use the soft bristle brush (P/N 27652) to:
  - Brush all camera head surfaces, paying close attention to the seam beneath the focus ring. Brush for 1 minute or until there is no visible debris on the brush.
5. While immersed, use the soft bristle brush (P/N 27652) to brush the following locations for 30 seconds each:
  - Entire silicone cable
  - Card edge connector
  - Camera head buttons
  - Cam-Lock mechanism, including its crevices within the coupling mechanism
6. While immersed, use a soft, lint-free cloth to wipe the entire camera head cable.
 

**NOTE:** If heavy debris collects on the brush while brushing, rinse the brush until there is no visible debris on the brush.

**NOTE:** If heavy debris collects on the brush while brushing, rinse the brush until there is no visible debris on the brush.

**CAUTION:** Do not brush the glass window of the camera head. Doing so may cause damage to the glass. Use a soft, lint-free cloth.

**NOTE:** If heavy debris collects on the brush while brushing, rinse the brush until there is no visible debris on the brush.

**NOTE:** If heavy debris collects on the brush while brushing, rinse the brush until there is no visible debris on the brush.

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7. Remove the camera head from the cleaning solution.
8. Under running cold tap water, rinse the entire camera head and cable for 3 minutes including:
  - Focus ring
  - Camera head buttons
  - Cam-Lock coupling mechanism
  - Crevices and seams
  - Entire cable
9. Repeat rinsing (step 8) for a total of 3 times.
10. Dry the camera head with a soft, lint-free, disposable cloth or compressed air.

### 8.6 Assembly, inspection and care

The cleaned medical device must be visually inspected for cleanliness, completeness, damage and dryness:

- If residues or contamination are still present, the medical device must be manually cleaned and subjected to a full cleaning procedure once more.
- Camera heads with damaged cables must be withdrawn from use.

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- Wipe the camera head connector with a cotton tip applicator soaked in 70 % isopropyl alcohol. Do not allow exposed glass windows to air dry. Glass surfaces can be cleaned using a cotton tip applicator soaked in 70 % isopropyl alcohol to prevent streaks and spots.

Before sterilization, coil the camera head cable into loops with a diameter of at least 6 inches (15 cm). In doing so, do not kink or twist the cable.

**WARNING:** The medical device must be sterilized using an approved and validated sterilization method following manual cleaning.

### 8.7 Packaging systems

Only standardized and approved packaging materials or systems may be used (EN 868 Parts 2 – 10, EN ISO 11607 Parts 1 + 2, DIN 58953). National regulations must be observed.

### 8.8 Sterilization

**WARNING:** Before sterilization, the instruments must be thoroughly cleaned and all visible organic material, blood and cleaning solution completely removed.

The procedure must be selected taking the respective applicable national requirements into account and in consultation with the device manufacturers.

**The following sterilization methods have been validated and approved by KARL STORZ for IMAGE1 S™ 4U:**

#### 8.8.1 Hydrogen peroxide (H<sub>2</sub>O<sub>2</sub>) sterilization – ASP STERRAD®

**CAUTION:** STERRAD® sterilization may cause cosmetic changes to the camera head that do not necessarily impact the camera's functionality.

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## Reprocessing (for US customers only)

 **WARNING:** Remember that there are restrictions concerning the instruments which can be sterilized in the different STERRAD® sterilization systems in respect of dimensions and material.

 **CAUTION:** For detailed information, please consult the user handbook of the respective device.

 **NOTE:** To ensure that the medical device concerned can be sterilized using the different STERRAD® units, refer to the ‘STERRAD® Sterility Guide’.

 **CAUTION:** Use only FDA-cleared polypropylene sterilization wraps and/or polyolefin pouches. Do not use paper pouches or sterilization wraps containing wood pulp or cotton.

1. Place the camera head in an FDA-cleared STERRAD® compatible instrument tray.
2. Wrap the sterilization tray with two single layers of FDA cleared polypropylene sterilization wrap, using a sequential double wrapping technique. If using a sterilization sealed container refer to the manufacturer's instructions for use on using the correct disposable filters for the indicated sterilization method.
3. Load the STERRAD® sterilizer, arranging the items such that the hydrogen peroxide plasma can surround them. Do not allow any items to touch the wall of the sterilizer.
4. Start the STERRAD® sterilizer. The following cycles have been validated:
  - STERRAD® 100S
  - STERRAD® NX® ‘Standard’ Cycle
  - STERRAD® 100NX® ‘Standard’ Cycle
  - ‘DUO’ cycle

### 8.8.2 Hydrogen peroxide (H<sub>2</sub>O<sub>2</sub>) sterilization – STERIS® AMSCO® V-PRO® 1

 **CAUTION:** V-PRO® sterilization may cause cosmetic changes to the camera head that do not necessarily impact the camera's functionality.

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**Reprocessing  
(for US customers only)**

1. Place the camera head in an FDA-cleared V-PRO® compatible instrument tray.
2. Wrap the sterilization tray with two single layers of FDA cleared polypropylene sterilization wrap, using a sequential double wrapping technique. If using a sterilization sealed container refer to the manufacturer's instructions for use on using the correct disposable filters for the indicated sterilization method.
3. Load the V-PRO® sterilizer, arranging the items such that the vaporized hydrogen peroxide can surround them. Do not allow any items to touch the wall of the sterilizer.
4. Start the V-PRO® sterilizer. The following cycles have been validated:
  - V-PRO® 1 'Lumen' Cycle
  - V-PRO® 1 Plus 'Lumen' and 'Non-lumen' Cycle
  - V-PRO® maX 'Lumen' and 'Non-lumen' Cycle

**8.8.3 Ethylene oxide sterilization  
(EO)**

1. Place the camera head in an FDA-cleared sterilization tray.
2. Double wrap the sterilization tray with polypropylene wrap. If using a sterilization sealed container refer to the manufacturer's instructions for use on using the correct disposable filters for the indicated sterilization method.
3. Start sterilization under the following parameters:

**Conditioning parameters**

Temperature: 55 °C (131 °F)  
Humidity: 70 % RH  
Conditioning dwell time: 30 minutes

**Sterilization parameters**

Sterilant: 100 % ethylene oxide  
Temperature: 55 °C (131 °F)  
Gas concentration: 735±30 mg/l  
Exposure time: 180 minutes

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**8.8.4 Chemical low-temperature  
sterilization with peracetic acid  
– STERIS® system 1E®  
(only valid for USA)**

1. Place the camera head in one of the following trays:
  - C1200E SS1E General Processing Tray/ Container
  - C1220E SS1E Directed Flow Processing Tray/Container
  - C1140E SS1E Flexible Endoscope Processing Tray/Container
  - C1160E SS1E Universal Flexible Processing (Open) Tray
2. Load the tray into the SS1E and start the sterilizer.

**8.9 Limits of reprocessing**

The end of the product's lifespan is largely determined by wear, reprocessing methods, the chemicals used and any damage resulting from use.

Always perform a functional test with the full IMAGE1 S™ system prior to using the camera head. The system should not be used if any of the accessories are damaged or non-functional.

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## 9 Maintenance

### 9.1 Maintenance

Preventive maintenance is not essential. Regular maintenance can, however, contribute to identifying potential problems before they become serious, thus enhancing the instrument's reliability and extending its useful service life. Maintenance services can be requested from your local representative or from the manufacturer (see below for functional testing).

### 9.2 Servicing and repair

Defective items of equipment must be serviced and repaired exclusively by KARL STORZ or by persons authorized by KARL STORZ; all repair work must employ original KARL STORZ parts only.

**CAUTION:** Do not attempt to open the camera head or lens. Opening these sealed assemblies affects the camera's soakability and voids all warranties. Do not remove the covers on the camera processor. Warranty is voided unless removed by authorized personnel.

### 9.3 Disposal

This unit has been marked in accordance with the European Directive on waste electrical and electronic equipment (WEEE).

At the end of its useful operating life, dispose of the unit as electronic scrap.

Please ask either KARL STORZ SE & Co. KG, a KARL STORZ subsidiary or your specialist dealer for information on your local collection point. Within the scope of application of this Directive, KARL STORZ SE & Co. KG is responsible for the proper disposal of this unit.

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**9.4 Repairing devices**

Only personnel authorized by the manufacturer may repair devices. Interventions described in these instructions for use are exempt from this rule.

In Germany, you can contact the following address for repairs:

KARL STORZ SE & Co. KG  
Abt. Reparaturservice  
Take-Off Gewerbepark 83  
78579 Neuhausen

Service hotline: +49 7461 708-980  
E-mail: technicalsupport@karlstorz.com

In other countries, the respective KARL STORZ branches or specialist dealers are responsible.

**9.5 Infection prevention**

Contaminated devices may not be shipped. To prevent contact infections and airborne infections, devices must first be decontaminated. KARL STORZ reserves the right to send back contaminated devices.

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### 9.6 Limitation of liability

KARL STORZ shall be liable for the safe operation, operational reliability, and performance of this equipment only subject to the following conditions:

- all assembly operations, system expansions, readjustments, modifications, or repairs have been performed by a person or persons duly authorized by KARL STORZ;
- all electrical installations at the location of use meet applicable national and local electrical codes, and
- the unit has been used in accordance with its operating instructions at all times.

### 9.7 Warranty

The guarantees provided can be found in the Standard Conditions of Business of KARL STORZ. The medical device must always be sent to your local subsidiary (see 'Subsidiaries' section), even during the warranty period.

Opening the equipment or performance of any repairs or modifications to the equipment by unauthorized persons shall relieve us of any liability for its performance. Any such opening, repair, or modification performed during the warranty period shall void all warranty.

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**Kompatible  
Kamera-Kontrolleinheit**

**Compatible  
camera control unit (CCU)**

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**10 Compatible  
camera control unit (CCU)**

Item	Cat. No.
IMAGE1 S™ 4U-LINK in combination with IMAGE1 S CONNECT®	TC 304
or IMAGE1 S CONNECT® II	TC 200
	TC 201*

\* IMAGE1 camera control units are available with various configurations and image outputs. These camera control units are compatible with IMAGE1 S™ camera heads. Please check your local region for device configuration availability.

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**11 Spare parts,  
recommended accessories**

Please check your local region for recommended accessory availability.

Article	Order no.
Adaptor, autoclavable, allows telescopes to be changed under sterile conditions	533 TVA
Integral rotating beamsplitter, for simultaneous viewing through the telescope and on the monitor	9530 BD
Dust cap for camera heads	6349190
Sterile drapes* for KARL STORZ cameras	
Instruction Manual	96286003D
Instruction manual IMAGE1 S™	96206286D

\* For safe and comfortable working with KARL STORZ cameras, we recommend the use of sterile drapes.

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**12 Information on  
electromagnetic  
compatibility (EMC)**

The IMAGE1 S™ 4U camera head TH 120 together with the IMAGE1 S™ is suitable for use within close proximity to an active HF electro-surgical device in professional healthcare facility environment. Professional healthcare facility includes physician offices, dental offices, limited care facilities, freestanding surgical centers, freestanding birth centers, multiple treatment facilities, hospitals (emergency rooms, patient rooms, intensive care, surgical rooms, outside the RF shielded room of an ME system for magnetic resonance imaging).

**WARNING:** This equipment is MR Unsafe. Keep the device away from the Magnetic Resonance Imaging (MRI) Scanner Room and mobile MRI scanner devices.

**WARNING:** Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the IMAGE1 S™ 4U camera head, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

**NOTE:** The emission characteristics of this equipment make it suitable for use in industrial areas as well as in hospitals (CISPR 11 Class A) and other professional healthcare environment. If it is used in a residential environment (for which CISPR 11 Class B is normally required) this equipment might not offer adequate protection to radio communication service. The user might need to take mitigation measures, such as relocating or re-orienting equipment.

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**Information on  
electromagnetic  
compatibility (EMC)**

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**i NOTE: HF surgical instrument compatibility**  
This equipment has been evaluated for compatibility with high-frequency (HF) surgical equipment. It has been tested according to Annex BB of the standard IEC 60601-2-2.

**i NOTE: RFID compatibility**  
This equipment has been evaluated for compatibility with Radio-Frequency Identification (RFID). It has been tested according to the standard AIM 7351731.

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<b>Table 1 – Compliance level for Immunity tests</b> <b>Guidance and manufacturer's declaration – electromagnetic immunity</b>			
The IMAGE1 S™ 4U camera head is intended for use in the electromagnetic environment specified below. The user of the IMAGE1 S™ 4U camera head should make sure that it is used in such an environment.			
<b>Immunity tests</b>	<b>EN/IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment – guidance</b>
Electrostatic discharge (ESD) acc. to IEC 61000-4-2	± 8 kV Contact discharge ± 15 kV Air discharge	± 8 kV Contact discharge ± 15 kV Air discharge	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Rapid transient electrical Interferences/bursts acc. to IEC 61000-4-4	± 2 kV for power lines ± 1 kV for input and output lines 100 kHz repetition	± 2 kV/1 kV ± 1 kV for input and output lines 100 kHz repetition	Mains power quality should be that of a typical commercial or hospital environment.
Surges acc. to IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and fluctuations of the supply voltage acc. to IEC 61000-4-11	<u>Voltage Dip:</u> Dip to 0 % for 1 cycles @ 0° phase angle Dip to 70 % for 25/30 cycles @ 0° phase angle Dropout to 0 % for 0.5 cycles @ 0°, 45°, 90°, 135°, 180°, 225°, 270° & 315° phase angles <u>Voltage interruption:</u> 100 % for 250/300 cycles	<u>Voltage Dip:</u> Dip to 0 % for 1 cycles @ 0° phase angle Dip to 70 % for 25/30 cycles @ 0° phase angle Dropout to 0 % for 0.5 cycles @ 0°, 45°, 90°, 135°, 180°, 225°, 270° & 315° phase angles <u>Voltage interruption:</u> 100 % for 250/300 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the IMAGE1 S™ 4U camera head require continued operation during power mains interruptions, it is recommended that the IMAGE1 S™ 4U camera head be powered from an uninterruptible power supply or a battery.
Magnetic field at the supply frequency (50/60 Hz) acc. to IEC 61000-4-8	30 A/m at 50 Hz/60 Hz	30 A/m at 50 Hz/60 Hz	If image distortion occurs, it may be necessary to position the IMAGE1 S™ 4U camera head further from sources of power frequency magnetic fields or to install magnetic shielding. The power frequency magnetic field should be measured in the intended installation location to assure that it is sufficiently low.
Radiated, radio-frequency, electromagnetic field immunity test acc. to IEC 61000-4-3	3 V/m 80 MHz to 2.7 GHz *Refer table 2 for wireless Proximity RF field test levels	3 V/m 80 MHz to 2.7 GHz	
Immunity to conducted disturbances, induced by radio-frequency fields acc. to IEC 61000-4-6	3 V <sub>rms</sub> on 150 kHz to 80 MHz 1 kHz 80% AM modulation 6 V <sub>rms</sub> in ISM bands	3 V <sub>rms</sub> on 150 kHz to 80 MHz 1 kHz 80% AM modulation 6 V <sub>rms</sub> in ISM bands	

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**Table 3 – Test levels for Radiated and conducted Immunity Tests**  
Guidance and manufacturer's declaration – electromagnetic immunity

The IMAGE1 S™ 4U camera head is intended for use in the electromagnetic environment specified below. The user of the IMAGE1 S™ 4U camera head should make sure that it is used in such an environment.

Immunity tests	EN/IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF disturbances acc. to IEC 61000-4-6	3 V <sub>rms</sub> on 150 kHz to 80 MHz 6 V <sub>rms</sub> in ISM bands between 0.15 MHz and 80 MHz	3 V <sub>rms</sub> on 150 kHz to 80 MHz 6 V <sub>rms</sub> in ISM bands between 0.15 MHz and 80 MHz	Portable and mobile RF communications equipment should be used no closer to any part of the IMAGE1 S™ 4U camera head, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1.2 \sqrt{P}$ where P is the nominal power of the transmitter in watts [W] according to the information provided by the manufacturer of the transmitter and d is the recommended separation distance in meters [m]. Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey <sup>a</sup> should be less than the compliance level in each frequency range <sup>b</sup> . $d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.3 \sqrt{P}$ 800 MHz to 2.7 GHz
Radiated RF disturbances acc. to IEC 61000-4-3	3 V/m 80 MHz to 2.7 GHz	3 V/m 80 MHz to 2.7 GHz	Interference may occur in the vicinity of equipment marked with the following symbol: 

Note: At 80 MHz and 800 MHz, the higher frequency range applies.

Note: These guidelines may not apply in all situations. The propagation of electromagnetic quantities is affected by absorptions and reflections of buildings, objects, and persons.

<sup>a</sup> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the 4U camera head is used exceeds the applicable RF compliance level above, the 4U camera head should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the 4U camera head.

<sup>b</sup> Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

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<b>Table 4 – Emission class and group</b> <b>Guidelines and manufacturer's declaration – electromagnetic emissions</b>		
<p>The IMAGE1 S™ 4U camera head is intended for use in the electromagnetic environment specified below. The customer or user of the IMAGE1 S™ 4U camera head should ensure that it is used in such an environment.</p>		
RF emissions as per CISPR 11	Group 1	The IMAGE1 S™ 4U camera head uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause interference in nearby electronic equipment.
RF Emissions as per CISPR 11	Class A	The IMAGE1 S™ 4U camera head is suitable for use in all establishments other than domestic and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions as per IEC 61000-3-2	Class A	
Voltage fluctuations/flicker Emissions as per IEC 61000-3-3	Complies	

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<b>Table 5</b>			
<b>Recommended separation distances between portable and mobile RF communications equipment and the IMAGE1 S™ 4U camera head</b>			
<p>The IMAGE1 S™ 4U camera head is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of the IMAGE1 S™ 4U camera head can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the IMAGE1 S™ 4U camera head as recommended below, according to the maximum output power of the communications equipment.</p>			
Rated maximum output power of transmitter [W]	Separation distance d [m] as a function of the transmitter frequency		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.7 GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23
<p>For transmitters rated at a maximum output power not listed above, the recommended separation distance D in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.</p> <p>Note: At 80 MHz and 800 MHz, the separation distance for higher frequency range applies.</p> <p>Note: These guidelines may not apply in all situations. The propagation of electromagnetic quantities is affected by absorptions and reflections of structures, objects and people.</p>			

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KARL STORZ Endoscopy Canada Ltd.  
7171 Millcreek Drive, Mississauga, Ontario  
L5N 3R3 Canada  
Phone: +1 905 816-4500, Fax: +1 905 816-4599  
Toll free phone: 1-800-268-4880 (Canada only)  
Toll free fax: 1-800-482-4198 (Canada only)  
E-Mail: info-canada@karlstorz.com

KARL STORZ Endoscopy-America, Inc.  
2151 East Grand Avenue  
El Segundo, CA 90245-5017, USA  
Phone: +1 424 218-8100, Fax: +1 424 218-8525  
Toll free phone: 800 421-0837 (USA only)  
Toll free fax: 800 321-1304 (USA only)  
E-Mail: communications@kse.com

KARL STORZ Veterinary Endoscopy-America, Inc.  
1 South Los Cameros Road  
Goleta, CA 93117, USA  
Phone: +1 805 968-7776, Fax: +1 805 685-2588  
E-Mail: info@karlstorzvet.com

KARL STORZ Endoscopy Latino-America, Inc.  
815 N. W. 57th Avenue, Suite 480  
Miami, FL 33126-2042, USA  
Phone: +1 305 262-8980, Fax: +1 305 262-8986  
E-Mail: info@ksela.com

KARL STORZ Endoscopia México S.A. de C.V.  
Edificio Atlántic, Oficina 3G  
Calle D e/ Tra y 3ra  
10400 Vedado, Havana, Cuba  
Phone: +537 836 95 06, Fax: +537 836 97 76  
E-Mail: kstorzuba@gmail.com

KARL STORZ Endoscopia México S.A. de C.V.  
Av. Ejercito Nacional No. 453 Piso 2,  
Colonia Granada, Alcaldía Miguel Hidalgo  
C.P. 11520 Ciudad de México  
Phone: +52 (55) 1101 1520  
E-Mail: mx-info@karlstorz.com

KARL STORZ Marketing América Do Sul Ltda.  
Rua Joaquim Floriano, nº. 413, 20º andar – Itaim Bibi,  
CEP-04534-011 São Paulo, Brasil  
Phone: +55 11 3526-4600, Fax: +55 11 3526-4680  
E-Mail: br-info@karlstorz.com

KARL STORZ Endoscopia Argentina S.A.  
Zufriategui 627 6° Piso, B1638 CAA - Vicente Lopez  
Provincia de Buenos Aires, Argentina  
Phone: +54 11 4718 0919, Fax: +54 11 4718 2773  
E-Mail: info@karlstorz.com.ar

KARL STORZ Endoskopi Norge AS  
Stamveien1  
1483 Hagan, Norway  
Phone: +47 6380 5600, Fax: +47 6380 5601  
E-Mail: post@karlstorz.no

KARL STORZ Endoskop Sverige AB  
Storsåtragränd 14  
127 39 Skårholmen, Sweden  
Phone: +46 8 505 648 00  
E-Mail: kundservice@karlstorz.se

KARL STORZ Endoscopy Suomi OY  
Taivaltie 5  
01610 Vantaa, Finland  
Phone: +358 (0)96824774, Fax: +358 (0)968247755  
E-Mail: asiakaspalvelu@karlstorz.fi

KARL STORZ SE & Co. KG  
Representation Office  
Keštučio st. 59 / Lenktoji st. 27  
08124 Vilnius, Lithuania  
Phone: +370 5 272 0448  
Mobile: +370 685 67 000  
E-Mail: info-lt-iv@karlstorz.com

KARL STORZ Endoskopi Danmark A/S  
Skovlytoften 33  
2840 Holte, Denmark  
Phone: +45 45162600, Fax: +45 45162609  
E-Mail: marketing@karlstorz.dk

KARL STORZ Endoscopy (UK) Ltd.  
415 Perth Avenue, Slough  
Berkshire, SL1 4TQ, United Kingdom  
Phone: +44 1753 503500, Fax: +44 1753 578124  
E-Mail: info-uk@karlstorz.com

KARL STORZ Endoscopiae Nederland B. V.  
Displayweg 2  
3821 BT Amersfoort, Netherlands  
Phone: +31 (0)33 4545890  
E-Mail: info-nl@karlstorz.com

KARL STORZ Endoscopy Belgium N. V.  
Phone: +31 (0)33 4545890  
E-Mail: info-be@karlstorz.com

KARL STORZ Endoscopiae France S. A. S.  
12, rue Georges Guynemer, Quartier de l'Europe  
78280 Guyancourt, France  
Phone: +33 1 30484200, Fax: +33 1 30484201  
E-Mail: marketing-fr@karlstorz.com

KARL STORZ Endoskop Austria GmbH  
Landstraßer Hauptstr. 148/1/G1  
1030 Wien, Austria  
Phone: +43 1 71 56 0470, Fax: +43 1 71 56 0479  
E-Mail: storz-austria@karlstorz.com

KARL STORZ Endoscopia Ibérica S. A.  
Parque Empresarial San Fernando  
Edificio Munich – Planta Baja  
28830 Madrid, Spain  
Phone: +34 91 6771051, Fax: +34 91 6772981  
E-Mail: info-es@karlstorz.com

KARL STORZ Endoscopia Italia S. r. l.  
Via dell'Artigianato, 3  
37135 Verona, Italy  
Phone: +39 045 8222000, Fax: +39 045 8222001  
E-Mail: info-ita@karlstorz.com

KARL STORZ Croatia d.o.o.  
Capraška 6  
10000 Zagreb, Croatia  
Phone: +385 1 6406 070, Fax: +385 1 6406 077  
E-Mail: info-hrv@karlstorz.hr

KARL STORZ Endoskopija d.o.o.  
Cesta v Gorice 34b  
1000 Ljubljana, Slovenia  
Phone: +386 1 620 5880, Fax: +386 1 620 5882  
E-Mail: pisarna@karlstorz.si

KARL STORZ Polska Sp. z o.o.  
ul. Bojkowska 47  
44-100 Gilwice, Poland  
Phone: +48 32 706 13 00, Fax: +48 32 706 13 07  
E-Mail: info-pl@karlstorz.com

KARL STORZ Endoskop Magyarország Kft.  
Toberek utca 2. fsz. 17/b  
HU-1112 Budapest, Hungary  
Phone: +36 195 096 31, Fax: +36 195 096 31  
E-Mail: info-hu@karlstorz.com

KARL STORZ Endoscopia Romania srl  
Str. Prof. Dr. Anton Colorian, nr. 74, Sector 4  
041393 Bukarest, Romania  
Phone: +40 (0)31 4250800, Fax: +40 (0)31 4250801  
E-Mail: info-ro@karlstorz.com

KARL STORZ Endoskope Greece M.E.P.E.\*  
Patriarhou Grigoriou E' 34  
54248 Thessaloniki, Greece  
Phone: +30 2310 304868, Fax: +30 2310 304862  
E-Mail: info-gr@karlstorz.com

\*Repair & Service Subsidiary

KARL STORZ Industrial\*\*  
Gedik Is Merkezi B Blok  
Kat 5, D 38-39, Bagdat Cad. No: 162  
Maltepe Istanbul, Turkey  
Phone: +90 216 442 9500, Fax: +90 216 442 9030

\*\*Sales for Industrial Endoscopy

000 KARL STORZ Endoscopy – WOSTOK  
Derbenyevskaya nab. 7, building 4  
115114 Moscow, Russia  
Phone: +7 495 983 02 40, Fax: +7 495 983 02 41  
E-Mail: Info-ru@karlstorz.com

TOV LLC KARL STORZ Ukraine  
Avenue Geroyiv Stalingrada Str. 2D, office 717  
Kyiv, 04210/Ukraine  
Phone: +38 095 000-895-0, +38-097-000-895-0,  
+38 073 000-895-0  
E-Mail: marketing@karlstorz.com.ua

KARL STORZ SE & Co. KG  
Representation Office  
Sabit Orudschow 1184, apt. 23  
1025 Baku, Azerbaijan  
Phone: +99 450 613 30 60  
E-Mail: info-az@karlstorz.com

KARL STORZ ENDOSKOPE –  
East Mediterranean and Gulf (Offshore) S.A.L.  
Spark Tower 1<sup>st</sup> floor  
Charles Helou St., Horch Tabet – Sin El Fil  
Beirut, Lebanon  
Phone: +961 1 501105, Fax: +961 1 501950  
E-Mail: info@karlstorz-emg.com

KARL STORZ Endoscopy (South Africa) (Pty) Ltd.  
P.O. 6061, Roggebaai 8012  
Cape Town, South Africa  
Phone: +27 21 417 2600, Fax: +27 21 421 5103  
E-Mail: info@karlstorz.co.za

TOO KARL STORZ Endoskopy Kasachstan  
Saryarka, 6, BC "Arman", off. 910  
010000 Astana, Republic of Kazakhstan  
Phone: +7 7172 552-549, 552-788, Fax: -444  
E-Mail: info@karlstorz.kz

KARL STORZ ENDOSKOPE  
East Mediterranean & Gulf (branch)  
Building West Side 7A – Unit 7WA – 3008  
Dubai Airport Free Zone, P.O. Box 54983  
Dubai - United Arab Emirates  
Phone: +971 (0)4 2958887, Fax: +971 (0)4 3205282  
Service Hotline: +971 (0)4 3415882  
E-Mail: info-gne@karlstorz-emg.com

KARL STORZ Endoscopy India Private Limited  
11<sup>th</sup> Floor, Dr. Gopal Das Bhawan  
28, Barakhamba Road  
New Delhi 110001, India  
Phone: +91 11 4374 3000, Fax: +91 11 4374 3010  
E-Mail: corporate@karlstorz.in

KARL STORZ SE & Co. KG  
Interchange 21 Tower, Level 33  
399 Sukhumvit Road,  
North Klongtoey, Wattana,  
10110 Bangkok, Thailand  
Phone: +66 2 660 3669  
E-Mail: info-th@karlstorz.com

KARL STORZ SE & Co. KG  
Resident Representative Office  
14<sup>th</sup> Floor, MPlaza Saigon  
39 Le Duan, District 1  
Ho Chi Minh City, Vietnam  
Phone: +84 28 3823 8000, Fax: +84 28 3823 8039  
E-Mail: infovietnam@karlstorz.com

KARL STORZ Endoscopy China Ltd.  
Room 2503-05, 25F AXA Tower, Landmark East,  
No. 100 How Ming Street, Kwun Tong, Kowloon,  
Hong Kong, People's Republic of China  
Phone: +852 28 65 2411, Fax: +852 28 65 4114  
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy (Shanghai) Ltd., Beijing Branch  
Room 1805-1807, Building B, 18F Beijing IFC  
No. 8, Jianguomenwai Street, Chaoyang District,  
100022, Beijing, People's Republic of China  
Phone: +86 10 5638188, Fax: +86 10 5638199  
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd., Shanghai Branch  
Room 701A Building 5 & Room 501 Building 7,  
No. 3000 Longdong Avenue, Pilot Free Trade Zone,  
201203, Shanghai, People's Republic of China  
Phone: +86 21 60339888, Fax: +86 21 60339808  
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd., Chengdu Branch  
Room 803-805, 8F Jin Jiang International Building  
No. 1 West Linjiang Road, Wuhou District,  
610044, Chengdu, People's Republic of China  
Phone: +86 28 86587977, Fax: +86 28 86587975  
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd., Shenyang Branch  
Room 2001-2005, 20F N-MEDIA International Center,  
No. 167 Youth Avenue, Shenhe District,  
110014, Shenyang, People's Republic of China  
Phone: +86 24 23181118, Fax: +86 24 23181119  
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd., Guangzhou Branch  
Room 02B & 03 & 04A, 35F Teem Tower,  
No. 208 Tianhe Road, Tianhe District,  
510620, Guangzhou, People's Republic of China  
Phone: +86 20 87321281, Fax: +86 20 87321286  
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy Asia Marketing Pte Ltd.  
No. 8 Commonwealth Lane #03-02  
Singapore 149555, Singapore  
Phone: +65 69229150, Fax: +65 69229155  
E-Mail: infoasia@karlstorz.com

KARL STORZ Endoscopy Singapore Sales Pte Ltd  
No. 8 Commonwealth Lane #03-02  
Singapore 149555, Singapore  
Phone: +65 69229150, Fax: +65 69229155  
E-Mail: infoasia@karlstorz.com

KARL STORZ SE & Co. KG  
Representative Office Indonesia  
Sinarmas MSG Tower Level 37  
Jl. Jend. Sudirman No. Kav. 21  
Jakarta Selatan  
DKI Jakarta 12920  
E-Mail: infoindonesia@karlstorz.com

KARL STORZ Endoscopy Korea Co. Ltd.  
9F Hyowon-Building  
97, Jungdae-ro, Songpa-gu  
05719 Seoul, Korea  
Phone: +82-70-4350-7474, Fax: +82-70-8277-3299  
E-Mail: infokorea@karlstorz.com

KARL STORZ Endoscopy Taiwan Ltd.  
12F, No. 192, Sec. 2, Chung Hsin Rd.,  
Sindian District, New Taipei City, Taiwan  
Phone: +886 933 014 160, Fax: +886 2 8672 6399  
E-Mail: info-tw@karlstorz.com

KARL STORZ SE & Co. KG  
Representative Office Philippines  
1901 Picadilly Star Bldg., 4<sup>th</sup> Avenue, BGC  
Taguig City 1636, Philippines  
Phone: +63 2 317 45 00, Fax: +63 2 317 45 11  
E-Mail: philippines@karlstorz.com

KARL STORZ Endoscopy Japan K. K.  
Stage Bldg. 8F, 2-7-2 Fujimi  
Chiyoda-ku, Tokyo 102-0071, Japan  
Phone: +81 3 6380-8622, Fax: +81 3 6380-8633  
E-Mail: info-jp@karlstorz.com

KARL STORZ Endoscopy New Zealand Ltd.  
31 Morningside Drive Mt Albert  
Auckland, 1025, New Zealand  
PO Box 56 511, Dominion Rd  
Auckland, 1446, New Zealand  
Phone: +64 9 846 6044, Fax: +64 9 846 6808  
Toll free: +64 508 84 84 84 (New Zealand only)  
E-Mail: sales@karlstorz.co.nz

KARL STORZ Endoscopy Australia Pty. Ltd.  
68 Waterloo Road, Macquarie Park NSW 2113  
P O Box 50 Lane Cove NSW 1595, Australia  
Phone: +61 (0)2 9490 6700  
Toll free: 1800 996 562 (Australia only)  
Fax: +61 (0)2 9420 0695  
E-Mail: karlstorz@karlstorz.com.au  
www.karlstorz.com

**STORZ**  
KARL STORZ—ENDOSKOPE

**KARL STORZ SE & Co. KG**

Dr.-Karl-Storz-Straße 34  
78532 Tuttlingen

Postfach 230  
78503 Tuttlingen  
Germany

Telefon: +49 7461 708-0  
Telefax: +49 7461 708-105  
E-Mail: [info@karlstorz.com](mailto:info@karlstorz.com)  
Web: [www.karlstorz.com](http://www.karlstorz.com)