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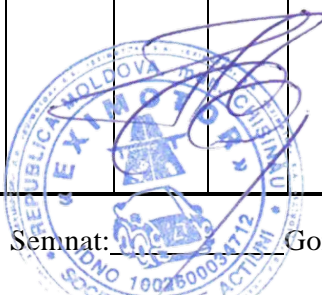
* Key feature

OLED Display

Helps to read various information including total test number, date, unit and other options

100 Test memory

Saves the test result including test date and time

User-Friendly interface

Enables to change device setting easily

Graph mode

Allows to track all test results efficiently

Calibration reminder

Helps get on-time calibration by letting users know about

* Specification

Indication of B.A.C	0.000 ~ 0.500 %BAC
	0.00 ~ 5.00 ‰
	0.00 ~ 2.50 mg/L
Accuracy	± 5% (at 0.05 %BAC, at 25°C)
Sensor type	Premium Fuel Cell
Blow sensing	Pressure sensor
Display	OLED (128 x 64 pixel)
Warm up time	Within 10 seconds (at 0.05 %BAC, at 25°C)
	Warm up time may vary depending on the BAC measured
Power supply	Two size alkaline batteries
Battery life	Approx. 1000 tests
Demensions(WxHxD)	60mm x 128mm x 24mm
Weight	130g (including batteries)
Calibration period	Every 12 months or after 500 tests
Operating temperature	-5 ~ 40 °C
Certificate(approved)	CE, RoHS
Certificate(in process)	EN 15964

* The specifications are subject to change without prior notice for functional improvements

DA TECH CO., LTD.

39, Pyeongcheon-ro 141 Beon-gil, Bupyeong-gu, Incheon, Korea (Zip :21310)

TEL: +82-32-868-0844 / FAX: +82-32-868-0846 / E-mail: sales@datech.co.kr / www.alcofind.com

ALCO FIND

Digital Breathalyzer AF-50

Instruction Manual

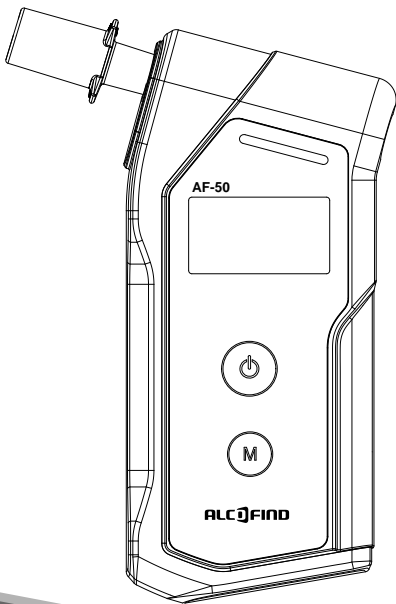
TABLE OF CONTENTS

1. Important Safety Information	3
2. Introduction	4
3. Contents	4
4. Name of each part	5
5. Basic Operation	6
5.1 Preparation	6
5.2 Operation	6
5.3 Symbols & Readings	9
6. Basic Menu Options	11
6.1 TEST RESULT	11
6.2 TEST RESULT-G	11
6.3 ALARM LIMIT	12
6.4 UNIT	12
6.5 BRIGHTNESS	13
6.6 SOUND	13
6.7 INFORMATION	14
6.8 PIN CODE	14
7. Advanced Menu Options	15
7.1 TEST LIMIT	15
7.2 SET TIME	15
7.3 DATE LIMIT	16
7.4 CHANGE PIN	16
8. Specification	17
9. Contact Information	18

1. Important Safety Information

- Before using the alcohol tester, read all instruction manual carefully and perform the test.
- Recommended to use calibration service every 12 months or after 500 times tests by experts. The use without periodic calibration service may result in invalid result.
- The test should be performed after 30 minutes after eating, drinking or smoking. Failure to wait 30 minutes may result in invalid results or incorrect results.
- Residual alcohol in the mouth can distort the measurement.
- Do not immerse in water.
- Aromatic drinks or mouth sprays containing alcohol can interfere with measurements. In this case, wait for 30 minutes or rinse the mouth with pure water.
- Do not permit to blow cigarette smoke into the tester which may damage the sensor.
- Don't do the drunk driving and even small amount of alcohol may cause impairment in normal driving operation.
- Do not use this device as a tool to determine "drink and drive". The manufacturer and the sellers will not take any legal responsibility for the test results.
- When not used for a long time, keep the device after taking out the batteries. In case of testing after long-term storage, there might be high tolerance of test results for the first test.
- Replace the batteries when the battery indicator icon reaches empty for the accuracy reason.
- In case the test result is higher than 0.1 %BAC (1.0 g/L) and you try again without interval, then the warm-up time might be delayed.
- Perform a test within the operating temperature, which is specified in the instruction manual.
- Blowing hard may cause damage to the sensor as saliva flows into the sensor.
- Do not store the device in the place where the temperature and humidity are high. Recommended it to store at room temperature.

- Do not suck in air during the test.
- Recommended using clean mouthpiece during the test for accurate measurements.
- The device including mouthpieces should be kept out of reach of children under 12 years of age.

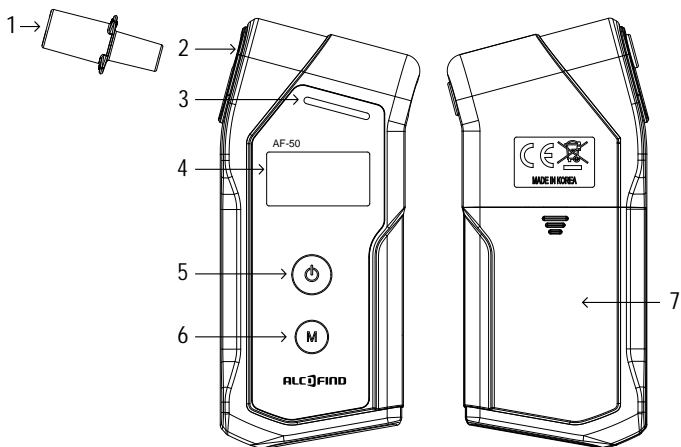
2. Introduction

Thank you for purchasing our alcohol tester, which is a professional use alcohol tester with high-reliability. It can be used in the law enforcement, hospital, school and many other industrial fields. It measures estimated B.A.C (Blood Alcohol Content) in human breath by means of a premium fuel cell sensor which is highly durable and providing accurate breath alcohol results

3. Contents

- Digital Breathalyzer: AF-50
- Two AA alkaline batteries
- 6 pieces of mouthpiece
- Carrying case
- Instruction manual


4. Name of each part


1. Mouthpiece
2. Mouthpiece slot
3. Status LED
4. High-Quality OLED Display
5. Power button
6. Menu button
7. Battery cover

5. Basic Operation

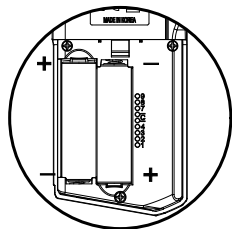5.1 Preparation

Insert two AA batteries
(1.5V Alkaline)

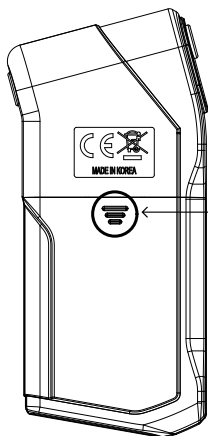Press
and push
down
the cover
to open

- Install two AA Batteries in the battery compartments by paying attention to the battery polarity.
- Insert a mouthpiece into the slot on the side of the alcohol tester.
- Ensure whether it securely fits or not.
- Extra mouthpieces should be used for other person's testing for sanitary reasons.

5.2 Operation

Step 1. Power On / Turn on the device

Press the power button for 2 seconds to turn on the device.

The alarm limit, date and time set in the device will be displayed.

- ALARM LIMIT
0.050 %BAC
- DATE / TIME
18-01-17 12:30

Step 2. Warm-up

The warm-up process will be started automatically. When the progress bar fills up, the 'START BLOW' will be displayed with double beep sound.

Note. Press the menu button shortly in this step if you want to check the 10 previous test results with a bar graph, which enables you to check visually whether your test results are higher than the alarm limit that you have set in the menu.

Step 3. Measurement

Blow through the mouthpiece until the device beeps twice.

Note. The blowing time may vary depending on set value.

Step 4. Analyzing

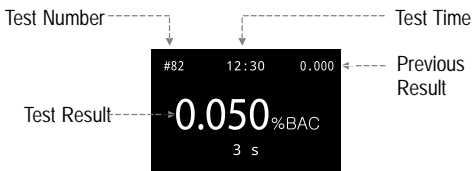
When the sampling is successful, the analyzing process will be started automatically.


Step 5. Test result

After a few seconds (depending on the alcohol level), the estimated BAC will be displayed for 4 seconds, and the power button and menu button will be deactivated during this time. Depending on the test result, the status LED will be flashed as followings:

- 1) Flashes red LED if the test result is equal or higher than the alarm limit
- 2) Flashes green LED if the test result is lower than the alarm limit

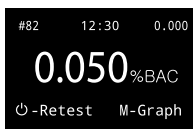Step 6. After Test

After displaying the test result for 4 seconds, the display will change to enable you to conduct the following functions:

- 1) Retest - Press the power button shortly if you want to test again,
- 2) Graph - Press the menu button shortly if you want to check the 10 previous test results with a bar graph, which enables you to check visually whether your test results are higher than the alarm limit that you have set in the menu.

Note. It also can be entered into the graph mode in Step 2 by pressing the menu button shortly.

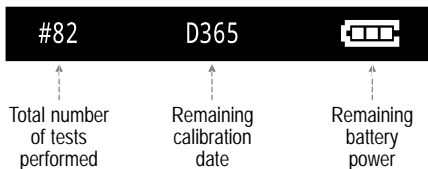
- ▲ : The most recent test result
- ◆ : The test result with an approved breath sample
- X : The test without an approved breath sample

- 3) POWER OFF - Press and hold the power button for 2 seconds if you want to turn off the device manually.

Note. The device will turn off automatically if it is not in use for 3 minutes

5.3 Symbols & Troubleshooting

Status Display

Troubleshooting guide

MESSAGE	WHAT IT MEANS	ACTION
BLOW ERROR	The sampling volume or blow power is not enough to analyze the breath sample.	Press the power button to perform a new retest.
LOW BATTERY	The testing cannot be performed because the battery is low.	Replace with new batteries.
CALIBRATION DUE DATE HAS PASSED	The calibration is required because the device has exceeded its recommended period of use.	Contact your local distributor or customer service center for proper recalibration service.

MESSAGE	WHAT IT MEANS	ACTION
OVER NUMBER OF TEST ALLOWED	The calibration is required because the device has exceeded its recommended number of test.	Contact your local distributor or customer service center for proper recalibration service.
BREATH SAMPLE WITH EXCESSIVE SALIVA	The excessive saliva went into the sensor during the test.	Wait 20 minutes, and then try again. Contact your seller for customer service if this error code is displayed for a long time.
OUT OF TEMP RANGE	The testing cannot be performed because the device is powered on outside of operating temperature.	Repeat the test after the device has reached the temperature within the operating range.
NO MEMORY CAPACITY AVAILABLE	The testing cannot be performed because the device has tested more than 10,000 times. Note. As soon as the total number reaches 9,000 times, there will be a pre-warning message that shows every time when the device is turned on in order to remind you to get calibration service before the device is blocked.	Contact your local distributor or customer service center for proper ecalibration service.

6. Basic Menu Options

*How to enter the menu options

- In Warm-up process (Step 2), press the menu button for 2 seconds.

*How to move/select/exit the menu options.

- To move the cursor up, press the power button shortly.

- To move the cursor down, press the menu button shortly.

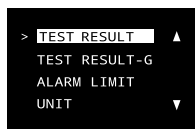
- To enter the selected option, press the power button for 2 seconds.

- To return to warm up process, press the menu button for 2 seconds.


*How to save the selected setting

- Press and hold the power button and the menu button together for 2 seconds until a beep is heard.

6.1 TEST RESULT

Test number	-----	> NO.0082	N
Test result	-----	> 0.050	%BAC
Test date	-----	> DATE	18/01/26
Test time	-----	> TIME	12:30:10

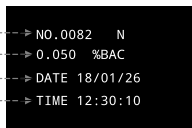The test results can be checked with test number, date and time.

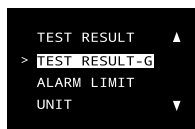
Note. The most recent test result will appear first.

* To see the previous result, press the power button shortly.


* To see the next result, press the menu button shortly.

* To return to the menu list, press the menu button for 2 seconds.

6.2 TEST RESULT-G

Current page#	-----	> 9
Total page#	-----	> 9
Alarm limit	-----	>

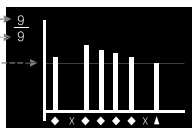▲ : The most recent test result

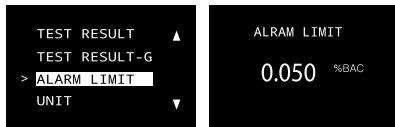
◆ : The test result with an approved breath sample

X : The test without an approved breath sample

The test results can be checked with a bar graph, which enables you to check visually whether your test results are higher than the alarm limit that you have set in the menu.

- * To see the previous page, press the power button shortly.
- * To see the next page, press the menu button shortly.
- * To return to the menu list, press the menu button for 2 seconds.

6.3 ALARM LIMIT


The limit value for alarm can be set in this menu.

The settable value range:

- 0.010 ~ 0.100 %BAC (0.10 ~ 1.00 ‰)
 - OFF
- * To raise the alarm limit, press the power button shortly.
 - * To reduce the alarm limit, press the menu button shortly.
 - * To save the setting and return to the menu list, press and hold the power button and the menu button together for 2 seconds.
 - * To return to the menu list without saving, press the menu button for 2 seconds.

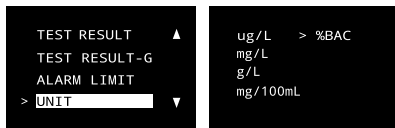6.4 UNIT


The measurement unit can be set in this menu.

The settable value range:

- ug/L, mg/L, g/L, mg/100mL, %BAC.
- * To move the cursor up, press the power button shortly.
 - * To move the cursor down, press the menu button shortly.

- * To save the setting and return to the menu list, press and hold the power button and the menu button together for 2 seconds.
- * To return to the menu list without saving, press the menu button for 2 seconds.

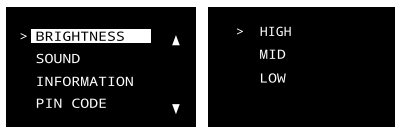6.5 BRIGHTNESS

The brightness of the display can be set in this menu.

The settable value range:

- HIGH, MID, LOW

- * To move the cursor up, press the power button shortly.
- * To move the cursor down, press the menu button shortly.
- * To save the setting and return to the menu list, press and hold the power button and the menu button together for 2 seconds.
- * To return to the menu list without saving, press the menu button for 2 seconds.

6.6 SOUND

The sound can be set in this menu.

The settable value range:

- HIGH, MID, LOW, OFF

- * To move the cursor up, press the power button shortly.
- * To move the cursor down, press the menu button shortly.

- * To save the setting and return to the menu list, press and hold the power button and the menu button together for 2 seconds.
- * To return to the menu list without saving, press the menu button for 2 seconds.

6.7 INFORMATION

The following information about the device can be checked in this option.

1. The version of firmware
2. The calibration date
3. The total number of tests performed

- * To return to the menu list, press the menu button for 2 seconds.

6.8 PIN CODE

In this menu, the correct PIN code must be entered to move into the advanced menu option.

The default PIN code is 0000, and the PIN code is adjustable from 0000 to 9999 in the advanced menu option.

- * To change digits of PIN code, press the power button for 2 seconds.
- * To raise the number of PIN code, press the power button shortly.
- * To reduce the number of PIN code, press the menu button shortly.
- * To enter the advanced menu option, press and hold the power button and the menu button together for 2 seconds.
- * To return to menu list, press the menu button for 2 seconds.

7. Advanced Menu Options

7.1 TEST LIMIT.

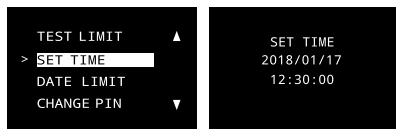
The maximum number of measurements allowed can be set in this menu.

The settable value range:

- 50 ~ 1000 tests
- OFF

- * To raise the test limit, press power button shortly.
- * To reduce the test limit, press the menu button shortly.
- * To save the setting and return to the advanced menu list, press and hold the power button and the menu button together for 2 seconds.
- * To return to the advanced menu list without saving, press the menu button for 2 seconds.


7.2 SET TIME

The device time can be adjusted in this menu.

- * To change digits of time and date, press the power button for 2 seconds.
- * To raise the number of time and date, press the power button shortly.
- * To reduce the number of time and date, press the menu button shortly.
- * To save the setting and return to the advanced menu list, press and hold the power button and the menu button together for 2 seconds.
- * To return to the advanced menu list without saving, press the menu button for 2 seconds.

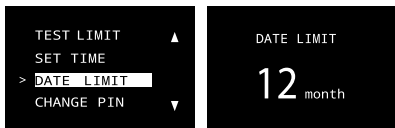7.3 DATE LIMIT

The date for the next calibration can be set in this menu.

The settable value range:

- 1 ~ 24 month

- * To raise the date limit, press the power button shortly.
- * To reduce the date limit, press the menu button shortly.
- * To save the setting and return to the advanced menu list, press and hold the power button and the menu button together for 2 seconds.
- * To return to the advanced menu list without saving, press the menu button for 2 seconds.

7.4 CHANGE PIN

The PIN code can be set in this menu.

The settable value range:

- 0000 ~ 9999

- * To change digits of PIN code, press the power button for 2 seconds.
- * To raise the number of PIN code, press the power button shortly.
- * To reduce the number of PIN code, press the menu button shortly.
- * To save the setting and return to the advanced menu list, press and hold the power button and the menu button together for 2 seconds.
- * To return to the advanced menu list without saving, press the menu button for 2 seconds.

8. Specification

Indication of B.A.C	0.000 ~ 0.500 %BAC 0.00 ~ 5.00 ‰ 0.00 ~ 2.50 mg/L
Accuracy	±0.005 %BAC (at 0.100 %BAC, at 25°C) ±0.05 ‰ (at 1.00 ‰, at 25°C) ±0.025 mg/L (at 0.50 mg/L, at 25°C)
Warm up time	Within 10 seconds at 0.050 %BAC * Warm up time may vary depending on the BAC measured
Sensor type	Premium Fuel Cell
Power supply	Two AA size alkaline batteries
Memory capacity	100 test results with test number, date and time
Battery life	Approx. 1000 tests
Frequency band	2402 ~ 2480 MHz
Maximum transmit power	1.6 dBm
Dimensions(WxHxD)	60 x 128 x 24 mm
Weight	130 g including batteries
Calibration	Every 12 months or after 500 tests
Operating temperature	-5 ~ 40°C

9. Contact information

For any questions, after-sales service, product calibration or to purchase additional mouthpieces, please kindly contact the nearest distributor or seller below.

Local distributor	
Manufacturer	DA Tech Co., Ltd 13, Seounsandan-ro 4-gil, Gyeyang-gu, Incheon, Korea, Republic of (Zip. 21072) Web: www.alcofind.com Tel: +82-32-868-0844 / Fax: +82-32-868-0844

DECLARATION OF CONFORMITY

Directive 2014/53/EU

We, DA Tech Co., Ltd having its principal office at 39 Pyeongcheon-ro 141 beon-gil, Bupyeong-gu, Incheon, Korea, declare under our sole responsibility that the following product:

Product Name : Breathalyzer

Model number : AF-50

is in conformity with the following essential requirements and other relevant provisions of the Directive 2014/53/EU:

Essential Requirement	Applicable Standards / Specifications
Health & Safety	EN 60950-1:2006+A11:2009+A1:2010+A12:2011+A2:2013
	EN 62479 : 2010
Electromagnetic Compatibility	Draft ETSI EN 301 489-1 V2.2.0 (03/2017)
	Draft ETSI EN 301 489-3 V2.1.1 (03/2017)
	Final draft ETSI EN 301 489-17 V3.2.0 (03/2017)
	EN 61326-1:2013,
	EN 55011:2009+A1:2010 Class B,
	EN 61326-2-3 : 2013
	EN 61000-4-2 : 2009
	EN61000-4-3 : 2006+A1:2008+A2:2010,
EN 61000-4-8 : 2010	
Effective Use of Spectrum	EN 300 328 V2.1.1
	EN 303 413 V1.1.0
Various Requirements	Risk Assessment

The product carries the CE Mark: 

Authorized Signature / Date:  / October 11, 2018
Name & Title of Signatory: Kwang Hee Park / President

DA Tech Co., Ltd.
39 Pyeongcheon-ro 141 beon-gil,
Bupyeong-gu, Incheon, Korea

CERTIFICATE *of* EXAMINATION

NOTIFIED BODY EU-TYPE EXAMINATION CERTIFICATE

ESTE78 / 24 Jun 2019 / Rev A

Obsoletes: ESTE51-1 / 13 Jul 2018 / Rev A

Radio Equipment Directive (RED) 2014/53/EU

MiCOM Labs Inc., Notified Body Number 2280 declares, on the basis of the assessment of the tests and the technical documentation provided by the applicant that the following product complies with the essential requirements of the above noted Directive.

Product Name:
Breathalyzer

Approval Holder Name:
DA Tech Co., Ltd

Gordon Hurst, Product Certifier

This Certificate is Issued under the Authority of:
MiCOM Labs Inc., 575 Boulder Court, Pleasanton, California 94566, USA

Notified Body Number: 2280

Product Name:

BreathalyzerProduct Model Numbers: **AF-50, Polaris, ACE X, Pro100, ALCODETECTOR S100****Approval Holder:** DA Tech Co., Ltd, 39 Pyeongcheon-ro 141 beon-gil, Bupyeong-gu, Incheon, Korea**Product Manufacturer:** DA Tech Co., Ltd, 39 Pyeongcheon-ro 141 beon-gil, Bupyeong-gu, Incheon, Korea**Standards**

Group	Name
Article 3.1(a) Health & Safety	EN 60950-1:2006+A11:2009+A1:2010+A12:2011+A2:2013 EN 62479: 2010
Article 3.1(b) Electromagnetic Compatibility	Draft ETSI EN 301 489-1 V2.2.0 (03/2017) Draft ETSI EN 301 489-3 V2.1.1 (03/2017) Final draft ETSI EN 301 489-17 V3.2.0 (03/2017) EN 61326-1:2013, EN 55011:2009+A1:2010 Class B, EN 61326-2-3 : 2013 EN 61000-4-2 : 2009 EN61000-4-3 : 2006+A1:2008+A2:2010, EN 61000-4-8 : 2010
Article 3.2 Effective Use of Spectrum	EN 300 328 V2.1.1 EN 303 413 V1.1.0
Article 3.3 (a) to (i) Various Requirements	Risk Assessment

Annex 1 to EU-Type Examination

**EU-Type examination on the essential requirements
Article 3**

Article 3.1 - a) Health and Safety	Assessed
Article 3.1 - b) Electromagnetic compatibility	Assessed
Article 3.2 - Effective use of radio spectrum	Assessed
Article 3.3 - a) interworks with Accessories/Chargers	Assessed
Article 3.3 - b) interworks with Radio Networks	Assessed
Article 3.3 - c) can connect to interfaces	Assessed
Article 3.3 - d) does not harm the network, misuse network resources	Assessed
Article 3.3 - e) privacy protections	Assessed
Article 3.3 - f) fraud protections	Assessed
Article 3.3 - g) emergency services access	Assessed
Article 3.3 - h) assist users with disabilities	Assessed
Article 3.3 - i) integrity of software	Assessed

Description of Apparatus

Company Name	DA Tech Co., Ltd
Certification No.	ESTE78
Issue Date / Rev	24 Jun 2019 / Rev A
Equipment Description	Breathalyzer

Emission Information

Technology	Frequency		Emission Designator	RF Power		
	From	To		Max.	Type	Field Strength
BLE	2402 MHz	2480 MHz		1.6dBm		--
GPS	1575.42 MHz			--	--	--

Technical Construction File Details: (Documents Reviewed)

Technical Report(s):

Article 3.1(a) Health & Safety:
ESTRCE1802-001(1)
ESTSSE1802-005

Article 3.1(b) Electromagnetic Compatibility:
ESTECE1802-001
ESTECE1802-002

Article 3.2 Effective Use of Spectrum:
ESTRCE1802-001

Article 3.3 (a) to (i) Various Requirements:
AF-50 RED Risk Assessment Rev 1_re1.pdf

Supporting Documentation:

Service Agreement
Agent Authorization
EU Application
EU Declaration of Conformity
Block Diagram
BOM or Parts List
External Photographs
Internal Photographs
Label and its Location
Operational Description
PCB Layout
Schematics
Test Setup - EU
User Manual
NB update request Letter

Notes

Update: The following models have been updated due to name change. They are identical to the original certified model, AF-50.

-BEFORE: AF-50, Polaris, ACE X Police, Alcotest Pro100

-NEW: AF-50, Polaris, ACE X, Pro100, ALCODETECTOR S100

Scope

This EU-Type Examination Certificate is given in respect of compliance of radio spectrum use Article 3 Paragraph 2 of the RED Directive 2014/53/EU. The scope of the evaluation and this certificate relates only to those items identified in 'Annex 1 to EU - Type Examination Certificate' for the specific product and Certificate number referenced above.

EU Type Examination was performed according to Module B: EU-type examination procedure per Annex III the Directive on the essential requirements in Article 3, for the specific product and Certificate Number referenced above.

This EU Type Examination Certificate is based upon the review of the Technical Documentation and supporting evidence for the adequacy of the technical design solution, it is only valid in conjunction with the attached Annexes. The scope of this statement relates to a single sample of the apparatus identified above and of the submitted documents only.

Annex 2 to EU-Type Examination Obligations of the Applicant

Ref RED 2014/53/EU Article 10 - Obligations of manufacturers

1. When placing their radio equipment on the market, manufacturers shall ensure that it has been designed and manufactured in accordance with the essential requirements set out in Article 3.
2. Manufacturers shall ensure that radio equipment shall be so constructed that it can be operated in at least one Member State without infringing applicable requirements on the use of radio spectrum.
3. Manufacturers shall draw up the technical documentation referred to in Article 21 and carry out the relevant conformity assessment procedure referred to in Article 17 or have it carried out. Where compliance of radio equipment with the applicable requirements has been demonstrated by that conformity assessment procedure, manufacturers shall draw up an EU declaration of conformity and affix the CE marking.
4. Manufacturers shall keep the technical documentation and the EU declaration of conformity for 10 years after the radio equipment has been placed on the market.
5. Manufacturers shall ensure that procedures are in place for series production to remain in conformity with this Directive. Changes in radio equipment design or characteristics and changes in the harmonised standards or in other technical specifications by reference to which conformity of radio equipment is declared shall be adequately taken into account.

When deemed appropriate with regard to the risks presented by radio equipment, manufacturers shall, to protect the health and safety of end-users, carry out sample testing of radio equipment made available on the market, investigate, and, if necessary, keep a register of complaints, of non-conforming radio equipment and radio equipment recalls, and shall keep distributors informed of any such monitoring.

6. Manufacturers shall ensure that radio equipment which they have placed on the market bears a type, batch or serial number or other element allowing its identification, or, where the size or nature of the radio equipment does not allow it, that the required information is provided on the packaging, or in a document accompanying the radio equipment.
7. Manufacturers shall indicate on the radio equipment their name, registered trade name or registered trade mark and the postal address at which they can be contacted or, where the size or nature of radio equipment does not allow it, on its packaging, or in a document accompanying the radio equipment. The address shall indicate a single point at which the manufacturer can be contacted. The contact details shall be in a language easily understood by end-users and market surveillance authorities.
8. Manufacturers shall ensure that the radio equipment is accompanied by instructions and safety information in a language which can be easily understood by consumers and other end-users, as determined by the Member State concerned. Instructions shall include the information required to use radio equipment in accordance with its intended use. Such information shall include, where applicable, a description of accessories and components, including software, which allow the radio equipment to operate as intended. Such instructions and safety information, as well as any labelling, shall be clear, understandable and intelligible.

The following information shall also be included in the case of radio equipment intentionally emitting radio waves:

- (a) frequency band(s) in which the radio equipment operates;
- (b) maximum radio-frequency power transmitted in the frequency band(s) in which the radio equipment operates.

9. Manufacturers shall ensure that each item of radio equipment is accompanied by a copy of the EU declaration of conformity or by a simplified EU declaration of conformity. Where a simplified EU declaration of conformity is provided, it shall contain the exact internet address where the full text of the EU declaration of conformity can be obtained.
10. In cases of restrictions on putting into service or of requirements for authorisation of use, information available on the packaging shall allow the identification of the Member States or the geographical area within a Member State where restrictions on putting into service or requirements for authorisation of use exist. Such information shall be completed in the instructions accompanying the radio equipment. The Commission may adopt implementing acts specifying how to present that information. Those implementing acts shall be adopted in accordance with the advisory procedure referred to in Article 45(2).

11. Manufacturers who consider or have reason to believe that radio equipment which they have placed on the market is not in conformity with this Directive shall immediately take the corrective measures necessary to bring that radio equipment into conformity, to withdraw it or recall it, if appropriate. Furthermore, where the radio equipment presents a risk, manufacturers shall immediately inform the competent national authorities of the Member States in which they made the radio equipment available on the market to that effect, giving details, in particular, of the noncompliance, of any corrective measures taken and of the results thereof.

12. Manufacturers shall, further to a reasoned request from a competent national authority, provide it with all the information and documentation in paper or electronic form necessary to demonstrate the conformity of the radio equipment with this Directive, in a language which can be easily understood by that authority. They shall cooperate with that authority, at its request, on any action taken to eliminate the risks posed by radio equipment which they have placed on the market.

Ref RED 2014/53/EU Article 11 - Authorised representatives

1. A manufacturer may, by a written mandate, appoint an authorised representative.

The obligations laid down in Article 10(1) and the obligation to draw up technical documentation laid down in Article 10(3) shall not form part of the authorised representative's mandate.

2. An authorised representative shall perform the tasks specified in the mandate received from the manufacturer. The mandate shall allow the authorised representative to do at least the following:

(a) keep the EU declaration of conformity and the technical documentation at the disposal of national market surveillance authorities for 10 years after the radio equipment has been placed on the market;

(b) further to a reasoned request from a competent national authority, provide that authority with all the information and documentation necessary to demonstrate the conformity of radio equipment;

(c) co-operate with the competent national authorities, at their request, on any action taken to eliminate the risks posed

Article 19 General principles of the CE marking

1. The CE marking shall be subject to the general principles set out in Article 30 of Regulation (EC) No 765/2008.

2. On account of the nature of radio equipment, the height of the CE marking affixed to radio equipment may be lower than 5 mm, provided that it remains visible and legible.

Article 20 Rules and conditions for affixing the CE marking and the identification number of the notified body

1. The CE marking shall be affixed visibly, legibly and indelibly to the radio equipment or to its data plate, unless that is not possible or not warranted on account of the nature of radio equipment. The CE marking shall also be affixed visibly and legibly to the packaging.

2. The CE marking shall be affixed before the radio equipment is placed on the market.

3. Member States shall build upon existing mechanisms to ensure correct application of the regime governing the CE marking and shall take appropriate action in the event of improper use of that marking.

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