**MOPEDIA®**

MA RP715-719-720-721-722-723-724-726 01 A\_06-2013

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I Class Medical Device according  
to Directive 93/42/EEC and further modifications

## 1. Introduction

Thank you for purchasing a MopediA THREE and/or FOUR LEG CANE by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

## 2. Intended use

The three-leg and four-leg canes Moretti are devices manipulated by one arm adapted to support and / or help the mobility of people with walking difficulties.

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

## 3. EC Declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the cane aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the above directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

## 4. General warnings

- Read carefully the entire manual before using the device .
- Make sure that the product is delivered complete and keep it packed away from heat sources
- Service life- the device use limit is defined by the wear of the parts
- Do not allow children to play or operate the device

## 5. General description

### 5.1 Three-leg cane codes

- RP715** Anodized aluminum three-leg cane - Closed handle-child
- RP719** Anodized aluminum three-leg cane - Brachiun rest
- RP720** Anodized aluminum three-leg cane - Open handle
- RP721** Anodized aluminum three-leg cane - Closed handle

### 5.2 Four-leg cane codes

- RP722** Anodized aluminum four-leg cane - Brachiun rest
- RP723** Anodized aluminum four-leg cane - Open handle
- RP724** Anodized aluminum four-leg cane - Closed handle
- RP726** Anodized aluminum four-leg cane - Narrow base (15x20cm)

The three-leg and four-leg canes MOPEDIA are realized in anodized aluminum tubeing. They are supplied with non-slip soft rubber tips. They are height adjustable every 2.5 cm. These devices make it easy standing and walking. Give stability to the person in terms of balance and support if there are problems of power. Maximum user's weight 130kg. for adult model ( for RP719-RP722 -100 kg) for RP715 ( child model) 80 kg.

### 5.3 To adjust leg height

To adjust the height, press the push button and slide the cane to the desired height. Be sure the push button is fully protruded through the desired hole when finished.

### 5.4 To adjust right or left hand use

Press the push button and rotate handle by 180 degrees. Be sure the push button is fully protruded through the desired hole on the other side of the cane.

## 6. Before use

- Check if the frame is damaged in order to guarantee a safe use for people and things
- Check the correct assembly of the device

## 7. Warnings for a correct use

- Use extra caution on wet and slippery surfaces
- DO NOT use with patient who exceed the weight indicated in the product label and in the technical features chapter of the present manual
- Avoid to leave the device in contact with water for long periods of time
- Periodically check its sturdiness, the stability and the rubber tips are in good conditions. Replace them once worn out
- In case of doubts about use, adjustment or maintenance, contact your dealer
- Keep it away from children to avoid any possible risks

8. How to use

Hold the device with the opposite hand to the weak leg and with the longer side of the support base directed toward the inside. Bring the device forward together with the foot of the leg weak loading the full weight on the stronger leg. During walking, be careful not to place the device nor too close to the body to avoid tripping over one of tips or too far away to avoid an imbalance. In case of use on stairs, verify that the size of the step are sufficient to contain the base of device. After that, the person will have to raise the first leg stronger, subsequently, the device, then the weaker leg. Down the stairs with the device must follow the injured leg, and finally the good leg. If necessary, get help from an assistant.

9. Maintenance

The Mopedia devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

10. Cleaning and disinfection

10.1 Cleaning

Use only a damp cloth to remove dust, use a damp cloth with a mild soap to remove more difficult dirt. Dry thoroughly before use.

10.2 Disinfection

If you need to disinfect the device use a common disinfectant cleaner

**NOTE:** Never use acids, alkalis or solvents such as acetone

11. Condition of disposal

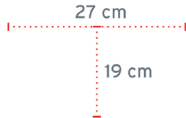
11.1 General condition of disposal

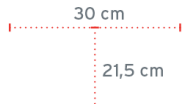
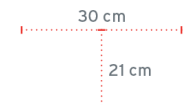
In case of disposal of the device don't use the common container for municipal waste. We recommend to dispose the device in the appropriate disposal areas for recycling.

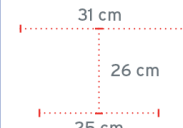
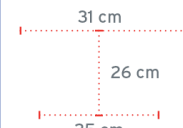
12. Technical features

12.1 Technical features - three-leg canes

| Codes | Max user's weight | Adjustable height | Handle                 | Base   |
|-------|-------------------|-------------------|------------------------|--|
| RP715 | 80 kg             | From 66 to 73 cm  | Closed anatomic handle |  |
| RP719 | 100 kg            | From 76 to 98 cm  | Brachium rest          |  |

|              |        |                  |                        |  |
|--------------|--------|------------------|------------------------|--|
| <b>RP720</b> | 130 kg | From 75 to 97 cm | Sponge open handle     |  |
| <b>RP721</b> | 130 kg | From 74 to 97 cm | Closed anatomic handle |  |

## 12.2 Technical features - four-leg canes

| Codes        | Max user's weight | Adjustable height | Handle                 | Base  |
|--------------|-------------------|-------------------|------------------------|---|
| <b>RP722</b> | 100 kg            | From 76 to 98 cm  | Brachium rest          |   |
| <b>RP723</b> | 130 kg            | From 75 to 95 cm  | Open anatomic handle   |   |
| <b>RP724</b> | 130 kg            | From 74 to 96 cm  | Closed anatomic handle |   |
| <b>RP726</b> | 130 kg            | From 75 to 95 cm  | Open anatomic handle   |  |

## 12.3 Spare parts

### RV7036 (4pcs)

Rubber tips for narrow base four-leg canes Ø14mm

### RV7033 (2pcs)

Rubber tips for large base four-leg and three-leg canes Ø18mm

### 13. Warranty

Moretti products are guaranteed from material or manufacturing faults for a period of ( two ) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair , damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti ), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

#### 13.1 Repairs

**Warranty repairs** In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location . The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

**Repair not covered by warranty** A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

**Non-defective device** The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer .

#### 13.2 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

#### 13.3 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

**WARRANTY CERTIFICATE**

Product \_\_\_\_\_

Purchased on (date) \_\_\_\_\_

Retailer \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

Sold to \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

**MORETTI S.P.A.**

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

**www.morettispa.com** email: **info@morettispa.com****MADE IN P.R.C**

Via Bruxelles, 3 - Meleto  
52022 Caviglia (Arezzo)  
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)