

| No. | Form | Name | Function |
|-----|------|---|---|
| 10 | | Lower tabletop button *2 | Lowers the tabletop. Reference P.4-60 |
| 11 | | Imaging unit shift button (towards the head) *2 | The imaging unit moves towards where the patient's head is. Reference P.4-64 |
| 12 | | Imaging unit shift button (towards the feet) *2 | The imaging unit moves towards where the patient's feet are. Reference P.4-64 |
| 13 | | Tabletop left shift button *2 | The tabletop moves towards the patient's left. Reference P.4-58 |
| 14 | | Tabletop right shift button *2 | The tabletop moves towards the patient's right. Reference P.4-58 |
| 15 | | Emergency stop button | Brings the movement of the tabletop to an emergency stop. Reference P.5-2 |


*1: In the Smart FOV mode (option), these buttons are not available.









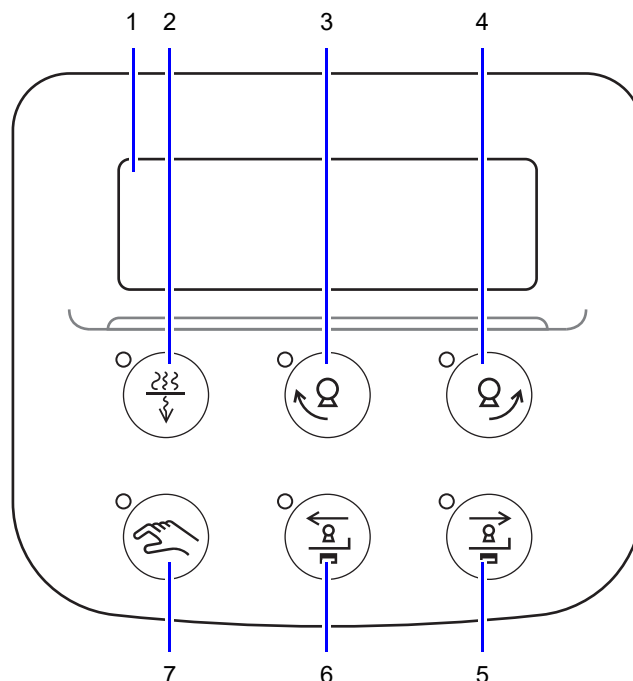





Reference "4.12.3.1 Smart FOV (Option)" P.4-130


*2: Only moves while the button is being pressed.

2.2.3 Collimator Control Panel


Control Panel

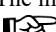

| No. | Form | Name | Function |
|-----|---|--|--|
| 1 |  | Display | Displays data about the irradiation field, SID, etc. The backlight comes on when the X-ray tube unit is selected. It goes out when the No.2 tube unit is selected.  Reference P.2-6 |
| 2 |  | Filter switchover button | Changes the BH filter.  Reference P.4-111 |
| 3 |  | X-ray tube unit rotation button | Rotates the X-ray tube unit in the direction of the arrow when the tabletop is in the vertical position.  Reference P.4-225 |
| 4 |  | | |
| 5 |  | Imaging unit shift button (towards the feet) | The imaging unit moves towards where the patient's feet are.  Reference P.4-225 |
| 6 |  | Imaging unit shift button (towards the head) | The imaging unit moves towards where the patient's head is.  Reference P.4-225 |
| 7 |  | Manual operation button | The irradiation field can be set while the LED is lit by using the collimator open/close knobs. Press the button to light the LED if  (customizable button) is assigned on the console when using the DR technique.  Reference P.4-80 |

 **Hint** The function on the C-leaf collimator (optional) or the Independent mask collimator (optional) can be allocated to some buttons on the control panel.
Contact your Shimadzu service representative if you wish to change the layout of the control panel buttons.

Display

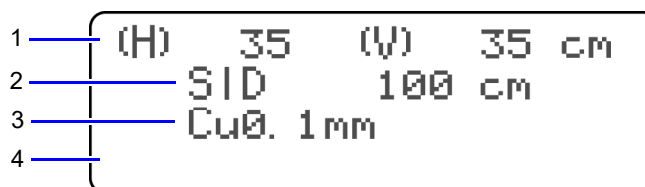

| No. | Name | Function |
|-----|-------------------|--|
| 1 | Irradiation field | Displays the irradiation field (in cm/inch). |
| 2 | SID | Displays the SID value (in cm/inch). Is not displayed while the DR technique is selected. |
| 3 | Filter type | Displays the filter type that has been selected. |
| 4 | Message | Displays error messages and other operational messages. |