# 24 100 00 Single Cuvettes - 500

# 24 200 00 Single Cuvettes - 5000

### Information of use

The Cuvette as general laboratory article is suitable to hold a reaction mixture for use in optical analyzers which are used in laboratories for in vitro diagnostic tests.

The cuvettes are ready for immediate use. They have unlimited shelf life if stored at 0 - 50°C.

### Content

Product	Single Cuvettes	Single Cuvettes
Cat.No.	24 100 00	24 200 00
Content	500 pcs (500 det.)	5000 pcs (5000 det.)

The Cuvettes can be used with Coatron X Analyzers.

For application with semi-automated Coagulation System Coatron X, each box contains a voucher label with a VIN and PIN code to generate a ticket on the web-based registration page ([www.teco-reg.com](http://www.teco-reg.com)).

The ticket information (VIN/PIN) must be entered for the respective device to release the number of tests for this device only. The (VIN/PIN) can only be used once per unit.

### Precautions and waste information

The Cuvette should only be used once in analyzers. To prevent contamination (sample/reagent), it is advised to avoid contact with skin and eyes. Suitable protective clothing and gloves are recommended.

Please also note the disposal of components in accordance with local regulations for infectious material.

Material: pure, clear Polystyrol (PS)  
 Maximum volume should be less than ~ 500µL  
 Minimum volume: 75 µL  
 Dimensions max.: Ø11,5 mm x 24 mm


### 24 100 00

Example Picture of the package – 500 Single Cuvettes

Packaging:

1. Log Bag, Dim.: (mm) 165 x 295 x 0,05, Mat.: LDPE;
2. Card Box, Dim.: (mm) 252 x 104 x 65
3. Paper Sleeve with Identification and Information

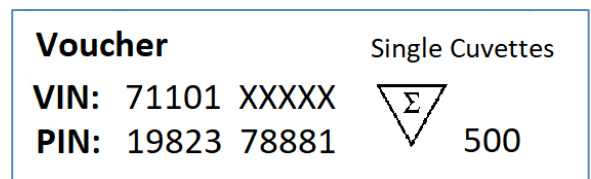Example of Voucher – 500 Single Cuvettes/Pack


### 24 200 00

Example Picture of the package – 5000 Single Cuvettes

Packaging:

1. Log Bag, Dim.: (mm) 600 x 400 x 0,05, Mat.: LDPE;
2. Card Box, Dim.: (mm) 400 x 250 x 150
3. Label with Identification and Information

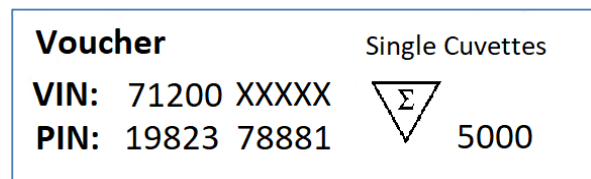Example of Voucher – 5000 Single Cuvettes/Pack

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REF

A0230-010, A0230-040, A0230-100

In-Vitro-Diagnostic reagent for professional use

**Intended Use**

Clotting test for the quantitative determination of prothrombin time (PT) in citrated human plasma according to Quick1,2. Intended to be used by professional laboratory personnel using coagulation analysers. The determination of PT is used for the global evaluation of the extrinsic pathway of coagulation.

Prolonged clotting times may be observed in the following situations: deficiency or inhibition of extrinsic coagulation factors II, V, VII, X, Vitamin-K, Fibrinogen, presence of Vitamin-K inhibitors like Phenprocoumon, Warfarin or others.

**Contents & Determinations**

Product	TEClot PT-S	TEClot PT-S	TEClot PT-S
Cat.No.	A0230-010	A0230-040	A0230-100
PT-S Reagent*	5x2 mL	10x4 mL	10x10 mL

**Determinations**

Coatron M**	200 Det.	800 Det.	2000 Det.
Coatron A4	100 Det.	400 Det.	1000 Det.
Coatron A6	200 Det.	800 Det.	2000 Det.

\*contains an extract of Rabbit brain with buffer, stabilizers and Calcium chloride.

\*\*Micro method (75µl in total)

**Preparation**

Reconstitute with high purity water with the volume stated on the vial label.

A0230-010	A0230-040	A0230-100
2 mL	4 mL	10 mL

Let stand at room temperature with occasional swirling for at least 15 min. Then place reagent into instrument and let incubate for further 15 min. The reagent sediments and must be swirled before each testing. On Coatron instruments, you can use a mixing bar for this.

**Storage & Stability**

Unopened reagents are stable until the expiration date shown on the label stored at 2°-8°C. Opened reagent:

PT Reagent	2-8 °C	20-25 °C	37°C
	5 days	36 hours	8 hours

**Precautions**

Avoid contact with skin and eyes. Wear suitable protective clothing. Dispose components in compliance with local regulations for infectious material. All components are checked for HIV, HBV, HCV. However products from human blood should be considered as potentially infectious.

**Specimen collection and storage<sup>4</sup>**

- Obtain venous blood by clean vein puncture.
- Immediately mix 9 parts blood with 1 part 3.2% sodium citrate (0.105M) and mix well
- Centrifuge the specimen at 1500g for 10 min. (platelet < 10000/µl)
- Separate plasma after centrifugation and store in plastic or siliconised glass tube.
- Use plasma within 4 hours, otherwise store frozen and thaw just prior to use.

Stability of plasma: 4h at 18-26°C 8h at 2-8° 30d at -20°C 6m at -70°C

**Procedure**

A. Coatron A procedure  
See application book of device

**B. Coatron X / Coatron M procedure**

- Incubate PT reagent at 37°C for at least 10 minutes
- Pipette 25 µl of sample into a test cuvette. Incubate at 37°C for 1-2 minutes.
- Swirl reagent short before use, if no magnetic stir bar is applied
- Add 50 µl of PT reagent (37°C) and simultaneously start measurement.

For other instrument, please refer to your instrument manual for more detailed instrument specific instructions.

**Expected Results**

Typical seconds: 11 – 18 sec  
Normal range: 70 - 130% 0.85 – 1.15 INR

However results are influenced by instruments, technique, calibration etc. Each laboratory is recommended to establish its own range on the specific instrument used.

**Standardisation and Calibration**

The PT result is expressed as seconds or activity (% Quick) or INR (International Normalised Ratio).

**INR results:**

were calculated from normal time and ISI value (international sensitivity index). First is obtained by running fresh plasma from a pool of healthy individuals. The ISI value is stated in the LOT specific certificate of analysis.

$$INR = \left( \frac{Patient\ PT}{Normal\ PT} \right)^{ISI}$$

**Activity % (Quick) result:**

were calculated from a calibration curve, which is prepared from reference plasma (e.g. TECAL N) and dilutions in saline solution like 0.9% NaCl<sub>2</sub> or TECLot IBS buffer. At least three or more calibration points are recommended. The calibration curve must be confirmed with control plasma in normal and abnormal range.

% of normal	100%*	50%	25%	12.5%**
diluted in saline	not dil.	1+1	1+3	1+7

\*The median of at least 21 healthy individuals is defined as 100%.<sup>5</sup>

\*\*12.5% dilution may cause "+++" results in some cases, because the level of fibrinogen is too high diluted for optical detection.

**Quality Control**

TEControl or other commercial control plasma should be used for reliable quality control of performance at a frequency in accordance with good laboratory practice (GLP). TEControl can be frozen one time after reconstitution. 120-150 µl stored in closed polypropylen tubes at -20°C is stable for 30 days

**Limitations**

Great care must be taken to minimize variations which may occur by seemingly insignificant factors.

**A. Specimen Collection. AVOID:**

- Use only plastic tubes or siliconised glass.
- Delayed mixing of blood with anticoagulant.
- Contamination with tissue thromboplastin.
- Improper ratio of anticoagulant with blood.
- Hemolyzed, icteric or lipemic samples may interfere optical systems
- Use only samples with less than 0.8 U/ml heparin concentration

**B. Laboratory Techniques**

- Perform tests at 37°C.
- Use only high purity water.
- Optimum pH is 7.0-7.5.
- ISI value is not constant within the first 30 min after reconstitution.
- Reagent sediments and must be swirled before each testing.

**Performance Characteristics**

Typical performance on instrument Coatron M4

Precision:	CV% (within run)	CV% (inter-runs)
Normal control	< 3,0	< 5,0
Abnormal control	< 3,0	< 5,0

**Warranty**

This product is warranted to perform in accordance with its labelling and literature. TECO disclaims any implied warranty of merchantability or fitness for any other purpose, and in no event will TECO be liable for any consequential damages arising out of aforesaid express warranty.

**References**

- Quick, A.J., The Hemorrhagic Diseases and the Physiology of Hemostasis. Charles C. Thomas: Springfield, IL. 1942.
- Quick, A.J., Hemorrhagic Diseases. Lea and Febiger: Philadelphia. 1957.
- Miale, J.B., Laboratory Medicine-Hematology, 4th Edition. C.V. Mosby: St. Louis. 1972.
- National Committee for Clinical Laboratory Standards: Guidelines for the Standardized Collection, Transport and Preparation of Blood Specimens for Coagulation Testing and Performance of Coagulation Assays.
- Besselaar A M H P van den, Lewis SM, Mannucci P n Poller L. 1993. Status of present and candidate International Reference Preparations (IRP) of thromboplastin for prothrombin time. Thromb Hemostas 69: 85
- Besselaar A M H P van den. 1991. The significance of the International Normalized Ratio (INR) for oral anticoagulant therapy. H17CC 3: 146153.

**Symbol keys**

Expiry date	In Vitro Diagnostica	Biological hazard	Catalogue Number	Reconstitute with dest. water	Consult accompanying documents
Store at 2-8°C	EU conformity	Manufacturer	Lot. Number	Ready to use	Authorized Representative

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### Bestimmungen

Coatron M**	200 Def.	800 Def.	2000 Def.
Coatron A4	100 Def.	400 Def.	1000 Def.
Coatron A6	200 Def.	800 Def.	2000 Def.

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### Referenzen

- Quick, A.J., The Hemorrhagic Diseases and the Physiology of Hemostasis. Charles C. Thomas: Springfield, IL. 1942.
- Quick, A.J., Hemorrhagic Diseases. Lea and Febiger: Philadelphia. 1957.
- Miale, J.B., Laboratory Medicine-Hematology, 4th Edition. C.V. Mosby: St. Louis. 1972.
- National Committee for Clinical Laboratory Standards: Guidelines for the Standardized Collection, Transport and Preparation of Blood Specimens for Coagulation Testing and Performance of Coagulation Assays.
- Besselaar A M H P van den, Lewis SM, Mannucci P n Poller L. 1993. Status of present and candidate International Reference Preparations (IRP) of thromboplastin for prothrombin time. Thromb Hemostas 69; 85
- Besselaar A M H P van der, 1991. The significance of the International Normalized Ratio (INR) for oral anticoagulant therapy. H17CC 3; 146153.

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**Referências**

- Quick, A.J., The Hemorrhagic Diseases and the Physiology of Hemostasis. Charles C. Thomas: Springfield, IL. 1942.
- Quick, A.J., Hemorrhagic Diseases. Lea and Febiger: Philadelphia. 1957.
- Miale, J.B., Laboratory Medicine-Hematology, 4th Edition. C.V. Mosby: St. Louis. 1972.
- National Committee for Clinical Laboratory Standards: Guidelines for the Standardized Collection, Transport and Preparation of Blood Specimens for Coagulation Testing and Performance of Coagulation Assays.
- Besselaar A M H P van den, Lewis SM, Mannucci P n Poller L. 1993. Status of present and candidate International Reference Preparations (IRP) of thromboplastin for prothrombin time. Thromb Hemostas 69; 85
- Besselaar A M H P van den. 1991. The significance of the International Normalized Ratio (INR) for oral anticoagulant therapy. H17CC 3; 146153.

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A0320-050

### Intended Use

Clotting test for quantitative determination of the Activated Partial Thromboplastin Time (APTT) in citrated human plasma using silicate as contact activator for factor XII. Intended to be used by professional laboratory personnel using coagulation analysers. The determination of the APTT is used for the global evaluation of the intrinsic pathway and detecting deficiencies of the intrinsic coagulation factors VIII, IX, XI, XII, and Fletcher Factor or other coagulation methods where an APTT reagent is required<sup>1,2</sup>.

The APTT reagent in the kit contains phospholipids and silica to ensure a highly consistent and stable product<sup>3</sup>. The APTT reagent is lupus anticoagulant insensitive. Lupus anticoagulant insensitive reagents yield more reliable factor assay results than reagents, which are sensitive to lupus inhibitors<sup>4</sup>.

Prolonged clotting times may be observed in the following situations: deficiency of intrinsic coagulation factors, presence of heparin or other anticoagulants, which affect the intrinsic pathway and in liver diseases.

### Contents

Product	TECLOT aPTT-S
REF	A0320-050
aPTT-S reagent	10x5 mL
Determinations*	2000

\*Micro method (75µL in total)

APTT-S reagent contains colloidal silicate with phospholipids, buffer and preservatives.

Recommended additional material (not included in package)

Auxiliary reagents	A0350-050 Calcium Chloride 0.025M, 10 x 5mL
Calibration	not required
Quality Control	P6001-010 TECControl N, 10 x 1mL P6101-010 TECControl A, 10 x 1mL

### Preparation

Ready to use. Swirl APTT reagent gently prior usage

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### Precautions

The reagent contains sodium azide (less than 0.1%) to prevent microbial growth. Avoid contact with skin and eyes. Wear suitable protective clothing. Dispose components in compliance with local regulations for infectious material.

### Specimen collection and storage<sup>5</sup>

- Obtain venous blood by clean vein puncture.
- Immediately mix 9 parts blood with 1 part 3.2% sodium citrate (0.105M) and mix well
- Centrifuge the specimen at 1500g for 10 min. (platelet < 10000/µL)
- Separate plasma after centrifugation and store in plastic or siliconised glass tube.
- Use plasma within 4 hours, otherwise store frozen and thaw just prior to use.

Stability of plasma: 4h at 18-26°C 8h at 2-8° 14d at -20°C 6m at -70°C

### Procedure

#### A. Automated Method: Coatron A

See application book of device

#### B. Manual Method: Coatron X

- Prewarm **CaCl<sub>2</sub>** (0.025M) at 37°C for at least 10 min
- Pipette **25 µl of sample** into a test cuvette. Prewarm at 37°C for 1-2 minutes.
- Add **25 µl APTT-S reagent** and incubate exactly for **3 min** at 37°C.
- Add **25 µl of CaCl<sub>2</sub>** (0.025M) and simultaneously start test.
- Record the clotting time in seconds.

### Expected Results

Typical normal results are 27-42 sec. However results are influenced by the method of clot detection and can vary from laboratory to laboratory. Each laboratory is recommended to establish its own normal range on the specific instrument used.

### Quality Control

TEControl or other commercial control plasma should be used for reliable quality control of performance at a frequency in accordance with good laboratory practice (GLP). TECControl can be frozen one time after reconstitution. 120-150 µL stored in closed polypropylen tubes at -20°C is stable for 30 days

### Limitations

#### A. Specimen Collection. AVOID:

- Use only plastic tubes or siliconised glass.
- Delayed mixing of blood with anticoagulant.
- Contamination with tissue thromboplastin.
- Improper ratio of anticoagulant with blood.
- Hemolyzed, icteric or lipemic samples may interfere optical systems

#### B. Laboratory Techniques

- Perform tests at 37°C.
- Use only high purity water.
- Optimum pH is 7.0-7.5.

#### C. Interfering substances

- Bilirubin 40mg/dL
- Haemoglobin 1000 mg/dL

### Performance Characteristics

#### Typical performance on instrument Coatron X

<b>Precision:</b>	CV% (within run)	CV% (inter-runs)
QC control	< 3,0	< 5,0

#### Factor & Heparin sensitivity:

Factor (%)	APTT Clotting time (s)		
	F VIII	F IX	F XI
< 1%	100	80	103
10%	53	52	58
40%	40	39	41
100%	35	35	358

Heparin (U/mL)	0 U/mL	0,2 U/mL	0,4 U/mL
APTT clotting time (s)	35	70	180

These values should be used as guidelines only. Each laboratory should establish factor or heparin sensitivity using its own instruments and techniques.

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### References

- Proctor RR, Rapaport SI. The partial thromboplastin time with kaolin. A simple screening test for first stage plasma clotting factor deficiencies. *Am J Clin Pathol* 36, 212-219 (1961).
- Triplett DA, Harns CS, Koepke JA. The effect of heparin on the activated partial thromboplastin time. *Am J Clin Pathol* 70, 556-569 (1978).
- Stevenson KJ, Easton AC, Thomson JM, Poller L. The reliability of activated partial thromboplastin time methods and the relationship to lipid composition and ultrastructure. *Thromb Haemost* 55, 250-258 (1986).
- Denis-Magdelaine A, Flahault A, Verdy E. Sensitivity of sixteen APTT reagents for the presence of lupus anticoagulants. *Haemostasis* 25, 98-105 (1995).
- NCCLS: Guidelines for the Standardized Collection, Transport and Preparation of Blood Specimens for Coagulation Testing and Performance of Coagulation Assays

Symbols key:

Expiry date	In Vitro Diagnostica	Biological hazard	Catalogue Number	Consult accompanying documents
Store at 2-8°C	EU conformity	Manufacturer	Lot. Number	Authorized Representative

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**Referenzen**

- Proctor RR, Rapaport SI. The partial thromboplastin time with kaolin. A simple screening test for first stage plasma clotting factor deficiencies. *Am J Clin Pathol* 36, 212-219 (1961).
- Triplett DA, Harms CS, Koepke JA. The effect of heparin on the activated partial thromboplastin time. *Am J Clin Pathol* 70, 556-569 (1978).
- Stevenson KJ, Easton AC, Thomson JM, Poller L. The reliability of activated partial thromboplastin time methods and the relationship to lipid composition and ultrastructure. *Thromb Haemostasis* 55, 250-258 (1986).
- Denis-Magdelaine A, Flahault A, Verdy E. Sensitivity of sixteen APTT reagents for the presence of lupus anticoagulants. *Haemostasis* 25, 98-105 (1995).
- NCCLS: Guidelines for the Standardized Collection, Transport and Preparation of Blood Specimens for Coagulation Testing and Performance of Coagulation Assays

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# Calcium Chloride 0,025M

# TECO

**REF****A0350-050***In-Vitro-Diagnostic reagent for professional use*

(EN)

**Information for Use**

Laboratory reagent - suitable to add calcium ions to the citrated human plasma samples, approved and recommended to combine with TEClot APTT-S / PT-B reagent.

**Contents**

Product	Calcium Chloride 0.025M
REF	A0350-050
CaCl <sub>2</sub> 0.025M	10x5 mL

**Preparation**

Ready to use.

**Storage and Stability**

Unopened product is stable until the expiration date shown on the label stored at 2-8°C.

Opened product is stable for 30 days if stored at 2-8°C in the original vial.

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Calcium Chloride Solution contains sodium azide (<0.1%) . Sodium azide under acid conditions yields Hydrazoic acid, a toxic compound. Azide compounds should be diluted with running water before being discarded. Upon disposal, azide compounds should be flushed with large volumes of water. These precautions are recommended to avoid deposits in metal pipes in which explosive conditions may develop.

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**Inhalt**

Product	Calcium Chloride 0.025M
REF.	A0350-050
CaCl <sub>2</sub> 0.025M	10x5 mL

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A0401-020

**Intended Use**

TEClot TT reagent can be used manually or on semi-automated and automated instruments. The test is commonly applied to detect various sources of interference with normal blood coagulation. Prolongation of the thrombin clotting time can be taken as a qualitative indication of abnormal fibrinogen levels (high or low), or the presence of interfering substances such as FDP's or heparin. Quantitative evaluation of the possible causes of prolonged thrombin clotting time should be performed as follow-up studies, such as APTT or chromogenic assay for heparin, Clauss fibrinogen, FDP determinations, heparin neutralization by protamine sulphate or polybrene<sup>1</sup>, normal plasma mixing studies<sup>2</sup> or reptilase assay<sup>3</sup> to distinguish between hypofibrinogenemia and FDP effects.

**Contents & Determinations**

Product	TEClot TT
Cat.No.	A0401-020
TT Reagent*	10 x 2mL

**Determinations**

Coatron M	Coatron A4	Coatron A6
800	400	800

\*Each vial contains a lyophilized preparation of 26 units bovine thrombin with buffers and stabilizers. The final thrombin concentration in test application for Coatron systems is 4.33 E/mL.

**Preparation**

Reconstitute the reagent with 2 mL purified water. Allow to stand for 5 minutes then mix gently by inversion.

**Storage & Stability**

Unopened reagents are stable until the expiration date shown on the label stored at 2°-8°C. Reconstituted reagent:

	-20 °C	12 °C	20-25 °C
TT Reagent	1 month	24 hours	8 hours

The vials can be only frozen once.

**Precautions**

Avoid contact with skin and eyes. Wear suitable protective clothing. Dispose components in compliance with local regulations for infectious material. All components are checked for HIV, HBV and HCV. However products from human blood should be considered as potentially infectious.

**Specimen collection and storage<sup>4</sup>**

1. Obtain venous blood by clean vein puncture.
2. Immediately mix 9 parts blood with 1 part 3.2% sodium citrate (0.105M) and mix well
3. Centrifuge the specimen at 1500g for 10 min. (platelet < 10000/ $\mu$ L)
4. Separate plasma after centrifugation and store in plastic or siliconised glass tube.
5. Use plasma within 4 hours, otherwise store frozen and thaw just prior to use.

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(Refer to instrument operation manual for detailed instructions)

**B. Manual Method: Coatron M system**

1. Pipette **50  $\mu$ l of sample** into a test cuvette. Incubate at 37°C for 1-2 minutes.
2. Add **25  $\mu$ l of TEClot TT** reagent and simultaneously start test.
3. Record the clotting time in seconds.

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**Expected Results**

Typical normal results are 12 – 24 s<sup>6</sup>. However results are influenced by the method of clot detection and can vary from laboratory to laboratory. Each laboratory is recommended to establish its own normal range on the specific instrument used.

**Quality Control**

TEControl or other commercial control plasma should be used for reliable quality control of performance at a frequency in accordance with good laboratory practice (GLP). TEControl can be frozen one time after reconstitution. 120-150  $\mu$ l stored in closed polypropylen tubes at -20°C is stable for 30 days

**Limitations****A. Specimen Collection. AVOID:**

1. Use only plastic tubes or siliconised glass.
2. Delayed mixing of blood with anticoagulant.
3. Contamination with tissue thromboplastin.
4. Improper ratio of anticoagulant with blood.
5. Hemolyzed, icteric or lipemic samples may interfere optical systems
6. Heparin below 1U/mL will not interfere results.

**B. Laboratory Techniques**

1. Perform tests at 37°C.
2. Use only high purity water.
3. Optimum pH is 7.0-7.5.

As well as the cause of elongated Thrombin Clotting Times indicated above, a recent report has suggested that many systemic amyloidosis patients with bleeding complications may have a circulating inhibitor which prolongs the Thrombin Clotting Time<sup>5</sup>. Also therapeutic levels of heparin may entirely abolish clotting in the Thrombin Clotting Time test, although neutralization with protamine sulphate or polybrene should correct the Thrombin Clotting Time<sup>1</sup>.

**Performance Characteristics**

<b>Precision:</b>	CV% (within run)	CV% (inter-runs)
Normal control	< 3.0	< 5.0
Abnormal control	< 3.0	< 5.0

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**References**

1. Laposata et al. The Clinical Haemostasis Handbook, Yearbook Medical Publishers Inc., p219, 1989.
2. Thompson, A.R. and Harker, L.A. Manual of Haemostasis and Thrombosis. 3rd Ed., F.A. Davis Co., p62,1983.
3. DeMott, W.R., in: Laboratory Test Handbook, 2nd Ed., Jacobs D.S. et al Eds., Lexi.Comp Inc., p432-433, 1990.
4. NCCLS: Guidelines for the Standardized Collection, Transport and Preparation of Blood Specimens for Coagulation Testing and Performance of Coagulation Assays
5. Gastineau, D.A. et al. Inhibitor of the Thrombin Time in Systemic Amyloidosis: A Common Coagulation Abnormality Blood, 1991, 77: 2637-2640.
6. Lothar Thomas, Labor und Diagnose, 6.Auflage,2005, Page846

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**B. Manuelle Methode: Coatron M system**

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**Referenzen**

- Laposata et al. The Clinical Haemostasis Handbook, Yearbook Medical Publishers Inc., p219, 1989.
- Thompson, A.R. and Harker, L.A. Manual of Haemostasis and Thrombosis. 3rd Ed., F.A. Davis Co., p62, 1983.
- DeMott, W.R., in: Laboratory Test Handbook, 2nd Ed., Jacobs D.S. et al Eds., Lexi-Comp Inc., p432-433, 1990.
- NCCLS: Guidelines for the Standardized Collection, Transport and Preparation of Blood Specimens for Coagulation Testing and Performance of Coagulation Assays
- Gastineau, D.A. et al. Inhibitor of the Thrombin Time in Systemic Amyloidosis: A Common Coagulation Abnormality Blood, 1991, 77: 2637-2640.
- Lothar Thomas, Labor und Diagnose, 6.Auflage, 2005, Page846

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### Intended Use

The TEClot FIB is intended for the quantitative determination of fibrinogen in human plasma according to method developed by Clauss.<sup>1</sup> Levels of fibrinogen can increase as a result of inflammation, pregnancy or oral contraceptive use<sup>2</sup>. Decreased levels can be found in certain states such as liver disease and DIC. Congenital deficiencies include afibrinogenemia (no detectable fibrinogen), hypofibrinogenemia (<1 mg/ml) and dysfibrinogenemia (abnormal fibrinogen molecule).

### Content & Determination

Product	TEClot FIB	TEClot FIB
REF	A0511-020	A0511-050
Thrombin reagent	10x2 mL	10x5 mL
Determinations*	800	2000

\*Micro method (75µL in total)

Thrombin reagent contains bovine thrombin (~80NIH) with stabilizers

### Additional required material (not included to package)

Auxiliary reagents	A0590-125 IBS buffer, 1 x125mL
Calibration	P8001-005 TECal N, 5 x 1mL
Quality Control	P6001-010 TEControl N, 10 x 1mL P6101-010 TEControl A, 10 x 1mL

### Preparation

A0511-020: Reconstitute with 2mL purified water  
A0511-050: Reconstitute with 5mL purified water

Swirl gently after reconstitution and allow standing for 15 minutes at room temperature. Mix well before use. Do not shake.

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\* Reagent must be protected from UV-light and evaporation

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See application book of device

#### B. Manual Method: Coatron X

- Preparation of Standard, Control and Patient Dilutions

Standard Dilution	Plasma	IBS Buffer
1:5	200µL Standard	800µL
1:10	500µL 1:5 STD	500µL
1:20	500µL 1:10 STD	500µL
1:40	500µL 1:20 STD	500µL
Patient or Control	100µL Plasma	900µL

- Pipette **50 µL diluted standard or patient plasma** (1:10) into a test cuvette. Prewarm at 37°C for 1-2 minutes.
  - Add **25 µL Thrombin reagent** and simultaneously start test.
- For other instrument, please refer to your instrument manual for more detailed instrument specific instructions.

### Calibration

TECal Normal or other commercially prepared plasma standard in which Fibrinogen has been determined should be used as reference (200-300mg/dL). Plot the clotting time obtained with each of the FIB standard dilutions on the y-axis against the concentration of FIB (mg/dL) on the x-axis using log-log graph paper. The line of best fit should be determined by linear regression analysis. The fibrinogen in plasma samples can be determined by interpolation from the calibration curve.

### Expected Results

Typical normal results are 180-450 mg/dL<sup>4,5</sup>. However results are influenced by the method of clot detection and can vary from laboratory to laboratory. Each laboratory is recommended to establish its own normal range on the specific instrument used.

### Quality Control

TEControl or other commercial control plasma should be used for reliable quality control of performance at a frequency in accordance with good laboratory practice (GLP). TEControl can be frozen one time after reconstitution. 120-150 µL stored in closed polypropylen tubes at -20°C is stable for 30 days

### Limitations

- A. Specimen Collection. AVOID:
- Use only plastic tubes or siliconised glass.
  - Delayed mixing of blood with anticoagulant.
  - Contamination with tissue thromboplastin.
  - Improper ratio of anticoagulant with blood.
  - Hemolyzed, icteric or lipemic samples may interfere optical systems
- B. Laboratory Techniques
- Perform tests at 37°C.
  - Use only high purity water.
  - Optimum pH is 7.0-7.5.

### Performance Characteristics

<b>Precision:</b>	CV% (within run)	CV% (inter-runs)
Normal control	< 5.0	< 5.0
Abnormal control	< 5.0	< 10.0

(Typical performance on instrument Coatron M4)

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### References

- Clauss, A., Gerinnungsphysiologische Schnellmethode zur bestimmung des Fibrinogens. Acta Haematol., 1957, 17: 237-246.
- Shaw, T.S., Assays for Fibrinogen and its Derivatives, CRC Crit. Rev. Clin. Lab. Sci., 1977, 8: 145-192.
- National Committee for the National Laboratory (NCCLS) Standards: Collection transport and preparation of blood specimens for coagulation testing and performance of coagulation assays. Document H21-A2, vol. 11, No. 23, 1991.
- Scully, R.E. et al., Normal Reference Laboratory Values, N. Eng. J. Med., 1980, 302(37) : 37-48.
- Okuno, T. and Selenko, V., Amer. J. Med. Tech., 1972, 38(6) : 196-201.

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### Referenzen

1. Clauss, A., Gerinnungsphysiologische Schnellmethode zur Bestimmung des Fibrinogens. Acta Haematol., 1957, 17: 237-246.
2. Shaw, T.S., Assays for Fibrinogen and its Derivatives, CRC Crit. Rev. Clin. Lab. Sci., 1977, 8: 145-192.
3. National Committee for the National Laboratory (NCCLS) Standards: Collection transport and preparation of blood specimens for coagulation testing and performance of coagulation assays. Document H21-A2, vol. 11, No. 23, 1991.
4. Scully, R.E. et al., Normal Reference Laboratory Values, N. Eng. J. Med., 1980, 302(37) : 37-48.
5. Okuno, T. and Selenko, V., Amer. J. Med. Tech., 1972, 38(6) : 196-201.

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(EN)

**Information for Use**

General Buffer Solution – suitable as Laboratory reagent for dilution of citrated human plasma samples - recommended in combination with TEClot FIB.

**Content**

Product	IBS Buffer
Cat.No.	A0590-125
IBS Buffer	1x125 mL

**Preparation**

IBS: pH 7.3 - 7.4, liquid  
Ready to use.

**Storage and Stability**

Unopened reagents are stable until the expiration date shown on the label stored at 2-8°C.

**Precautions**

Avoid contact with skin and eyes. Wear suitable protective clothing. Dispose components in compliance with local regulations for infectious material.

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**Intended Use**

Use as a normal control for following coagulation tests:

**PT, APTT, Thrombin time, Fibrinogen,  
Anti-thrombin and D-Dimer**

**Contents**

10 x 1mL freeze dried citrate-anticoagulated human plasma

**Preparation**

Reconstitute individual vials with **1,0 ml** distilled water. Allow to stand at room temperature, with occasional swirling, for 15 min before use. Be certain all particulate matter is well dissolved.

PT whole blood (TEClot PT-B): Reconstitute individual vials with **1,7 ml** distilled water.

**Storage & Stability**

Unopened vials are stable until the expiration date shown on the label stored at 2°-8°C.

Dissolved plasma change analytic levels below 10% if stored as following:

-20 °C	2-8 °C	20-25 °C
1 month	8 hours	4 hours

Dissolved plasma can be refrozen only one time in aliquots (120-150µL). Stored at -20°C in closed polypropylene tubes, the aliquots must be used within 30 days.

**Precautions**

This product contains substance from human origin!  
Avoid contact with skin and eyes. Wear suitable protective clothing. Dispose components in compliance with local regulations for infectious material. All components are checked for HIV, HBV and HCV. However products from human blood should be considered as potentially infectious.

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### Intended Use

Use as a calibrator or normal control for following coagulation tests:

**PT, APTT, Thrombinfime, Fibrinogen,**  
**Factors: II, V, VII, VIII, IX, X, XI, XII,**  
**Antithrombin, Protein-C, free Protein-S,**  
**D-Dimer**

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PT whole blood (TECLOT PT-B CAT=A0260 xxx): Reconstitute individual vials with **1,7 ml** distilled water.

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### Precautions: Potential Biohazardous material

This product contains substance from human origin! Avoid contact with skin and eyes. Wear suitable protective clothing. Dispose components in compliance with local regulations for infectious material. All components are checked for HIV, HBV and HCV. However products from human blood should be considered as potentially infectious.

### Performance Characteristics:

Refer to "Certificate of Analysis".

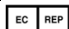
### Limitations:

The control plasma is subject to the limitations of the assay system (reagent + instrument). Results out of expected range may indicate deterioration, false test calibration or problems with one or more components of the test system

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Symbols key:

 Expiry date	 In Vitro Diagnostica	 Biological hazard	 Catalogue Number	 Consult accompanying documents
 Store at 2-8°C	 EU conformity	 Manufacturer	 Lot. Number	 Authorized Representative

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