

INSTRUCTION FOR USE

ONESEPT ENZYM ENZYMATIC CLEANER FOR MEDICAL INSTRUMENTS AND ENDOSCOPES



General Features

- It is compatible with glass, ceramic, silicone, plastic, PVC, acrylic, polymer, polycarbonate, polystyrene, HDPE, porcelain and stainless steel materials. Anticorrosive effective.

Product package content; ONESEPT ENZYM 5LT (REF: KAF G40)

Chemical Composition

Contents
%1-%5 (protease, amylase, lipase) Glycerin, Monopropylene glycol, Nonionic surfactants, Complexing agents, Anticorrosive substances.

Purpose of usage/usage areas:

Suitable for endoscope washing machines, ultrasonic washing machines and manual use. It provides the cleaning and pre-disinfection of all kinds of invasive surgical instruments, dental surgical instruments, surgical laparoscopy instruments, flexible and rigid endoscopes. Provides a more effective disinfection process by cleaning organic wastes on the surfaces such as blood, fat, protein and starch. Compatible with glass, rubber, wood, ceramic, silicone, plastic and stainless steel.

Usage

- It is used by diluting in % 0,5 concentration. All instruments are immersed into the solution. Shows maximum activity at 20- 45°C. It is effective in 5 minutes in manual and ultrasonic usage. The instruments, have been removed from the solution at the end of the process, are rinsed thoroughly.



WARNINGS and PRECAUTIONS

- Wear gloves when using. Do not use after the expiry date. Read the usage instruction. In case of contact with eyes, rinse immediately with plenty of water and consult a doctor. Do not use for cleaning hand, face, body and foodstuffs. Causes severe burns. Do not mix with any other product or substance. Keep away from foodstuffs. Keep out of reach of children. Send to the recycling process after the packaging has been emptied completely. In case of poisoning, call the National Poison Information Centre number for Emergency Health Service.

- Do not breathe dust/fume/gas/mist/vapours/spray.
- Wash thoroughly with water after handling.
- Wear protective gloves/protective clothing/eye protection/face protection.
- IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if fitted and easy to do. Continue rinsing.
- Take off contaminated clothing and wash before reuse.
- Dispose of contents/container in accordance with local regulations.

Storage and Shipping Conditions:






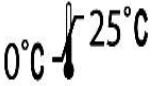
- Store at 0°C -25°C/30°C in its original packaging in a cool and dry place.
- Product shelf life is 2 years from the date of production.
- Use the opened product within 6 (six) months.
- Keep out of reach of children




➤ Sterilization

Our products are used non-sterile.



➤ Symbols Used on the Label and Their Meanings

	Production date		Expiration date
	Read the Instructions for Use.		Protect from sunlight.
	Store in dry place.		Storage temperature range

	Manufacturer Information	LOT	Lot number
	Barcode number	REF	Reference number
	Danger		