


## Setting Moving Speed


The moving speed can be changed in the longitudinal and lateral directions.

- 1 Press the position information area of the X-ray diagnostic table on the touch panel.

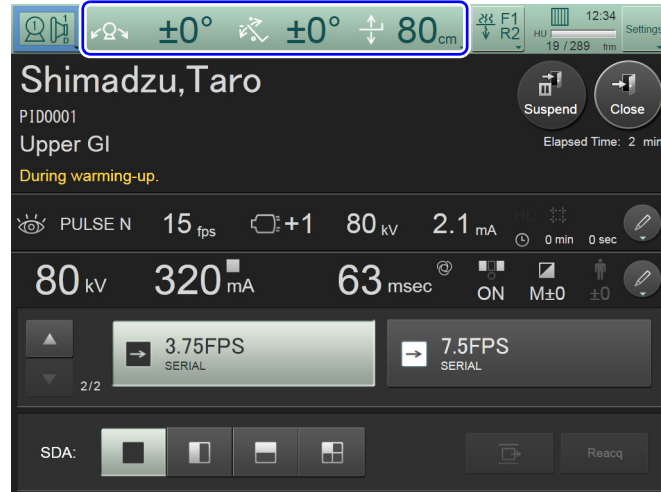

The [R/F Table Setting] screen is displayed.

- 2 Select the each moving speed.  
"1" is the lowest moving speed. Increasing the number increases the speed.

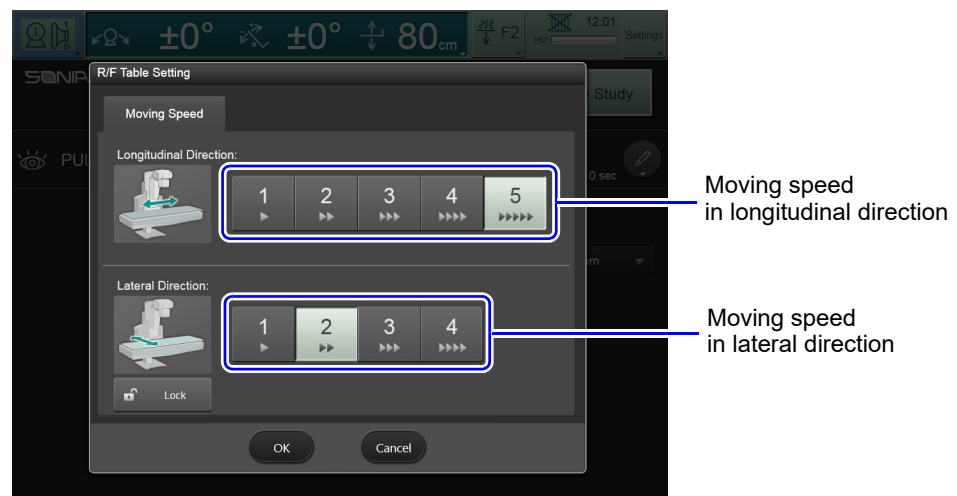

- The moving speed cannot be changed while the X-ray diagnostic table is in operation.
- The moving speed returns to the initial value when the power is turned off.

**Hint** Contact a Shimadzu service representative if you wish to change the initial moving speed.

- 3 When the setup is finished, press [OK].  
The [R/F Table Setting] screen closes.