POWERGRIP

Bipolare Koagulationszange
Bipolar Coagulation Forceps

POWERGRIP

Bipolare Koagulationszange

Bipolar Coagulation Forceps

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Quick exchange of electrode inserts

The different inserts can be exchanged in only a few seconds:
The electrode is screwed into the shaft, inserted in the handle and locked in place by means of a click-stop. At the same time the connection to the integrated electric connector is established.

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Simple one handed operation

The extended flanges on the electrode shaft, give the surgeon enhanced single finger control over the full 360° rotation available, resulting in precise control of the electrode.

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Standard flat plug
with common bipolar cable fitting

Powergrip Griff Powergrip handle	Powergrip Schaft Powergrip shaft	
Art. Nr. / Art. No.	Länge / length	Art. Nr. / Art. No.
82410001	34cm	82410034

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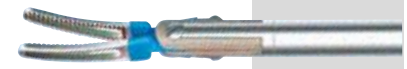
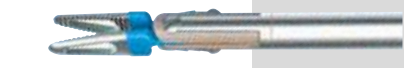
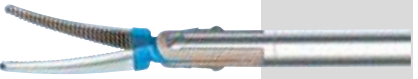
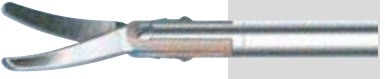
Precise coagulation, more powerful grasping, better cutting and more exact work: the Bissinger POWERGRIP

The Bissinger POWERGRIP is an instrument used for grasping, cutting and bipolar coagulation in Minimally Invasive Surgery. It is characterized by a mechanism enabling well adjustable opening and closing of the jaws with very high pressure while grasping and cutting. This is accompanied by precise coagulation of the tissue. The jaws are moved by a double hinge which is connected electrically and mechanically to the handle by a slide rod. The connector for standardized bipolar cables is integrated in the particularly ergonomic handle made of high quality PEEK polymer. This saves the purchase of expensive special cables and prevents them from being connected incorrectly. We supply standardized bipolar cables for connection to practically all common high-frequency generators. The Bissinger POWERGRIP can be dismantled in three parts. This enables thorough and reliable cleaning and facilitates the execution of repair work or the replacement of defective parts. Beyond that, the Bissinger POWERGRIP offers a number of important further advantages:

- exchangeable electrode inserts for different applications
- easy handling
- excellent cleaning and ease of repair

	Große Fasszange Large grasping forceps	82400010
	Pinzette gebogen Forceps, curved	82400011
	Fasszange gebogen Grasping forceps, curved	82400015
	Delta Fasszange Delta Grasper	82400014

	Dissektor Maryland gebogen Maryland dissector, curved	82400019
	Schere gebogen Scissors, curved	82400018
	Delta Fasszange Delta Grasper	82400014
	Duck Bill Fasszange Duck Bill Grasper	82400008

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Bissinger – Focus on better solutions

Bipolar and monopolar instruments
Accessories for electrosurgery
Surgical instruments
Design and development of instruments
to customer specifications
Service and repair

2-Bananenstecker*

3 m

Art. No. 80100010

5 m

Art. No. 80100013

U20

4 m

Art. No. 80100260

Erbe

3 m

Art. No. 80100017

5 m

Art. No. 80100018

Martin, Berchtold
Aesculap GK 55, GK 60

3 m

Art. No. 80100019

5 m

Art. No. 80100020

Valleylab, Lamidey, EMC

3 m

Art. No. 80100021

5 m

Art. No. 80100022

C € 0297

Günter Bissinger

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Hans-Theisen-Str.1

79331 Teningen/Germany

Tel. +49 7641 9 14 33 0

Fax +49 7641 9 14 33 33

eMail: info@bissinger.com

Internet: <http://www.bissinger.com>

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*Non-compliant with EU standards

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Medizintechnik GmbH