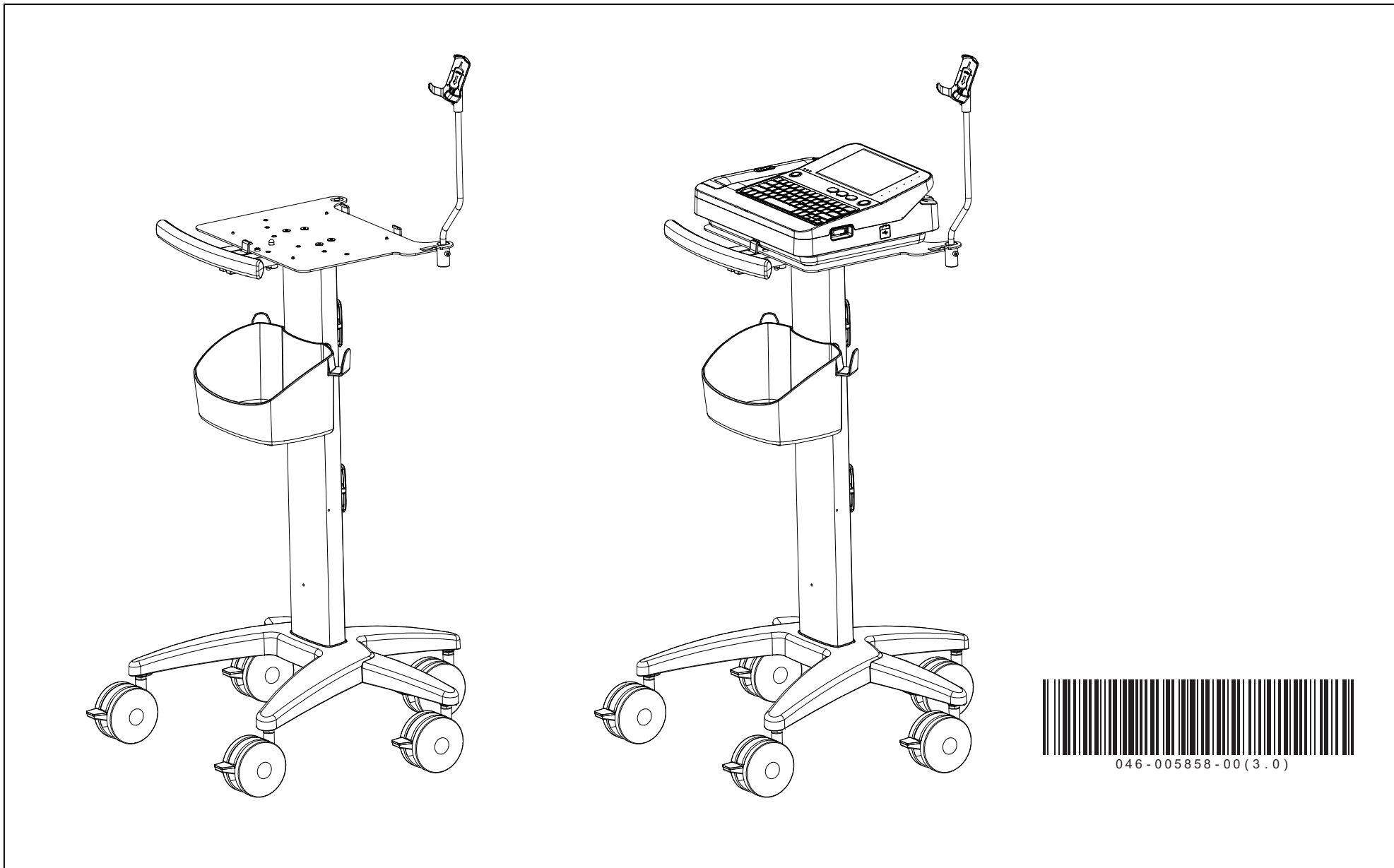
BeneHeart R12/R12A心电图机推车使用说明/Rolling Stand Indication for use for BeneHeart R12/R12A Electrocardiograph

046-005858-00 (3.0)

Statement

This manual provides the instruction necessary to operate the product in accordance with its function and intended use. Observance of this manual is a prerequisite for proper performance and correct operation, and ensures patient and operator safety.

Safety Information

The safety statements presented in this chapter refer to the basic safety information that the operator of wall-mount bracket shall pay attention to and abide by. There are additional safety statements in other sections that are the same as, similar to, or specific to the operations.

- **Danger:** Indicates an imminent hazard situation that, if not avoided, will result in death or serious injury.
- **Warning:** Indicates a potential hazard or unsafe practice that, if not avoided, could result in death or serious injury.
- **Caution:** Indicates a potential hazard or unsafe practice that, if not avoided, could result in minor personal injury or product/property damage.
- **Note:** Provides application tips or other useful information to ensure that you get the most from your product.

There are no dangers that refer to the product in general.

Warnings:

- ✧ During the application, the plastic button of the Trolley Tray Kit must be fastened to the machine reliably.
- ✧ The tilt angle of the plane on which the rollstand is placed shall be not greater than 10°.

Caution:

- ✧ Use the standard parts and components (such as screws and nuts) accompanying the rollstand for assembly. Or use the specifications specified in the instruction.

Notes:

- ✧ Keep this manual close to the rollstand so that it can be obtained conveniently when necessary.
- ✧ Loosen the brake of the wheels before moving the rollstand.

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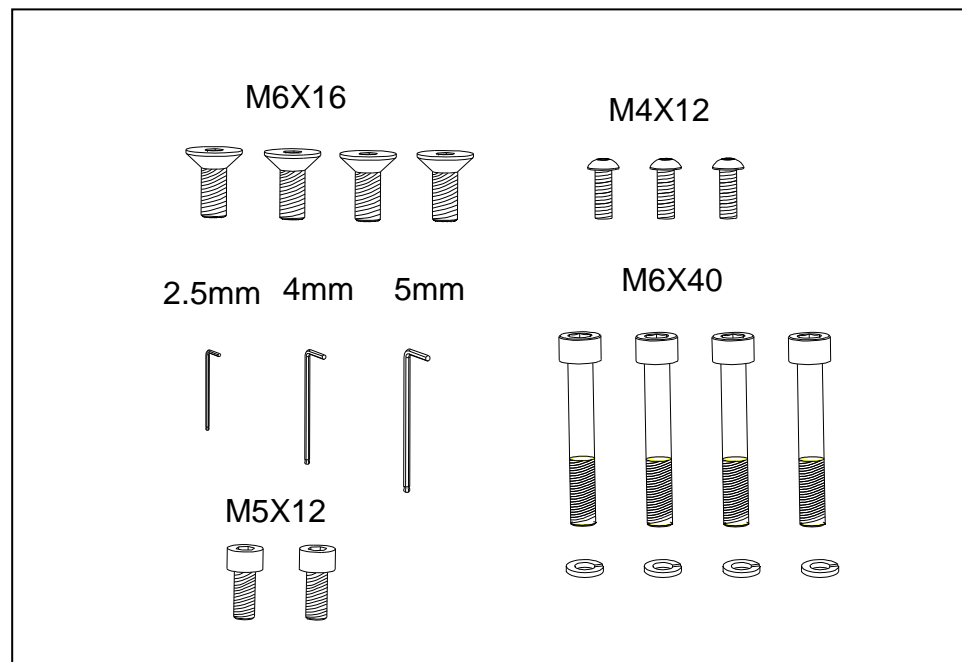
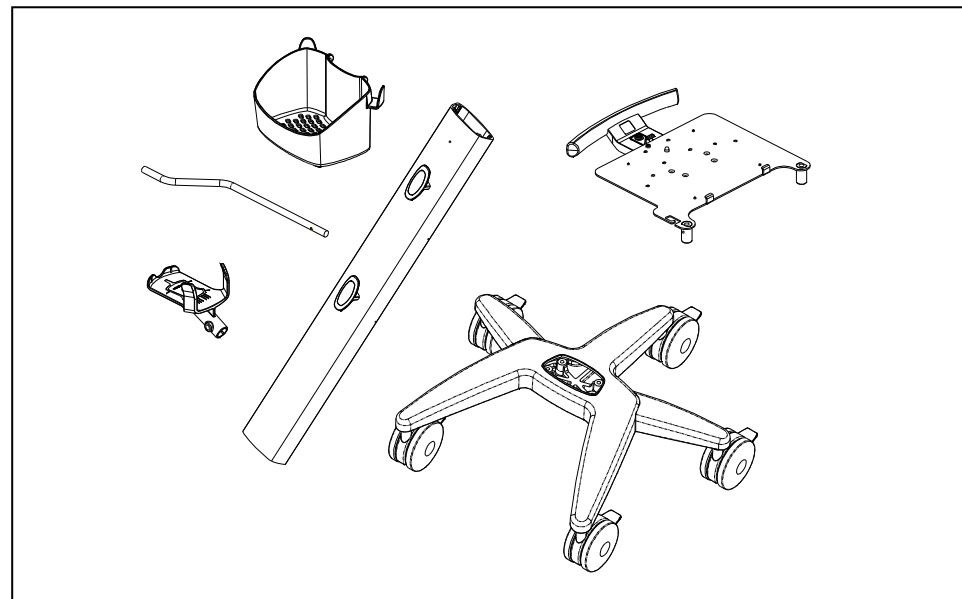
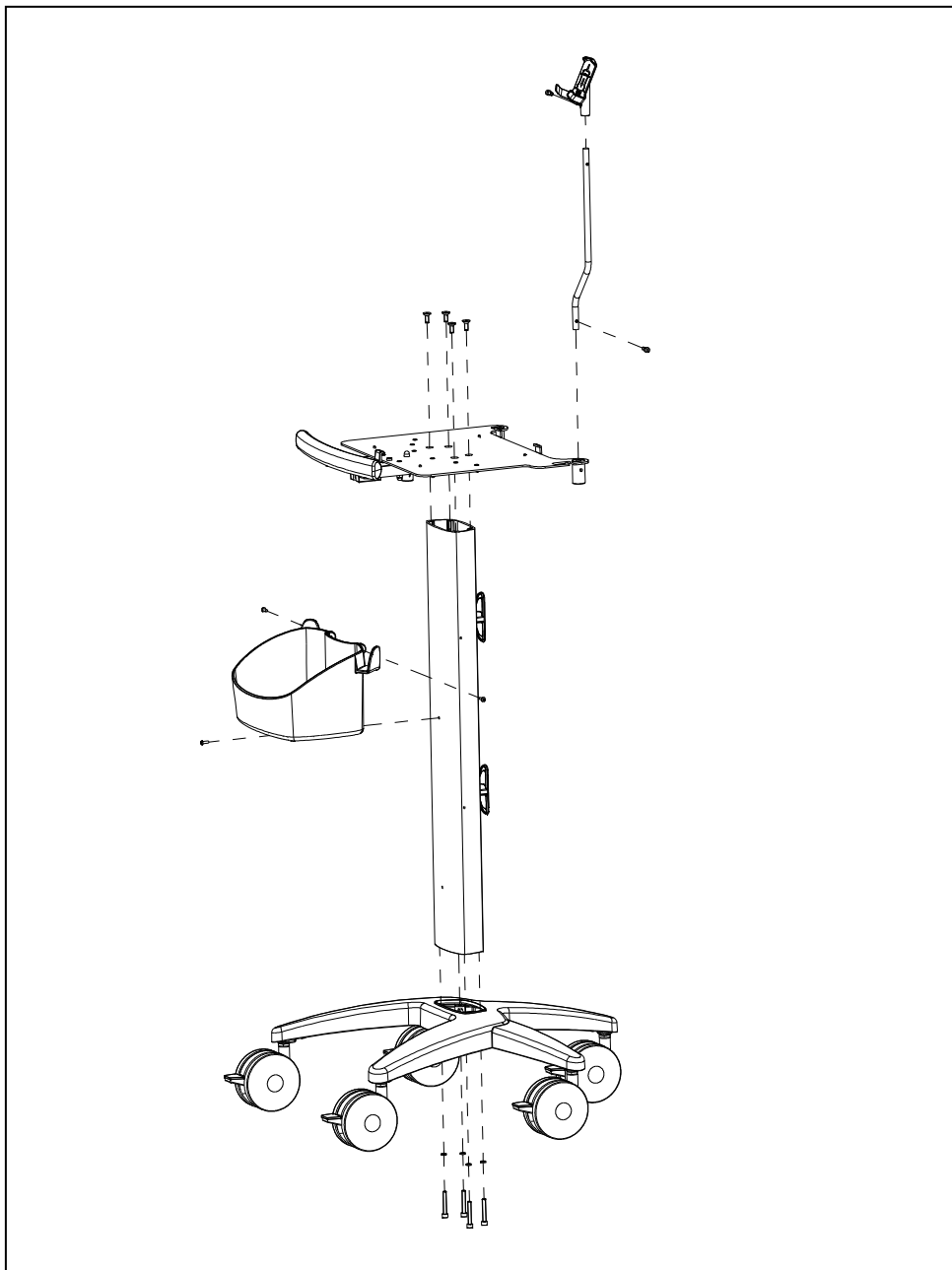
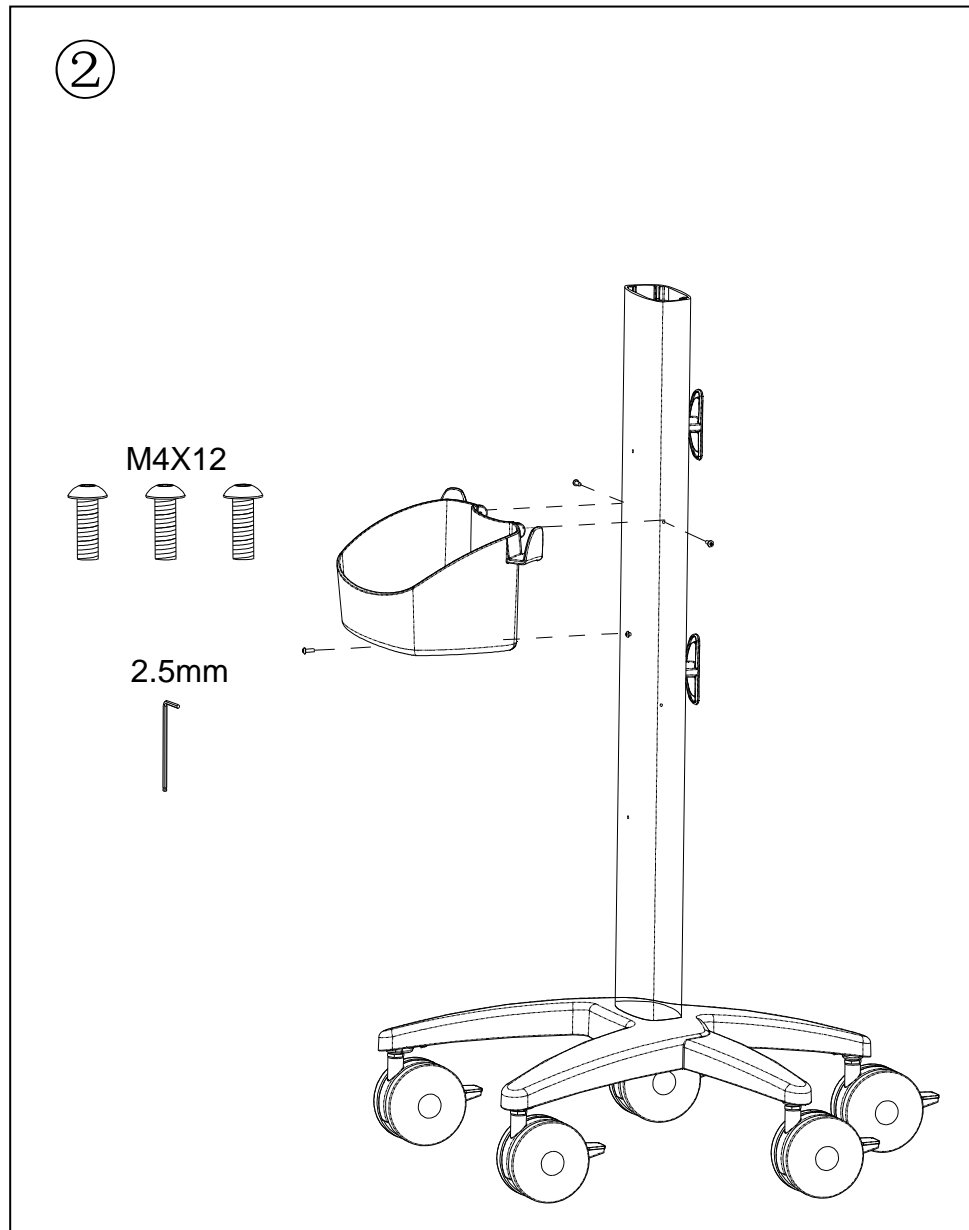
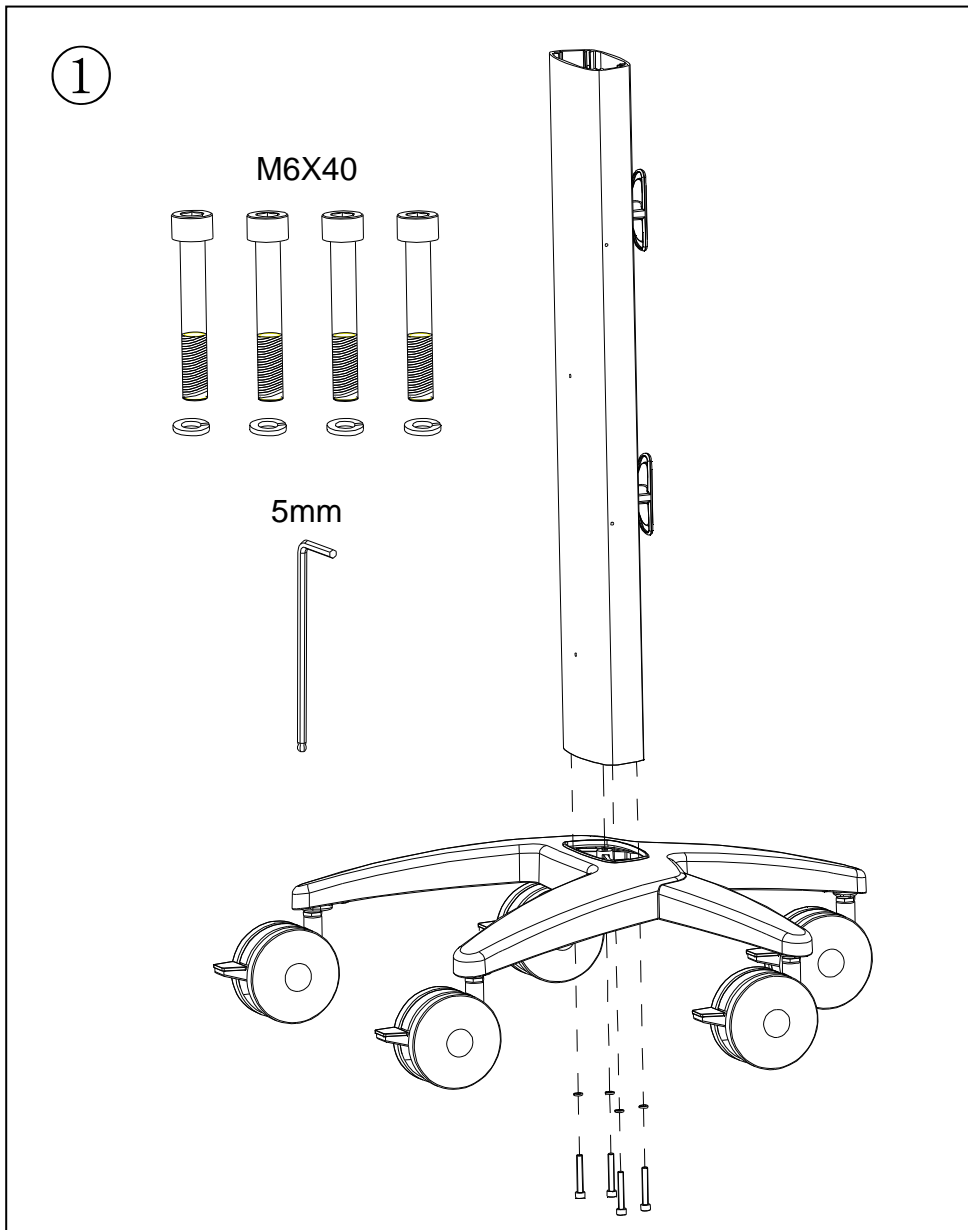
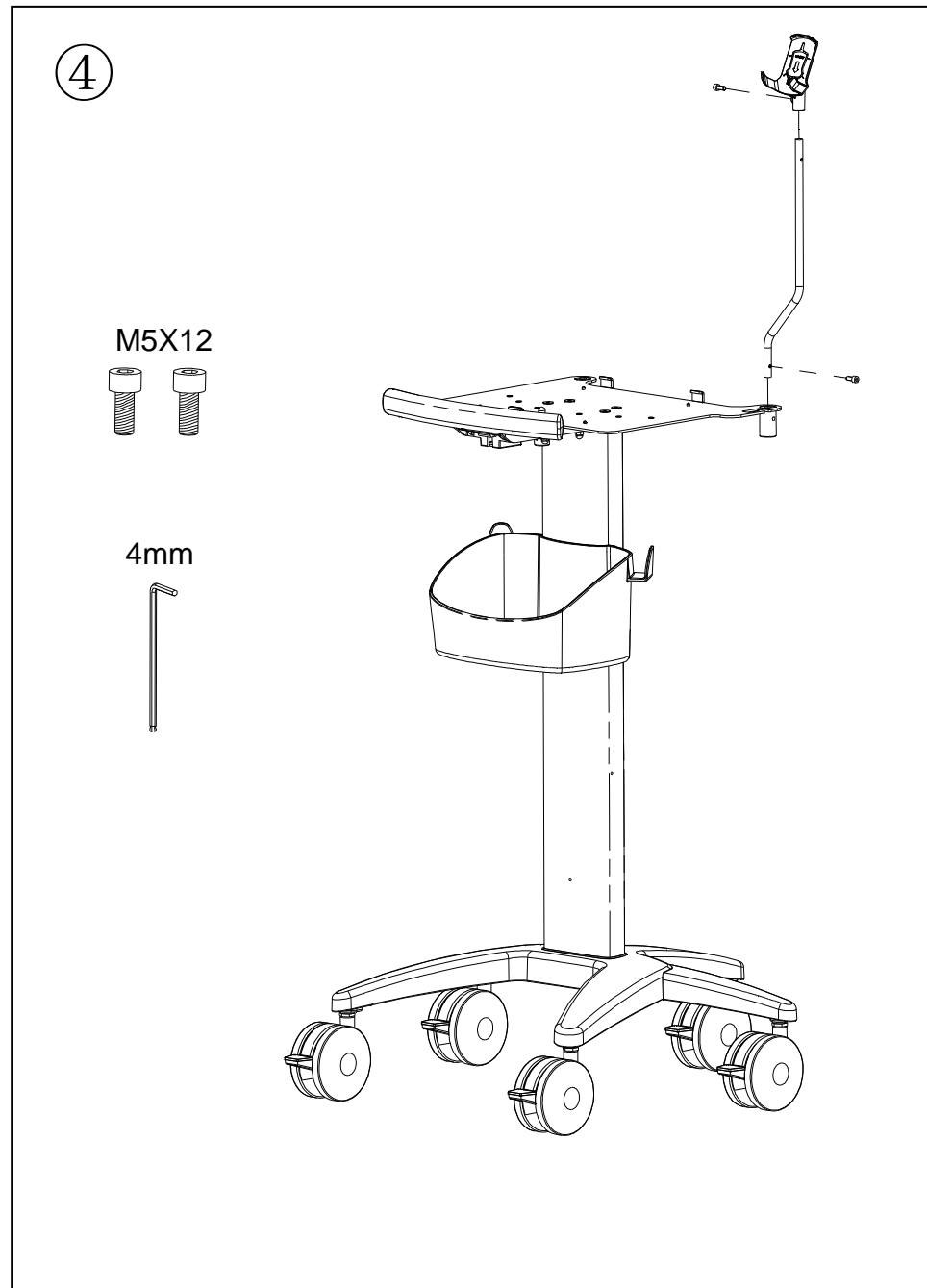
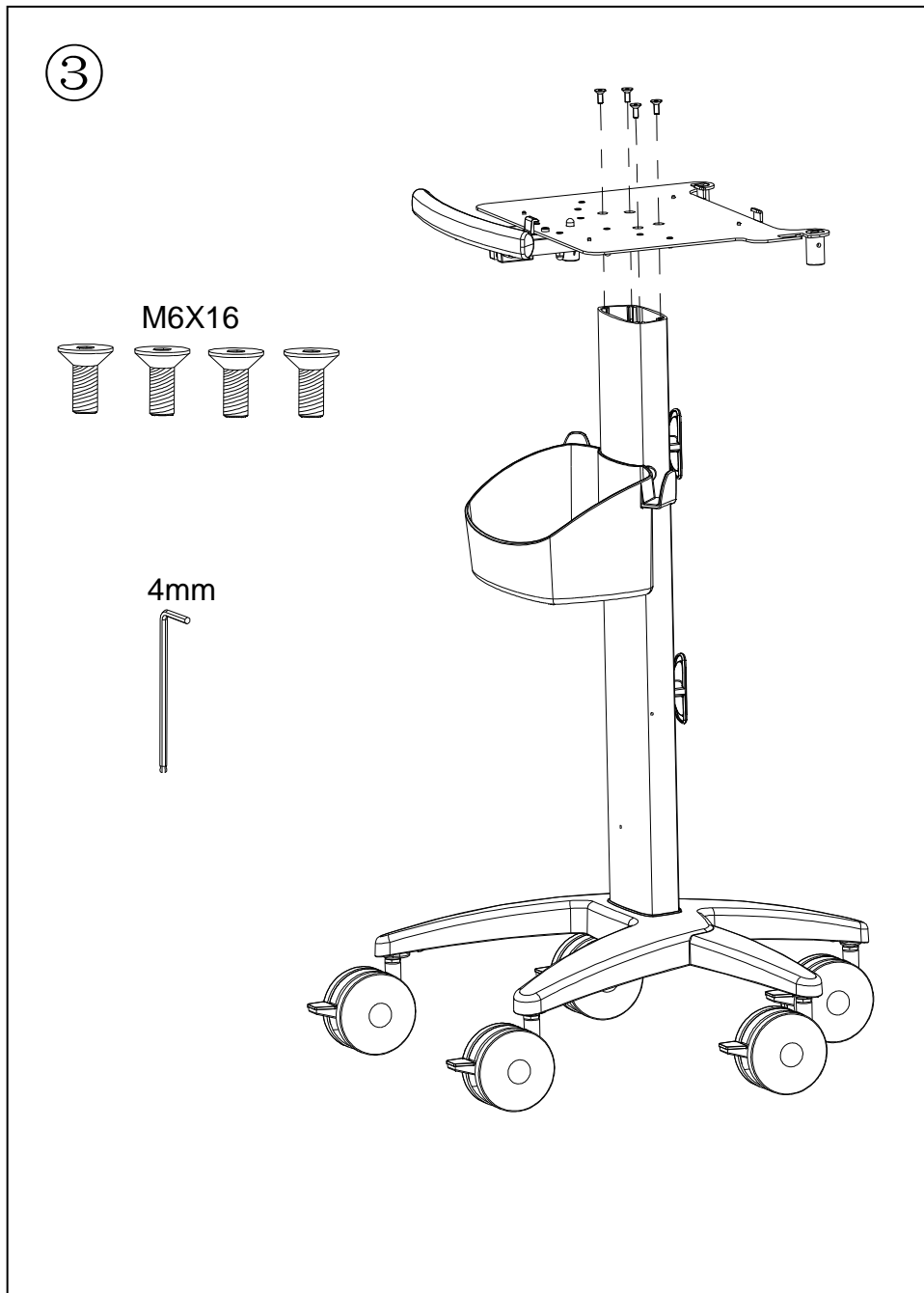
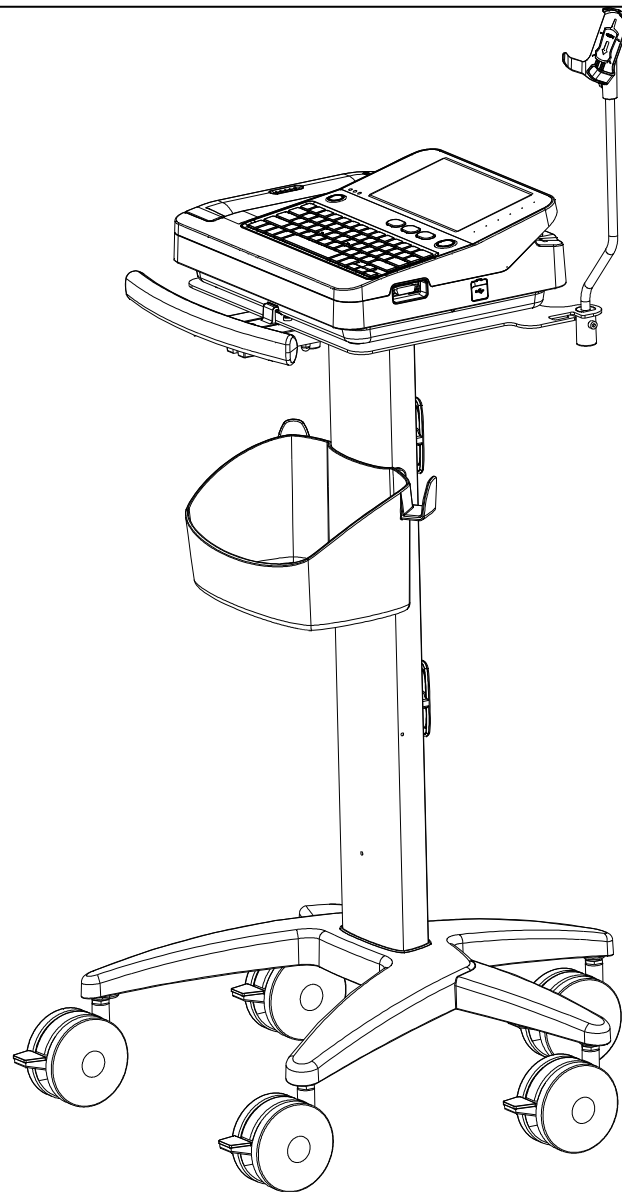
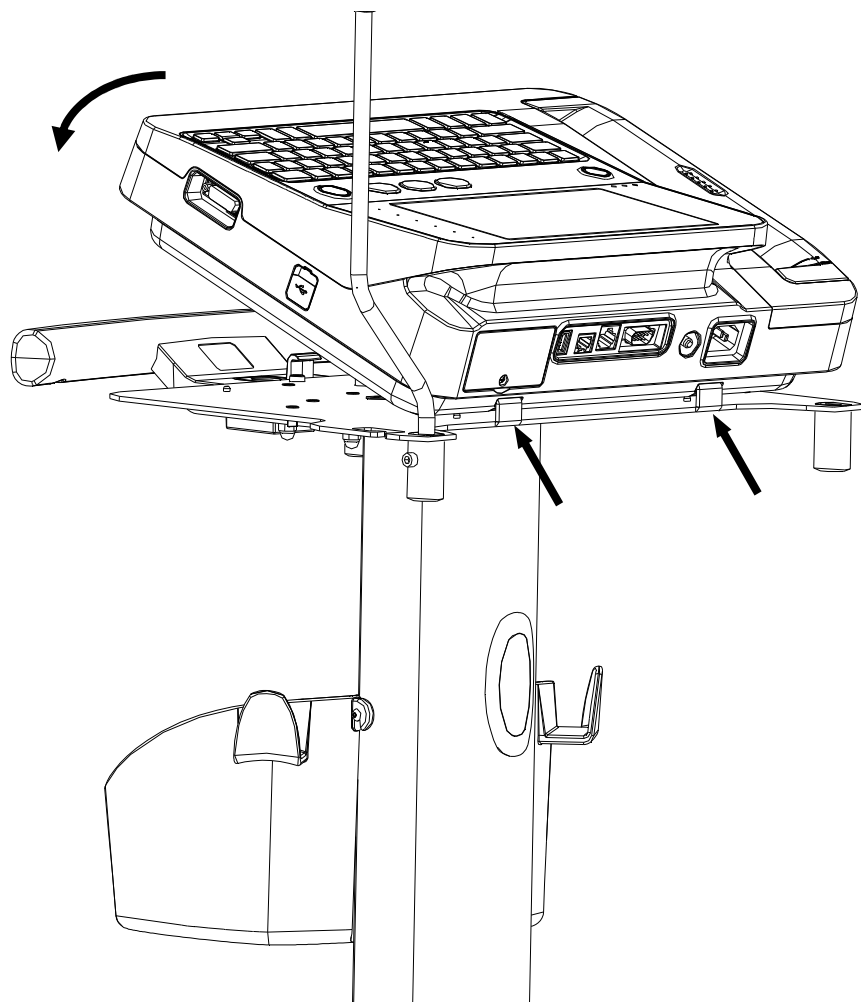
⑤  警告：请确保扣紧机器！
Warning: Make sure the machine have been fixed!

Cleaning\Disinfection 清洁消毒

Cleaning\ Disinfection:

1. clean the stand and accessories with cotton, sponge, or soft cloth moistened with water.
2. After cleaning, wipe off the water with a dry cloth.
3. The recommended disinfectants include:

| | |
|------------------------------------|---------------------|
| 70% isopropyl | Descosept AF |
| 70% Ethanol | Descosept forte |
| 3% Hydrogen peroxide | Mikrozid® AF liquid |
| 1%perform® classic concentrate OXY | Virkon. |
| Dismozon® plus | Terralin Liquid |

- Besides, the bin can also use the following disinfectants:
 Super Sani-cloth, Sani-cloth bleach and Sani-cloth AF3;
4. Clean the stand and accessories as instructed above.
 5. Disinfect the stand with cotton or soft cloth moistened with one of the recommended disinfectants.
 6. After disinfection, be sure to wipe off the disinfectant left on the stand with a soft cloth moistened with water.

CAUTION:

- Remove the patient monitor and its accessories before cleaning and disinfection.
- Do not use abrasive materials (such as steel wool and silvery polishing agent) or any strong solvents (such as acetone).
- Do not immerse or soak the stand or wet its surface.
- we recommend that the stand be disinfected only when necessary as determined by your hospital's policy, to avoid damage to the stand.

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