

Umbilical Cord Clamp

Code No.:

- Premium Quality (Code: 9002-2).
- Standard Quality (Code: 9002-1).

1- Description:

• The Cord Clamp is used to clamp the umbilical cord immediately after the birth of the child.

2- Advantages & Features:

- Suitable for clamping the umbilical cord after birth.
- Security locks to prevent accidental opening after clamping.
- Grooved clamping area to prevent slipping of umbilical cord.

3- Indications:

• Used to clamp the umbilical cord immediately after the birth of achild.

4- Contraindications:

• Can cause irritation to the skin.

5- The purpose:

- The Cord Clamp is used to clamp the umbilical cord immediately after the birth of the child.
- For single use only.

6- Instruction for Use:

- Clamp the umbilical cord of new born baby one inch from the distal end using the Umbilical Cord Clamp.
- Cut the end of umbilical from mother.
- Clean the body of infant & wrap and fix the umbilical cord with medical dressing.

Technical File

ISSUE/REV: - 1/0

Page 1 of 2

Umbilical Cord Clamp



ULTRA FOR MEDICAL PRODUCTS CO.

• The Clamp will be taken off after 24 hours by cutting off the clamp from the hole with a special clamp scissor or it can also fall down by itself, as it is dry.

7- Care instructions:

- The Umbilical Cord Clamp is not reopened and kept not less than twenty four hours.
- Do not use if package is open or damaged.
- Discard after single use.

8- Storage conditions:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Suitable temperature.

9- Product code with size and measures:

- Vaso Clamp with Premium Quality (Code: 9002-2).
- Cord Clamp with Standard Quality (Code: 9002-1).

10- List of product components:

• Umbilical Cord Clamp.

11- Sterility:

• Sterilized by Ethylene oxide Gas (20% ETO +80 % Carbone dioxide).

12-Shelf life: 5 Years



Assiut Factory: Part No. 304, 305, 306, 307, 308, 309, 310, 312, Arab El Awamer- industrial zone, Abnoub, Assiut, Egypt.. **Tel**: 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

& Fax:002-088-4964222**& Mob:** 002-01001558853, 002-01068832355

Cairo Head Office: 64, Nakhla El Motaiy Triumph Square Heliopolis **Tel:** 022/4171621-4143794 **& Fax**: 022/4171613 **& Mob**: 01223988200

Assuit Office: 23, July Str. Tel: 088/2364111 - 2364222 & Fax: 088/2334964 & Mob.: 01223988202

Alexandria Office: 212Abd El salaam Airef Str, Luran Tel: 03/5856202 – 5856458 & Fax: 03/5828988& Mob: 01223948666

E-mail: info@ultramedumic.com , shady@elaggargroup.com Website: www.ultramedumic.com

Technical File

ISSUE/REV: - 1/0

Page 2 of 2

Umbilical Cord Clamp