

# DETROSAN SFC CONCENTRATE SURFACE DISINFECTANT USER MANUAL

## DETROSAN SFC

---

- SURFACE DISINFECTANT – QAC AND AMINE BASED
- CONCENTRATE

### Usage Areas

- It is used for cleaning and disinfection of surfaces existing at hospitals and health foundations, like surgery rooms, clean rooms, intensive care units, patient rooms, new born units etc.

### Usage form

- Detrosan SFC is diluted at %0,25 concentration (2,5 ml product into 1 L water)
- Floor or medical device surfaces are cleaned with moistened wipe or mop with Detrosan SFC.
- Wait for 15 min. effect time.
- Life of prepared solution is 24 hours. In the case of intensive contamination, it is recommended to change solution.

### Ingredient

- Didecylidimethyl ammonium chloride, Benzalkonium chloride, Bis(3aminopropyl) dodecylamine

### Microbiologic effect

- Bactericidal (incl.Tbc), fungicidal, virucidal effect in 15 minutes

### First aid informations

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.

### Storage Conditions

Shelf life is 2 years after production date. Keep away from direct sunlight. Store containers tightly closed and in a well ventilated area at 0-30 °C

### Package

5 L