


2.1 System Appearance

2.1.1 R-300

■ Front / Right Side

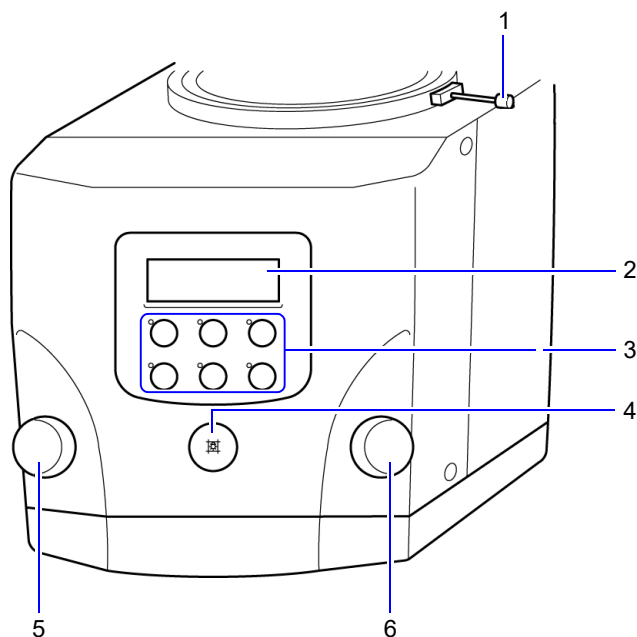

* : Equipment suitable for use in the patient environment.

No.	Name	Function	See
1	Stopper	Used to secure the collimator.	P.28
2	Display panel	Displays the irradiation field, SID and so on.	P.19
3	Operation panel	Used to change the SID indication, to automatically select the X-ray filter and to perform other operations.	P.14
4	Lamp button	On pressing this button, the collimator lamp and line marker (option) are turned on for the set amount of time. On pressing it again, the collimator lamp and line marker are turned off.	P.25, P.39
5	Irradiation field adjustment knob (for H-leaves)	Used to adjust the irradiation field size.	P.26
6	Irradiation field adjustment knob (for V-leaves)		