


# LIMB HOLDER

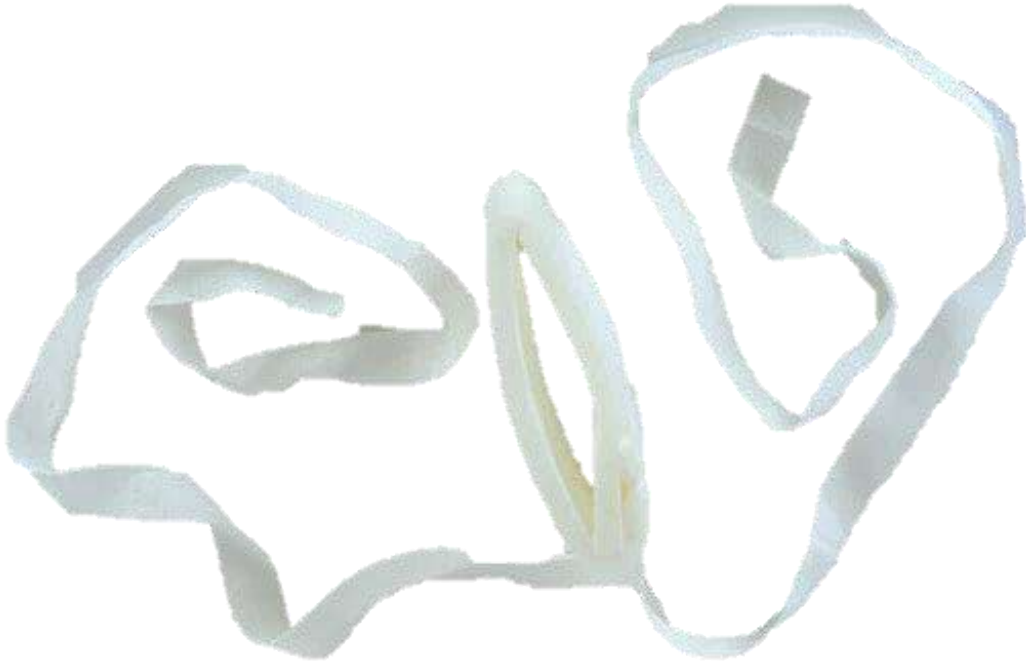

## Limb Holder

Limb holders are used to secure the patient's hands and feet to the bed to prevent hospitalised and operated patients from unknowingly injuring themselves or their surroundings.

### Available Sizes

|                           |          |
|---------------------------|----------|
| Limb Holder (Adult/Hand)  | DLM-1001 |
| Limb Holder (Adult/Ankle) | DLM-1002 |
| Limb Holder (Pediatric)   | DLM-1003 |
| Limb Holder (Children)    | DLM-1004 |

### Features

- Ideal for emergency room use.
- It has a secure fastening system.
- It does not irritate the skin.


### Intended Use

Limb Holders are used to prevent the patient from moving during or after a surgical operation and adversely affecting the operation or treatment process, and to fix the patient's hands and feet to where she/he lies. Limb Holders are disposable devices.

### Instructions for Use

1. Place the limb holder(hand/ankle) over the patient's hand/ankle.
2. Bring the two ends together to secure the limb holder on the patient using the hook.
3. Pass the belt toward where the patient lies and secure it.

### Technical Data

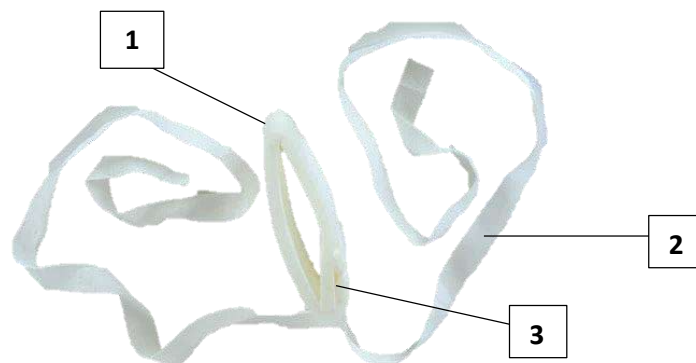

|   | Part                    | Qty | Material |
|---|-------------------------|-----|----------|
| 1 | Velcro fastening fabric | 1   | Textile  |
| 2 | Belt                    | 1   | Textile  |
| 3 | Hook                    | 1   | Textile  |

### Manufacturing Methods

- Cutting
- Sewing (Assembly)
- Packaging - Boxing

## Specifications

- Manufactured under ISO 13485 Quality Management System standard.
- Latex free.
- Phthalate free.
- Medical device for single use: Do not reuse.

## User Information

### Storage:

Storage condition only valid for storing in original packaging. Do not use the product if the original packaging is damaged.

Storage conditions: Keep away from direct sunlight  
Keep dry

### Packaging:

Packaging Unit: Packaging with single pouches  
Box Qty: 500 pcs  
Inner Bag Qty: 50 pcs (hand) / 100 pcs (ankle)

### Regulatory Compliance:



#### Dilamed Tıbbi Ürünler A.Ş.

Yazıbaşı Mah. Fatma Tonkuş Cad. No:24 Torbalı/İzmir

+90 232 281 11 25

[info@dilamedtibbi.com](mailto:info@dilamedtibbi.com) [www.dilamedtibbi.com](http://www.dilamedtibbi.com)

[sales@dilamedtibbi.com](mailto:sales@dilamedtibbi.com)

