UK

Lymphostabil

For stabilization of lymphocytes (free of Ca and Mg)

Package Size

REF 824 020 Lymphostabil 100 ml

IVD Diagnostic reagent for in vitro use only

Product Description

Lymphostabil is a ready-to-use modified McCoy5A Medium (free of Ca and Mg). It is a tissue culture medium used for suspension and storage of lymphocytes.

Stability and Storage

Unopened, Lymphostabil can be stored at -20°C up until the expiry date indicated on the label. Shipment of liquid Lymphostabil (> -20°C) does not cause product deterioration.

In order to maintain sterility and to avoid repeated freeze-thaw cycles, after opening the container, Lymphostabil should be divided into appropriate size aliquots and stored at -20°C or colder. Thaw the aliquot at 2...8°C before use.

If crystals should appear, warm at 37°C until dissolved.

Indications of deteriorations

Lymphostabil is coloured red. A yellow colouration of Lymphostabil which still remains after thawing may indicate a change of the pH value. Yellow coloured Lymphostabil should **not** be used.

Application

In order to obtain the most viable and pure lymphocyte suspension possible, lymphocytes should be isolated according to the procedure described for the lymphocytotoxicity test (e.g. Bio-Rad Lymphotype HLA).

The isolated lymphocytes should be suspended in Lymphostabil to a concentration of 2000 – 3000 cells per microliter.

The viability of lymphocytes suspended in Lymphostabil is 4 – 6 days when stored at 2...8°C.

Literature

1. McCoy TA, Maxwell M, Kruse PF Jr. (1959): Amino acid requirements of the Novikoff hepatoma in vitro. Proc Soc Exp Biol Med. 100(1): 115-8.
2. Hsu TC, Kellogg DS Jr. (1960): Primary cultivation and continuous propagation in vitro of tissues from small biopsy specimens. J Natl Cancer Inst. 1960 Aug; 25: 221-35.
3. Iwakata S & Grace J T, Jr. (1964): Cultivation in vitro of myeloblasts from human leukemia. NY State J. Med. 64: 2279-82

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Literatur

4. McCoy TA, Maxwell M, Kruse PF Jr. (1959): Amino acid requirements of the Novikoff hepatoma in vitro. Proc Soc Exp Biol Med. 100(1): 115-8.
5. Hsu TC, Kellogg DS Jr. (1960): Primary cultivation and continuous propagation in vitro of tissues from small biopsy specimens. J Natl Cancer Inst. 1960 Aug; 25: 221-35.
6. Iwakata S & Grace J T, Jr. (1964): Cultivation in vitro of myeloblasts from human leukemia. NY State J. Med. 64: 2279-82