


## 8.15 Sending Images

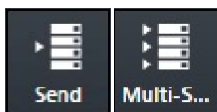

- You can select study or image and send it to the DICOM storage server.
- **Send**: Send images to the server configured in **SCP List**.
- **Multi Send**: Send images to the server configured in **Protocol List**.



- Refer to Storage Menu in VXvue Service Manual for the information about **SCP List** and **Protocol List**.

### Send

- Select the study or image to be sent.
  - You can select multiple studies or images with the **Ctrl** or **Shift** key.
  - Check the connection state of the, **DICOM Storage** server before sending images.
- Click on the **Send** button or **Multi Send** button.
  - Select **Protocol** from the **Storage Protocol** box if you click on the **Multi Send** button.



- Click on the **All Image, Selected** or **Cancel** button from the **Send** box.

### Check the sending status

- Check the image sending status from the status bar.
- Check the image sending status at the **Queue** tab.



- Refer to Queue Menu in VXvue Service Manual for the detailed information about the **Queue** tab.

### Use automatic sending / closing functions

- If the study is closed, it is sent to the DICOM storage server automatically according to the setting of auto sending function.
- If all images of the study are sent to the DICOM storage server, the study is closed automatically according to the setting of auto close function.
  - This function is activated even though the study indicated on the screen currently is different from the study which has been sent to the DICOM storage server.
- The auto sending function and the auto closing function cannot be used at the same time.