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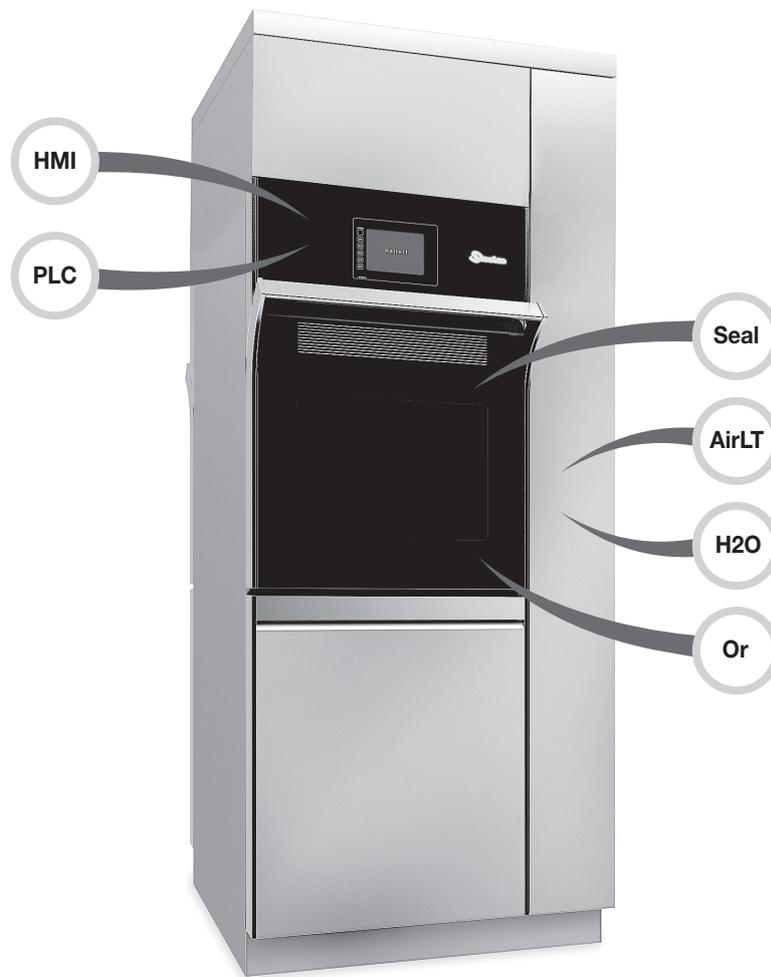
---

### **ATTENTION**

For further information, please refer to the USER MANUAL

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### **ATTENTION**

The manufacturer declines all responsibility for personal injury or material damage resulting from the non-observance of SAFETY PRECAUTIONS.

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### **ASSISTENCE**

Should your machine not work properly even after maintenance has been carried out, contact the Technical Support Centre of reference, describing the fault and giving the machine model and serial numbers.

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---

### WARNING – PERSONAL INJURY

Allow maintenance to be performed only by trained and authorized personnel, equipped with appropriate clothing and Personal Protective Equipment (PPE).

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Disconnect the electrical supply to the machine before undertaking maintenance. Make sure the chamber is empty and neither basket nor instruments are present

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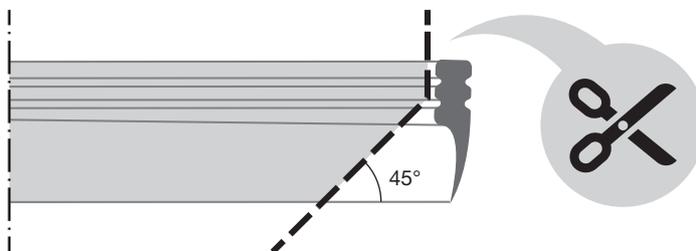
Remove the old gaskets. Clean carefully the gaskets housings

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
Leaving a slight leading edge at the top of the gasket, cut slightly down and then 45° as shown in the picture

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
### 2a

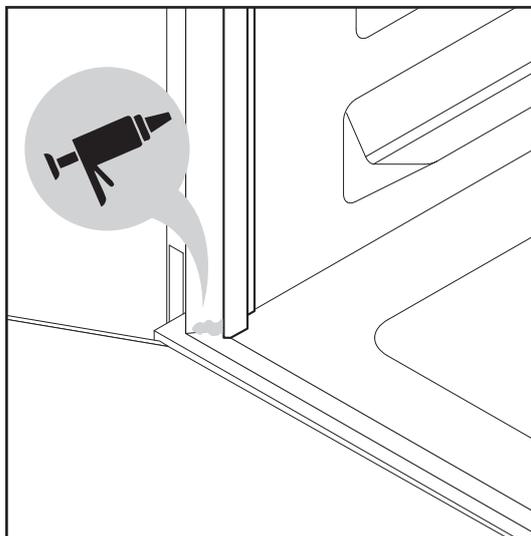
Put a small amount of silicone in the corner of the door on the inside of the gasket seat

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

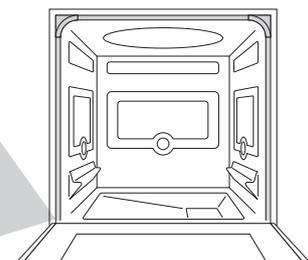
3a

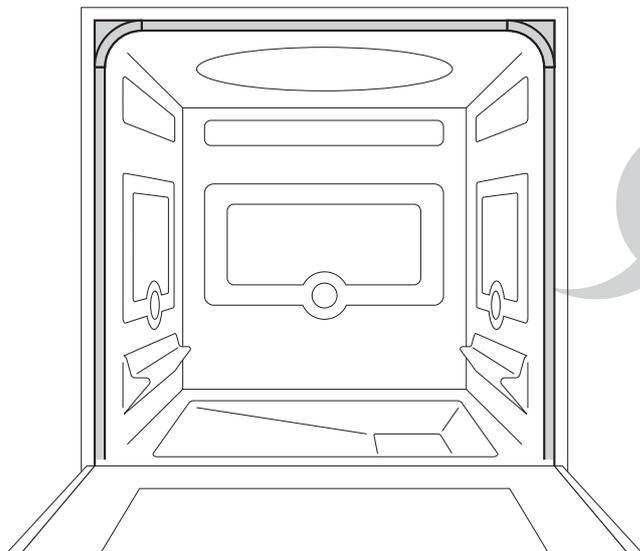Wet a little the gasket housing with a soap and water solution to help the gasket insertion

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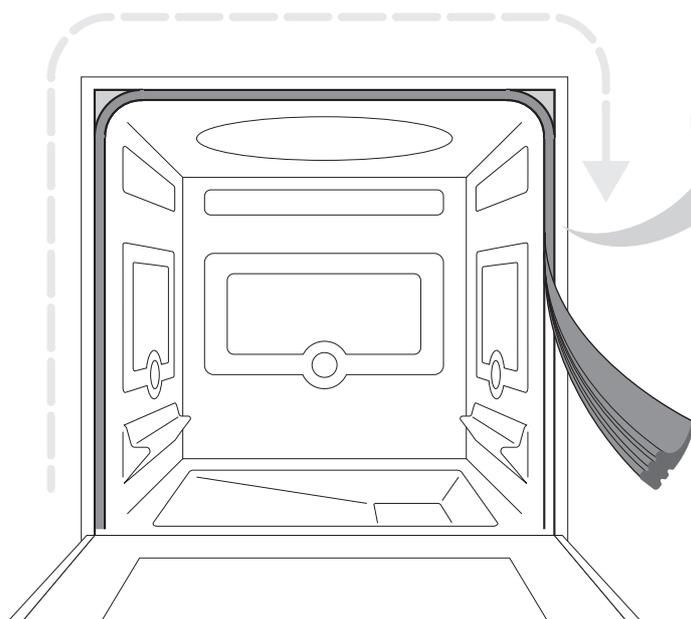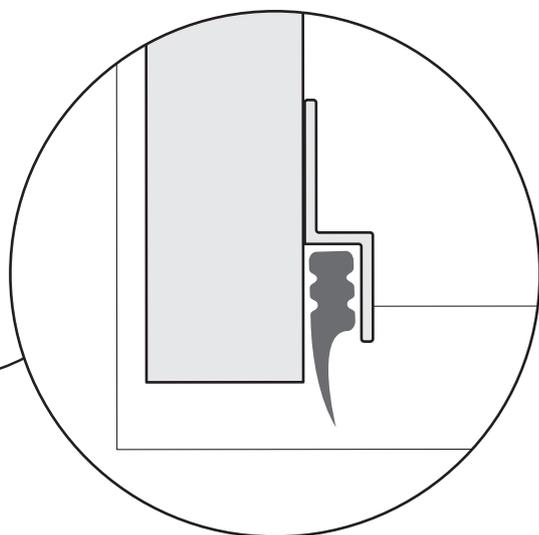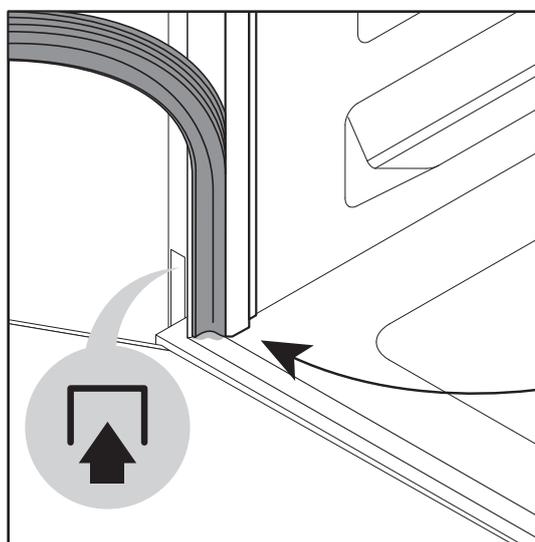
4a

Insert the perimeter gasket in its housing starting from the point where silicone was applied

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

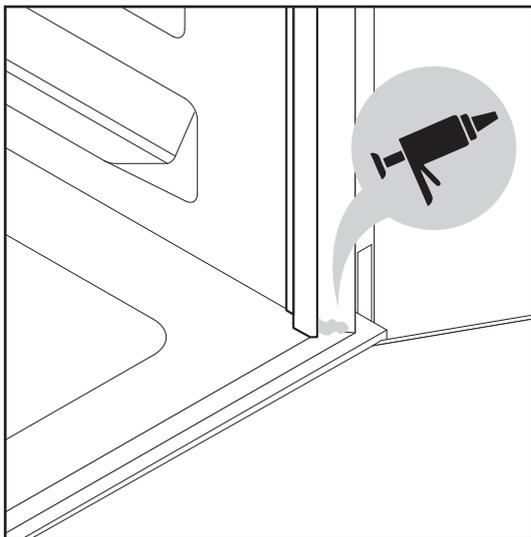
5a

Put a small amount of silicone in the opposite corner of the door on the inside of the gasket seat

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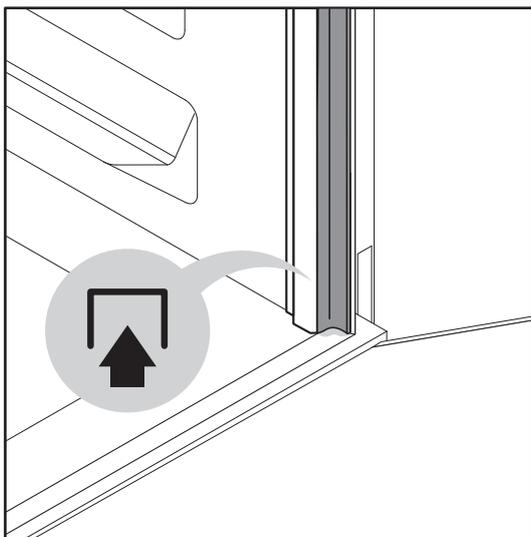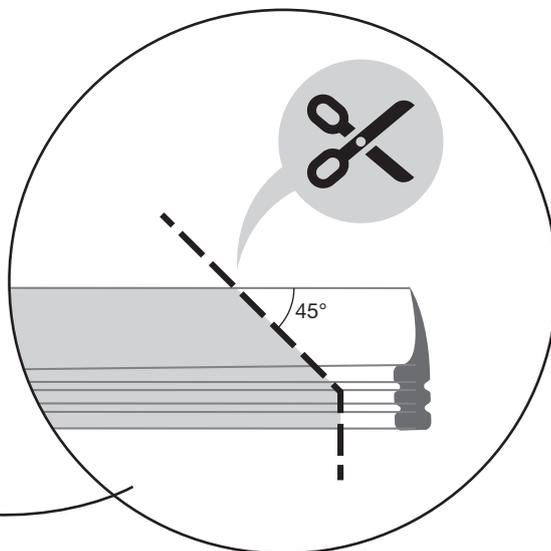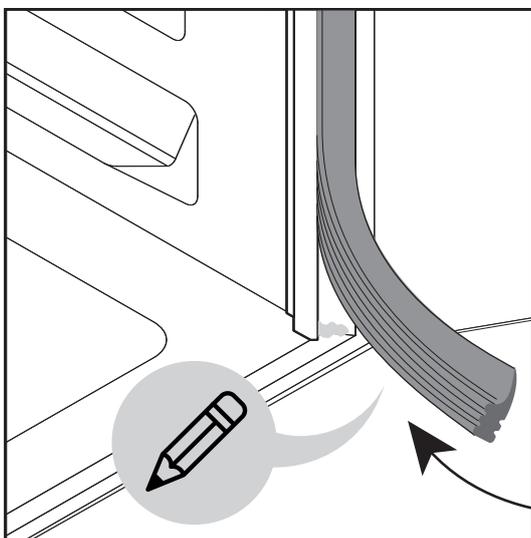6a




Cut as shown previously in figure 1A the exceeding gasket. Insert the remaining part of the gasket in the housing

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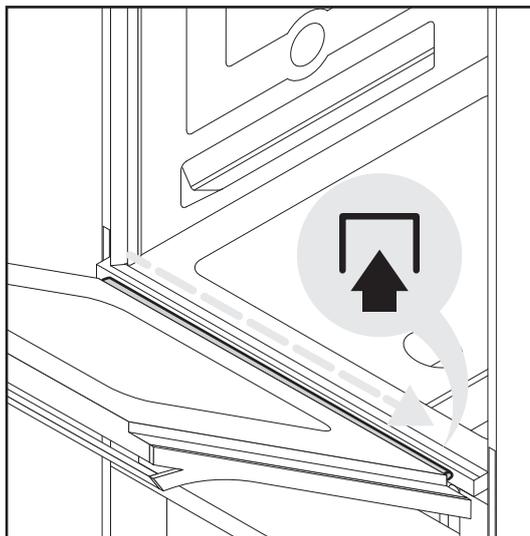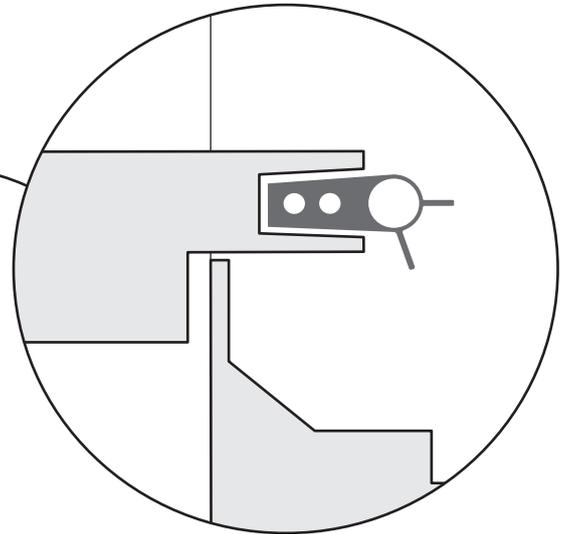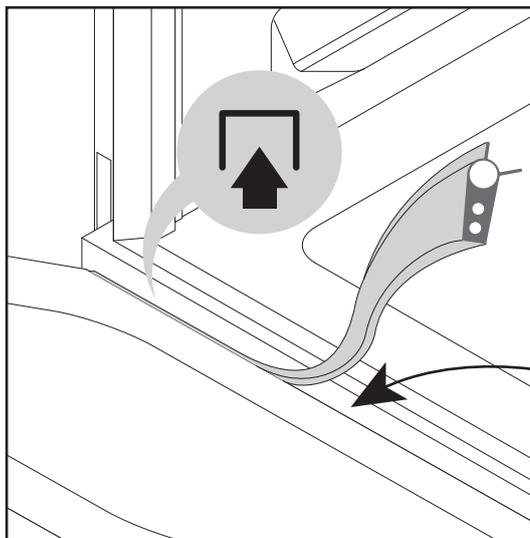Insert the base gasket on the bottom of the door

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### INSTALLATION COMPLETED

Repeat installation on the opposite side for double door machines

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Steelco is not responsible for incorrect installation or damage due to installation by unauthorized or untrained personnel

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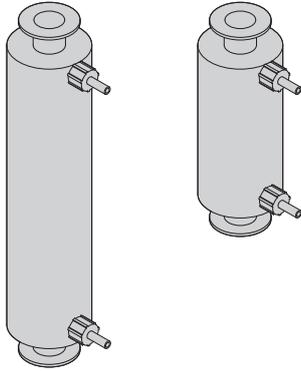
# Water and channel filters replacement

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Water filter, channel filter  
*Filtro acqua, filtro canale*

Disconnect the electrical supply to the machine before undertaking maintenance and make sure that the water feeds are all closed

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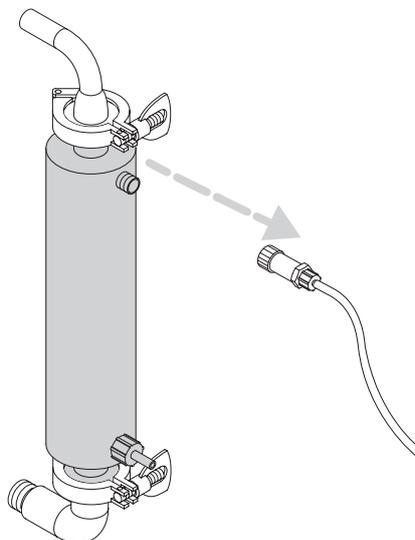
Unscrew and remove the cover of the machine to get access to the filters

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
1

If present, disconnect the e breather tube

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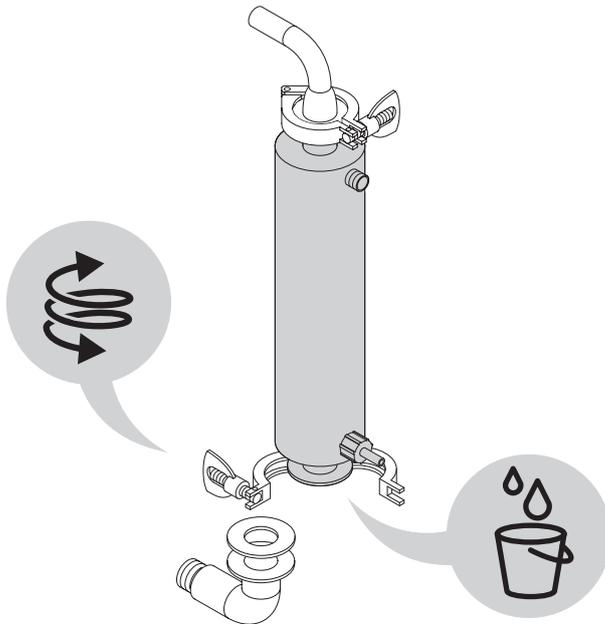
2

Place a bucket under the filter to capture any water during filter replacement. Unscrew and remove the bottom clip, seal and filter from the circuit

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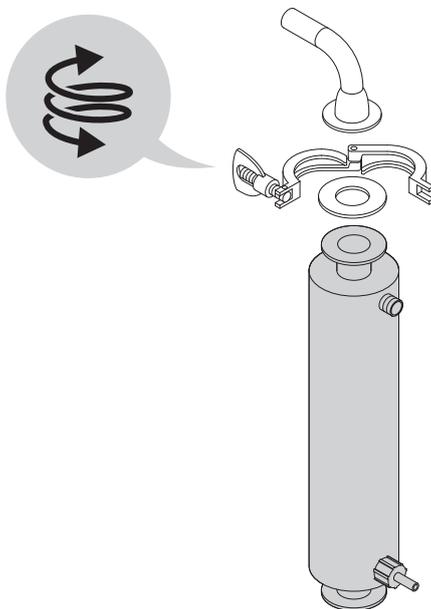
3

Unscrew and remove the top clip, seal and filter from the circuit.

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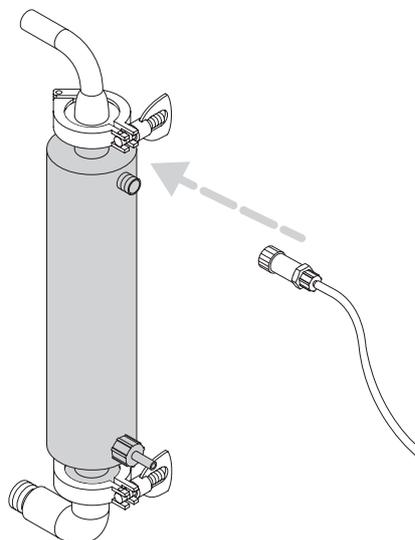4

Insert the upper and lower seals and connecting tubing followed by the retaining clips and tighten. If present reinsert the breather tube

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#### INSTALLATION

Repeat installation for every water and channel filter of the machine

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Disconnect the electrical supply to the machine before undertaking maintenance

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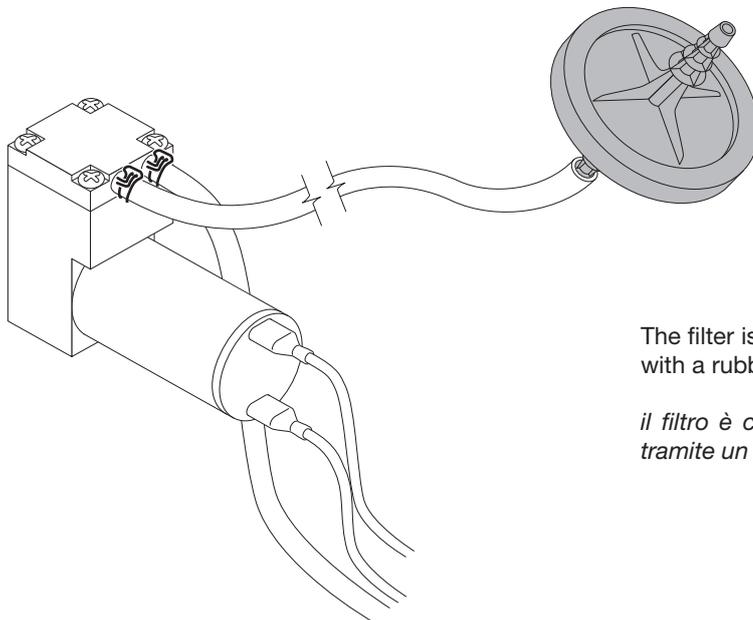
Unscrew screws and remove the panel to have access to the membrane pump and leak air test filter

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

1

The filter is attached to the membrane pump with a rubber tube

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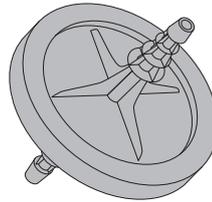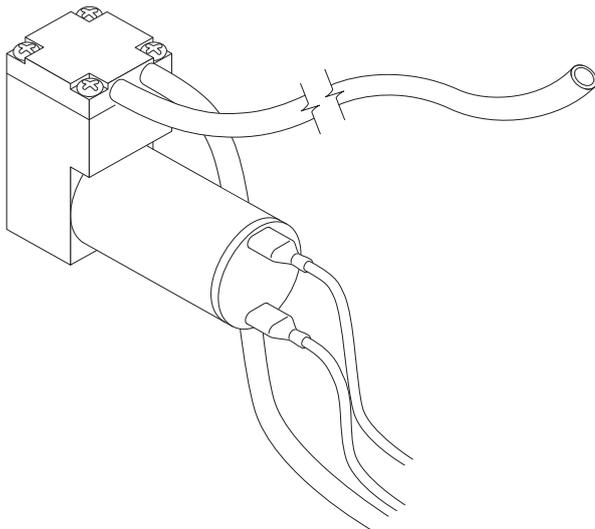2

Remove the filter and replace with one of the same type

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CUTTING  
TAGLIO

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# Programmable Logic Controller (PLC) battery replacement

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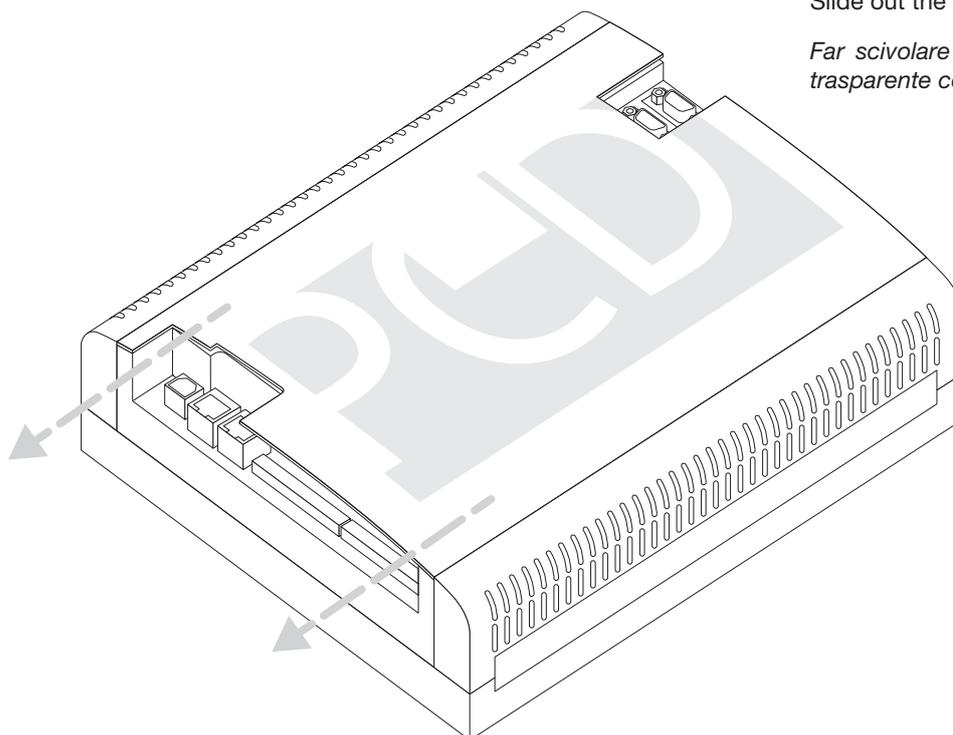
Unscrew and remove the panel getting access to the PLC (Programmable Logic Controller)

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1


Slide out the transparent protective cover

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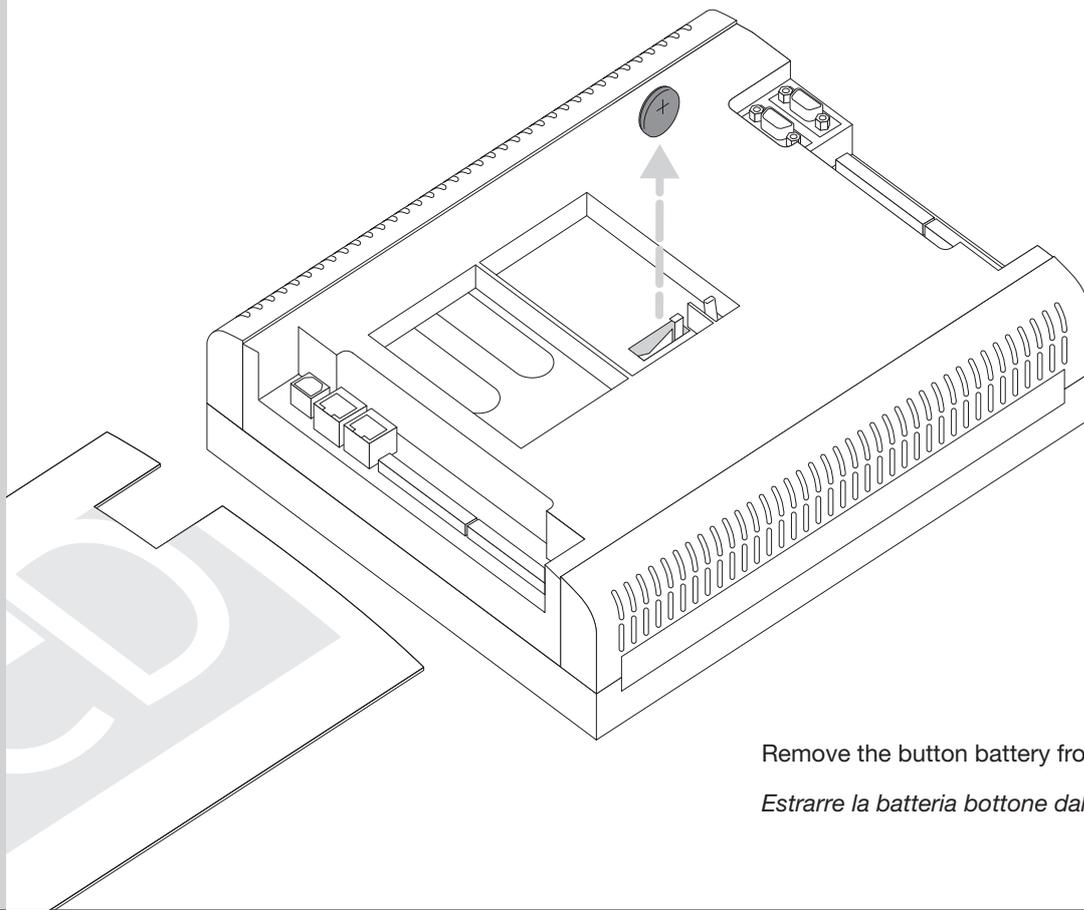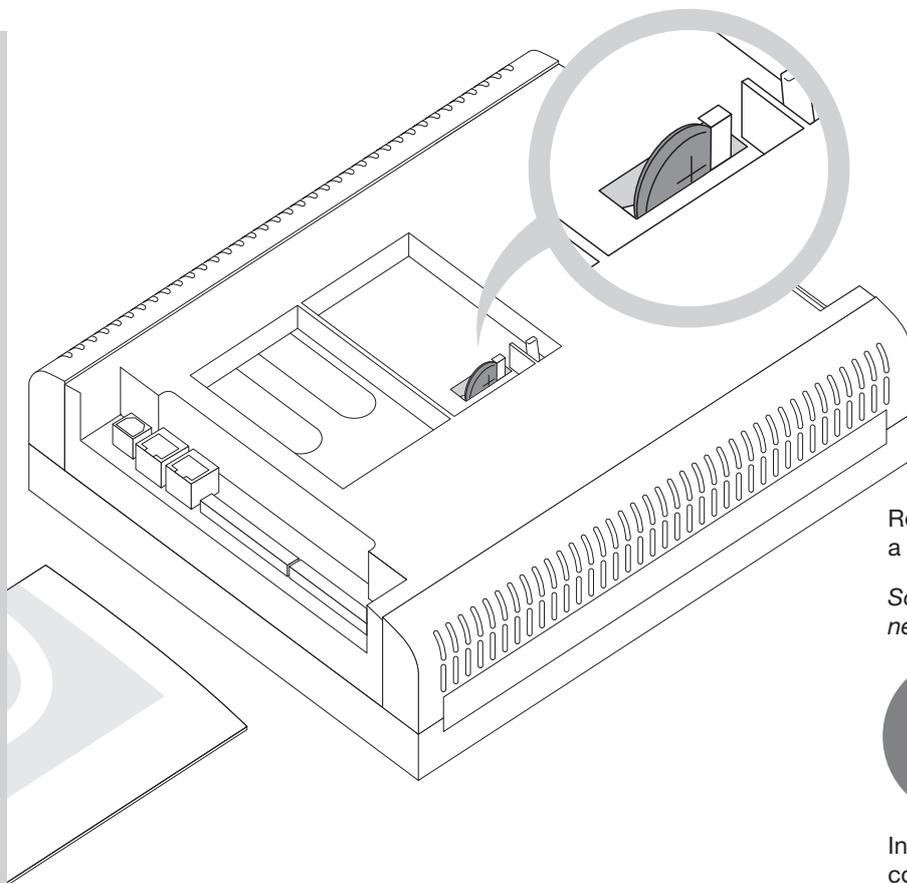
3

Replace the old button battery with  
a new one of the same type

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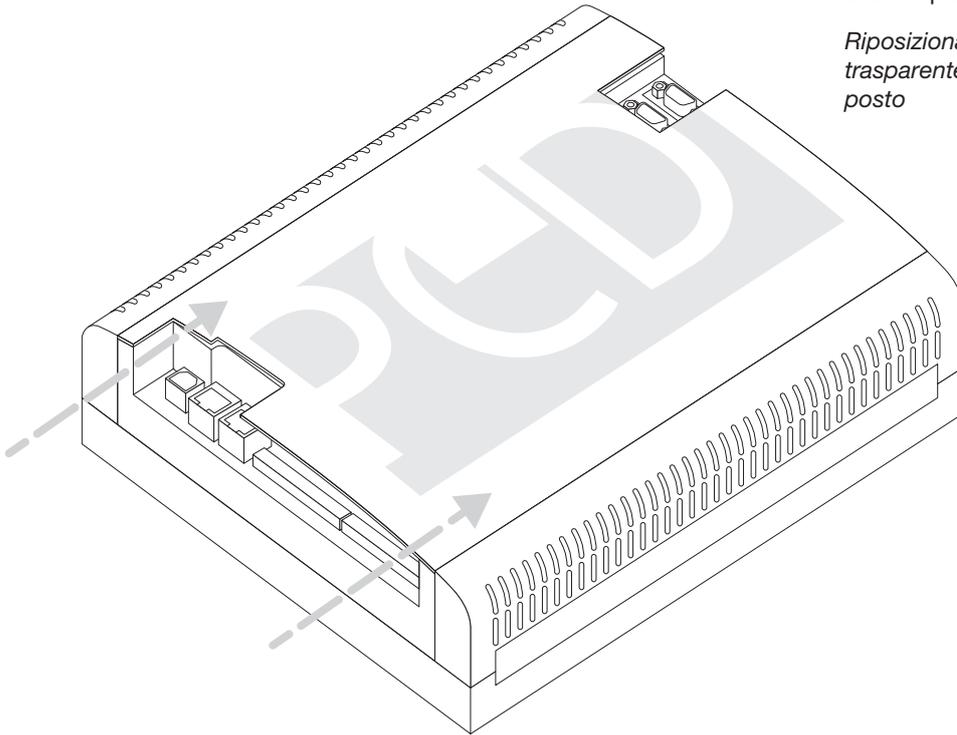
Inserting the new battery, observe  
correct polarity

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4

Slide the transparent protective cover back in place

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CUTTING  
TAGLIO

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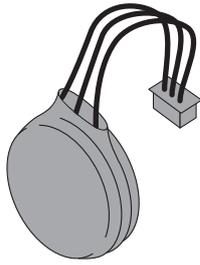
# Human machine interface (HMI) battery replacement

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Perform memory back-up: all data will be lost during battery replacement operations

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Switch the main disconnecting device of the machine on "off" position

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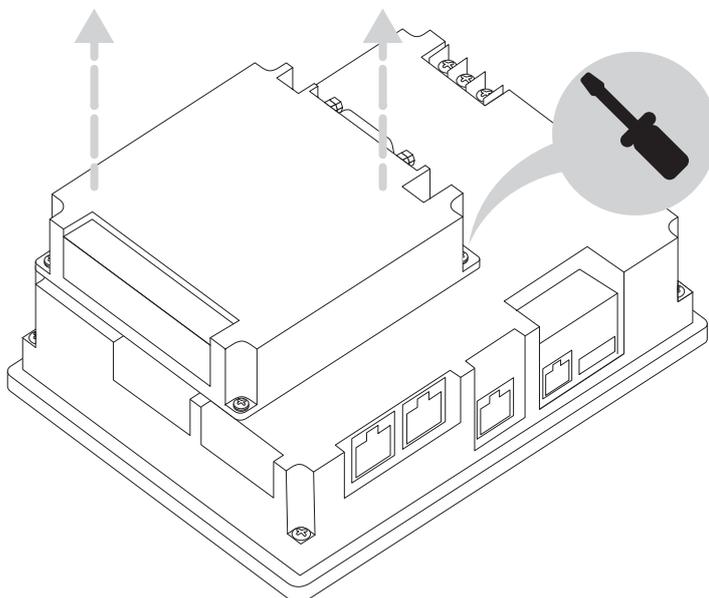
Unscrew and remove the panel to get access to the HMI (Human Machine Interface)

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
1

Unscrew the 4 screws and disconnect the expansion module

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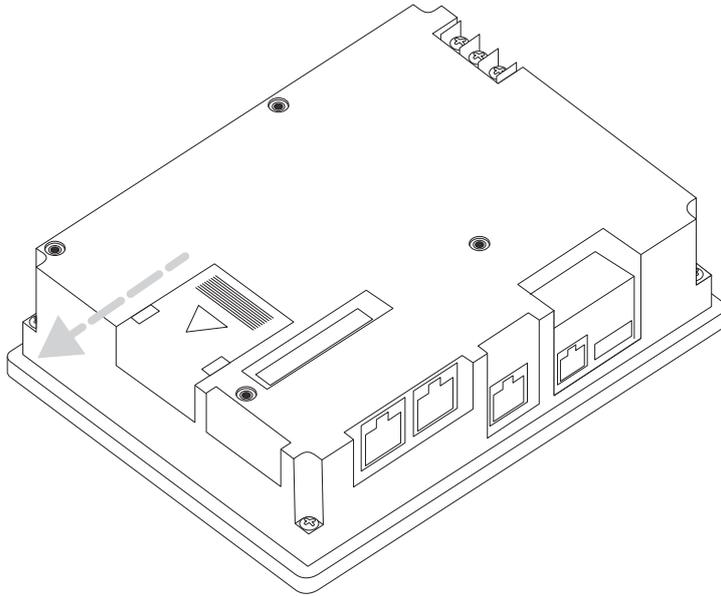
2

Slide out and remove the protection cover of the battery compartment

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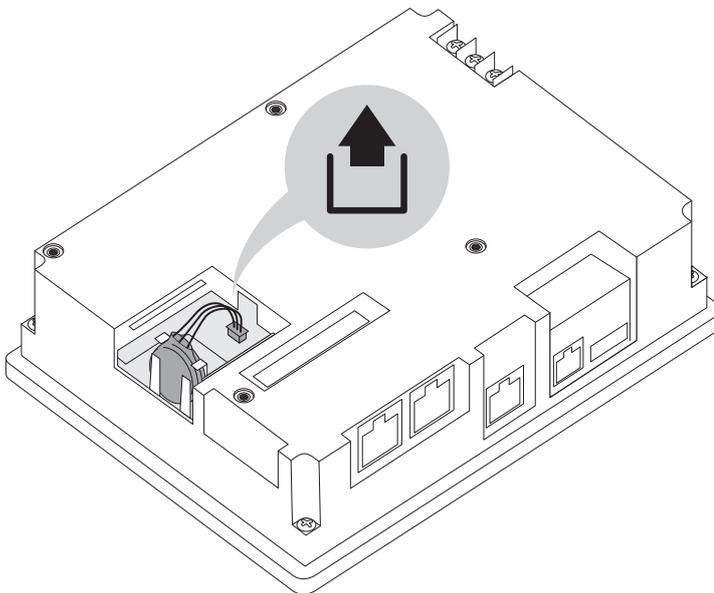
3

To disconnect the battery connector, gently pull it out

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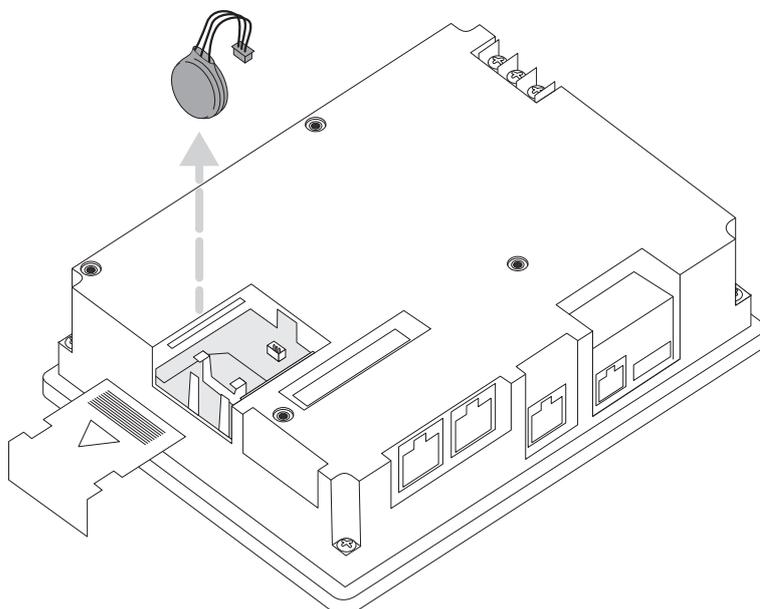
3

Remove the old battery and replace it with another of the same type

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# 3b

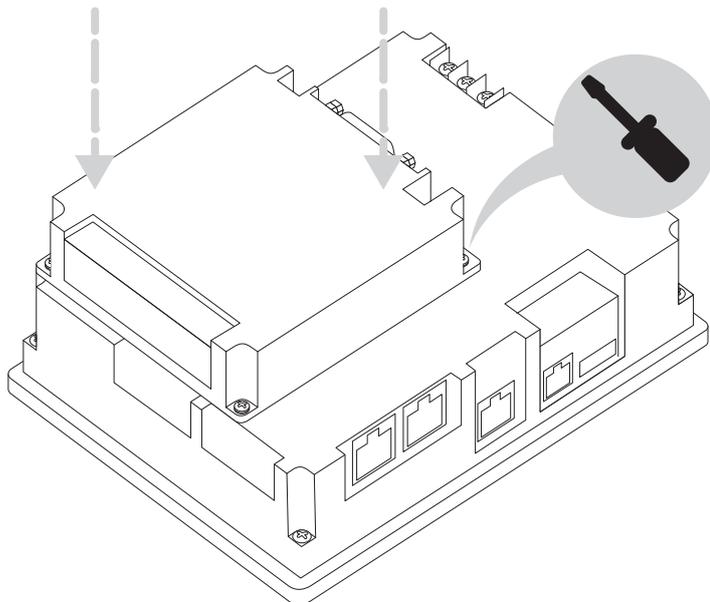Insert and connect the new battery to the device. Close the battery compartment. Reconnect and fasten with the screws the expansion module. Finally re-establish the ground cable.

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**INSTALLATION COMPLETED**  
Repeat for each HMI unit of the device

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
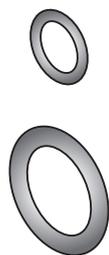
# Replacement of the channels connection O-rings

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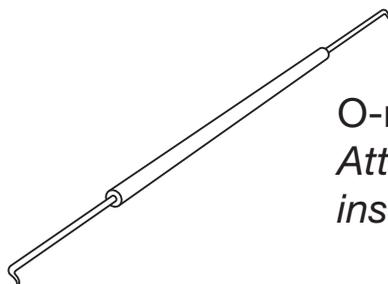1

Open the device door. Pull out and remove the wash-cart

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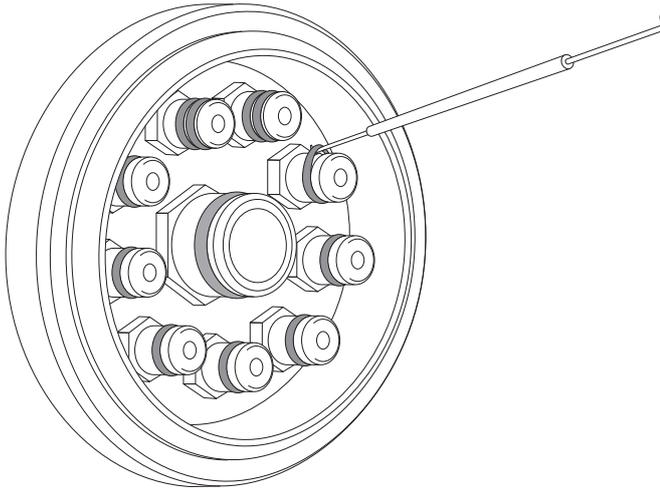
2

The channels connctons to the wash cart are inside the chamber on the left side (loading door)

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
3

Use the O-ring removal/insertion tool to remove the O-rings from their housing. The leak-test channels are provided with two O-rings: remove them both

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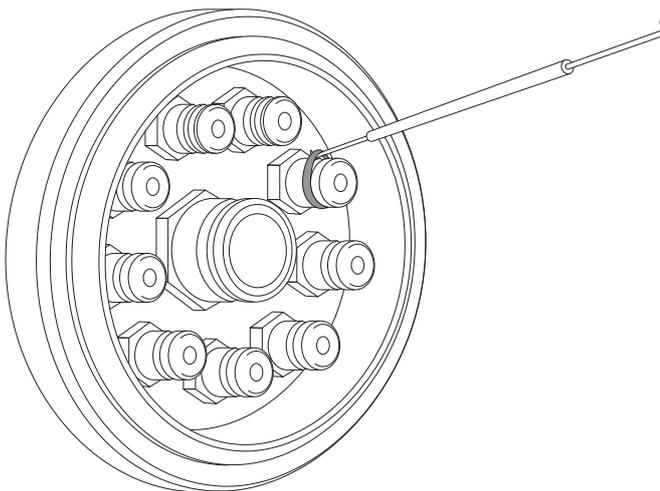3

Use the O-ring removal/insertion tool to insert the replacement O-rings in their housing. The leak-test channel requires two O-rings. The O-ring of the central channel is bigger than the other ones; replace it with a new one of the proper size

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