


4.1.2 Thumbnail View

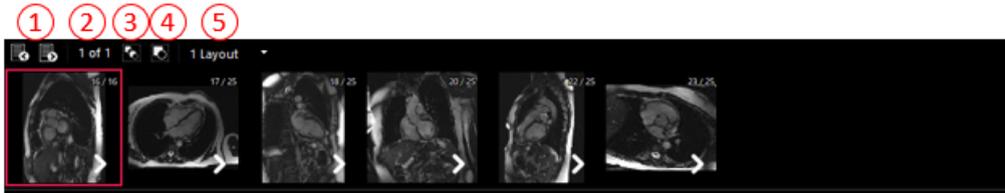


No.	Name	Function
1	Previous page / Next page	Displays the previous or next thumbnail image based on the current thumbnail image.
2	Page Count	Displays the page of the currently displayed thumbnail image
3	Show All Images	Displays all images.
4	Show First Image	Groups images in series unit and shows the first image.
5	Thumbnail Layout	Adjusts the layout of thumbnail view.



- When displaying thumbnails in the first image view, the number of the series number / image is displayed in the upper right corner.
 - Displays view type and number of images on Mammography mode.
- Click  displayed on the image to expand the bound images.
- Click  displayed on the image to group the images.

4.1.3 Simple Print View

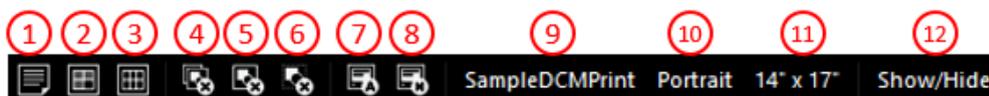


No.	Name	Function
1	New Sheet	Creates a new sheet.
2	Sheet Layout	Set the layout for the created sheet in the Output window.
3	Print Layout	Sets the layout of the output window for the selected sheet.
4	Clear All Sheets	Deletes all images in the output window and moves to the viewer window.
5	Clear Current Sheet	Deletes the selected sheet in the Output window.
6	Clear Selected Image(s)	Deletes the selected image in the output window.
7	Print All Sheets	The images of all the sheets in the output window are transferred to the DICOM printer. DICOM output can be stopped during output.
8	Print Current Sheet	The image of the currently selected sheet is sent to the DICOM printer.
9	Print SCP	Displays the currently set print SCP.

10	Film Orientation	Sets the orientation of the output paper. (Portrait / Landscape)
11	Film Size	Set the film size.
12	Show / Hide option	Sets the printing information. (Image Information / Annotation / Scale bar)



- Using the icon  displayed on the image, you can drag the image of the study viewer and add it to the Simple Print view.
- The settings for Print SCP can be set in the preset window.

Simple Print View Layout

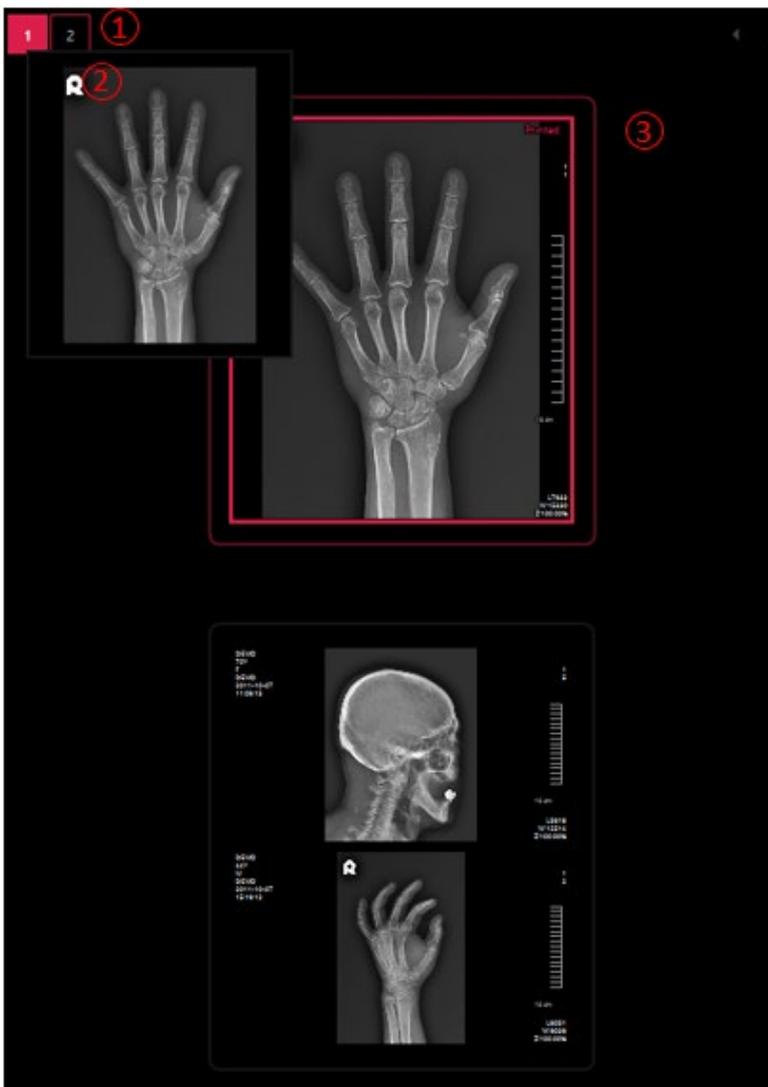

No.	Name	Function
1	Sheet Number	Displays the number of the created sheet.
2	Sheet Preview	If you put the mouse on the sheet number, you can preview the sheet.
3	Sheet List	Displays a list of generated sheets.