

## 4.11 Setting X-ray Conditions for Fluoroscopy and Radiography

### 4.11.1 Setting Conditions for Fluoroscopy

The conditions for fluoroscopy is displayed on the touch panel.

#### 4.11.1.1 Selecting Fluoroscopy Mode


This unit allows multiple fluoroscopy modes to be registered during installation and adjustment. Multiple fluoroscopy modes are useful in the following cases:

- To change the correlation between fluoroscopy tube voltage and current for use in abdominal or orthopedic examinations.
- To reduce irradiation dose as practically low as possible for infant patients.

A fluoroscopy mode has four settings below.

- Fluoroscopy tube current that interlocks with fluoroscopy tube voltage
- Upper limit for fluoroscopy tube voltage
- Selecting Pulsed Fluoroscopy/Continuous Fluoroscopy
- Limitation of maximum rate for pulsed fluoroscopy

1 Press the upper  icon on the touch panel.

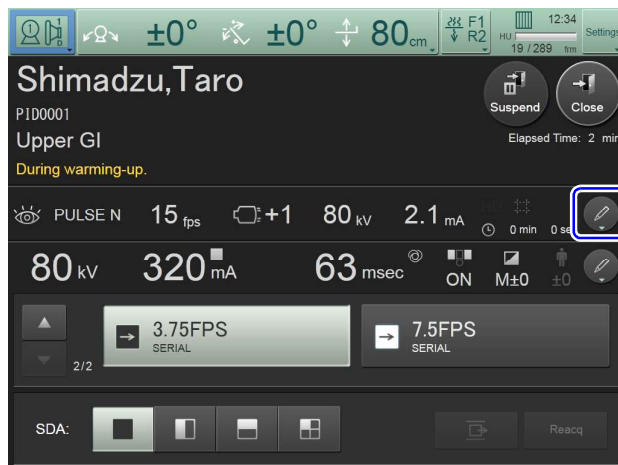

The [Fluoroscopy Condition Details] screen is displayed.