

# THICK WIPES IMPREGNATED WITH A CLEANING AND DISINFECTING LOTION



## PHAGO'WIPES MD



- Medical device
- Large wipes
- Great anti microbial spectrum of activity

### USE

PHAGO'WIPES MD is a medical device. Those wipes are designed to clean and disinfect non-invasive and non-submersible medical devices (small medical and dental equipment).

### COMPOSITION

Active disinfectant:  
Ethanol (N° CAS : 64-17-5)  
Alkyldimethylbenzylammonium chloride (n° CAS 68424-85-1)  
Propan-2-ol (n° CAS 67-63-0)

### CHARACTERISTICS

Nonwoven wipes  
Wipes color: white  
Fragrance: eucalyptus

### INSTRUCTIONS OF USE

Single use wipes; ready to use.

- When using for the first time: open the flap, remove the tamper-evident seal, thread the first wipe through the hole, then close the flap.
- Clean the equipment with the wipe.
- Contact time between 30 seconds to 15 minutes depending on the requested antimicrobial activity.
- Allow the surface to dry.
- Do not rinse.
- Close the flap after each use to prevent the wipes from drying out.

### COMPATIBILITY

Always test on a small surface before using it.

### PRESERVATION

Shelf life : 30 months after production.  
we advice to use it within 3 months after the opening of the flow pack. Keep the product according to the producer recommendations.

### MICROBIOLOGICAL ACTIVITIES

SPECTRUM OF ACTIVITY		CONTACT TIME
Bactericidal	EN 13727* - EN 14561* EN 14561 : MRSA	5 minutes
Fungicidal	EN 14562 : <i>Aspergillus fumigatus</i>	5 minutes
Yeasticidal	EN 13624 - EN 14562 : <i>Candida albicans</i>	5 minutes
Tuberculocidal	EN 14348 - EN 14563 : <i>Mycobacterium terrae</i>	15 minutes
Virucidal	EN 14476* : HBV - HCV - Adenovirus - Coronavirus - Norovirus - VRS - H1N1 - HSV1 EN 14476 : Polyomavirus EN 14476 : Rotavirus	1 minute 15 minutes pour le Rotavirus
ADDITIONNAL TESTS		
Bactericidal	EN 1276 EN 13697	30 secondes 5 minutes
Yeasticidal	EN 16650* - EN 13697 : <i>Candida albicans</i>	5 minutes

\*In dirty conditions

### PRECAUTIONS FOR USE

Product for professional use.  
safety data sheet is available on [www.quickfds.com](http://www.quickfds.com)  
Dangerous.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### REFERENCE

P60013 Flowpack of 100 white wipes (dim: 13cm x 20cm)  
Palet of 50 box of 12 flowpacks