M860

Carucior si targa
– separabile

areequipment

EN 1865-1
10G

M860. Carucior si targa – separate.

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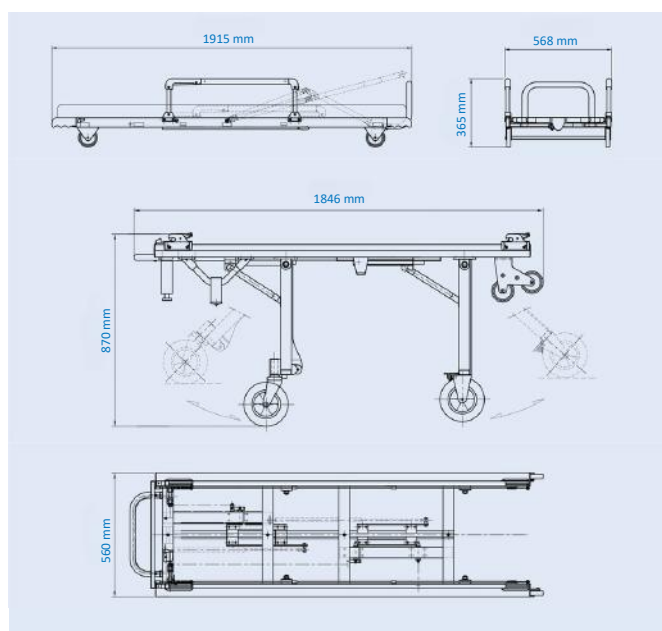
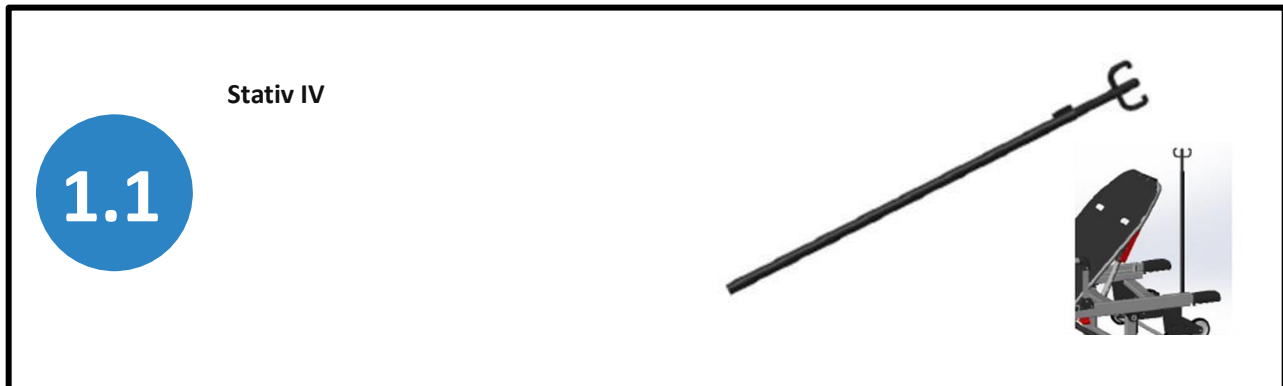
ACCESORII

1.5

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The image shows two views of a black pediatric safety vest with high-visibility yellow-green straps. On the left, the vest is shown on a blue chair. On the right, the vest is laid flat, showing its full structure including shoulder straps, chest straps, and leg straps.

Sistem CAM

1.6

The image shows a silver metal medical gurney with four wheels and a white, box-like protective enclosure (CAM) mounted on top. The enclosure has a clear front panel and is supported by a frame.

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M860

IFU-TROLLEY-STRETCHER-M860-REV.02

EDITION DATE: 05/01/2023
REVISION DATE: 15/09/2025
REVISION: 02

INDEX

Customer Relations	3
Contacts	3
Property Notice	3
Legal Notice	3
Limited Warranty Conditions	3
Adverse Reaction Warning	3
Unique Device Identification	3
Symbol Labeling and Legend	4
Indications, Contraindications, and Target Groups	5
Target Patient Population and Clinical Condition to be Diagnosed	5
Installation, Maintenance, Assembly, and Calibration Instructions	5
Contacts	5
Instructions for Safe Use	5
Storage and Handling Conditions	5
Expiry Date	5
Security Recommendations	6
Intended Purpose of the Device	7
Applied Standards	7
M860	8
M860 General Data	9
Accessories	10
Compatible Fastening Systems	14
Front Leg Control	15
Rear Leg Control	16
Front Leg Control	16
Stretcher Locking System	17
Adjustable Leg Support	18
Telescopic Handles	19
Folding Side Support	19
Folding Headboard	20
Maintenance	21
Lubrication	21
Checklist	22
Disinfection/Cleaning	23
Warnings or Precautions to be Taken for Safe Disposal of the Device	24
Training Registration	25
Maintenance Registration	26

Customer Relations

For assistance with your order or general information:

Contacts

Auto Ribeiro, Lda. - Rua S. Caetano, 551
4410 - 494 Canelas
Vila Nova de Gaia, Portugal.

Phone: (+351) 227 157 100

Website: www.arehousement.com

Property Notification

The information presented in this manual is the property of Auto Ribeiro and/or its authorized partners. Auto Ribeiro reserves all rights relating to intellectual property, exclusive design, manufacture, reproduction, and marketing of the product and the information contained therein, unless expressly assigned to third companies.

Legal Notice

This manual contains general instructions for use, operation, and maintenance of the product, and is not exhaustive. Safe and correct use is the sole responsibility of the user.

The safety information is provided for guidance only, and it is the user's responsibility to comply with applicable standards and protocols.

It is essential to undergo appropriate training before using the product in a real-world context.

Keep this manual and ensure that it accompanies the product in case of transfer. Additional copies can be requested from Customer Support.

Limited Warranty Conditions

Auto Ribeiro's products are covered by a limited warranty, the terms and conditions of which are set out in the documentation provided at the time of purchase. Full information about the warranty and its limitations can be requested directly from Auto Ribeiro Customer Service.

Adverse Reaction Warning

In the event of an adverse event or serious incident related to the use of this product, the user must report the incident to Auto Ribeiro and the competent authorities, in accordance with Regulation (EU) 2017/745 on medical devices.

To report directly to Auto Ribeiro, use the contact channels indicated in this manual.

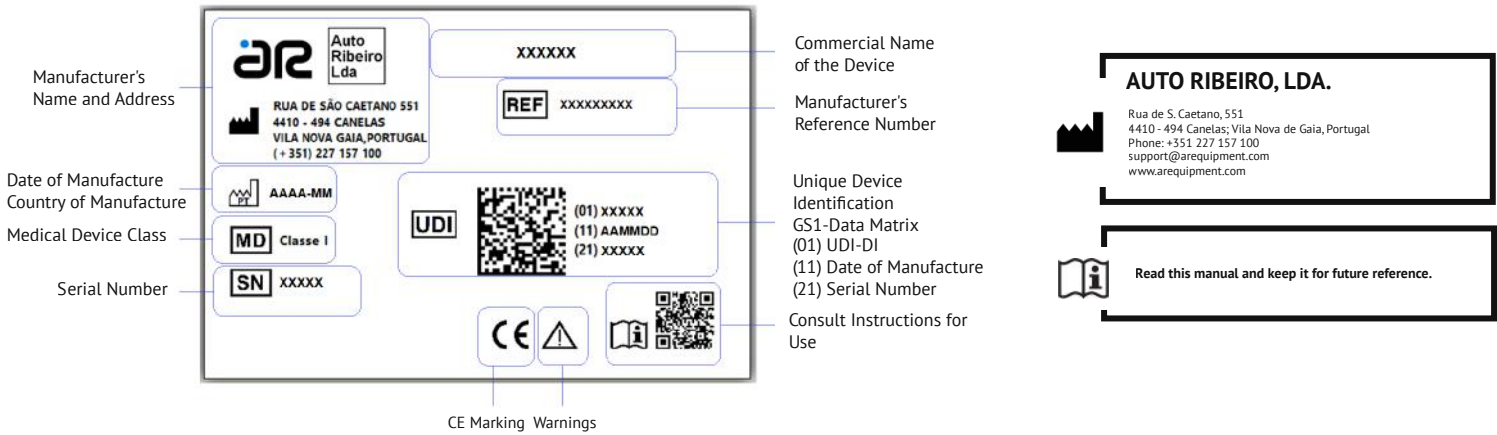
Unique Device Identification

Auto Ribeiro complies with the requirements of Unique Device Identification (UDI), in accordance with European Union regulations, allowing for the clear identification of medical devices.

The UDI label contains information in human- and machine-readable formats and is affixed to the product as indicated in this manual.

Information about the device is recorded in the European Database of Medical Devices (EUDAMED) and the National Authority for Medicines and Health Products (INFARMED), accessible to competent authorities and end users, as applicable.

LABELING AND SYMBOL LEGEND

Symbol	Description
	Indicates the manufacturer of the medical device.
	Indicates the date and country in which the medical device was manufactured.
	Indicates that an item is a medical device.
	Indicates the manufacturer's serial number so that a medical device can be identified.
	Indicates the manufacturer's reference number so that the medical device can be identified.
	Indicates a medium that contains information relating to the unique identification of the device.
	It means European technical conformity.
	Indicates that special attention is required when using the device or control located near where the symbol is placed, as the current situation requires the operator's attention or action in order to avoid undesirable consequences.
	Indicates that the user should consult the instructions for use.

Indications, Contraindications, and Target Groups

Transportation must always be carried out using all restraint systems supplied with the equipment. The load capacity must not be exceeded. Safety warnings are mentioned in the equipment's operating instructions and are also added in the form of a sticker on the device itself.

Target Patient Population and Clinical Condition to be Diagnosed

Patients in pre-hospital, hospital and ambulance transport. Applicable whenever the user considers that the patient and their clinical condition allow safe transport.

Installation, Maintenance, Assembly, and Calibration Instructions

Ensure that only components and services approved by Auto Ribeiro are used for maintenance and repairs. Also use only accessories approved by Auto Ribeiro, thus ensuring the safety and functionality of the product. It is recommended that maintenance be carried out at the ARSII technical assistance service.

Contacts

ARService - Rua da Urtigueira, n.º 288 e 298
4410-304 Canelas,
Vila Nova de Gaia, Portugal.

Phone: (+351) 227 157 104
Mobile: (+351) 910 556 363

E-mail: geral@arservice2.com

Instructions for Safe Use

This device is intended for use by qualified operators who have been trained for this purpose. Its operation facilitates its operation, being adaptable to a wide range of vehicles.

The instructions for using the equipment must be read in full before use.

Expiration Date

It is estimated that the device's useful life will not exceed 10 years, as regulatory criteria are constantly updated, thus preventing the product from becoming obsolete. During its useful life, the maintenance and cleaning cycles defined in the instructions for use must be followed to safeguard and prevent failures.

Storage and Handling Conditions

It is packaged in such a way that its characteristics and performance during its intended use are not adversely affected during transport and storage.

Packaged in a cardboard box, covered with protective bubble wrap on the top surface, and sealed with metal staples.

This product is not affected by temperature or humidity variations.

Information about compatibility with other products

Description of accessories and other products that, although not devices, are intended to be used in combination with the device:

Reference	Commercial Name
M051013033	ERGONOMIC MATTRESS - M SERIES LBS
M051013036	MATTRESS LBS
M051013037	MATTRESS WITH FLAPS
M051005014PRT	PEDIATRIC VEST
P012007007	IV SUPPORT
P012007008	IV SUPPORT
F014001001	OXYGEN SUPPORT
F014001010	OXYGEN SUPPORT
P012014050PRT	OXYGEN SUPPORT
F011001001PRT	INSTRUMENTS TRAY
F001009001ACP	LBS STRETCHER
F099001001	CAM SYSTEM
F099001002	CAM SYSTEM
K200999005	BELTS
K200999020	BELTS

Compatible fixtures: E250, E250BB, E250F, E300, A150, A150F, A200.

SAFETY RECOMMENDATIONS

NOTICE

Important Safety Guidelines

Follow the instructions provided to ensure safe operation.

Risk of falling at the same level – Take special care when handling the equipment, considering the risk of falling inside the vehicle.

Risk of falling at different levels – Take special care when handling the equipment, considering the risk of falling from inside to outside the vehicle.

Risk of collision with moving objects – Perform regular maintenance to reduce and/or eliminate risks arising from parts with significant wear or looseness.

Risk of overload or strain on the operator – Perform periodic maintenance to minimize excessive strain during operation.

Biological risks – Clean regularly to reduce the risk of contamination.

The use of unauthorized parts or services may cause accidents or equipment failure.

Ensure that only components and services approved by Auto Ribeiro are used for maintenance and repairs. Also, use only accessories approved by Auto Ribeiro, thus ensuring the safety and functionality of the product.

The operator must have experience in using the stretcher and basic first aid knowledge. During use, you must strictly follow the instructions; otherwise, the company is not responsible for any risks.

Warnings indicate potentially dangerous situations.

When unloading the ambulance stretcher, it must be placed on level ground. To ensure safety, the legs of the wheels must be fully extended in the air before the wheels leave the vehicle, avoiding falls and injuries to the patient when landing on the stretcher.

Illegal or incorrect operation as well as overloading is strictly prohibited to avoid damage caused by improper use of the stretcher.

If the bolts and nuts of the stretcher are loose, they must be tightened in time and lubricated when necessary.

Disinfection / Cleaning

Use detergents/disinfectants that are compatible with aluminum, stainless steel, and rubber.

When cleaning, carefully check the concentration recommended by the manufacturer to avoid damage to the equipment materials.

Rinse with a limited amount of water, dry as much as possible, and lubricate the guides, bars, and sliding parts.

If the equipment is used in aggressive environments (e.g., saline environments), daily washing and lubrication cycles are recommended to protect exposed parts.

The manufacturer is not responsible for damage caused by the use of inappropriate cleaning products that may deteriorate surface materials.

NOTIFICATION

Notifications highlight important information that is not related to risks.

INTENDED PURPOSE OF THE DEVICE

The product was designed to load and transport patients with the least possible effort in pre-hospital, hospital and ambulance transport.

STANDARDS APPLIED

EN 1865-1: Patient handling equipment used in road ambulances.
Specification for general stretcher systems and patient handling equipment.

EN 1789: Medical vehicles and related equipment – Road ambulances.

M860

M860

TROLLEY-STRETCHER

M860 TROLLEY-STRETCHER

About

The M860 stretcher trolley, with 4 parts, is a device with a removable, manually operated stretcher. This product is fully developed and produced by Auto Ribeiro.

Its structure was developed from scratch, entirely made of high-strength aluminum alloy. Likewise, the aluminum profiles were designed taking into account their mechanical resistance and final appearance, being produced exclusively for Auto Ribeiro.

It is distinguished by its robustness, versatility and quality of final finish.

Technical Characteristics

- Aluminum structure;
- 4 175 mm extendable handles;
- 4 wheels with a diameter of 200 mm (2 rotating and 2 fixed);
- Adjustable headrest inclination;
- Adjustable inclination of the foot covering in 4 positions.

Weight (without accessories): 32.7 kg (trolley) + 18 kg (stretcher)

Maximum Load Capacity: 250 kg

Dimensions (closed): 1846 x 560 x 455 mm

Dimensions (open): 1846 x 560 x 870 mm

Dimensions (stretcher): 1915 x 568 x 365 mm


Accessories included:

- Serum support;
- Folding armrests;
- Foot support.

Warranty: 2 years

Different Configurations/Variants

Reference	Commercial Name	UDI-DI
F002003001ACP	TROLLEY M860	05600857500699
F002003007ACP	TROLLEY M860	05600857500705
F002003006ACP	TROLLEY M860	05600857500712
F002003009ACP	TROLLEY M860	05600857500729
F002006001ACP	TROLLEY M860	05600857500736
F002006005ACP	TROLLEY M860	05600857500743
F002006002ACP	TROLLEY M860	05600857500750
F002006004ACP	TROLLEY M860	05600857500767
F002016002ACP	TROLLEY M860	05600857500774
F002016003ACP	TROLLEY M860	05600857500781
F002005001ACP	TROLLEY M860	05600857500798
F001002001ACP	STRETCHER M860	05600857500897

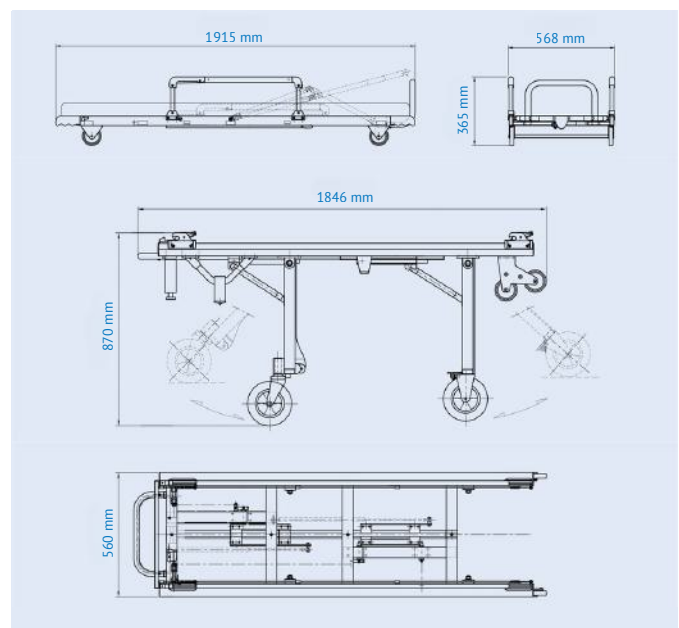ACCESSORIES

IV support

1.1

The image shows a long, black, adjustable IV support pole with a hook at the end. To the right, a smaller inset image shows the pole attached to the side of a table, demonstrating its use.

Instruments tray

1.2

The image displays a black, square instruments tray table with four legs. A smaller inset image shows the tray table attached to the side of a table, illustrating how it can be used to hold surgical instruments.

LBS

1.3

The image shows three views of the LBS (Load Bearing System). On the left, a diagram shows a black pad being placed on a table with red arrows indicating the load-bearing area. In the middle, a smaller table is shown with the LBS pad attached. On the right, a larger table is shown with the LBS pad attached, demonstrating its use for patient support.


Ergonomic mattress

1.4

The image shows a blue, rectangular ergonomic mattress designed for use on a table. It has a contoured shape to provide support and comfort for patients.

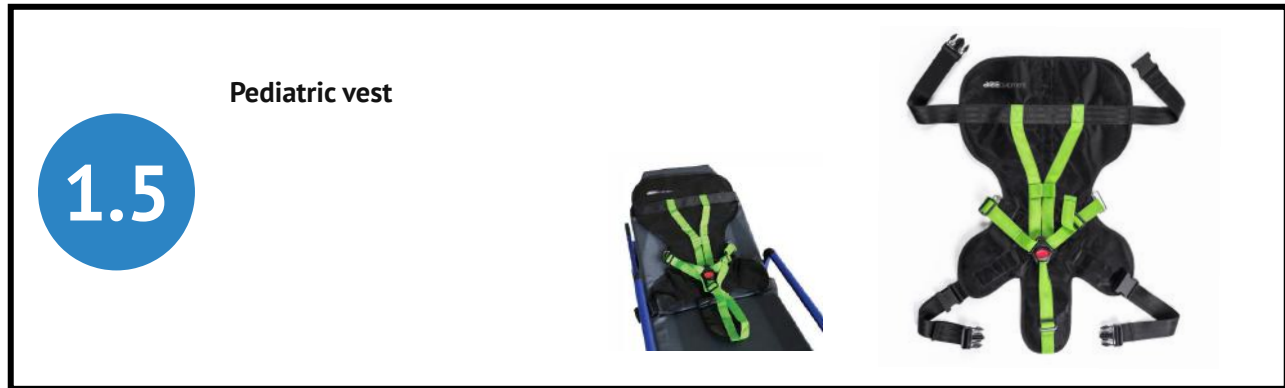ACCESSORIES

ACCESSORIES

Fastening belts

1.7

1.º - Tie the safety tapes to the internal bars.

2.º - Pass the safety tapes between the mattress grooves.

3.º - Fasten your seat belt.

ACCESSORIES

Pillow

1.8

1.º - Pass the strap through the mattress slots.

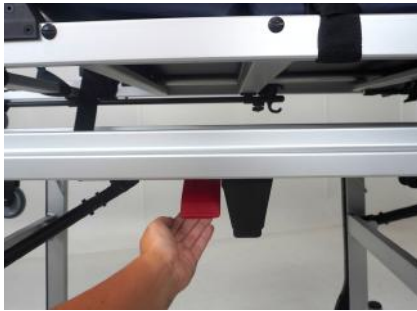
2.º - Join the two sides of the velcro.

COMPATIBLE FIXING SYSTEMS

FRONT LEG CONTROL (INTERMEDIATE POSITION)

Security system

1.º – Lift the lever (located on the right side of the trolley).

Final position

Wheels

2 swivel wheels

2 fixed wheels

Fixed wheel locking system
1.º – Activate the red system with your foot to lock the wheels.

REAR LEG CONTROL

1.º – Press the left lever until it moves and keep it pressed.

(Note: To position the rear leg at mid-stroke, you must press the lever and release it immediately.)

FRONT LEG CONTROL

2.º – Press the right lever until it moves and keep it pressed.

STRETTER LOCKING SYSTEM

1.º – Squeeze the lever and leave it pressed.

2.º – Pull the stretcher to release it from the cart.

ADJUSTABLE LEG SUPPORT

Position with footrest and knees bent

Lower leg raises

Leg elevation

TELESCOPIC HANDLES

1.º – Press the button with your finger.

2.º – Push or pull to achieve desired length.

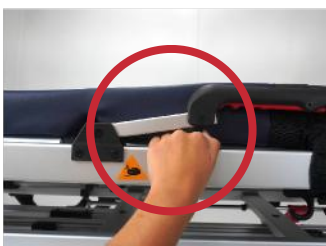
FOLDABLE SIDE SUPPORT

1.º – Press the button by hand.

2.º – Push or pull to reach the open position.

Crushing hazard

FOLDING HEADBOARD

It is possible to adjust the headboard panel in different positions.

- 1.º – Press the red lever.
- 2.º – Pull or push the headboard panel to achieve the desired position.

MAINTENANCE

- The equipment requires regular maintenance to maintain correct operation;
- You must follow a regular maintenance plan to avoid failures;
- Keep maintenance records for your equipment;
- Use original replacement parts recommended by your manufacturer;
- Read the manufacturer's safety data for each product.

The following table presents recommended minimum maintenance intervals for your equipment.

Disinfection	Cleaning	Inspection	Lubrication
After each use	After each use When necessary	3 months	3 months

LUBRICATION

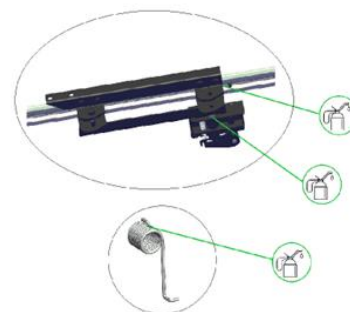
- Clean and disinfect equipment before lubricating;
- Clean the slide guides using a soft cloth to remove lubricant and residue (before lubricating);
- Lubricate the mechanical elements with high viscosity lubricant;
- Only lubricate the points recommended by the supplier without applying excessive amounts.

Points to lubricate

Do not apply lubrication

CHECKLIST

- Check that all components are in the equipment;
- Check for any excessive or abnormal wear;
- Check that all screws, nuts, bolts, rivets and springs are in the equipment;
- Check that all moving and rotating parts are operating smoothly;
- Check that the stretcher moves smoothly;
- Check the wheels for abnormal wear;
- Check that the extendable handles work correctly;
- Check that the equipment has the belts installed correctly;
- Check that the belts are in good condition, without cuts or scratches;
- Check that the belt fittings work correctly;
- Check whether the equipment is being used in an Auto Ribeiro fastener;
- Check if the equipment has abnormal gaps in the area where it fits into the ambulance fastener;
- Check that the equipment has all the necessary accessories.

Do not use water jets.

Do not use detergents/disinfectants containing sodium hypochlorite as a component for cleaning.

DISINFECTION/CLEANING

1. Disinfection/Cleaning of belts

- Remove the equipment belts;
- Clean and disinfect the belts with a compatible detergent/disinfectant according to the concentration indications suggested by the manufacturer;
- Clean with a small amount of warm water, dry as much as possible and reapply to the equipment as indicated in the instruction manual.

2. Disinfection/Cleaning of the mattress

- Remove the mattress from the equipment;
- Clean and disinfect the mattress with a compatible detergent/disinfectant and according to the concentration indications suggested by the manufacturer;
- Clean with a small amount of warm water, dry as much as possible and reapply to the equipment.

3. Disinfection/Cleaning of equipment (Each use)

- Use detergent/disinfectant compatible with aluminum, stainless steel and rubber;
- To clean, carefully check the concentration suggested by the manufacturer on the product, to avoid damaging the equipment (note: first test the product in a non-visible area and check if it causes damage);
- Clean with a small amount of warm water, dry as much as possible and lubricate the guides, bars and sliding parts;
- If operating in an aggressive environment (e.g. saline environment), daily cleaning/lubrication of the equipment is recommended to protect exposed parts;
- The manufacturer is not responsible for any damage caused by the use of a cleaning product that may damage the surface material;
- Do not use any type of solvent or diluent to clean the equipment.

4. Inspection

- A regular inspection plan must be followed to avoid equipment failures, so as not to put patient and operator safety at risk;
- It is necessary to check the parts used for immobilization purposes;
- Checking the tightness of the connection elements. Make adjustments if necessary.

5. Lubrication

- Clean and disinfect equipment before lubricating;
- Clean the slide guides using a soft cloth to remove lubricant and residue (before lubricating);
- Lubricate the mechanical elements with high viscosity lubricant;
- Only lubricate the points recommended by the supplier without applying excessive amounts.

Do not use water jets.

Do not use detergents/disinfectants containing sodium hypochlorite as a component for cleaning.

WARNINGS OR PRECAUTIONS TO BE TAKEN FOR THE SAFE DISPOSAL OF THE DEVICE

This device contains materials that must be disposed of in accordance with applicable local environmental regulations. Incorrect disposal may pose risks to public health and the environment.

Specific instructions:

1. Separation of components

• Before disposal, separate the main materials:

- Metal frame (aluminum);
- Rubber and plastic components;
- Canvas.

• Recycling and reuse

- Whenever possible, send metal components (aluminum) to authorized recycling centers;
- Plastic and rubber components may have specific solutions for energy recovery or recycling.

Do not delete:

Do not dispose of this device in household or unsorted waste. The end user is responsible for ensuring safe disposal through authorized operators.

Device disposal:

This device consists of a metal frame (aluminum), rubber components, plastic, and canvas.

At the end of its useful life, disposal must be carried out by separating the components by type of material and sending them to duly licensed waste management operators.

The metal frame must be recycled as aluminum scrap.

The canvas should be treated as non-hazardous waste, but should be sent for disposal in accordance with guidelines for hospital or contaminated waste.

Do not dispose of with household waste. Improper disposal may endanger public health and the environment.

For more information, consult the applicable legislation in force or contact the manufacturer.

M860

IFU-TROLLEY-STRETCHER-M860-REV.02

EDITION DATE: 05/01/2023
REVISION DATE: 15/09/2025
REVISION: 02

M860  Auto
Ribeiro
Lda

EU DECLARATION OF CONFORMITY

For medical devices

according to Regulation (EU) 2017/745

Auto Ribeiro Lda., with main facilities at Rua S. Caetano, 459 and 519 – 4411-701 Canelas, Vila Nova de Gaia, Portugal, declares under its own responsibility that the following product complies with the Essential Requirements of Medical Devices Directive 2017/745 and, where appropriate, with the following mentioned standards.

Auto Ribeiro Lda. maintains a Quality Management System that fulfills the requirements of NP EN ISO 9001. Copies of Auto Ribeiro, Lda. ISO 9001 certificate issued by SGS are available upon request.

Product: Trolley and Stretcher

Manufacturer: Auto Ribeiro Lda.

Model: M860 Trolley and Stretcher

Risk Class of the Device: Class I

Device ID: 78251

Basic UDI-DI: 5600786656TROLLEY68
5600786656STRETCHERC5

CDM: 57967814

Based on Standards:

- | | |
|-------------|--|
| EN 1865-1 | Patient handling equipment used in road ambulance.
-Specification for general stretcher systems and patient handling equipment. |
| EN 1789 | Medical vehicles and their equipment – Road Ambulances |
| EN ISO 9001 | System of quality management |

Place and date: Canelas - Vila Nova de Gaia, 27-12-2021

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Modification added to the stretcher :

- *No modification added to stretcher M760 and fixation E250.*
- *Product is identical to product cover with UTAC report 13/03264*

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Modifications of EN 1865-1:2010 +A1:2015 standard based on EN1865-1:2010 standard does not concern paragraph 4.2.10 Fixation. The stretcher AUTORIBEIRO M760 and E250 fixation comply with paragraph 4.2.10 EN1865-1:2010 + A1:2015 standard.

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Modification added to the stretcher :

- *No modification added to stretcher M860 and fixation E250.*
- *Product is identical to product cover with UTAC report 13/05648*

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Modifications of EN 1865-1:2010 +A1:2015 standard based on EN1865-1:2010 standard does not concern paragraph 4.2.10 Fixation. The stretcher AUTORIBEIRO M860 and E250 fixation comply with paragraph 4.2.10 EN1865-1:2010 + A1:2015 standard.

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Modification added to the stretcher :

- *No modification added to stretcher M760 and fixation E250.*
- *Product is identical to product cover with UTAC report 14/07567*

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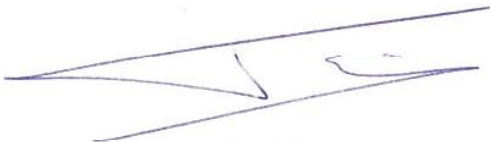
Modifications of EN 1865-1:2010 +A1:2015 standard based on EN1865-1:2010 standard does not concern paragraph 4.2.10 Fixation. The stretcher AUTORIBEIRO M764 4SW and E250 fixation comply with paragraph 4.2.10 EN1865-1:2010 +A1: 2015 standard.

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UTAC shall not be liable for any industrial or commercial applications that occur as a result of these tests. This test report may only be reproduced in the form of a full photographic facsimile. Test results are only available for the material submitted to tests or material identified in the present test report. Accreditation by the Cofrac test department only certifies the technical competence of the laboratory for the tests or analyses covered by this accreditation. () In order to modify the conformity or no conformity in the requirements, the uncertainty of the test results as not been explicitly taken into account.*

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