

# NEUTRALIZATION SOLUTION FOR AUTOMATIC INSTRUMENT WASHER MACHINES

---

## DETRO MATIC NS – USER MANUAL

- **Neutralization solution**
- **Concentrate**

### **PRODUCT PROPERTIES**

Detro Matic NS is used for removing alkali remnants after alkali cleaning of instruments at washer machines.

Removes Alkali Remnants  
Provides Effective Cleaning  
Compatible For All Washer Machines  
Compatible for All Water Hardness  
Protection Against Corrosion  
Low Usage Concentration  
Powerful Effect

### **INGREDIENT**

Citric acid, complexing agents, corrosion inhibitors

### **DOSING**

It is used at %0,05-0,1 concentration. (5-10 ml product into per 10 lt water)

### **WARNING**

Read usage instruction before usage. Don't use for cleaning of hand, face and food. Keep out of reach children. Irritant agent. Don't contact with skin and mucous membranes. In the case of eye contact, wash plenty of water immediately.

### **STORAGE CONDITIONS**

Keep away from direct sunlight. Store containers tightly closed and in a well ventilated area at 0-30 °C

### **PACKAGE**

5 L

**DETRO HEALTHCARE KİMYA SANAYİ A.Ş.**

☎ 212 659 77 62 📞 212 659 77 63

✉ info@detrox.com.tr – www.detrox.com.tr