

INSTRUCTION FOR USE

PRODUCT NAME

Medster SafeCheck Sterilization Reel and Pouch

PRODUCT DESCRIPTION

Medster SafeCheck sterilization reel and pouch is constructed from PET/PP copolymer film and medical kraft paper. A process indicator is applied on the surface of the reel and pouch. Indicators show the method of sterilization.

APPLICATION

1. Medster SafeCheck sterilization pouches are read for use. Medster sterilization reels need to be cut to the required length and sealed at one side of the pouch.
2. Cut the reel at the desired length.
3. Place the item that is to be sterilized into the pouch. Do not fill reels to more than 75% of their capacity
4. Always leave minimum 1,5cm gap between the material and each seal edge of the pouch for a smooth airflow.
5. Seal the two sides of the pouch with a calibrated and validated sealer. Strength of the seal must assure a clean peel.
6. When loading into the sterilizer place the pouches into the basket in a way that film faces to film and paper faces to paper.
7. Process the load according to the established procedures.
8. Do not use a wet pouch. Repeat the sterilization process again after changing the pouch.
9. Always open the pouch from the direction of the arrow. For a clean peel open the pouch slowly.
10. Do not use if the pouch is torn or damaged.

STORAGE CONDITIONS

Store at max. +35°C and %70 RH humidity. Sterilization reel have a shelf life of 3 years when stored at recommended conditions.

The pouches, either sterilized or not, must be kept away from moisture, excessive airflow, direct sunlight. Once the paper is wet, pouch must not be used anyone.

MEDSTER TIBBİ CİHAZ VE SAĞLIK HİZMETLERİ LTD. ŞTİ.

Malıköy Başkent OSB Mah. 19. Cad. No:51 Sincan/Ankara/Türkiye

Tel: +90 312 387 05 50 Fax: +90 312 387 05 51

E-mail: info@medster.com.tr Web: www.medster.com.tr