



# DISINFECTION WIPE FOR SURFACES USER MANUAL

## **DETROSAN AF WIPES**

---

- **READY TO USE ALCOHOL FREE DISINFECTION WIPE**
- **EFFECTIVE AT 1 MIN.**

### **Product Properties**

- Effective at 1 min.
- Aldehyde, Phenol, Alcohol free.

### **Usage Areas**

It is aldehyde and phenol free and a fast effective alcohol free disinfection wipe. It is used at medical equipments (patient beds, doors, medical instruments, cabinets etc.) and surfaces existing at surgery rooms, intensive care rooms, dental units. It is effective especially in alcohol sensitive surfaces like baby incubators, acrylic surfaces etc.

### **Usage form**

- Take sufficient wipe according to surface size that will be wiped.
- Visible contaminations (blood etc.) existing on the surface are cleaned firstly.
- Then take some wipes again and wipe equipment completely.
- Wait for 1 minutes for effect time.

**Uyarı:** Appy electronic devices after switch off.

### **Ingredient**

Quarternary ammonium compounds and Amine ingredients

### **Advantages**

- Low effect duration
- Baktericidal (incl. TBC ve MRSA), fungucidal ve virucidal (HIV, HBV, HCV)
- Suitable for alcohol sensitive surfaces
- Not Irritant
- Not harmful for surfaces.
- Easy drying and nice odour.

### **First aid informations:**

- In the case of eye contact, wash plenty of water minimum 15 min. Appy to physician.
- In the case of swallowing, not vomit. Drink plenty of water and shake your mouth and throat with water. Immediately appy to physician.

**DETRO HEALTHCARE KİMYA SANAYİ A.Ş.**

☎ 212 659 77 62 📞 212 659 77 63

✉ info@detrox.com.tr – www.detrox.com.tr