**Instruction - If you observe any discrepancy in artwork hard copy and artwork in CD.
Please do not process the artwork. Revert back to us for Clarification.**

			Dept.	Name.	Sign/Date.
Product Name	BIO DECALIN	Prepared By PMT/MKT	QC		
Item Name	INSERT				
Artwork Code / version	SP1998 Rev 00				
Supersedes	NA	RA			
Size	230mm X 272mm				
Material Specification	54 GSM / Maplitho Paper	QA HEAD			
Artwork History	Change as per (CC-QAD-19-010)				
Special Instruction	CE2460	Head Purchase			
Style of Carton	NA				
Colors	Black Color	CMD			
Type of File	CDR file				

 C:0 | M:0 |Y:0 |K:100

 C:0 | M:0 |Y:0 |K:90

BIO DECALIN

Perfluorodecalin (C₁₀F₁₈)

Description:

Bio Decalin is sterile non-pyrogenic ultra purified 100% fluorinated perfluorocarbon liquid as an aid for retinal surgery. It is chemically inert & non-toxic in nature.

Packaging:

Bio Decalin is supplied non-pyrogenic & purified, in a 5.0 ml / 7.0ml glass vial or in a prefilled syringe filled with 5.0 ml / 7.0ml volume in sterile blister packaging. Patient card & extra labels are supplied with each box for optimal traceability. This product does not contain latex.

Indication:

Intra-operative device for retinal surgery. It is used in the surgical management of:
•Retinal detachment with giant tears
•Retinal detachment with proliferative vitreoretinopathy (PVR) including diabetic retinopathy.
•Retrieval of luxated natural or artificial lens in the Vitreous.

Warnings:

•Single use only.
•Do not resterilize.
•Do not use after the expiration date.
•Do not reuse. The used product should be considered as a biological waste. It may lead to any biological reactions including but not limited to inflammation, infection, injury or any unknown clinical condition.

Bio Decalin is an Intra-operative device, for vitreoretinal surgery. It should not be left in place in the eye at the end of the procedure. Care should be taken to maintain the product inside of the bubble during the injection and just in front of the retinal tear. This will prevent the product from dispersing and infiltrating the sub-retinal tissues. The volume to be inject should be adapted to the procedure.

It should not be injected directly into the vitreous, or injected simultaneously with aspiration of the vitreous, as severe intraocular damage may occur.

Contraindications:

The product should not be allowed to infiltrate under the subretinal tissues since there are no specific studies on tolerance. Not for long term use.

Storage:

Store at room temperature (15°C to 30°C)
Keep away from sunlight.

Instruction For Use:

Perform this activity in sterile field only. Fit the needle onto the syringe and pierce the stopper after having removed the metal cap. Transfer the product from the vial into the syringe. Replace the needle with cannula for infra-vitreous injection. Use a fluid or gas exchange procedure to remove the product.

Avoid spillages & collect spilled material with an absorbent.

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biotech

■ Bio-Tech Vision Care Pvt. Ltd.
Plot no. 555, 556, 557 Khatraj-Vadsar Road,
Opp. Shubham Tex-O-Pack, Village Khatraj,
Taluka Kalol, Dist. Gandhinagar, Gujarat, India.
Customer Care No. - +91 79 66823000
www.biotechhealthcare.com
intsales@biotechhealthcare.com
sales@biotechhealthcare.com

CE
2460

EC REP
Biotech Europe Meditech Inc Limited
AF2, IDA Business & Technology Park,
Roscommon, Ireland.

SP 1998_Rev.00 12.12.20

biotech

■ Bio-Tech Vision Care Pvt. Ltd.
Plot no. 555, 556, 557 Khatraj-Vadsar Road,
Opp. Shubham Tex-O-Pack, Village Khatraj,
Taluka Kalol, Dist. Gandhinagar, Gujarat, India.
Customer Care No. - +91 79 66823000
www.biotechhealthcare.com
intsales@biotechhealthcare.com
sales@biotechhealthcare.com

CE
2460

EC REP
Biotech Europe Meditech Inc Limited
AF2, IDA Business & Technology Park,
Roscommon, Ireland.

biotech

■ Bio-Tech Vision Care Pvt. Ltd.
Plot no. 555, 556, 557 Khatraj-Vadsar Road,
Opp. Shubham Tex-O-Pack, Village Khatraj,
Taluka Kalol, Dist. Gandhinagar, Gujarat, India.
Customer Care No. - +91 79 66823000
www.biotechhealthcare.com
intsales@biotechhealthcare.com
sales@biotechhealthcare.com

CE
2460

EC REP
Biotech Europe Meditech Inc Limited
AF2, IDA Business & Technology Park,
Roscommon, Ireland.

biotech

■ Bio-Tech Vision Care Pvt. Ltd.
Plot no. 555, 556, 557 Khatraj-Vadsar Road,
Opp. Shubham Tex-O-Pack, Village Khatraj,
Taluka Kalol, Dist. Gandhinagar, Gujarat, India.
Customer Care No. - +91 79 66823000
www.biotechhealthcare.com
intsales@biotechhealthcare.com
sales@biotechhealthcare.com

CE
2460

EC REP
Biotech Europe Meditech Inc Limited
AF2, IDA Business & Technology Park,
Roscommon, Ireland.