EN FOLDING WHEELCHAIRS WITH QUICK RELEASE
F FAUTEUILS ROULANTS PLIANTS AVEC DÉMONTAGE RAPIDE
ES SILLAS DE RUEDAS PLEGABLES DE DESMONTAJE RÁPIDO
P CADEIRAS DE RODAS DOBRÁVEIS COM DESMONTAGEM RÁPIDA

EN Painted steel frame, seat and backrest in padded and washable nylon. PU tubeless rear aluminium wheels with quick release, safety and anti-tipper wheels. Front PU wheels elevating, flip back and detachable PU armrests. Swing away and adjustable footrests with heel stopper. Back pocket.

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CP200 series
 Self-propelling wheelchair
 interchangeable into transit
 wheelchair. Double cross
 PU tubeless tires

Rear wheels Ø 60 cm
 Front wheels Ø 20 cm

CP200-38 Seat cm 38
CP200-40 Seat cm 40
CP200-43 Seat cm 43
CP200-45 Seat cm 45

CP200-46 Seat cm 46
CP200-48 Seat cm 48
CP200-50 Seat cm 50

Dual

Front

Lateral

Back

CP205 series
 Transit wheelchair
 interchangeable into
 self-propelling wheelchair
 Double cross

Rear wheels Ø 30 cm
 Front wheels Ø 20 cm

CP205-38 Seat cm 38
CP205-40 Seat cm 40
CP205-43 Seat cm 43
CP205-45 Seat cm 45

CP205-46 Seat cm 46
CP205-48 Seat cm 48
CP205-50 Seat cm 50

Dual go!

Front

Lateral

Back

CPA110-45
WC seat 45 cm

**CPA100-40, CPA100-43,
CPA100-45, CPA100-50**
Foam cushion h 5 cm

CAR900
Headrest

CPA501
Linear table for
wheelchair 55x32 cm

CPA506
Table for wheelchair
58x38 cm

CPA516
Table for wheelchair
Large 68x57 cm for
CP200-50, CP205-50

CPA600
Adjustable
IV pole

CPA606
Oxygen bottle
holder - Ø
Max 120 mm

CPA605
Oxygen bottle holder
Ø Max 120 mm

CPA130
Footrest for sizes
45-46-48-50 only

CPA120C **CPA120V**
Chrome-plated
elevating foot-
rests - PAIR Painted
elevating
footrests -PAIR

CPA234
One hand drive kit
Only for CP200 Series

CAR605
Antitheft
bar


CPR150
Anti-tipper
wheels
4 cm - PAIR

KOALA
Restraining system line

RP245
Safety belt

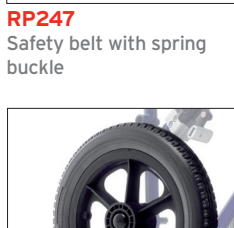RP247
Safety belt with spring
buckle

CPR108
Armrest adjustable
height 73÷81 cm from
floor - PAIR

CPR111
Swing away footrests -
PAIR

CPA135
Swing away lateral footrests
- PAIR for size 38 only

CPA242
PU rear wheel Ø 60 cm
- PAIR

CPA240
PU rear wheel Ø 30 cm
- PAIR

CPR143
PU front wheel Ø 20x3 cm
- SINGLE

TECHNICAL FEATURES

Weight and dimension

Measures in CM															
CP200-38								38		58					
CP200-40								40		60					
CP200-43								43		63					
CP200-45	113	81	42	90	72÷80	49	41÷56	45	41	65	29	17,5	60	20	4 150
CP200-46								46		65,5					
CP200-48								48		68					
CP200-50								50		70					
CP205-38								38		52					
CP205-40								40		54					
CP205-43								43		57					
CP205-45	104	76	42	90	72÷80	49	41÷55	45	41	59	25	17,5	30	20	4 150
CP205-46								46		59,5					
CP205-48								48		62					
CP205-50								50		64					

EN FOLDING TRANSIT WHEELCHAIRS
F FAUTEUILS ROULANTS PLIANTS DE TRANSIT
ES SILLAS DE RUEDAS PLEGABLES DE TRÁNSITO
P CADEIRAS DE RODAS DOBRÁVEIS DE TRÂNSITO

EN Painted steel frame, seat and backrest in padded and washable nylon, holder back pocket. Mag rear wheels with PU tires. Flip-back. PU armrests. Swing away adjustable footrests with heel stopper.

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CP500 series
Single cross,
rear solid tires

Rear wheels Ø 30 cm
Front wheels Ø 20 cm

CP500-40 Seat cm 40

CP500-43 Seat cm 43

CP500-45 Seat cm 45

Go!

CP520 series
Double cross, rear solid tires,
folding backrest

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CPA501
Linear table for wheelchair 55x32 cm

CPA506
Table for wheelchair 58x38 cm

CPA150
Armrest adjustable height 73÷81 cm from floor - PAIR
Only for CP500 series

CPR108
Armrest adjustable height 73÷81 cm from floor - PAIR
Only for CP520 series

CPA600
Adjustable IV pole

CPA606
Oxygen bottle holder - Ø Max 120 mm

CPA605
Oxygen bottle holder Ø Max 120 mm

CPA130
Footrest for sizes 45-46-48 only

CAR605
Antitheft bar

CPR150
Anti-tipper wheels 4 cm - PAIR

CPA120C
Chrome-plated elevating footrests - PAIR

CPA120V
Painted elevating footrests - PAIR

KOALA
Restraining system line

RP245
Safety belt

RP247
Safety belt with spring buckle

CPR102
Complete armrest - PAIR
Only for CP520 series

CPR111
Swing away lateral footrests - PAIR

CPR131
PU rear wheel Ø 30 cm - PAIR

CPR143
PU front wheel Ø 20x3 cm - SINGLE

CPR215S Left
CPR215D Right
Brake Only for CP520 series

TECHNICAL FEATURES

Weight and dimension

Measures in CM																
CP500-40								40		55						
CP500-43								43		58						
CP500-45	94	70	42	92	72	48	42÷54	45	43	60	26	17,5	30	20	-	120
CP500-46								46		60,5						
CP500-48								48		63						
CP520-40								40		55						
CP520-43								43		58						
CP520-45	99	73	42	93	72	48	42÷56	45	43	60	26	17,5	30	20	-	150
CP520-46								46		60,5						
CP520-48								48		63						

EN FOLDING WHEELCHAIRS WITH RECLINING BACKREST
F FAUTEUILS ROULANTS PLIANTS, DOSSIER INCLINABLE
ES SILLAS DE RUEDAS PLEGABLES CON RESPALDO RECLINABLE
P CADEIRAS DE RODAS DOBRÁVEIS COM ENCOSTO INCLINÁVEL

EN Seat and backrest in black padded nylon. Backrest with adjustable inclination. PU wheels and anti-tipper castors. Elevating, detachable footrests with height adjustable leg rest. Handles with brakes and brakes on the rear wheels.

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CP800 series
 Steel frame, single cross bar
 Rear wheels Ø 60 cm
 Front wheels Ø 20 cm

Comfy-S

CP800-40 **CP800-43** **CP800-46**
 Seat 40 cm Seat 43 cm Seat 46 cm

CP805 series
 Steel frame, single cross bar
 Rear wheels Ø 30 cm
 Front wheels Ø 20 cm

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
KOALA
 Line restraining system

RP245
 Safety belt only for CP810

RP247
 Safety belt with spring buckle only for CP800 - CP805

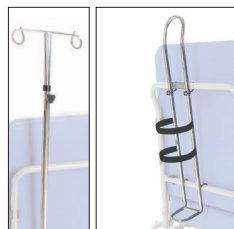RCA120 RCA135
 IV pole Chromium steel oxygen bottle holder
 Ø max 150mm
 (image for illustrative purpose only)

CPA130
 Footrest for sizes 50 only

CBR500
 Linear table

CAR520
 Molded table for CP810

CPA507
 Table for wheelchair
 58x38 cm

CAR831
 Armrest desk type - PAIR
 Only for CP800 - CP805

CAR171
 Elevating footrest - PAIR
 Only for CP800 - CP805

CAR174
 PU rear wheel Ø 60 cm -
 PAIR Only for CP800

CAR172
 PU front wheel Ø 20 cm -
 PAIR Only for CP800 - CP805

CAR173
 Anti-tipper wheel
 Ø 5 cm - PAIR
 Only for CP800 - CP805

CAR830
 Elevating armrest - PAIR
 Only for CP810 series

CAR168
 Elevating footrest - PAIR
 Only for CP810 series

CAR217
 PU rear wheel - PAIR
 Only for CP810 series

CAR226
 PU front wheel - PAIR
 Only for CP810 series

CAR314
 Anti-tipper wheel - PAIR
 Only for CP810 series

CAR175
 PU rear wheel - PAIR
 Only for CP805 series

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ARDEA ONE WHEELCHAIRS INSTRUCTION MANUAL

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Via Bruxelles, 3 - Melegnano 20122 Cavriaglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

[illegible]

INDEX

1. Codes	pag.18
2. Introduction	pag.18
3. Intended use.....	pag.18
4. EC declaration of conformity	pag.19
5. General warning	pag.19
6. General description.....	pag.19
6.1 Parts view	pag.19
7. Assembling/ disassembling	pag.20
7.1 Opening.....	pag.20
7.2 Closing	pag.21
7.3 Install/remove the armrest.....	pag.21
7.4 Install/remove the swing away and elevating footrest.....	pag.22
7.5 Adjust the swing away legrest	pag.22
7.6 Adjust the elevating legrest.....	pag.22
7.7 Install/remove the 30cm wheels	pag.23
7.8 Adjust the brakes with 30cm wheels	pag.23
7.9 Install/remove the 60cm wheels.....	pag.23
7.10 Adjust the brakes with 60cm wheels.....	pag.24
8. Before use.....	pag.24
9. Using warnings.....	pag.24
10. How to use	pag.24
10.1 Get in the wheelchair.....	pag.25
10.2 Get out of the wheelchair.....	pag.25
10.3 Stairs and height difference (how to face them).....	pag.25
10.4 Use the armrest.....	pag.25
10.5 Use the 60cm wheels.....	pag.25
10.6 Use the 30cm wheels.....	pag.25
10.7 Use the casters for narrow ways	pag.26
10.8 Carrying	pag.26
11. Maintenance	pag.26
12. Cleaning and disinfection	pag.26
12.1 Cover and stuffing.....	pag.26
12.2 Frame and metal parts.....	pag.26
12.3 Brakes and wheels	pag.26
12.4 Disinfection	pag.26
13. Conditions of disposal	pag.26
14. Spare parts/ accessories	pag.26
15. Technical features.....	pag.27
15.1 Dimensions	pag.27
15.2 Technical specifications	pag.28
16. Warranty.....	pag.29
16.1 Repairing.....	pag.29
16.2 Spare parts.....	pag.29
16.3 Exempt clauses	pag.29

CE I Class Medical Device
according to Directive 93/42/EEC
and further modifications

1. Codes

CP100-xx	Series START 60 cm wheels - Self-Propelled
CP102-xx	Series START1 60 cm wheels - Self-Propelled
CP110-xx	Series NEXT 60 cm wheels - Self-Propelled
CP115-xx	Series NEXT GO 30 cm wheels - Transit
CP200-xx	Series DUAL 60 cm wheels - Self-Propelled
CP205-xx	Series DUAL GO 30 cm wheels - Transit
CP300-xx	Series PLUS 60 cm wheels - Self-Propelled - Heavy duty
CP310-xx	Series PLUS GO 30 cm wheels - Transit - Heavy duty
CP500-xx	Series GO! 30 cm wheels - Transit
CP520-xx	Series GO UP! 30 cm wheels - Transit
CP620-xx	Serie SKINNY 50 cm wheels - Small spaces
CP625-xx	Serie SKINNY GO! 30 cm wheels - Transit - Small spaces
CP880-35	Series KIDDY 56 cm wheels - Self-Propelled

(XX =Refers to size available for each wheelchair model ex. 38-40-43-45-46-48-50-55-60 cm)

2. Introduction

Thank you for purchasing a **ARDEA ONE** wheelchair by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

Note: check all parts for shipping damage and test before using. In case of damage, do not use the device. Contact your dealer for further instruction.

3. Intended use

This wheelchair is designed for the mobility of people with mobility problems and can be used for self-propelling (if equipped with big rear wheels) or with the aid of an operator.

WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without notice

4. EC declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the wheelchair category are compliant with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of **directive 93/42/EEC** as prescribed by Annex VII of the said directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES.**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with **Directive 93/42/EEC** and further modifications to the proper authorities

5. General warning

- **DO NOT** use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before using this equipment otherwise injury or damage may occur.
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the wheelchair
- Keep the packed wheelchair away from heat sources
- **SERVICE LIFE**- the wheelchair use limit is defined by the wear of the parts
- **ALWAYS** keep hands and fingers clear of moving parts to avoid injury
- **DO NOT** allow children to play on or operate the wheelchair
- Close the wheelchair before lifting it.
- **DO NOT** attempt to lift the wheelchair by any removable parts

6. General description

6.1 Parts view

The components can change depending on the model you choose:


A - Frame


B - Wrenches kit for assembling


C - Swing away legrest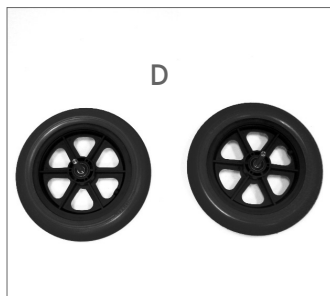


D - 30 cm casters
(diameter 30 cm) Transit model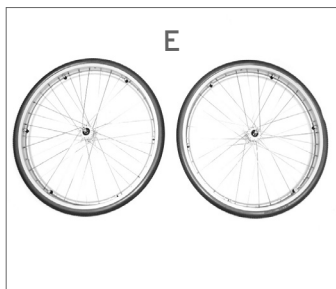

E - 60 cm Wheels
(diameter 60 cm) Self-propelled model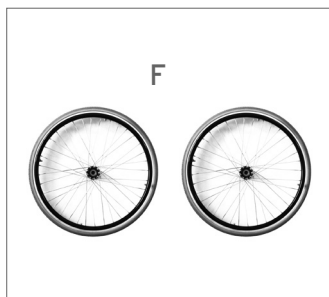


F - 50 cm Wheels for small spaces
(diameter 50 cm) Self-propelled model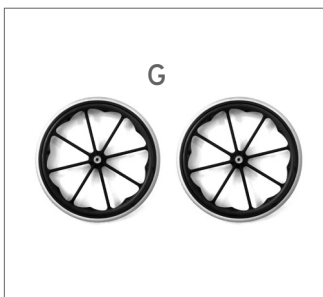

G - 56 cm Wheels - Kids model
(diameter 56 cm) Self-propelled model

7. Assembling/disassembling

7.1 Opening

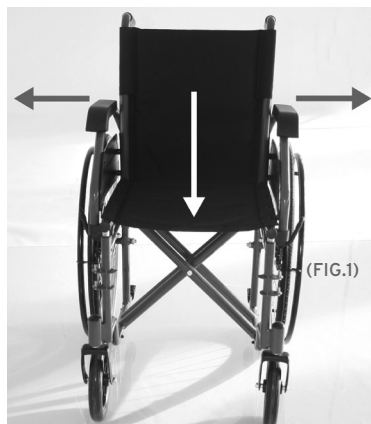Note: make reference to safety cautions in this manual

- Get the handles and pull outward, then press the seat until the wheelchair is completely open (FIG.1)

7.2 Closing

- Lift powerfully the seat and close the wheelchair. Push the handles close (FIG.2)

7.3 Install/remove the armrest


CP100-xx / CP102-xx / CP110-xx / CP115-xx / CP500-xx / CP520-xx / CP880-35

CP300-xx / CP310-xx / CP200-xx / CP205-xx

CP620-xx / CP625-xx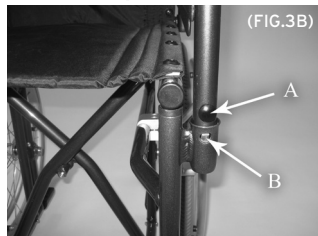

A - Push button
B - Pin for stop


- Place the armrest rear side into the rear, then hook them into the front by overturning them (FIG.3, FIG.3A)
- Insert the tubes in their supports triggering properly locking clip (FIG. 3B)
- To remove, unlock the armrest by pushing the front release lever, then overturn them and extract by pulling the rear release lever (FIG.4, FIG.4A)
- For removal press the release button (A) and then lift to up (FIG. 3B)

7.4 Install/remove the swing away or elevating legrest


- Assembly the swing away or elevating legrest by installing the hinge plates on the footrest onto the hinge pins on the wheelchair frame. Push the legrest toward inside of the wheelchair until it locks into place (FIG.5)
- Remove the legrest by pushing the legrest release lever in ward, rotate outward, lift the legrest off the hinge pins

7.5 Adjust the swing away legrest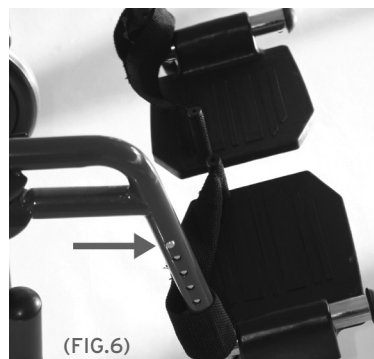

- To adjust the length of the footrest, unlock the safety lever at the bottom on the left side, then press the push button, slide the tube until the desired hole, then relock the safety lever. (FIG.6)


7.6 Adjust the elevating legrest (OPTIONAL)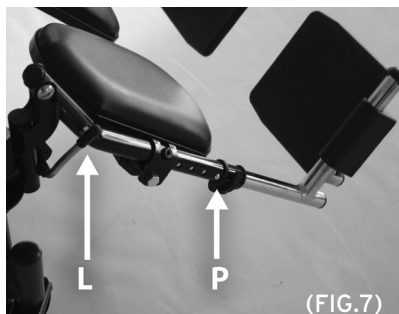

- To adjust the length of the footrest, unlock the safety lever at the bottom on the left side, then press the push button, slide the tube until the desired hole, then relock the safety lever. (FIG.7) (Rif.P)
- Adjust the legrest height by pushing the lever (FIG.7) (Rif.L), and release to lock at the desired height

7.7 Install/remove the 30cm wheels (only for NEXT, DUAL and SKINNY)

Place the caster into the hole on the wheelchair frame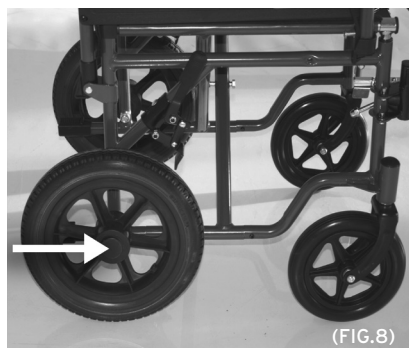

(FIG.8)


Only for quick release wheels

- Remove the plastic cap and push the wheel until it locks into the place (FIG.8)
- Remove the wheels: push the button on the centre of the wheel, and extract the wheel (FIG.8)

Only for wheels with fixed axle

- Place the small wheel and block it with the nut
- To remove the wheel, unscrew the nut and remove the small wheel

7.8 Adjust the brakes with 30cm wheels (only for NEXT, DUAL and SKINNY)

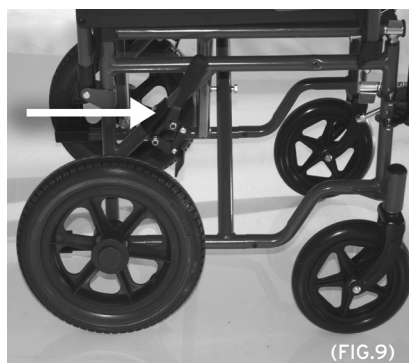(FIG.9)

- When the 30 cm wheels are installed, place the brake on the milling. Lock the device by tightening the Allen wrenches (FIG.9)

Note adjust the brake device so that it can make the right pressure to securely lock the wheel

7.9 Install/remove the 60cm wheels (only for NEXT GO, DUAL GO and SKINNY GO)

Place the wheel into the central hole on the wheelchair frame

(FIG.10)

Only for quick release wheels

- Remove the plastic cap and push the wheel until it locks into the place (FIG.10)
- Remove the wheels: push the button on the centre of the wheel, and extract the wheel (FIG.10)

Only for wheels with fixed axle (SKINNY GO)

- Place the big wheel and block it with the nut
- Unscrew the nut and remove the big wheel

7.10 Adjust the brakes with 60cm wheels (just for DUAL GO)

- When the 60 cm wheels are installed, place the brake on the milling. Lock the device by tightening the Allen wrenches (FIG.11)

Note adjust the brake device so that it can make the right pressure to securely lock the caster

8. Before use

- Inspect the frame for any damage. Check the tubing for cracks and rust, in order to guarantee the safe for people ant thing
- Check the correct assembly, in order to guarantee a safe use
- Check the seat is completely open. If not push the seat down from the lateral frame.
- Check the brake correctly works

9. Using warnings

- Check the automotive parts usage, in order to guarantee a safe use of the device for people and things
- Assure that the seat is completely open and locked
- Assure that all parts are correctly assembled. Checking carefully the correct assembling of the wheels
- Assure the brakes correctly works

10. How to use

Note: make reference to "GENERAL WARNINGS" chapter

WARNING!

- Getting in and out of the wheelchair may be performed independently, provided you have a adequate mobility and upper body strength
- Do not stand on removable parts

DO NOT STAND ON THE FOOTRESTS WHEN GETTING IN OR OUT OF THE WHEELCHAIR, OVERTURNING RISK!!!

10.1 Get in the wheelchair

1. Position the wheelchair as close as possible to you
2. Engage wheel locks . We suggest to remove the armrest to ease the patient getting in the wheelchair
3. Lean on the frame with arms and slowly go down towards the seat

10.2 Get out of the wheelchair

1. Position the wheelchair as close as possible along side the seat to which you are transferring
2. Engage wheel locks. We suggest to remove the armrest to ease the patient getting out of the wheelchair
3. Shift body weight into seat with transfer

10.3 Stairs and height difference

**WARNING!**

The following methods must be used only from experienced patients. If you are not experienced, let you help from an assistant!

GET ON

Position the wheelchair at the curb to be crossed. Balance the wheelchair on the rear wheels, in order to lift the front wheels, enough to lean them on the obstacle. Bend your trunk forward and vigorously push the handrail. Continue pushing until the rear wheels are climbing over the obstacle.

GET OFF

To get off the stair follow the reverse procedure, positioning the rear side of the wheelchair at the curb, keep the handrails vigorously and get off the obstacle slowly; then balance the wheelchair on the rear wheels in order to complete the descent

**WARNING!**

We recommend to use this method only in presence of obstacle lower than 10cm!

10.4 Use the armrest

The armrest can be flip back, detachable or flip back and detachable

Flip back - this feature is useful when you have to get in or out of the wheelchair.

Push the release lever then turnover the armrest

10.5 Use the 60cm wheels

The rear 60cm wheels are used when self-propelled.

Grasp the handrail and wheel through the desired direction

10.6 Use the 30cm wheels

The rear 30cm wheels are used when wheel chair is driven by an assistant through the handles

10.7 Use the narrow casters (only on models with quick release wheels)

The safety wheels are used to cross narrow spaces (elevator, etc.)

To use the safety wheels remove the rear 60cm wheels

10.8 Carrying

The wheelchair has been designed to be easily carried.

Once the armrest and the footrest are removed, can be closed and easily positioned in the car boot, for example. If this can't be possible, position the wheelchair in front of the car rear seat so that it is stable and cannot slip or slide. Lock the wheelchair with the seatbelt

11. Maintenance

The **ARDEA ONE** devices by Moretti are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair use only original spare parts. Before using follow the "BEFORE USE" chapter.

Moretti **ARDEA ONE** wheelchairs require few elementary and periodic maintenance, below we list the checks to be carried out periodically

braking capacity	monthly
seat cleaning	as needed

12. Cleaning and disinfection

12.1 Cover and padding

The cover and padding are in Nylon. Clean with warm water and mild soap.

12.2 Frame and metallic parts

Clean with warm water and mild soap. Dry with a soft cloth in order to not scratch the painting.

12.3 Brakes and wheels

Clean the wheels with warm water and mild soap. Pay attention in drying the wheels, it's dangerous use the wheelchair with wet wheels.

12.4 Disinfection

if you need to disinfect the device use a common disinfectant cleaner

Note: do not use solvent and abrasive product

13. Disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the wheelchair in the appropriate disposal areas for recycle.

14. Spare parts/accessories

For spare parts only refer to MORETTI master catalogue

15. Technical features

15.1 Dimensions

DIMENSION AND WEIGHT

CP100-38	113	83	43	90	74	49	58	38	43	56	27	17,5	60	20	-	120
CP100-40								40		58						
CP100-43								43		61						
CP100-45								45		63						
CP100-46								46		63,5						
CP100-48								48		65						
CP102-38	113	83	43	90	74	49	58	38	43	56	27	17,5	60	20	-	120
CP102-40								40		58						
CP102-43								43		61						
CP102-45								45		63						
CP102-46								46		63,5						
CP102-48								48		65						
CP110-40	105	81	43	90	73	49	56	40	42	64	26	18,5	60	20	-	150
CP110-43								43		67						
CP110-45								45		69						
CP110-46								46		69,5						
CP110-48								48		71						
CP110-50								50		74						
CP200-38	113	81	42	90	71	49	59	38	41	58	29	17,5	60	20	4	150
CP200-40								40		60						
CP200-43								43		63						
CP200-45								45		65						
CP200-46								46		65,5						
CP200-48								48		68						
CP200-50	50	70														
CP300-50	118	90	48	92	76	52	59	50	41	73	32	26	60	20	-	200
CP300-55								55		78						
CP300-60								60		83						
CP620-40	100	73	41	92	80	53	47	40	37	51	26	16,5	50	18	-	110
CP620-43								43		54						
CP880-35	94	74	36	86	62	42	38	35	37	56	31	14,5	56	15	-	100

DIMENSION AND WEIGHT

CP115-40								40		53						
CP115-43								43		56						
CP115-45								45		58						
CP115-46	98	75	41	92	76	50	56	46	43	58,5	25	18,5	30	20	-	150
CP115-48								48		61						
CP115-50								50		63						
CP205-38								38		52						
CP205-40								40		54						
CP205-43								43		57						
CP205-45	104	76	42	90	71	49	59	45	41	59	25	17,5	30	20	4	150
CP205-46								46		59,5						
CP205-48								48		62						
CP205-50								50		64						
CP310-50	104	77	48	92	70	52	59	50	39	63	30	26	32	20	-	200
CP310-55								55		68						
CP500-40								40		55						
CP500-43								43		58						
CP500-45	94	70	42	92	72	48	54	45	43	60	26	17,5	30	20	-	120
CP500-46								46		60,5						
CP500-48								48		63						
CP520-40								40		55						
CP520-43								43		58						
CP520-45	99	73	42	93	72	48	52	45	42	60	26	17,5	30	20	-	150
CP520-46								46		60,5						
CP520-48								48		63						
CP625-40	88	61	41	92	80	53	47	40	37	51	26	16,5	30	18	-	110
CP625-43								43		54						

*The dimensions may change $\pm 0,5$ cm

15.2 Technical specifications

- **Steel frame**
- **Max weight for CP100, CP102, CP500: 120 kg - for CP110, CP115, CP200, CP205, CP520: 150 kg - for CP300, CP310: 200 kg - for CP620, CP625: 110 kg - for CP880-35: 100 kg**
- **Front wheels: 20cm PU**
- **Front wheels (kids model): 15cm PU**
- **Rear small wheels: 30cm PU**
- **Rear big wheels: 60cm PU**
- **Rear big wheels (kids model): 56cm**
- **Fabric: Nylon**
- **Resistance to ignition:** the materials used for cover and padding have been treated with methods specified in Directive EN1021-1/2
- **Reference standards:** EN12183
- **Environmental temperature:** -20°C/+ 40°C
- **Storage:** store in a cool place, away from heat sources, such as sunlight (suggested temperature between +10°C and +40°C)

16. Warranty

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to mistaken installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

16.1 Repairing

Warranty repair: If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

Repair not cover by warranty: A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

Non defective device: The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

16.2 Spare parts

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

16.3 Exempt clauses

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

MORETTI®

WARRANTY CERTIFICATE

ARDEA®

Product _____

Purchasing date _____

Authorized dealer _____

Street _____ **Place** _____

Sold to _____

Street _____ **Place** _____

MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

Notes

This image shows a full page of blank, lined paper. It features approximately 20 evenly spaced horizontal grey lines across its entire width, providing a guide for handwriting or typing. The paper itself is a clean, off-white color.

Via Bruxelles, 3 - Melegnano
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com

THE INTERNATIONAL CERTIFICATION NETWORK

CERTIFICATE

ICMQ S.p.A. has issued an IQNet recognized certificate that the organization:

MORETTI S.p.A.

Via Gandhi, 9 – 25030 Erbusco BS

has implemented and maintains a

Quality Management System

for the following scope:

**design and construction of buildings; construction with glulam structural elements;
transport and assembling of concrete precast structural elements, waterproofing and
sealing of precast structures**

which fulfils the requirements of the following standard:

UNI EN ISO 9001:2015

Issued on: **24/04/2018**

First issued on: **30/03/1993**

Expires on: **30/04/2021**

This attestation is directly linked to the IQNet Partner's original certificate and shall not be used as a stand-alone document

Registration Number: IT- 1273

Alex Stoichitoiu
President of IQNET

Ing. Claudio Provetti
President of CISQ

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This certificate is valid
for the following scope:

Design, production and management of the production, distribution and placing on the market of durable medical equipment in the rehabilitation and home healthcare and devices for patient positioning and transport. Marketing and distribution of general non-active, non-implantable medical devices and Devices for wound care (sterile and non-sterile), Monitoring devices, device for home care, active rehabilitation devices

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Nicola Privato
Management Representative

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