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EN

**BASIC PRESCRIBING
INFORMATION FOR
Rubber-textile**

Purpose. Rubber-textile is intended for sanitary and hygienic
purposes as underlay impermeable material and is reusable.

Rubber-textile corresponds to GOST 3251-91 and is considered to be
operable.

Depending on basic fabric the drawsheet could be of two types:

A – on the basis of cotton fabrics;

B – on the basis of polyester and rayon fabric.

Scope. Sanitary, hygiene for medical institutions and for individual
use.

Bacterial purification. Bacterial purification of rubber-textile in
medical institutions is performed in 1% by mass of chloramine
solution by methods and procedures in accordance with actual
normative documents. Domiciliary bacterial purification of the
product after use is in thorough washing of drawsheet by hot water
with soap or soap powder.

Implementation. The rubber-textile is to be washed in warm soapy
water and rinsed with flowing water before use. Then it is to be dried
at room temperature. The rubber-textile is to be purified after use in
the way above mentioned and dried. It is not recommended to use a
heat radiating appliances or direct sun beams for drying the
rubber-textile.

Storage. Rubber-textile is stored at the temperature of from 0 °C to
25 °C and relative humidity of not more than 80% at the distance of
not less than 1 m from heating appliances protected from influence of
direct sun rays. During storage the drawsheet should not be subjected
to influence of oils, petrol and other substances destroying rubber.

Manufacturer's warranties/

Manufacturer warrants the quality of drawsheet if the storage and
operation conditions are adhered to.

Storage warranty period of rubber-textile type A is 24 months and a
shelf time of rubber-textile type B is 26 months from the day of
production.

Guaranteed service life of the rubber-textile type A is 1 month in
medical institutions and is 6 months at individual use.

Guaranteed service life of the rubber-textile type B is 2 month in
medical institutions and is 8 months at individual use from the date of
receipt by consumer.

Authorised Representative's in EU:

OU Scanvic Chem, 10119, Roosikrantsi 9 - 22, Tallinn,
Estonia, Tel: +37253451100, +3725073615,
E-mail: du@scanvicchem.ee, au@scanvicchem.ee