



Product Specification of:

Medical Oxygen Sensor

Type: M-12 Part Number: 410017

Nuova ID: E-17/A2

RoHS compliant and SVHC free

Meets the applicable requirements of ISO 80601-2-55

Comes with CE marking. further regulatory registrations available upon request

Produced under EN ISO 13485 Quality Management System

DOCUMENT PURPOSE

The purpose of this document is to present the performance specification and key features of the sensor.

This document should be used in conjunction with the Operating Manual of the instrument and the Product Safety Data Sheet of the sensor.

KEY FEATURES

Long-life galvanic gas sensor with high signal stability at low cross sensitivity to anesthesia gases combined with superior linearity over the entire measurement range.



This data sheet is subject to change without prior notice Specification_M-12_Rev_2

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany





TECHNICAL SPECIFICATIONS

MEASUREMENT

Operating Principle:	Partial pressure electrochemical cell				
Measurement Range:	0 100 Vol.%O ₂				
Initial Output Signal:	8.0 12.5 mV	at dry ambient air			
Response Time t90:	< 15 s				
Signal Drift (long-term):	≤ ± 1 Vol.%O₂/month	at ambient air			
Linearity Error:	≤ ± 3 %	at 100 Vol.%O ₂ applied for 5 min			
Zero Signal Offset:	≤ 0.3 Vol.%O ₂	at 100 Vol.%N ₂ applied for 5 min			
Repeatability Error:	$\leq \pm 1 \text{ Vol.%O}_2$	at 100 Vol.%O2 applied for 5 min			
Influence of Gas Humidity:	- 0.03 % of signal per %rH				
Signale Temperature Compensation	NTC on sensor PCB				
Cross-Sensitivity:	fullfills DIN EN ISO 80601-2-55				
ELECTRICAL					
Electrical Connector	3-Pin Molex				
Recommended Load Resistor:	≥ 10 kOhm				
MECHANICAL					
Weight:	≤ 24 g				
Material in Contact with Media:	PA, PPS, PTFE, Stainless Steel, NBR				
Gas Connector:	fits for M16x1 DIN 13 or 5/8-24 UNEF				
ENVIRONMENTAL					
Operating Temperature Range:	10 40 °C				
Ambient Pressure Range:	700 1250 hPa				
Ambient Humidity Range:	up to 100 %rH	non-condensing			
LIFETIME					
Expected Operating Life:	3 years	at ambient air, depending on applicatio			
	- /	at ambient air, depending on applicatio			

Important Note: All characteristics are based on conditions at 25 °C, 50 %rH, 1013 hPa and a gas flow of > 2.5 L_s/min. For sensor performance data under other conditions, contact ITG.

This data sheet is subject to change without prior notice

Specification_M-12_Rev_2

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany





Ambient Temperature Range:	15 25 °C	recommended
	-20 50 °C	maximum (≤ 10 h)
Ambient Pressure Range:	700 1250 hPa	
Ambient Humidity Range:	50 100 %rH	recommended non-condensing
	0 30 %rH	maximum one week
ELATED PRODUCTS		
Product	Part Number	Other Specifics
D ₂ Sensor M-12	41 00 17	-
D ₂ Sensor M-12	47 01 17	double pack
D ₂ Sensor M-12T	47 00 29	translucent housing
O ₂ Sensor M-12A	41 02 17	for Atom device

Cleaning and Disinfection

The sensor outer housing can be cleaned with a dry wipe. Do not wipe the sensor's gas entrance side. Do not use any chemical disinfectant or sanitizer on the sensor.

Poisoning

ITG sensors are designed to operate in a wide range of environments. For optimal sensor lifetime and performance it is important that exposure to high concentrations of solvent vapors is avoided during storage, installation into instruments and operation of the sensor. Do not use adhesives directly on or near the sensor as the solvents may cause stress corrosion on the plastic parts.

Intended Use

The electrochemical oxygen sensors for use in medical technology are used as accessories with a limited shelf life exclusively in conjunction with other medical products (ventilators, incubators, anesthesia machines, portable oxygen monitors, oxygen therapy devices and gas mixing units) so that they can fulfill the manufacturer's intended purpose.

The oxygen sensors measure the oxygen partial pressure in gas mixtures. Commissioning and use is carried out exclusively by expert clinic or service personnel. Direct contact with these items occurs during installation or maintenance by trained personnel.

Stabilisation Time

When installing a new sensor refer to the instrument manual for stabilization time before calibration. If not specified otherwise wait at least 15 minutes to ensure the sensor has stabilized in the instrument.

Calibration Interval

ITG sensors are designed to have minimal signal drift over their functional lifetime. For optimal performance and maximum measurement accuracy however they should be calibrated before each use.

This data sheet is subject to change without prior notice

Specification_M-12_Rev_2

Page 3 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 .: Fax: +49 (0)3841 220052 2 .: E-Mail: info@itg-wismar.de





If the Sensor is dropped

If a sensor is dropped, please check for visible mechanical damage of the sensor or if the sensor is leaking electrolyte. If this is the case, do not try to install the sensor into the device. Take safety precautions and immediately dispose the cell.

If the sensor shows no visible damage it should be placed in quarantine for at least 24 hours. Afterwards a follow-up check made by a two point calibration should be done.

Installation in Device

A gas tight sealing is ensured when the sensor is screwed in hand-tight. Don't use any mechanical tools to install the sensor. Using excessive force may damage the sensor.

Optimal mounting position of the sensor is when the gas sensing area faces downwards. A horizontal position is acceptable. It is not recommended to use the sensor with the gas sensing area facing upward.

Connection should be made via recommended electrical and mechanical connectors only. The specified load resistance must be taken into account. Soldering the sensor will damage it and void the warranty. Please contact ITG for further information.

Since temperature has an influence on the output signal it is not advisable to place any heat or vibration sources (i.e. electric- pumps or valves, coolers, etc.) in close proximity to the cell.

Avoid proximity of the cell to any EMC radiating units as those emit frequencies which might interfere with the sensor's electrical connection and connecting cable.

RFI/EMI Susceptibility

ITG sensors contain metal parts and might be susceptible to RFI or EMI. Before use in MRI environments please contact ITG for further information.

Disposal

At the end of the sensors lifetime the sensor should not be disposed of in normal public waste as it may contain hazardous materials and caustic electrolyte (for more information refer to the PSDS). Please contact your local authorities for environmental legislation to relevant local waste disposal.

This data sheet is subject to change without prior notice Specification_M-12_Rev_2

Page 4 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany






TECHNICAL DRWAING


Sensor ITG		Color			Electrical pin assignment		
3611501	Part number	Сар	Housing	Threaded ring	Pin 1	Pin 2	Pin 3
M-12	410017	white	white	white	minus (-)	minus (-)	plus (+)



Bottom view

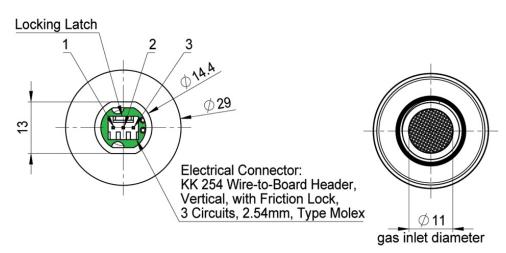

Front view

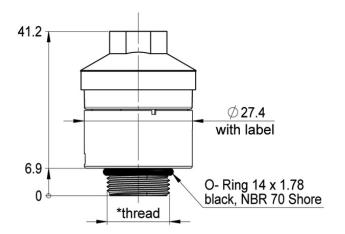

[* thread is compatible with DIN 13 M16x1 and UNEF 5/8-24] Dimension unit: mm Dimension tolerances: linear ± 0.5 mm , diameter ± 0.3 mm

This data sheet is subject to change without prior notice

Specification_M-12_Rev_2

Page 5 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 .: Fax: +49 (0)3841 220052 2 .: E-Mail: info@itg-wismar.de