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REF: MSEQSI1
REF: MSEQSI4
REF: MSEQSI12

Includes:
MSEQSCH1, 1 vial - MSEQSCH4, 4 vials - MSEQSCH12, 12 vials

Human lyophilized SERUM. Reconstitute with distilled water according to the quantity specified on the vial label.

The serum has been tested for Human Immunodeficiency Virus (HIV) antibody, Hepatitis B Surface Antigen (HbsAg), and Hepatitis C Virus (HCV) antibody and found to be NON-REACTIVE.
However this material should be handled carefully.

Storage and stability:

Store kit at 2-8°C The reconstituted serum stability is 7 days at 2-8°C and 3 weeks at -20°C.

PROCEDURE:

- Carefully open the vial and reconstitute the serum using distilled water, according to the amount specified on the vial label.
- Ensure the distilled water used is of good quality.
- Let the solution sit for 30 minutes, protected from light and at room temperature. Then gently shake until completely dissolved.
- Identify the analytes that exceed the maximum detection threshold during the analysis.
- Take a known volume of the sample (e.g., 1 mL).
- Dilute it with saline solution in a 1:4 ratio (e.g., add 3 mL of saline solution to 1 mL of sample).
- Mix thoroughly to obtain a homogeneous solution.
- Repeat the analysis on the diluted sample.
- Multiply the obtained result by the dilution factor (4) to calculate the correct value of the analyte in the original sample.

Calculation Example:

If the detected value for the analyte in the diluted sample is 50 mg/L, the correct value in the original sample will be:
 $50 \text{ mg/L} \times 4 = 200 \text{ mg/L}$

- Enter the obtained values on the website qs-veg.it.

Notes:

- Ensure the saline solution used is sterile and of analytical quality.
- Always follow good laboratory practices when handling samples.
- PSA remains stable for 2 days if the serum is stored at a temperature of 2–8°C.

**ATTENTION PLEASE WE ASK FOR A PARTICULAR CARE ENTERING ON THE WEB SITE
THE MONTH LOT AND OF ASSAY**

**N.B. ATTENTION THE BOTTLES CONTAIN MATERIAL OF HUMAN ORIGIN. IT IS RECOMMENDED TO IMPLEMENT ALL THE
RECOMMENDED PROCEDURES FOR THE PURPOSE OF PREVENTION AND DISPOSAL**

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