

## Instructions for Use

**Name:** Plaster of Paris bandage

**Intend Use:** The Plaster of Paris bandage is used for external fixation of fractures or soft tissue injuries.

**Composition:** The Plaster of Paris bandage is made of high-quality white natural gypsum, non-toxic gum material and high-quality, clean gauze.

**Operation:** It should be operated by a professional, otherwise it may easily cause the patient to be injured again.



### Warning:

1. Use with caution if allergic to gypsum, if allergic symptoms occur, consult a doctor immediately.
2. Do not use when the package is leaking or bandage is hardened.
3. The product does not come into direct contact with the wound.
4. Single use.
5. Open one bag at one time and use the cast immediately. Don't open multi-bags at the same time, so that to avoid weakening the strength.

### Instruction for use:

1. **Dipping:**
  - a. For product with modeling time of 4 to 7 minutes and for product with modeling time of 8 to 12 minutes: Use warm water 20°C - 40°C.
  - b. For bandages with modeling time up to 3 minutes: Use warm water 20°C - 30°C.

Hold one end of the plastic inner core with your fingers, gently dip the bandage at an angle of 45 in the water for no less than 15 seconds until there was no air bubble in the bandage
2. **Squeezing out the water:** Carefully take the bandage out of the water and move it into another vessel., gently squeeze out the excess water from both ends to the center with both hands. Do not twist bandage or squeeze excessively, to avoid losing too much gypsum powder.
3. **Molding:** After squeezing out the water, The bandage should be used immediately, so as not to lose its plasticity. Generally use the binding methods of overlapping circles, do not make bandage too tightly. Bind 6-8 layers in general place and 8-10 layers in force parts.
4. **Smoothing and/or enswathing with smoothing:** in order to remove the bubble in the bandage and make the bandage adhere evenly layer to layer, and modify the appearance to achieve smoothly and uniformly. Don't touch it when the plaster starts to condense.
5. **Shaping:** Shape the wrapped cast according to the need. The solidify time depends on the type of the product and is as indicated on the label:
  - a. modeling time up to 3 minutes
  - b. modeling time minimum 4 minutes and maximum 7 minutes
  - c. modeling time minimum 8 minutes and maximum 12 minutes

20 minutes later the cast reaches the enough strength.
6. **Removal of the product:** Use the electric cast saw to cut and remove it.

### Storage:

1. Be careful not to break the package.
2. Stored at Room Temperature.
3. Stored in a dry and cool place, away from direct sunlight.

**Transportation:** Heavy pressure, direct sunlight, rain or snow should be prevented.

**Package:** Put it in OPP bag, 1 roll/bag; then put it in a carton, 60-240 rolls/carton.

**Shelf life:** 3 years.


安吉县阳光医药用品有限责任公司  
Anji Sunlight Medical Products Co., Ltd.

Symbols meaning:

Symbol	Introductions	Symbol	Introductions
	Batch Code		Do not reuse" are "single use, "Use only once
	Don't use when packing damaged		non-sterile
	Warnings and Precautions		Manufacture Date
	medical device		Consult instructions for use
	Use until year & month (Expiration date)		Keep dry
	Keep away from sunlight		Name and Address of European Union Representative
	Manufacturer Name Address		CE Symbol

Manufacturer Information:

	Company: <b>Anji Sunlight Medical Products Co., Ltd.</b> Address: No.499, North Raocheng Road, Dipu Street, Anji, Zhejiang Province, PEOPLE'S REPUBLIC OF CHINA
	Company: SUNGO Europe B.V. Address: Olympisch Stadion 24, 1076DE Amsterdam

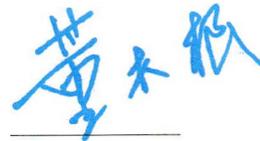

安吉县阳光医药用品有限责任公司  
Anji Sunlight Medical Products Co.,Ltd.