

## 8-7-4 Swivel and Up-Down Handle Replacement Procedure

### 8-7-4-1 Tools

Flat and Phillips screwdrivers as needed.

### 8-7-4-2 Time Required

60 min

### 8-7-4-3 Preparation

Shut down the ultrasound system as described in [Power Shut Down on page 4 - 9](#).

### 8-7-4-4 Swivel and Up-Down Handle Removal Procedure

- 1) Remove the Operating Panel as described in the [Operating Panel Keyboard Assembly Removal Procedure on page 8 - 43](#).
- 2) Remove the Probe Shelf as described in the [Probe Shelf Removal Procedure on page 8 - 157](#).
- 3.) Unscrew the handle shaft support. Rotate it upwards 90 degrees to remove the support by sliding it to the left or right side.

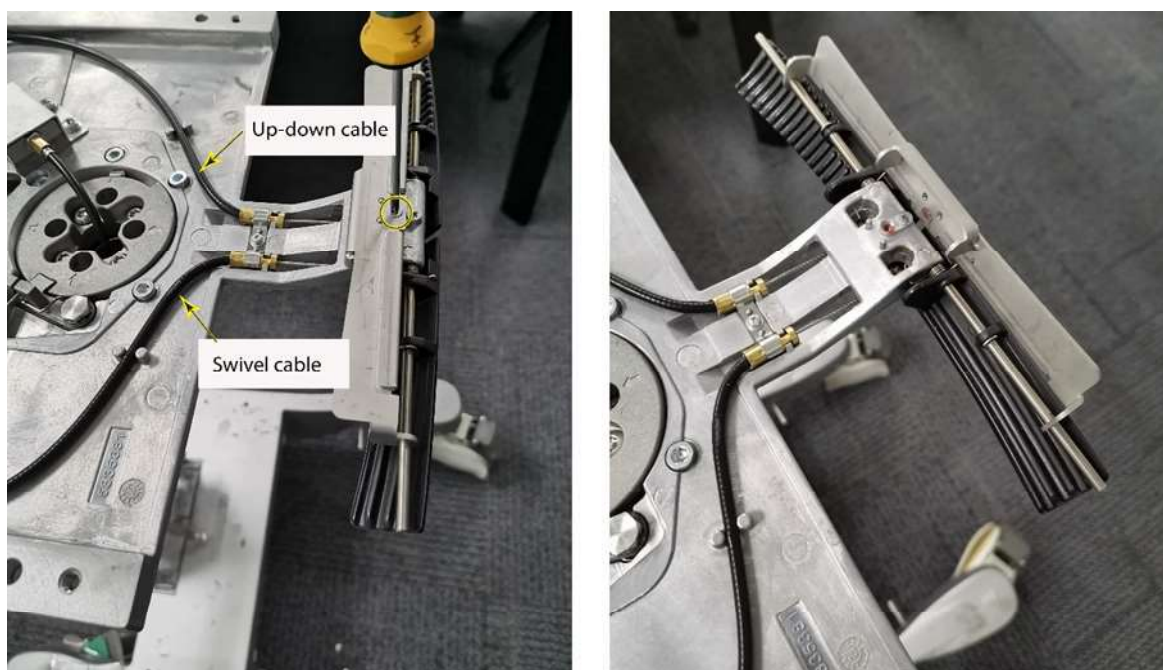


Figure 8-128 Protective Metal Plate Rotated 90 Degrees