OCUBLU-TRY TRYPAN BLUE®	TR
--------------------------------	-----------

Açıklama:

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Description:

Ocublu-Try Trypan Blue is a sterile tissue dye for anterior segment surgery. Ocublu-Try Trypan Blue has been developed as an auxiliary device for performing capsulorhexis during cataract surgery. Ocublu-Try Trypan Blue is used to selectively stain the anterior lens capsule, which helps with facilitating capsulorhexis and minimizing the risk of tearing during cataract surgery.

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Composition

1 mL Ocublu-Try Trypan Blue Contains:

0,2 mg of Trypan Blue	0,4 mg of Trypan Blue	0,6 mg of Trypan Blue
Sodium chloride	Sodium chloride	Sodium chloride
Disodium hydrogenphosphate	Disodium hydrogenphosphate	Disodium hydrogenphosphate
Sodium dihydrogenphosphate	Sodium dihydrogenphosphate	Sodium dihydrogenphosphate
Water for injection	Water for injection	Water for injection

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Instructions For Use :

This product is designed for use by ophthalmologists only. Ocublu-Try Trypan Blue is injected into the anterior chamber using a tuberculin syringe between the air bubble and the lens capsule. Ocublu-Try Trypan Blue is achieved as soon as the dye comes into contact with the anterior lens capsule. The anterior chamber is then flushed to remove any residual dye. A viscoelastic solution is then injected. The cataract operation is then performed in the usual manner.

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Indications

Ocublu-Try Trypan Blue is indicated for intraocular use for staining the capsular tissues during ophthalmic surgery, especially involving cataract extraction using phacoemulsifications. Ocublu-Try Trypan Blue permits clear visualization of the anterior capsule in contrast to non-stained lens cortex. It is intended for intraocular use only.

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Contraindications

No contraindications are known when used in accordance with recommendations.

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Presentation and Sterilisation

Ocublu-Try Trypan Blue is available in vial or syringe containing 1.0 ml of sterile solution. Each box contains 10 units. It is sterilized using steam.

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Storage Conditions

Ocublu-Try Trypan Blue must be stored at 10 - 30°C, protected from light.

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Warnings and Precautions for use

Product is only intended for single intraocular application only, it should not be administered by intravenous and intramuscular routes. Do not resterilize. Do not use this product after the expiry date. Any sterile pack that has been opened and/or damaged must not be used. The medical device must be removed from its primary package aseptically.

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Adverse Reactions

Unexpected adverse reactions and complications related to Ocublu-Try Trypan Blue must be reported to ATAKAN DEDE – MİRAY MEDİKAL.

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