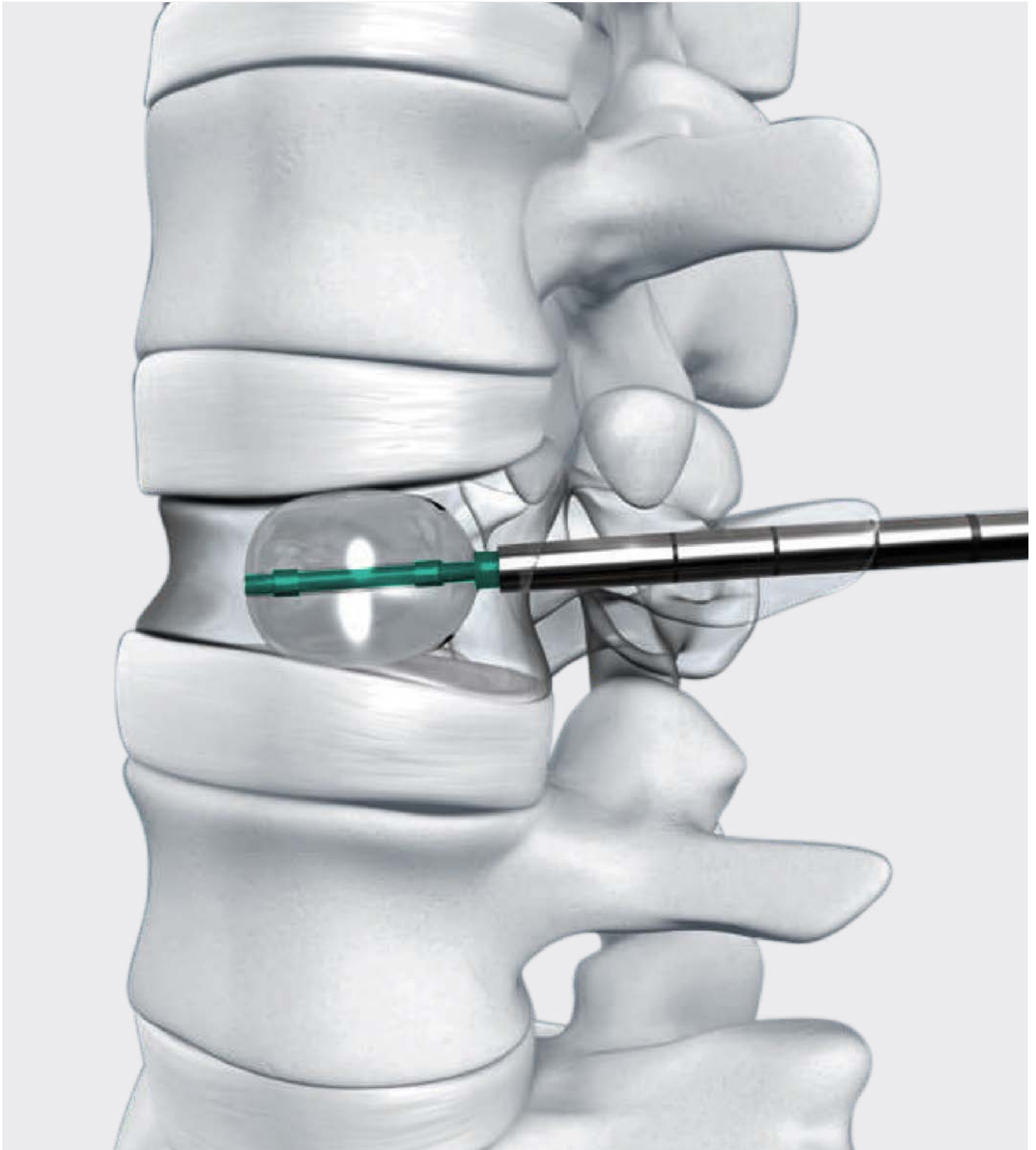
ADITUS KYPHOPLASTIE KIT

ADITUS KYPHOPLASTY KIT

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
AVAILABLE SIZES 8G / 11G / 13G

KYPHOPLASTIE 13G

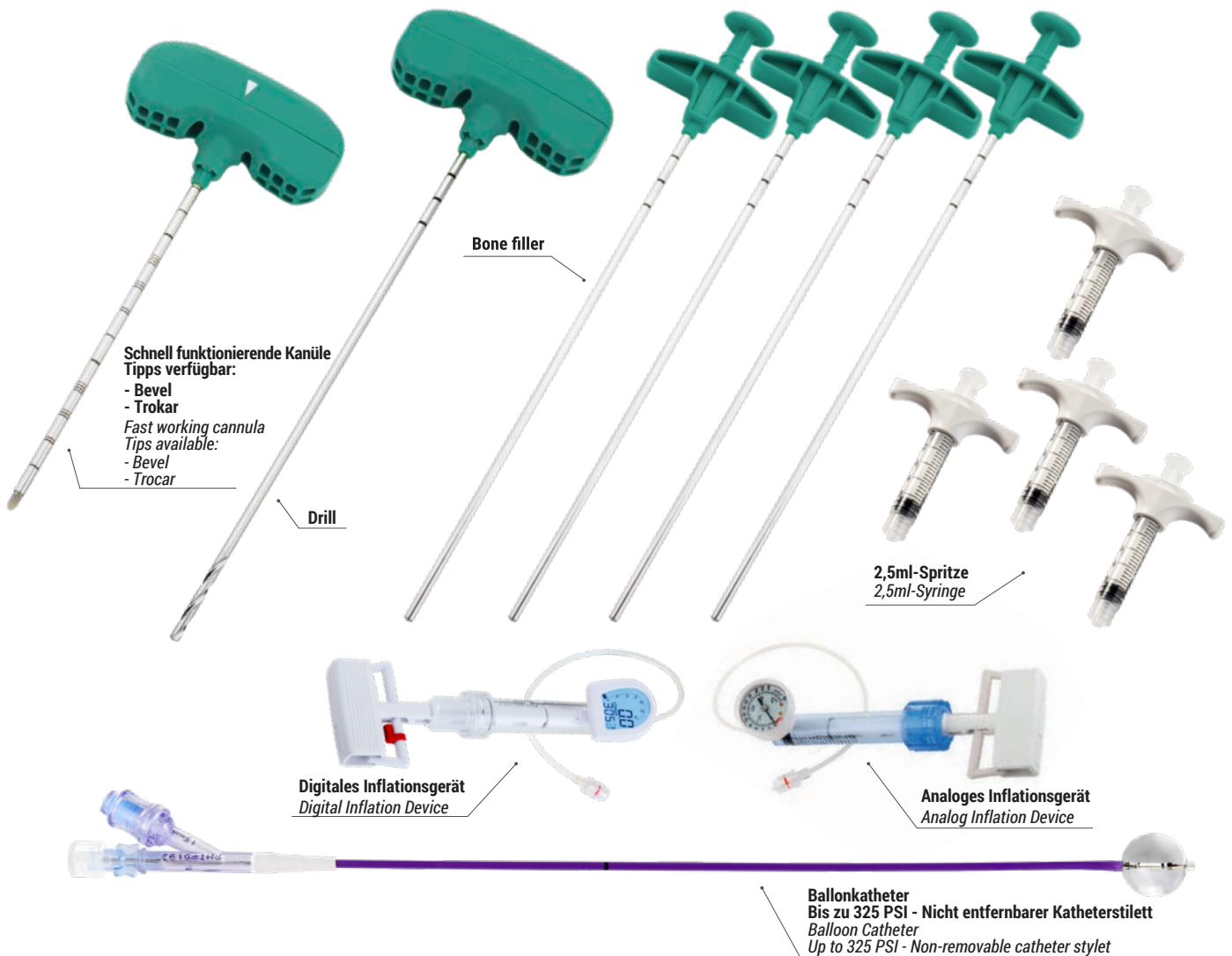KYPHOPLASTY 13G

KYPHOPLASTY 13G BENEFITS

- Ultimate minimally invasive solution
- A single treatment solution from the cervical to the lumbar vertebrae
- 13G balloon catheter inflation up to 325 PSI
- Reduction of back pain
- Fast recovery after treatment
- Cement injection cannulas (bone fillers) included in the kit are compatible with low and medium viscosity cement. With high viscosity cement, Easynject is recommended.

Kyphoplasty System is a minimally invasive system to treat fractures, restore vertebral body height and reduce back pain. A vertebral compression fracture (VCF) occurs when the vertebral body fractures and collapses due to osteoporotic conditions, tumors (eg. myeloma) or trauma.

After inserting and inflating the balloon catheter into the cavity, the initial anatomical vertebral body condition can be restored.

The balloon catheter is then deflated and the cavity created is now filled with bone cement avoiding any leakage.

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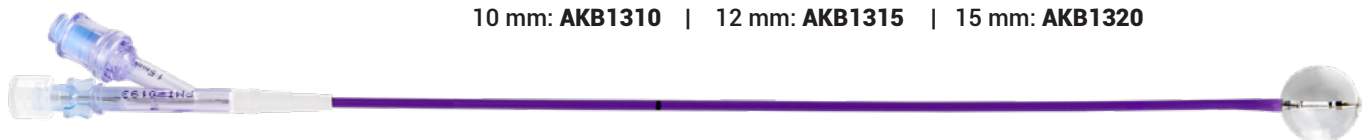
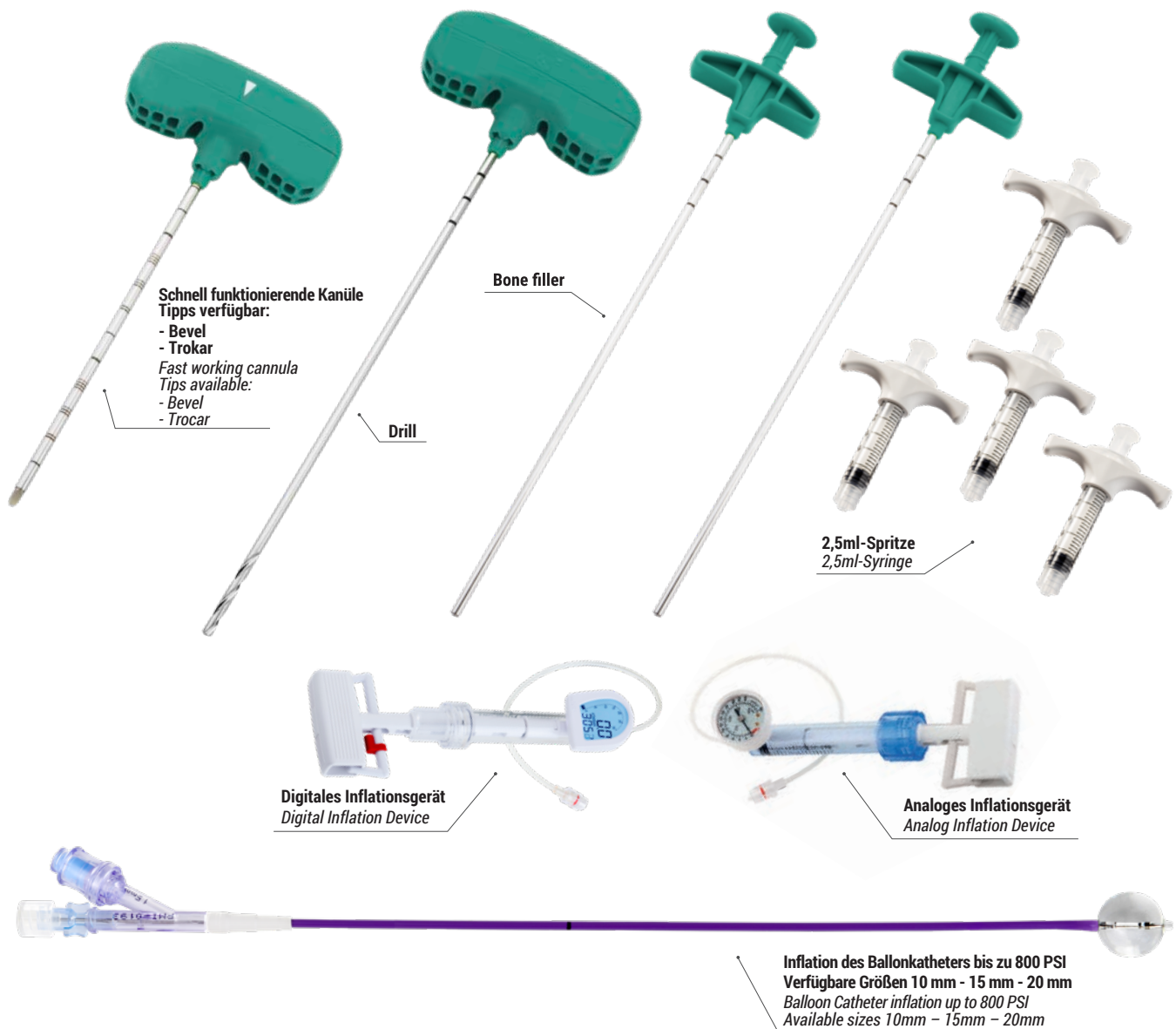
KYPHOPLASTY 11G

BENEFITS COMPARED TO 8G KIT:

- Reduced profile of cannula and balloon catheter
- Fast approach: direct percutaneous vertebral access and reduced number of surgical steps
- Less invasive and less traumatic treatment
- From lower thoracic to lumbar vertebrae

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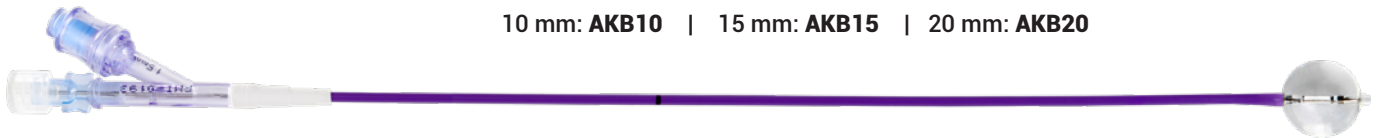
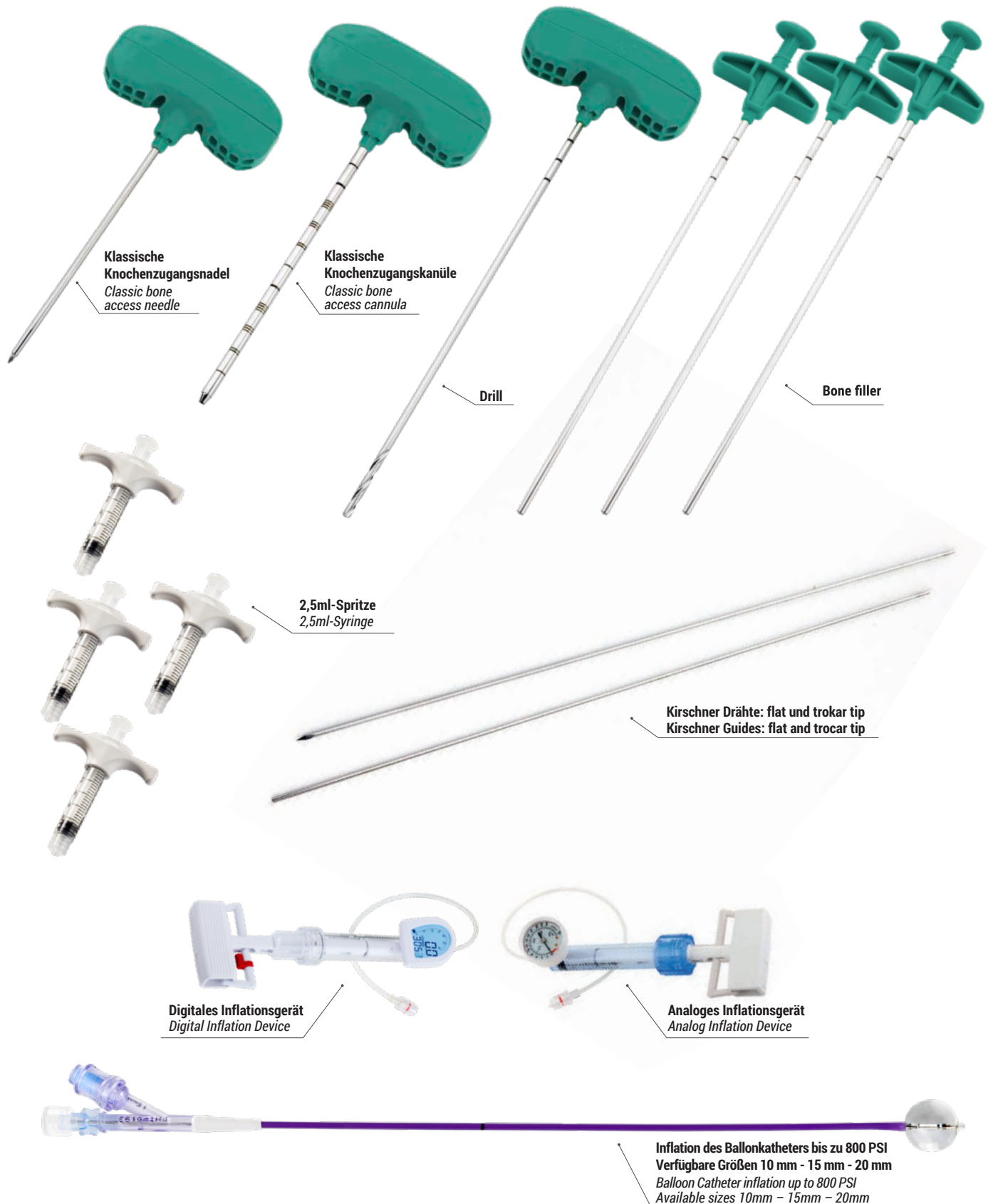
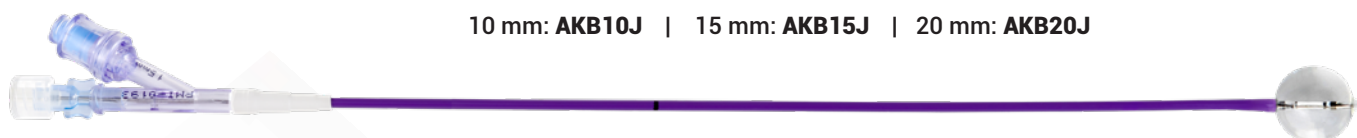
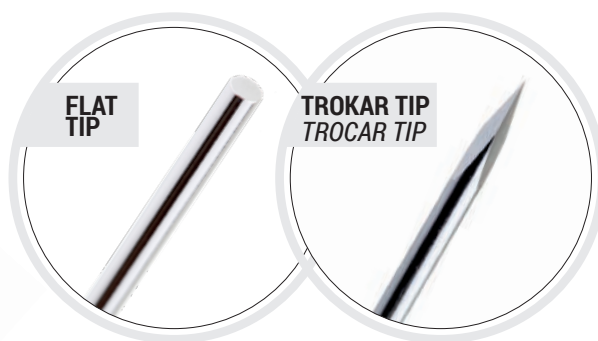
ADITUS VERTEBROPLASTIE

ADITUS VERTEBROPLASTY

VERTEBROPLASTY NEEDLE WITH PLASTIC HANDLE

VERTEBROPLASTIC is used in Vertebroplasty procedures to inject cement into the vertebral body. Vertebroplasty is a very well established technique for treating back pain caused by severe osteoporosis or tumors with consequent loss of height or fracture of the vertebral body. Coaxial biopsy can also be performed.

FEATURES:

1. ERGONOMIC GRIP WITH LOCKING SYSTEM
2. THE ENHANCED DESIGN OF THE PLASTIC HANDLE ALLOWS EASY HANDLING OF THE NEEDLE
3. TIP DIRECTION INDICATOR
4. AISI 304 STAINLESS STEEL CANNULA AND STYLET

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CODE: A-MIX

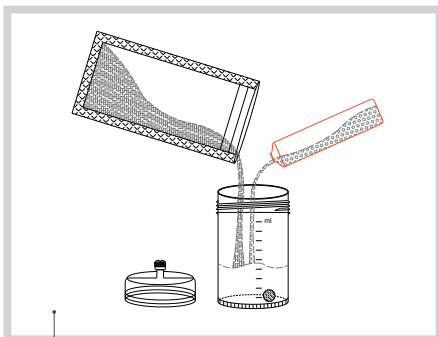
CEMENT MIXING SYSTEM

ADITUS is a disposable, single-use device, indicated for optimal and homogeneous mixing of PMMA. Thanks to its small size, ADITUS can be refrigerated to extend PMMA working time. ADITUS is a completely closed system with zero emission during PMMA handling, beneficial for both patient and OR staff. ADITUS can be used both in vertebroplasty and kyphoplasty treatments.

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
SURGICAL TECHNIQUE - CHIRURGISCHE TECHNIK


1. Mix PMMA components (polymer and monomer) into the ADITUS.

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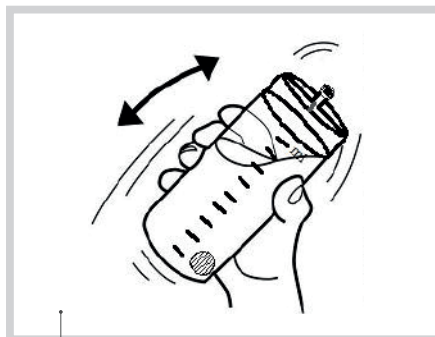2. Shake the ADITUS to reach ideal viscosity according to PMMA cement IFUs.

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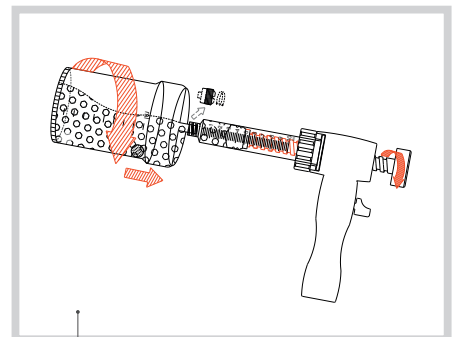
3. Connect the ADITUS to the suitable injection system and fill it up.

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ADITUS KNOCHENZEMENT DELIVERY SYSTEM

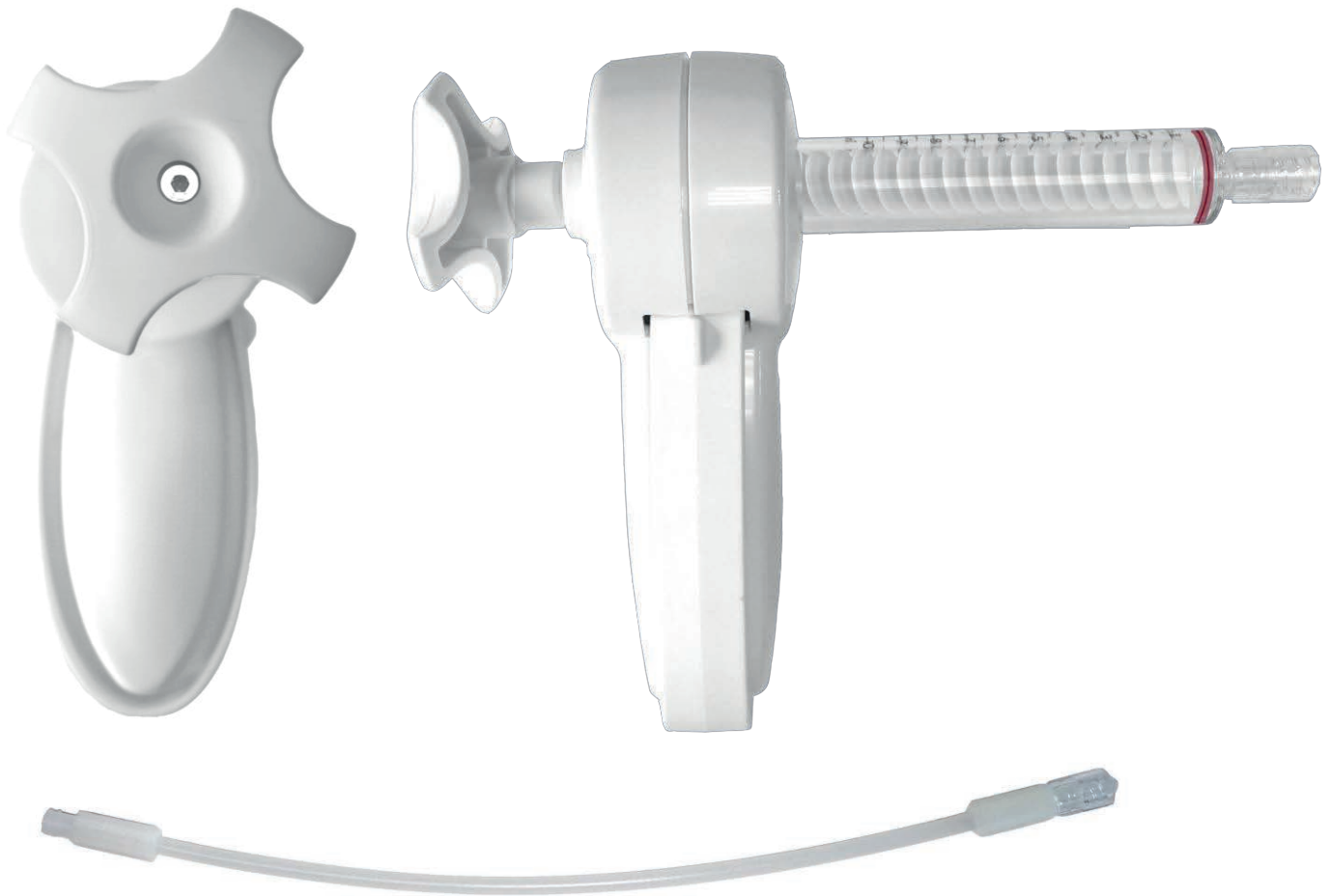ADITUS BONE CEMENT DELIVERY SYSTEM

CODE: A-VINS

CEMENT DELIVERY SYSTEM

- Compatible with low, medium and high viscosity bone cement.
- Ergonomic handle.
- 10ml single use syringe included.
- Millimetric advancement of the plunger, that allows a very accurate and controlled injection of cement.
- Allows fast and direct injection, simply pressing the button on the handle and pushing the plunger.
- Allows rapidly stopping the cement flow by turning 180° counterclockwise the plunger.

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