

4

Registering and Changing Radiography Programs

This chapter describes how to register and change radiography programs.

4.1 Registered Contents of Radiography Programs



Several examples of radiography programs are factory-installed.
 Before using the equipment, set appropriate radiography programs according to the applications.



When setting radiography programs for pediatric patients, include the name such as "child" or "infant" in the programs or steps.


The contents registered in radiography programs are shown below.

Tab

- Registered name

Radiography program

- Name
- Body thickness
- Slice angle and slice speed data
 Only when a tomography unit can transfer slice angle and slice speed data.

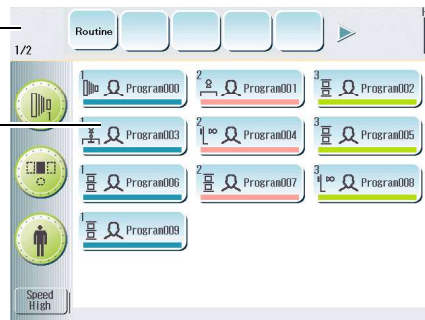

Radiography Program Screen


Radiography steps

- Name
- Technique
- Focus
- kV, mA or mAs, sec
- AEC (optional)
 - AEC ON/OFF
 - Density
 - Photo pickup field
 - Speed
- Comment




Radiography Step Screen

4.1 Registered Contents of Radiography Programs

 The maximum number of steps and registered number of character in name or comment for on radiography steps can be selected on site installation.

Maximum number of registerable radiography steps	Number of character for name on radiography steps	Number of character for comment on radiography steps
400	14	14
600	14	8
600	12	10
800	10	6
800	8	8

 **Reference** When using the ceiling tube support CH-200 unit's auto positioning option, refer to [4.3.3 "Ending Radiography Program Registration"](#)

4