

DOUBLE BLOOD BAG 450ML



For collection of 450ml blood.



600 ml
CPDA-1 63 ml
450 ml

CPDA-1 63ml

Each 100ml CPDA-1 contains:
Citric Acid (anhydrous) 0.299g.
Sodium Citrate (dihydrate) 2.63g.
Monobasic Sodium Phosphate (monohydrate) 0.222g.
Dextrose (monohydrate) 3.19g.
Adenine 0.0275g.
Water for injection to 100ml.

Needle :16 gauge .

Do not vent.

For single use only.

Do not use if there is any visible evidence of deterioration.

Blood bag in unopened overwraps should be used within 15days' from opening the aluminium foil pouch and prior to the expiry date.

Sterile non -toxic non -pyrogenic fluid path Steam sterilization .

Rx ONLY



2021.12



2024.11

Donation No.

Collection Date Expiry Date

ABO /RhD Group

Centre Identity

JIAXING TIANHE PHARMACEUTICAL CO., LTD.
Zhongfa Foreign Trade Industrial Zone, Fengqiao
Town, Nanhu District, 314008, Jiaxing City,
Zhejiang Province, China.

EC REP OBELIS S.A.
Bd Général Wabis, 53
1030 Brussels, Belgium

REF ODC-450SS



1TH02DA03Z

LOT 2112517




0002112517

TRANSFER BAG



Do not vent.
For single use only.
Do not use if there is any visible evidence of deterioration.
Sterile non -toxic non -pyrogenic fluid path Steam sterilization .

Rx ONLY

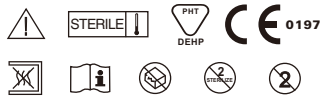

2021.12



2024.11

Donation No.

Collection Date Expiry Date

ABO /RhD Group

Centre Identity



JIAXING TIANHE PHARMACEUTICAL CO., LTD.
Zhongta Foreign Trade Industrial Zone, Fengqiao
Town, Nanhui District, 314008, Jiaxing City,
Zhejiang Province, China



OBELIS S.A
80 General Waha, 53
1030 Brussels, Belgium

REF ODC-450SS



ZTH02DA03Z

LOT 2112517




0002112517



CPDA-1 DOUBLE BLOOD BAG 450ML

NEEDLE:16G

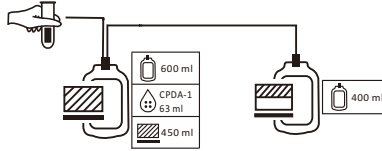

Each 100ml of CPDA-1 contains:

Citric Acid (anhydrous)	0.299g.	Sodium Citrate (dihydrate)	2.63g.
Monobasic Sodium Phosphate (monohydrate)	0.222g.	Dextrose (monohydrate)	3.19g.
Adenine	0.0275g.	Water for injection to 100ml.	

Sterile, non-toxic, non-pyrogenic fluid path. Steam sterilization.

CAUTIONS :

1. Do not use if there is any visible sign of deterioration.
2. Do not use blood bag more than 15 days after opening the aluminum foil pack.

STORAGE :

1. Unused pack may be stored at room temperature, relative humidity less than 90%.
2. Store the blood components at the specified temperature.
2. Avoid excessive heat and direct sunlight.



4 UNITS

2021.12

2024.11

REF ODC-450SS



1TH02DA03Z

JIAXING TIANHE PHARMACEUTICAL CO., LTD.
Zhongfa Foreign Trade Industrial Zone, Fengqiao
Town, Nanhu District, 314008, Jiaxing City,
Zhejiang Province, China

EC REP **OBELIS S.A**
Bd. Général Wahis, 53
1030 Brussels, Belgium

LOT 2112517



0002112517