

# KONIX® Sterile First Aid **BURN GEL** and **DRESSING**

KONIX® Sterile First Aid Burn Gel & Dressing are first aid products designed for first aid in all kinds of burn injuries (heat burns, sunburns, electrical burns, scalds, cuts, abrasions, etc.), which stops burning feeling and prevents further burns from progressing on the skin.

- ✓ Helps to relieve the sensitivity of the burned area shortly after the application.
- ✓ It cools and moisturizes the burnt area.
- ✓ It prevents the deepening of the burn by stopping the movement of heat to the lower layers of the skin.
- ✓ It prevents the infection caused by the external environment by covering the burn completely. It starts the healing phase of the tissue.
- ✓ Protects the burnt area against abrasion, friction and drying.



## COOLS & SOOTHES

the Minor Burns, Scalds & Sunburn  
Relieves and Moistens Sensitivity  
Helps Prevent Infections with  
Tea Tree Oil & Aloe Vera





# **KONIX** Sterile First Aid **BURN GEL** and **DRESSING**

## PRODUCT NAME

Konix® Sterile First Aid Gel and Burn Dressing

## DESCRIPTION

Konix® Sterile First Aid Gel and Burn Dressing is a first aid product group designed for first aid in all kinds of burn injuries (heat burns, sun burns, electrical burns, scalds, cuts, abrasions, etc.), which stops burning feeling and prevents further burns from progressing on the skin. Konix® Sterile First Aid Gel and Burn Dressing consists of sterile gel-impregnated burn dressings and gel bottles and sachets applied in emergency first aid and burn treatment. Konix® Sterile First Aid Gel and Burn Dressing quickly cools the burn to prevent further tissue damage and aid the healing process, and offers the patient a rapid and continuous reduction in sensitivity. Konix® Sterile First Aid Gel and Burn Dressing is used in emergency first aid for all kinds of burns by relieving sensitivity, rapidly removing heat, cooling the burn and stopping the progress of the burn. Konix® Sterile First Aid Gel and Burn Dressing contains Melaleuca (tea tree) oil and Aloe Vera. Konix® Sterile First Aid Gel and Burn Dressing consists of pads Konix First Aid Burn Gel-impregnated with 3 mm thick. When placed on the burn, a 3-6 mm gel layer is formed on the skin. The cooling properties of the dressing help stop the progression of the burn and minimize tissue damage. The dressings are made of a soft and flexible material, elastic enough to accommodate possible swelling or the need to stretch joints. It does not stick to the burn area and does not shed yarn or fibers.

## INDICATION

The indications for Konix® Sterile First Aid Gel and Burn Dressing product are as follows;

- Helps to relieve the sensitivity of the burned area shortly after the application.
- It cools and moisturizes the burnt area.
- It prevents the deepening of the burn by stopping the movement of heat to the lower layers of the skin.
- It prevents the infection caused by the external environment by covering the burn completely. It starts the healing phase of the tissue.
- Protects the burnt area against abrasion, friction and drying.
- Konix® Sterile First Aid Gel and Burn Dressing dissolves easily with normal water and does not leave any residue after washing the injured area.
- Used for first aid in all kinds of burn injuries (heat burns, sunburns, electrical burns, scalds, cuts, abrasions).

## CONTRAINDICATIONS

Konix® Sterile First Aid Gel and Burn Dressing is contraindicated in the situations described below;

- It should not be used by people who are sensitive and allergic to the ingredients, especially Melaleuca (Tea Tree) oil.

## SIDE EFFECT

In very rare cases, local sensitivity reactions such as dryness or burning may occur. Propylene glycol can cause skin inflammation. Melaleuca (Tea Tree) oil can cause allergic reactions.

## PRECAUTIONS

Konix® Sterile First Aid Gel and Burn Dressing is suitable for children, adults and infants over 1 month old. It should be used under physician control in pregnant and breastfeeding mothers. The product is for human use only. It should not be used for animals. The products are sterile and single-use. Keep out of the sight and reach of children. Do not use after the expiry date indicated on the packaging and bottle. If an unexpected effect is seen or irritation occurs, stop using it and consult your doctor. Do not use when there is extreme sensitivity to any of the ingredients. Is sterilized with radiation after the packaging. In case the package damaged, do not use since it will lose the sterility.

## APPLICATION

In cases where medical intervention cannot be applied immediately, Konix® Sterile First Aid Gel and Burn Dressing keeps the burn moist and protects it from the external environment. Konix Burn Gel is applied topically on the skin by massaging the affected area as a thin layer. It is applied externally to the affected area as needed. It is important that the area to be applied is cleaned and dried. It is sterilized by gamma radiation. Once opened, the product cannot be used again as the sterilization is disrupted. It should be used under physician control in pregnant women and nursing mothers. Konix Burn Dressing is an effective and simple application to treat burns and reduce sensitivity. The package is opened from the prescribed section, the dressing is removed from the package and placed directly on the burn without rubbing. It is lightly fixed with fingertips and left in place for up to 4 hours. At the end of the application, the dressing is simply removed and the remaining gel is wiped off with a dry and clean cloth. If the burn is in an area that makes it difficult to place the dressing fixed, Konix Sterile Burn Dressing is fixed with the help of a standard bandage. Small size dressings (5 cmx5 cm, 5 cmx15 cm, 10 cmx10 cm, 20 cmx20 cm, 25 mmx500 mm, 50 mmx1000 mm) are suitable for hand, arm and finger burns. Larger size dressings (30 cmx30 cm, 10 cmx40 cm, 20 cmx45 cm, 30 cmx40 cm, 40 cmx40 cm, 40 cmx60 cm, 60 cmx40 cm) are used for foot, chest, back and limb burns. For smaller burns, 50 ml, 59 ml, 100 ml, 118 ml, 125 ml, 3.5 gr; 5 gr; 10 gr; 20 gr are used. The dressing is applied only once as first aid and saves time for healthcare personnel. It is sterilized by gamma radiation. Once opened, the product cannot be used again as the sterilization is disrupted. Before using Konix® Sterile First Aid Gel and Burn Dressing products in chemical burns, the injured area should be neutralized and then Konix Sterile First Aid Gel and Burn Dressing Products should be used. In electrical burns, the electric current must be turned off and the injured person must first be rescued from the electric current, then Konix® Sterile First Aid Gel and Burn Dressing can be applied on the injured area.

