

Ref Doc: PSD0019		No.: QP-0591X0-X; QP-0741X0-X		Rev. 3	
Product:		Lateral Transfer Boards		Application: Manual Handling	
Technical Qualifications:					
Raw Materials:		Body of board - Polypropylene		Adhesive primer - Plastic primer	
		Non-slip pads - PVC upper and PET base, solvent acrylic adhesive		Screen print - Printing ink	
Packaging:		Primary – Bubble Wrap Secondary - Pure Cellulose Kraft, from a PEFC-approved source and 100% recycled waste-based paper with added chemicals. Tertiary – Banding Shrink Wrap, Card			
Regulatory:		Class I Non-Sterile			
Cleaning instruction:		Thoroughly clean the device as soon as possible using alcohol-based cleaning solutions (up to 70%) or normal disinfectant solutions. Manually wipe down the device until the device is visibly clean. If the device does not appear visibly clean, repeat the cleaning step. If the device remains contaminated upon visual inspection following multiple cleaning attempts, discard of the device in line with local hospital protocol.			
Safe Working Load:		N/A			
Other		Latex Free Reusable			
Products:					
Code:		Description:			
QP-059150-F		Lateral Transfer Board Folding 59 x 150cm			
QP-059150-R		Lateral Transfer Board Rigid 59 x 150cm			
QP-059180-F		Lateral Transfer Board Folding 59 x 180cm			
QP-059180-R		Lateral Transfer Board Rigid 59 x 180cm			
QP-074150-F		Lateral Transfer Board Folding 74 x 150cm			
QP-074150-R		Lateral Transfer Board Rigid 74 x 150cm			
QP-074180-F		Lateral Transfer Board Folding 74 x 180cm			
QP-074180-R		Lateral Transfer Board Rigid 74 x 180cm			
Intended Use:					
<p>GBUK Banana Lateral Transfer Boards are intended to assist in the lateral transfer of patients across a gap between two surfaces.</p>					

Pictures:

Figure 1: QP-059150-R

*** Variations: QP-059180-R, QP-074150-R, QP-074180-R***

Figure 2: QP-059150-F

*** Variations: QP-059180-F, QP-074150-F, QP-074180-F***



Figure 1



Figure 2

Features and Benefits:	Two widths and lengths available
	Easy to wipe clean
	Lightweight with handles for easy manoeuvrability
	Anti slip pads ensure maximum grip during transfer
	Requires minimum storage space
	High strength material for maximum durability


Description:		Packaging	
Code:	Description	Packaging	Units
QP-059150-F	Lateral Transfer Board Folding 59 x 150cm	Primary 6 x 1980 x 950mm	1
		Secondary 25 x 2000 x 850mm	
QP-059150-R	Lateral Transfer Board Rigid 59 x 150cm	Primary 6 x 1980 x 950mm	1
		Secondary 25 x 2000 x 850mm	
QP-059180-F	Lateral Transfer Board Folding 59 x 180cm	Primary 6 x 1980 x 950mm	1
		Secondary 25 x 2000 x 850mm	
QP-059180-R	Lateral Transfer Board Rigid 59 x 180cm	Primary 6 x 1980 x 950mm	1
		Secondary 25 x 2000 x 850mm	
QP-074150-F	Lateral Transfer Board Folding 74 x 150cm	Primary 6 x 1980 x 950mm	1
		Secondary 25 x 2000 x 850mm	
QP-074150-R	Lateral Transfer Board Rigid 74 x 150cm	Primary 6 x 1980 x 950mm	1
		Secondary 25 x 2000 x 850mm	
QP-074180-F	Lateral Transfer Board Folding 74 x 180cm	Primary 6 x 1980 x 950mm	1
		Secondary 25 x 2000 x 850mm	
QP-074180-R	Lateral Transfer Board Rigid 74 x 180cm	Primary 6 x 1980 x 950mm	1
		Secondary 25 x 2000 x 850mm	
Code:	GTIN Single Unit		
QP-059150-F	05060405744607		
QP-059150-R	05060405744614		
QP-059180-F	05060405744621		
QP-059180-R	05060405744638		
QP-074150-F	05060405744645		
QP-074150-R	05060405744652		
QP-074180-F	05060405744669		
QP-074180-R	05060405744676		

Drawings:


THIS DRAWING IS CONFIDENTIAL AND THE PROPERTY OF THE COMPANY "GBUK Group" IT CANNOT BE GIVEN, COMMUNICATED OR COPIED WITHOUT OUR AUTHORIZATION

DIMENSIONS THE ON THE DRAWING NOT TO SCALE

Example:

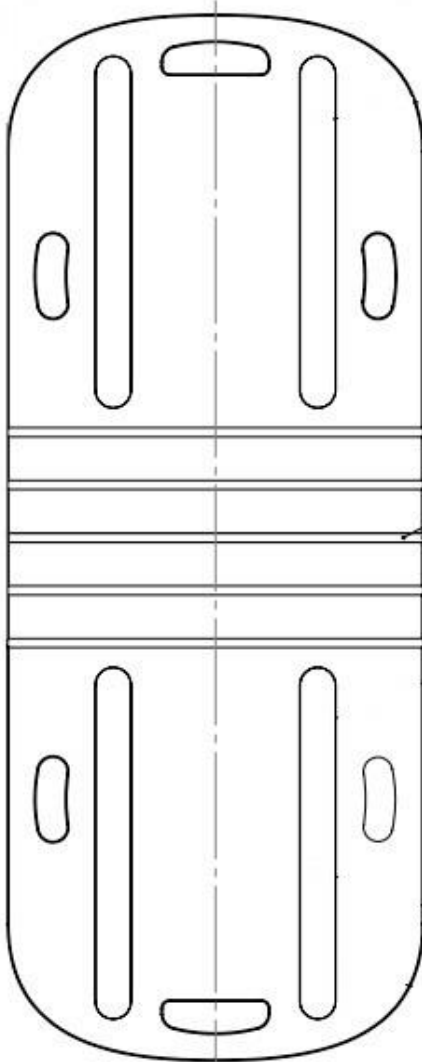

QP-059150-F

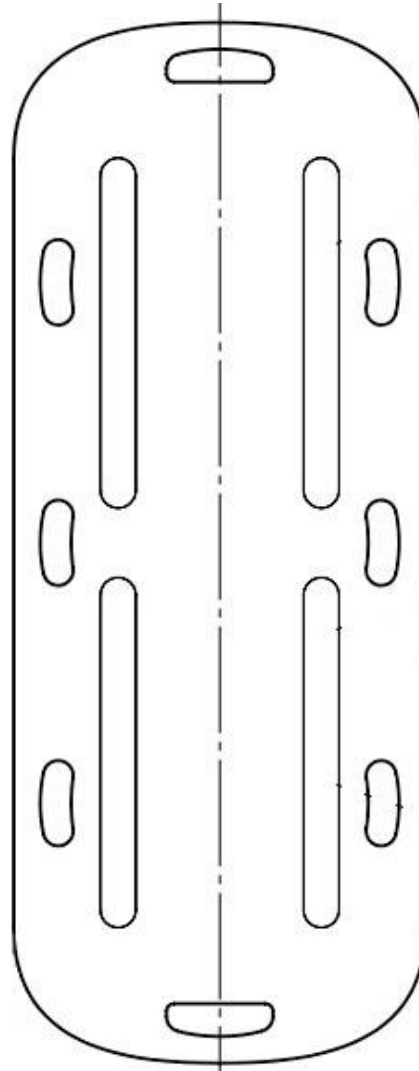

QP-059150-R

Description:

Product Print

Figure 3: QP-059150-R

*** Variations: QP-059180-R, QP-074150-R, QP-074180-R***

Figure 4: QP-059150-F

*** Variations: QP-059180-F, QP-074150-F, QP-074180-F***



Figure 3

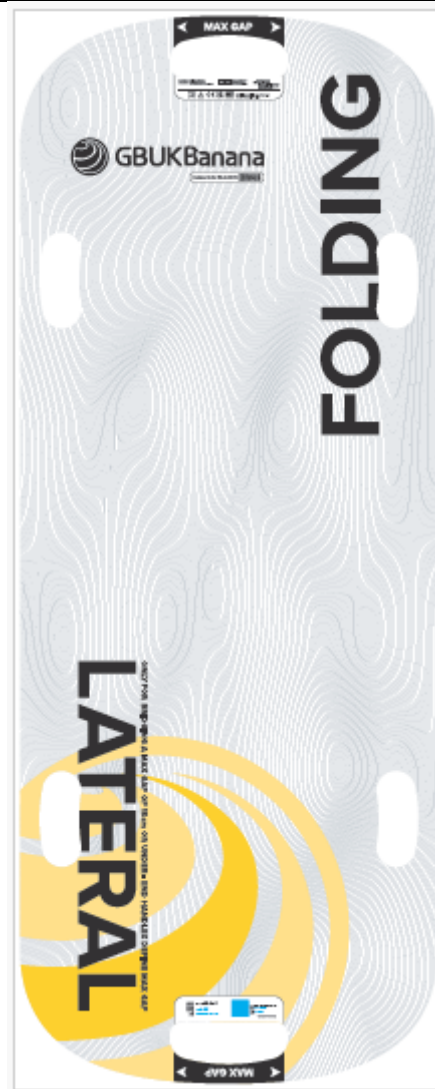


Figure 4

Description:	Primary Artwork
<p>Figure 5: QP-059150-R *** Variations: QP-059180-R, QP-074150-R, QP-074180-R***</p> <p>Figure 6: QP-059150-F *** Variations: QP-059180-F, QP-074150-F, QP-074180-F***</p>	

Board Lateral Transfer Board - Rigid 59cm x 150cm

Reusable

GBUK Banana Lateral Transfer Boards are intended to assist in the lateral transfer of patients across a gap between two surfaces.

Inspect Transfer Board before each use:

- Ensure that the correct Transfer Board is being used
- Do not use if Transfer Board is worn, damaged or wet

WARNING

- Always use appropriate equipment and sizes
- Carry out full Risk Assessment
- Follow RCN (or equivalent) guidelines
- Do not use the boards for lifting
- Do not use the boards to bridge a gap greater than 15 cm.

- FR** Planche de transfert latérale - Rigide
- IT** Pianale per lo spostamento laterale - Rigido
- ES** Tabla de traslado lateral - Rígida
- DE** Brett für seitliche Verlegung – Fest
- DA** Lateral overførselsplade - fast
- NO** Lateral overføringsbrett – stivt
- SV** Lateral förflyttningsbräda - Fast
- NL** Bord voor laterale transfer - stijf
- PL** Deska do transferu boczna - sztywna
- PT** Tábua de transferência lateral, rígida
- EL** Σανίδα για πλάγια μεταφορά - άκαμπτη
- TR** Sert Lateral Transfer Tahtası
- CS** Deska pro boční přenos – pevná
- ET** Lateraalne tõstelaud – jäik
- HU** Oldalfekvő beteg áthelyező lap - merev
- LV** Dēlis pārvietošanai uz sāniem – nelokāms
- MT** Tavla għal Trasferiment Laterali - Rígida
- RO** Placă de transfer pacient laterală - Rigidă
- SL** Deska za prestavljanje v stranskem položaju - toga



Figure 5

GBUK Group Ltd.
Woodland House
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0)1757 288 587
US: 18665 289 422
www.gbukbanana.com
info@gbukbanana.com

EC REP **QualRep Services BV**
Utrechtseweg 310 B42
6812 AR Arnhem, The Netherlands

CH REP **Swiss AR Services AG**
Industriestrasse 47,
6300 Zug, Switzerland



Symbol glossary available at www.gbukbanana.com

Not made with natural rubber latex.

SWL: N/A

Cleaning: Actichlor or Alcohol Wipes

Tumble Dry: N/A

Quantity: 1 units

REF **QP-059150-R**

LOT **123456**

YYYY-MM-DD

Made in UK.

EXAMPLE BARCODE (01)05060405744614
(11)YYMMDD
(10)123456

QP-059150-R-04 2409 24

Board Lateral Transfer Board - Folding 59cm x 150cm

Reusable

GBUK Banana Lateral Transfer Boards are intended to assist in the lateral transfer of patients across a gap between two surfaces.

Inspect Transfer Board before each use:

- Ensure that the correct Transfer Board is being used
- Do not use if Transfer Board is worn, damaged or wet

WARNING

- Always use appropriate equipment and sizes
- Carry out full Risk Assessment
- Follow RCN (or equivalent) guidelines
- Do not use the boards for lifting
- Do not use the boards to bridge a gap greater than 15 cm.

- FR** Planchette de transfert latérale - Pliable
- IT** Pianale per lo spostamento laterale - Pieghevole
- ES** Tabla de traslado lateral - Plegable
- DE** Brett für seitliche Verlegung – Faltbar
- DA** Lateral overførselsplade - foldbar
- NO** Lateral overføringsbrett – foldbart
- SV** Lateral förflyttningsbräda - Vikbar
- NL** Bord voor laterale transfer - opklapbaar
- PL** Deska do transferu boczna - składana
- PT** Tábua de transferência lateral, dobrável
- EL** Σανίδα για πλάγια μεταφορά - αναδιπλούμενη
- TR** Katlanır Lateral Transfer Tahtası
- CS** Deska pro boční přenos – skládací
- ET** Lateraalne tõstelaud – kokkupandav
- HU** Oldalfekvő beteg áthelyező lap - összehajtható
- LV** Dēlis pārvietošanai uz sāniem – salokāms
- MT** Tavla għal Trasferiment Laterali - Tiitewa
- RO** Placă de transfer pacient laterală - pliabilă
- SL** Deska za prestavljanje v stranskem položaju - preklopljiva



Figure 6

GBUK Group Ltd.
Woodland House
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0)1757 288 587
US: 18665 289 422
www.gbukbanana.com
info@gbukbanana.com

EC REP QualRep Services BV
Utrechtseweg 310 B42,
6812 AR Arnhem, The Netherlands

CH REP Swiss AR Services AG
Industriestrasse 47,
6300 Zug, Switzerland



Symbol glossary available at www.gbukbanana.com
Not made with natural rubber latex.

SWL: N/A

Cleaning: Actichlor or Alcohol Wipes

Tumble Dry: N/A

Quantity: 1 units

REF QP-059150-F

LOT 123456

YYYY-MM-DD



Made in UK.

EXAMPLE BARCODE
(01)05060405744607
(11)YYMMDD
(10)123456

QP-059150-F-04 2409 23

Demonstration Video and Product Brochure visit:

<https://www.gbukgroup.com/brands/gbuk-banana/>

Modified by:	Name: Aleksandra Malek	Signature: 	Date: 02-Dec-24 <small>I have reviewed this document 02-Dec-24 3:00:37 PM GMT</small>
Reviewed by:	Name: Nicola Neal	Signature: 	Date: 02-Dec-24 <small>I approve this document 02-Dec-24 3:49:36 PM GMT</small>



GBUK Banana is a trading name of GBUK Ltd.
GBUK Ltd. Is part of GBUK Group Ltd.

GBUK Banana
Woodland House
Blackwood Hall Business Park
North Duffield, North Yorkshire
YO8 5DD, UK

TEL: +44 (0) 1757 288 587
<https://www.gbukgroup.com/>
<https://www.gbukgroup.com/brands/gbuk-banana/>