Sanhill

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工厂环境一角
Beautiful factory-environment

Sanhill Medical Instrument Co. Ltd. is specialized in manufacturing consumable medical materials and enjoys good reputation among its worldwide customers, especially for latex foley catheter.

The company is developing on the principle of "excellent image, initiative spirit and top quality". With its sophisticated manufacturing equipments, detecting instruments as well as its mature manufacturing technique and well-committed employees, the company has been constantly producing reliable medical materials. Most of our materials have been authorized with CE&ISO certification and have been selling a great deal to the domestic market as well as overseas market through our strong sales network. They are widely used by clinical hospitals all over the world due to their safety and effectiveness and are well received by clinicians and patients worldwide with popularity and praise.

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引流接口安全，单向阀注入液体不漏水，不同颜色代表不同规格，容易区分。
Safe and excellent sealing property, different colours represent different size, easily distinguish it.

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DIRECTION

1. Generously lubricate tip and shaft of the catheter.
2. Carefully insert catheter tip into the bladder.
3. Using a syringe without needle, inflate balloon with sterile distilled water or physiological saline. Recommended volume to use is marked on funnel of catheter.
4. For deflation, cut off the inflation funnel above the valve, or using a syringe without needle push into valve to facilitate drainage.

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WARNING

1. For single use only.
2. Sterile for use if package is damaged.
3. The size and balloon capacity is marked on outer unit pack and funnel of catheter.
4. Store in a dark cool and dry place.

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Three Characteristics:

- Avoid impurity and calcifying on the surface of catheter.
- Ease irritation and allergy, relieve pain.
- Good biological compatibility.