


The Main Screen and Information Bar

The Main Screen provides access to all of the functions that are required to initiate programs and flushes, check the status of the reagents and filters, and configure the instrument to meet the requirements of your own laboratory.

Menu Options

The main menu is located on the right side of the Main Screen:

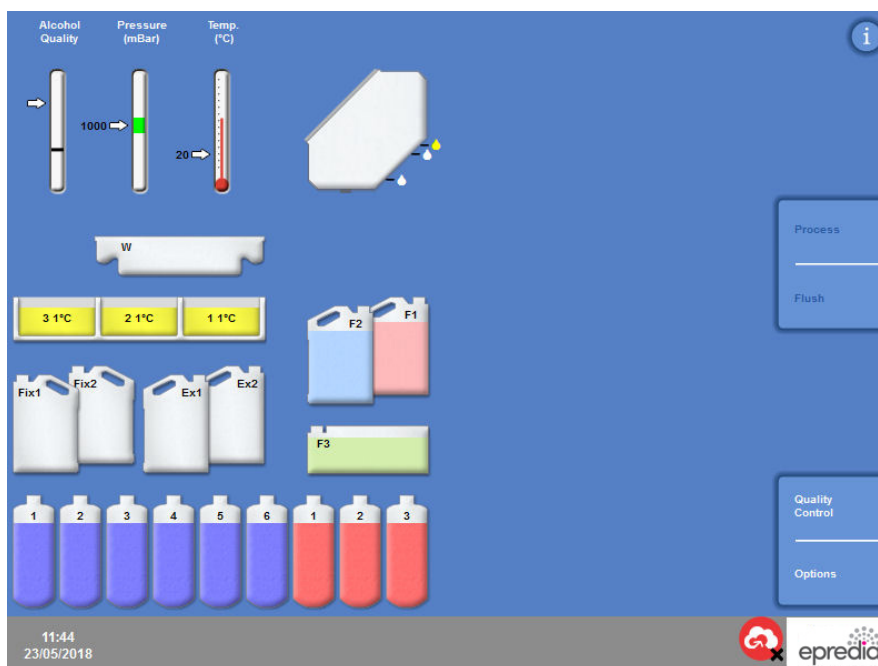

The Main Screen

The following menu options are available:

- Process:** This opens either the Reaction Chamber Available screen, from where you can start a processing program, or the Reaction Chamber Not Available screen, if there are no reagents loaded or the chamber is not prepared to start.
- Flush:** This opens the Select Flush screen. From here, you can start Flush programs to clean the Reaction Chamber.
- Quality Control:** This opens the Quality Control screen. From here, you can visually inspect reagents in the chamber, review use counts for reagents and filters and view and print quality control reports. Warning triangles provide visual alerts to problems that may affect processing.
- Options:** This opens the Options menu, which provides access to menus and settings that enable the instrument to be customised and configured.