



- Be careful in using **Allow long accession number** option. As the maximum number of letters implied in DICOM standards are 16, the image cannot be sent if you use this option depending on the PACS system.



- All DICOM-related logs are saved as a file in the specific location. If any problem occurs, contact the manager.
- All logs which were generated more than 90 days ago are removed automatically.


Station Name, Station AE Title

The station name, station AE Title and station port are used as the DICOM information.

System	
Station Name	
Station AE Title	VXVUE
Station Port	9104

4.9.2 Queue Menu

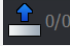

- Click on  button at the bottom of **VXvue** screen to move to the dialog for checking **Queue**.

- When you start to send or print a DICOM image, it is registered in **Queue** automatically.
- You can check the sending or printing status of DICOM image.
 - **Storage Queue** – Indicates the sending status of DICOM image to the DICOM storage server.
 - **Print Queue** - Indicates the printing status of DICOM image to the DICOM print server.

Storage Queue					Print Queue				
State	Server	AE Title	Patient ID	Patient Name	State	Server	AE Title	Patient ID	Patient Name
<input type="button" value="Retry"/> <input type="button" value="Delete"/>					<input type="button" value="Retry"/> <input type="button" value="Delete"/>				