 Integrated Disinfection Solutions	DETRO ENZYM PLUS DW KULLANMA KILAVUZU DETRO ENZYM PLUS DW USER MANUAL			
	Doküman No Document No	Yayın Tarihi Release date	Revizyon No Revision No	Revizyon Tarihi Revision Date
	KK.46.09	03.02.2025	0	-

DETRO ENZYM PLUS DW

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HEDEF KULLANICI/TARGET USER

Uzman Sağlık Personeli
Professional Medical Staff

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1%-5% nonionic surfactant, enzymes (protease, amylase, lipase, cellulase, mannanase), solvents, complexing agent, preservative.

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It is used for pre-cleaning of flexible endoscopes in DETROWASH series endoscope washing and disinfection devices. It provides effective cleaning of protein, carbohydrate and lipid based organic residues. Compatible with material structures such as stainless steel, synthetic rubber, aluminum, titanium, copper, polyamide, polyvinyl chloride, polyethylene, polyacetal, methyl polymethacrylate, polysulfone, polyurethane, polypropylene, ABS, silicone, latex, teflon and glass.

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It is designed to be used with the Detrowash series endoscope washing and disinfection device. The product container is placed in the disinfectant compartment of the device. The concentrated cleaner inlet of the device is connected to the container. Detro Enzym Plus DW is automatically used by the DETROWASH endoscope washing and disinfection device by diluting it to 0.5% (5ml/L). The ready-to-use solution is for single use only. After the disinfection process, the ready-to-use solution is sent to waste.

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Before using the product, carefully read the user manuals to ensure proper use and understanding of safety protocols. After reviewing, store the manuals in a safe place for future reference. Always store the product in its original packaging and out of reach of children. Avoid direct sunlight, heat and freezing. Visually check that the bottles are undamaged and leak-proof. Wear gloves when using the product to protect skin and avoid contact with eyes; in case of contact, rinse thoroughly with clean water, remove contact lenses if present and easy to remove, seek medical attention if irritation persists. If swallowed, rinse mouth and do not induce vomiting, call a national poison control center or physician. Use the product in well-ventilated areas to reduce fume inhalation and never mix with other cleaning agents. Also, check the product's Safety Data Sheet for more safety information.

SAKLAMA ve BERTARAF/STORAGE and DISPOSAL

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The expiration date is 3 years from the date of production. Keep the packaging tightly closed, in a well-ventilated environment at 5-30°C. Disposal operations must be carried out in full compliance with relevant local official regulations and environmental protection standards. For detailed information on product disposal, please review Article 13 of the Product Safety Data Sheet.

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If you encounter a negative situation regarding the product, you must inform the authorized institutions of the country where the product is located and the contact addresses below.

Detro Healthcare Kimya Sanayi A.Ş Atatürk Mah. Cemal Gürsel Cad. No:8, 34522 Esenyurt/İstanbul/Türkiye Tel: +90(212) 659 77 62 /Fax: +90 (212) 659 77 63/ info@detrox.com.tr	2 / 3
---	--------------

 Integrated Disinfection Solutions	DETRO ENZYM PLUS DW KULLANMA KILAVUZU DETRO ENZYM PLUS DW USER MANUAL			
	Doküman No Document No	Yayın Tarihi Release date	Revizyon No Revision No	Revizyon Tarihi Revision Date
	KK.46.09	03.02.2025	0	-

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