

Before quick starting the Cycle, you should check whether the type and parameters of the currently selected Cycle on the right side of the screen are correct.



Attention

When the operating conditions are fully met, and better sterilization effect will be obtained.

After the Cycle starts, it will jump to the cycle running screen:



Figure 4-4 Cycle Run Screen

#### 4.4.3 Value Display



Click the  to view display of the current Cycle.



Figure 4-5 View Display Screen

## 4.5 Auxiliary

### 4.5.1 Repeated Print

Press the Auxiliary Function key in the Home screen to enter the auxiliary function screen, and click the  REPRINT to print the printing record of the previous pot repeatedly. This screen requires level 1 Operator or above permission to enter.

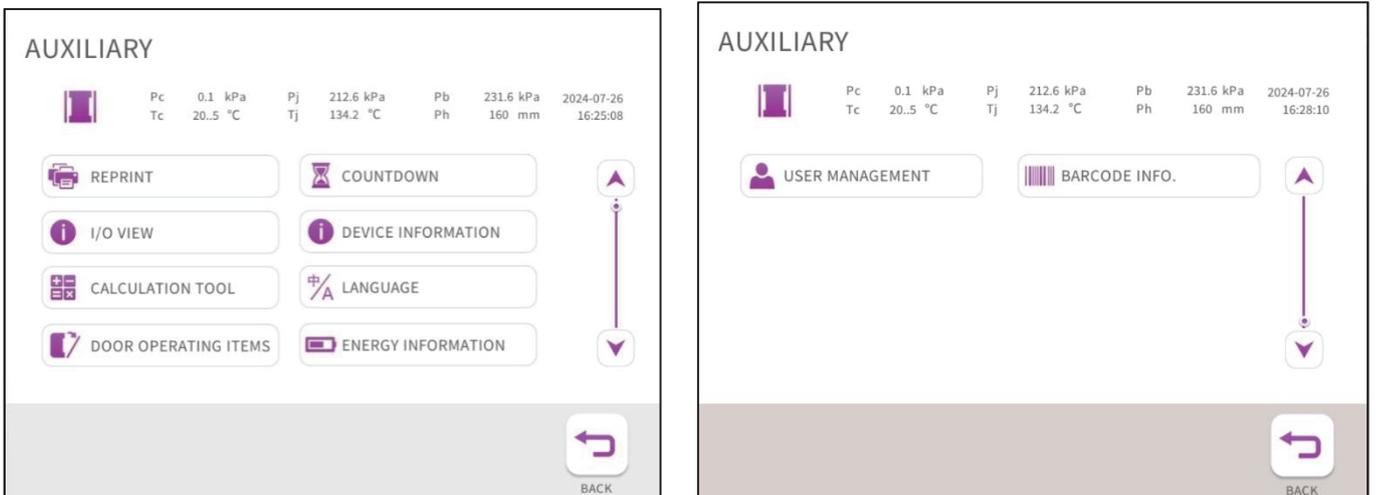


Figure 4-6 Auxiliary Screen