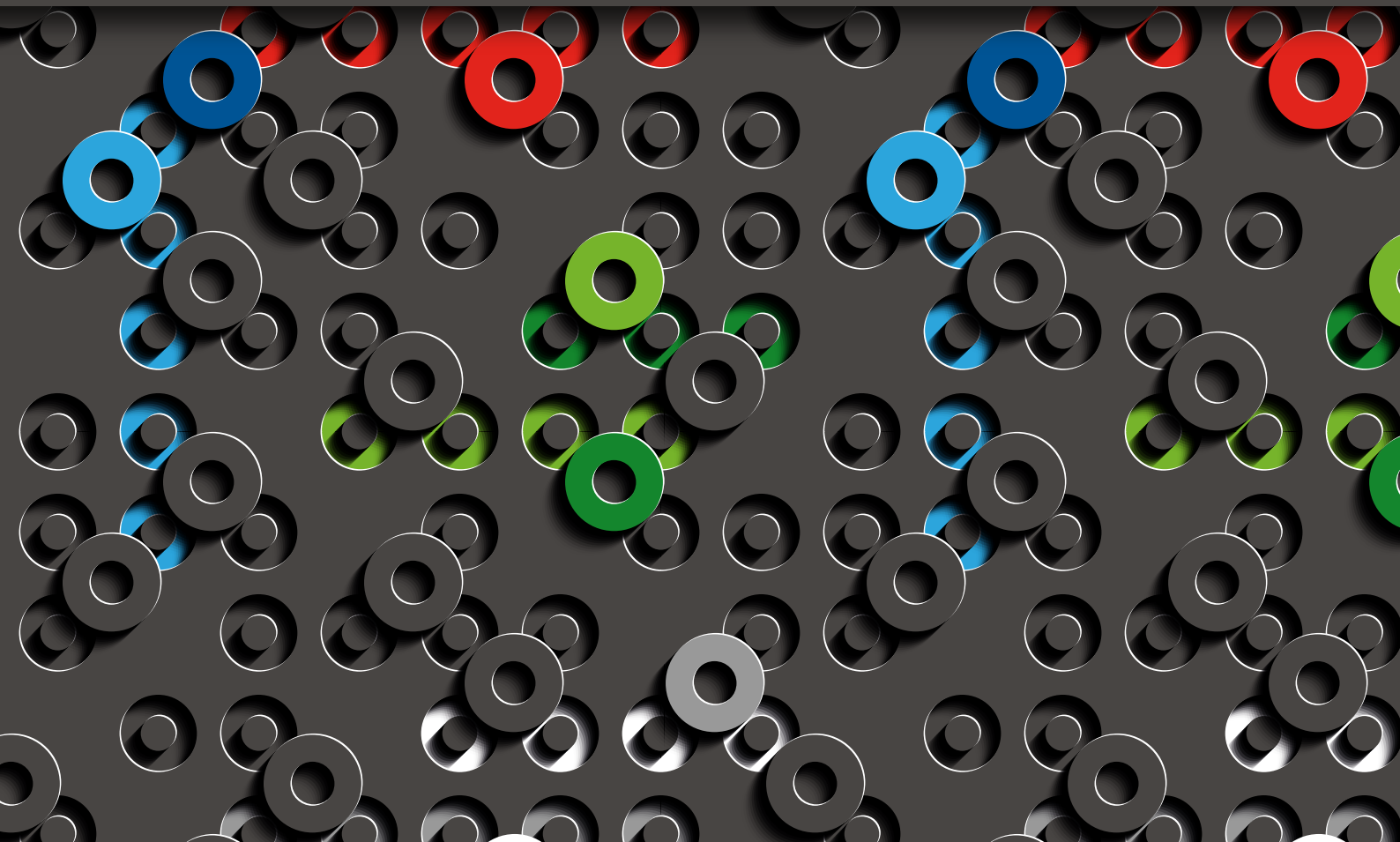
vacuoptaca[®]

**COMPANY PROFILE
L'ENTREPRISE
L'AZIENDA**

APTACA, established in 1976, gained its position in these years thanks to its great professional attitude in the national and international market of single use and reusable products for laboratory of analysis, of research and for hospitals, becoming a leader in this field.
The most modern productive technologies together with a highly qualified staff and thirty years experience in this field allow APTACA a great flexibility and the possibility to satisfy the most wide requests of his Customers and to offer an excellent service and a good quality product with competitive prices.
APTACA got, since 1998, the conformity of its Quality System to ISO 9001 and, from 2007, to ISO 13485 thanks to his efficient production system and his great quality attention.
The constant commitment in all these years in research and development of products allows us to offer a range always more complete and attentive to the Customers and End Users needs in Clinical Chemistry, Serology, Haematology, Microbiology, Bacteriology and Cryogenics fields.
In this context of continuous innovation is born VacuAptaca®, a complete line of Vacuum Devices for blood and urine sampling.
All VacuAptaca® Devices are subject to the most strictly quality controls and are manufactured with high quality materials, meeting the strictest European and international Rules (CE IVD - CE MD - ISO 6710 - UNI EN 14820 - UNI EN 14254 - UNI EN ISO 11137, etc.) in order to ensure the maximum safety and efficiency to Health workers.

OUR MISSION: CUSTOMER SATISFACTION AND PRODUCT SAFETY

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
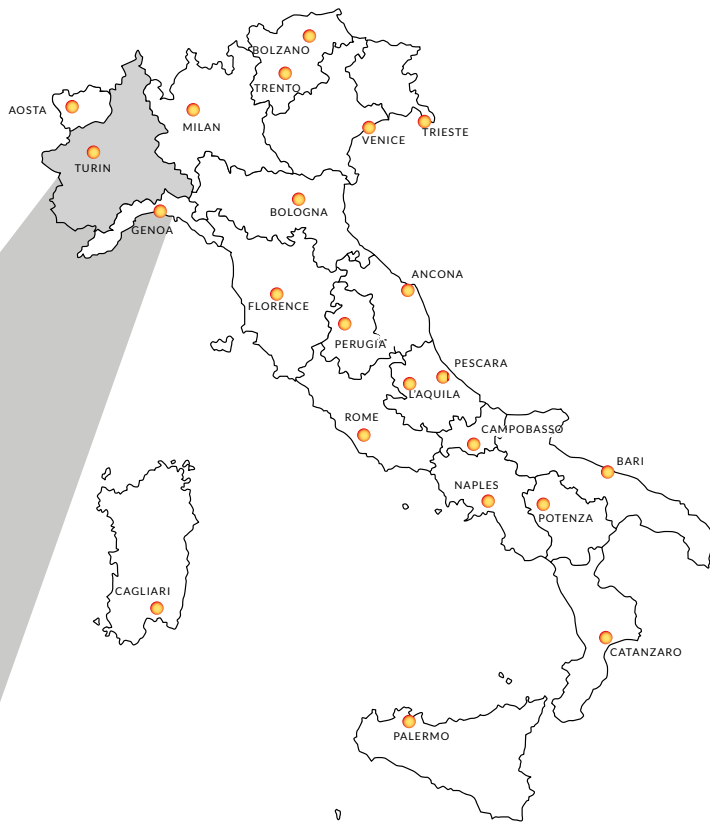
www.aptaca.com
www.vacuaptaca.it

Piedmont region
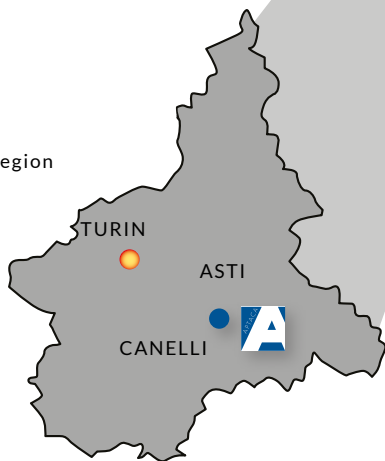
<p>SALES & CUSTOMER CARE OFFICE (EXPORT)</p> <p>export@aptaca.com</p>	<p>SALES & CUSTOMER CARE OFFICE (ITALY)</p> <p>info.italia@aptaca.com</p>	<p>PLANNING & LOGISTICS DEPARTMENT</p> <p>ordini@aptaca.com</p>	<p>PURCHASING DEPARTMENT</p> <p>acquisti@aptaca.com</p>
<p>QUALITY ASSURANCE & REGULATORY AFFAIRS</p> <p>qualita@aptaca.com</p>	<p>ACCOUNTING DEPARTMENT</p> <p>amministrazione@aptaca.com</p>	<p>RESEARCH AND DEVELOPMENT</p> <p>s.buono@aptaca.com</p>	<p>SECRETARIAT</p> <p>segreteria@aptaca.com</p>

Certified e-mail (PEC) aptaca@pec.aptaca.eu

End users are responsible for the Devices they select and shall verify both suitability and completion of technical information provided, as products must comply in material, shape and dimension to their intended use.
Nuova Aptaca has faculty to modify, without any previous notice, technical and aesthetic features of the articles described in this present catalogue.

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vacuoptaco

VACUUM TEST TUBES

A complete range of vacuum blood collection tubes, available in different sizes and with appropriate additives.

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Tubes without additive are suitable for use in Clinical Chemistry, in Immunohematology and for viral markers test. They can also be used as secondary test tubes to collect aliquots of the sample.

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CE 

4 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, no additive, for 4 ml of blood, white cap, sterile.

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CE 

6 ml

VacuAptaca® Ø13x100 mm vacuum test tube, in PET, no additive, for 6 ml of blood, white cap, sterile.

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CE 

2 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with Clot Activator, for 2 ml of blood, red cap, sterile.

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CE 

4 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with Clot Activator, for 4 ml of blood, red cap, sterile.

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CE 

6 ml

VacuAptaca® Ø13x100 mm vacuum test tube, in PET, with Clot Activator, for 6 ml of blood, red cap, sterile.

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CE 

9 ml

VacuAptaca® Ø16x100 mm vacuum test tube, in PET, with Clot Activator, for 9 ml of blood, red cap, sterile.

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CE IVD

3,5 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with Gel Separator + Clot Activator, for 3.5 ml of blood, red cap and red rubber, sterile.

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CE IVD

5 ml

VacuAptaca® Ø13x100 mm vacuum test tube, in PET, with Gel Separator + Clot Activator, for 5 ml of blood, red cap and red rubber, sterile.

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CE IVD

8 ml

VacuAptaca® Ø16x100 mm vacuum test tube, in PET, with Gel Separator + Clot Activator, for 8 ml of blood, red cap and red rubber, sterile.

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CE IVD

3,5 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with Gel Separator + Clot Activator, for 3.5 ml of blood, gold cap and red rubber, sterile.

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CE IVD

5 ml

VacuAptaca® Ø13x100 mm vacuum test tube, in PET, with Gel Separator + Clot Activator, for 5 ml of blood, gold cap and red rubber, sterile.

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CE IVD

8 ml

VacuAptaca® Ø16x100 mm vacuum test tube, in PET, with Gel Separator + Clot Activator, for 8 ml of blood, gold cap and red rubber, sterile.

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Tubes containing a buffered solution of Trisodium Citrate 3,2% (105 mM) or 3,8% (129 mM). Suitable for analysis of coagulation parameters in Clinical Chemistry and Hematology.

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COAGULATION TUBES
TUBES À ESSAI POUR COAGULATION
PROVETTE PER COAGULAZIONE

cod.
30500

2,7 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with Sodium Citrate 3.2%, for 2.7 ml of blood, light blue cap, sterile.

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cod.
30501

3,6 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with Sodium Citrate 3.2%, for 3.6 ml of blood, light blue cap, sterile.

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cod.
30500/38

2,7 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with Sodium Citrate 3.8%, for 2.7 ml of blood, light blue cap, sterile.

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cod.
30501/38

3,6 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with Sodium Citrate 3.8%, for 3.6 ml of blood, light blue cap, sterile.

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CE 

vacuaptaca
VACUUM TEST TUBES

Tubes for the erythrocyte sedimentation rate (ESR) determination containing a buffered solution of Trisodium Citrate 3.8%.

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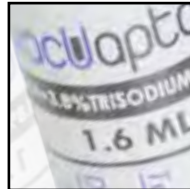
CE 

1,6 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with Sodium Citrate 3.8%, for 1.6 ml of blood, black cap, sterile.

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CE MD

2 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with K_2 EDTA, for 2 ml of blood, lavender cap, sterile.

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CE MD

3 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with K_2 EDTA, for 3 ml of blood, lavender cap, sterile.

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CE MD

4 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with K_2 EDTA, for 4 ml of blood, lavender cap, sterile.

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CE MD

6 ml

VacuAptaca® Ø13x100 mm vacuum test tube, in PET, with K_2 EDTA, for 6 ml of blood, lavender cap, sterile.

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CE 

2 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with K_3 EDTA, for 2 ml of blood, purple cap, sterile.

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CE 

3 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with K_3 EDTA, for 3 ml of blood, purple cap, sterile.

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CE 

4 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with K_3 EDTA, for 4 ml of blood, purple cap, sterile.

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CE 

6 ml

VacuAptaca® Ø13x100 mm vacuum test tube, in PET, with K_3 EDTA, for 6 ml of blood, purple cap, sterile.

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Tubes containing Lithium Heparin that inhibits the blood sample coagulation activating antithrombins. Suitable for analysis in Clinical Chemistry and Immunology.

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CE 

2 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with Lithium Heparin, for 2 ml of blood, green cap, sterile.

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CE 

4 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with Lithium Heparin, for 4 ml of blood, green cap, sterile.

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CE 

6 ml

VacuAptaca® Ø13x100 mm vacuum test tube, in PET, with Lithium Heparin, for 6 ml of blood, green cap, sterile.

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vacuaptaca
VACUUM TEST TUBES

Tubes containing Lithium Heparin that inhibits the blood sample coagulation activating antithrombins, with a polymer gel for plasma separation which is obtained by centrifugation of heparinized blood. Suitable for analysis in Clinical Chemistry and Immunology.

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CE IVD

4 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with Gel Separator + Lithium Heparin, for 4 ml of blood, light green cap, sterile.

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CE IVD

5 ml

VacuAptaca® Ø13x100 mm vacuum test tube, in PET, with Gel Separator + Lithium Heparin, for 5 ml of blood, light green cap, sterile.

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vacuaptaca

The tubes contain an anticoagulant and a glucose stabilizer: Sodium Heparin and Sodium Fluoride. Suitable for determining glucose and lactate.

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CE IVD

2 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with Sodium Heparin + Sodium Fluoride, for 2 ml of blood, grey cap, sterile.

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CE IVD

4 ml

VacuAptaca® Ø13x75 mm vacuum test tube, in PET, with Sodium Fluoride + Sodium Heparin, for 4 ml of blood, grey cap, sterile.

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vacuoptaco

NEEDLES AND ACCESSORIES

A wide range of accessories to perform the total safety vacuum blood collection.

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A full range of disposable Devices to be used in conformity to the procedures of use on force at the health structures and it is designed to guarantee the maximum safety of the operators. They are compatible with all VacuAptaca® tubes and needles.

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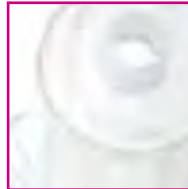
cod.
31300

CE (MD)

Vacuum collection disposable Holder, in polypropylene.

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CE (MD)

Butterfly needles, without Luer adapter, consist of an intravenous cannula, two fins, a tube equipped with a terminal fitting with unscrewable Luer/Lock cap suitable to use with luer adapter cod. 31200. Available as well with safety Device. Sterile, non-toxic, Pyrogen free.

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cod. **3153**

23G (Ø 0.60 x 19 mm) SAFETY DEVICE – Blue/Bleu/Blu

Multi-sample needles in ultra thin steel wall for vacuum system collection.

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cod.
31120

CE (MD)

Vacuum tube multi-sample needle 20G x 1", yellow, sterile.

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cod.
31220

CE (MD)

Vacuum tube multi-sample needle 20G x 1 1/2", yellow, sterile.

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cod.
31121

CE (MD)

Vacuum tube multi-sample needle 21G x 1", green, sterile.

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cod.
31221

CE (MD)

Vacuum tube multi-sample needle 21G x 1 1/2", green, sterile.

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cod.
31122

CE (MD)

Vacuum tube multi-sample needle 22G x 1", black, sterile.

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cod.
31222

CE (MD)

Vacuum tube multi-sample needle 22G x 1 1/2", black, sterile.

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Blood collection set composed by butterfly needle in ultra thin steel wall, Luer adapter compatible with the most part of the holders in commerce and small fitting tube.

Butterfly needles with safety device are designed to prevent accidental punctures in order to guarantee maximum safety of the Operators during the collection and disposal phases of the needle. Once the collection is finished it is possible to slide the needle inside the plastic protection and avoid the risk of accidental puncture. The needle will be fully remoted and securely locked into a closed system.

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vacUaptaca
NEEDLES AND ACCESSORIES

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CE (MD)

Butterfly needle with Luer adapter mounted and tube length 190 mm, sterile, individually wrapped. Non-toxic, Pyrogen free.

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23G - Blue/Bleu/Blu

CE (MD)

Safety Butterfly needle with Luer adapter mounted and tube length 190 mm, sterile, individually wrapped. Non-toxic, Pyrogen free.

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23G - Blue/Bleu/Blu

CE (MD)

Safety Butterfly needle with Luer adapter mounted, tube length 190 mm and with pre-attached holder. Sterile, individually wrapped. Non-toxic, Pyrogen free.

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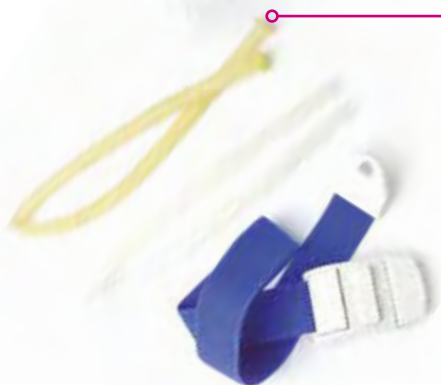
cod. **8104**

Sharp pocket size containers, provided with ergonomic shape and possible side fastening. Lid with double opening system, temporary and permanent closure. Maximum filling level indicator. Conform to ISO 23907.
Volume 0.4 liter - Dimensions 123 x 62 x 165 mm

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Storage boxes for vacuum tubes. Manufactured in ABS, they can contain up to 100 tubes, stackable. Available for both Ø13x75 mm and Ø13x100 mm (Ø max 14 mm) tubes version. The test tubes can be placed, after picking, without the cap, and can be sealed with two different methods, supplied with the storage box:

> Silica Gel Pad: Flexible and non-deformable, it seals the tubes while the lid is closed.
> Sticky Foam pad: Strong adhesion to the tubes while closing the lid, easy and convenient for disposal without contamination risk.

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vacuaptaco
NEEDLES AND ACCESSORIES

A line of containers conform to UN 3373 (biological substance Cat. B), studied for a safe transport of biological material, starting from samples collection to the arrival in the laboratory of analysis. All bags are provided with handle, zip closure and sealing possibility, internal compartment and external transparent pocket for documents, compartment for cool accumulators.

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Cylindrical container with hermetic closure and foam insert for transporting of n°12 test tubes Ø 12-16 mm. Classified as secondary container according to ADR/RID regulations. Round bag for transporting of one container cod. 6701. Classified as external container (3° container) according to ADR/RID regulations.

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bag / sac / borsa
Ø 120 x 150 mm

cod. **6840**

40 places test tubes rack with hermetic lid with seal and safety locks, complete of foam insert for test tubes Ø 12-16 mm. Classified as secondary container according to ADR/RID regulations. Dim. 215 x 125 x 180 mm.

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Isothermal bags for biological samples transportation. Available in different sizes, they are ideal to contain cod. 6840 for transporting up to 240 test tubes Ø 12-16 mm. Classified as external container (3° container) according to ADR/RID regulations.

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cod. **6748**

Dim. 450 x 270 x 400 mm
Test tubes / Tubes / Provette n° 240

Water freezing packs (ice packs), thin type, particularly suitable for use in inner pockets of the isothermal bags.

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vacuAptaca

URINE SAFETY COLLECTION SYSTEM

Close system for urine sampling in total safety. This system increases the safety of the operators due the contamination is virtually impossible while the transfer of the sample from the container to the tube. The health operator should no longer open the containers that reach the laboratory of analysis to effect the transfer of physiological fluid from the specimen container, thanks to automatic suction of VacuAptaca® vacuum tube.

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Test tubes for urine analysis to be carried out within two hours from the collection (without additive) or two hours after collection (with additive Boric Acid 1%)

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CE IVD

9 ml

VacuAptaca® Ø 16x100 mm cylindrical vacuum tube, in PET, with preservative (Boric Acid 1%), for urine collection sampling, for 9 ml, yellow cap, sterile.

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CE IVD

9 ml

VacuAptaca® Ø 16x100 mm cylindrical vacuum tube, in PET, no additive, for urine collection sampling, for 9 ml, yellow cap, sterile.

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vacuaptaco
URINE SAFETY COLLECTION SYSTEM

Containers with integrated system for the collection by vacuum test tube. Such closed system increases the safety of the operators due the contamination is virtually impossible during the sample transfer from the specimen container to the test tube.

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CE IVD

Graduated urine container vol. 120 ml, in polypropylene, yellow screw cap with sampling device for suction with vacuum tubes. Label on the cap for sampling system protection.

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CE IVD

Graduated urine container vol. 60 ml, in polypropylene, yellow screw cap with sampling device for suction with vacuum tubes. Label on the cap for sampling system protection.

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CE IVD

Straws with urine collection device for vacuum test tubes. Suitable for containers without collection device. Available with straw in two sizes.

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24 hours urine containers with integrated system for the collection by vacuum test tube. Such closed system increases the safety of the operators due the contamination is virtually impossible during the sample transfer from the specimen container to the test tube.

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CE IVD

2000 ml

24h urine collection container, with ergonomic handle. Screw cap with device for suction with vacuum tubes. Supplied with or without 150 mm sampling probe. Graduated from 100 to 2,000 ml. Also available in brown colour, suitable for photosensitive tests. Vol. 2,000 ml - Dim. 120 x 90 x 230 mm. Material: polyethylene.

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cod. **5471**

Without sampling probe/Sans sonde/Senza sonda prelievo
Neutral colour/Couleur neutre/Colore neutro

cod. **5472**

Without sampling probe/Sans sonde/Senza sonda prelievo
Brown colour/Couleur brune/Colore marrone

cod. **5671**

With sampling probe/Avec sonde/Con sonda prelievo
Neutral colour/Couleur neutre/Colore neutro

cod. **5672**

With sampling probe/Avec sonde/Con sonda prelievo
Brown colour/Couleur brune/Colore marrone

CE IVD

3000 ml

24h urine collection container, graduated, amber colour for light-sensitive tests, screw cap with device for suction with vacuum tubes. Available with or without 185 mm sampling probe. Vol. 3,000 ml. - Dim. 125 x 125 x 245 mm. Material: polyethylene.

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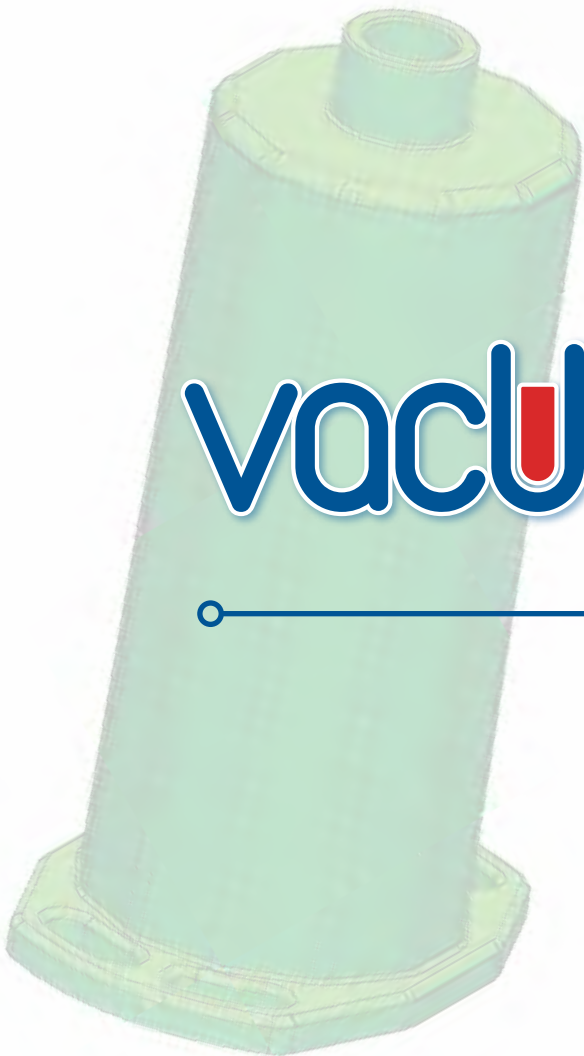
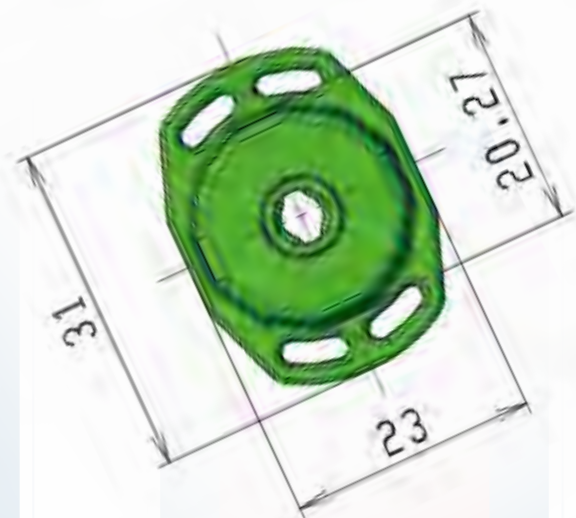
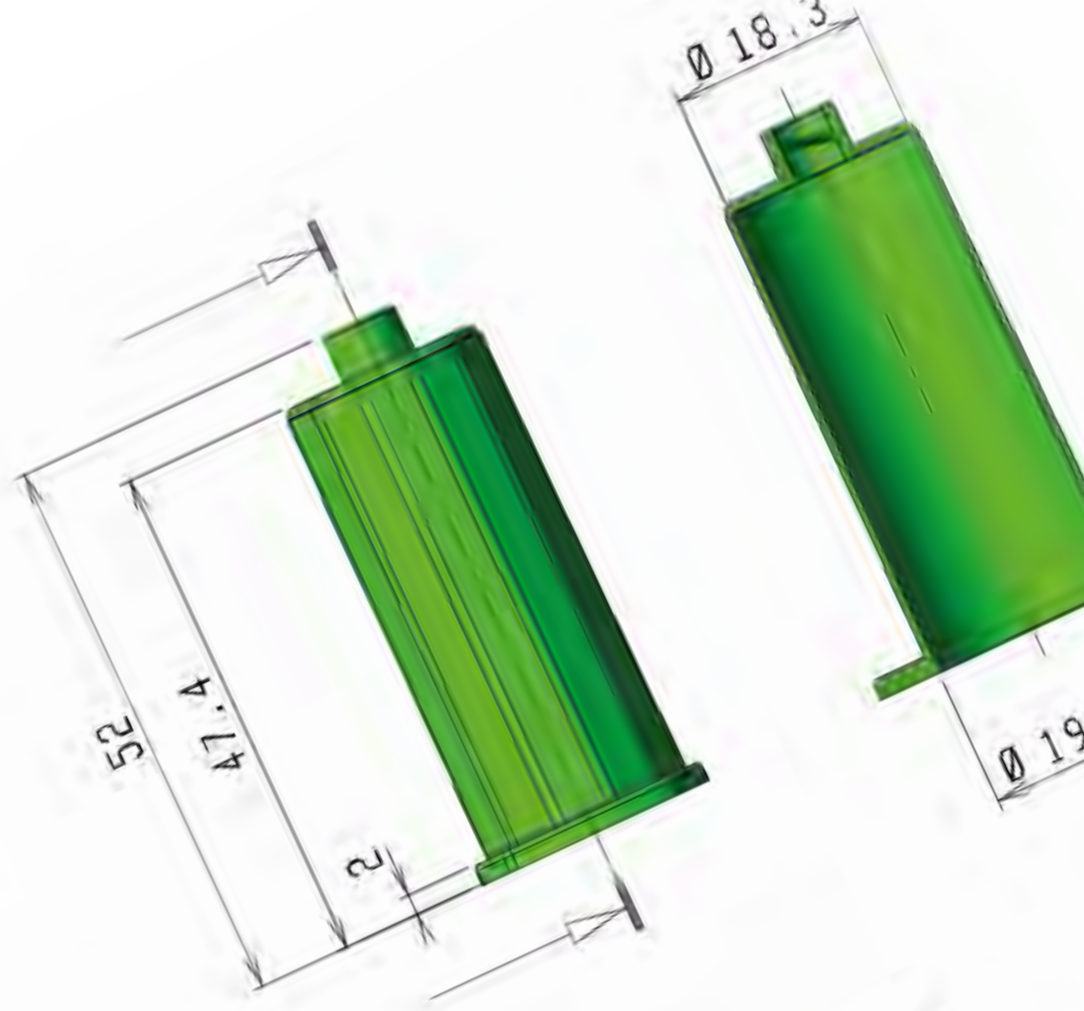
cod. **5731**

Without sampling probe/Sans sonde/Senza sonda prelievo

cod. **5732**

With sampling probe/Avec sonde/Con sonda prelievo

vacuoptaco

TECHNICAL INFORMATION

General characteristics: VacuAptaca® test tubes are manufactured in PET with cap and label of specific colour as per the additive content, in accordance to ISO 6710. Inside the cap there is an easily pierceable pharmaceutical rubber. Materials used for the production are latex-free, heavy metal free and phthalates and sulphates free. The tubes are inside sterile with a SAL (Sterility Assurance Level) of 10⁻⁶ in accordance to UNI EN ISO 11137, UNI EN ISO 11737 Rules. The tubes are classified as in Vitro Diagnostic Medical Devices according to Directive 98/79/CE and subsequent modifications and additions. Collection needles are classified as Medical Devices according to Directives 93/42/CE, 2007/47/EC and subsequent modifications and additions.

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¹When the outside temperature is higher than 35°C, the inverting time should be increased accordingly

²PRP (Platelet Rich Plasma) - PPP (Platelet Poor Plasma) - PFP (Platelet Free Plasma)

Storage: keep all the tubes to a temperature between +4 °C / +30 °C. Storage outside the up mentioned temperatures can cause a loss of efficiency of the tubes and therefore modify the results of the analysis test. Do not use the product beyond the expiration date.

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Sample collection and handling: most blood collection tubes contain chemical additives. It is of most importance to prevent a possible backflow from the tube in the veins, which could have negative effects on the patient. You should take the following precautionary measures:

1. patient arm must be oblique lying and face-down.

2. hold the tube so that the cap is facing upwards.

Make sure that the test tube content (for example additive or blood sample) does not touch the stopper or the end part of the needle during blood collection.

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Photosensitive-samples: in the case of bilirubin, erythrocyte protoporphyrin and carotene, samples must be transported under appropriate conditions in order to avoid exposure to sunlight.

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

Sequence for multiple collections: when the first test tube is full and the blood flow stops, gently remove it from the holder and introduce the next according to the following recommended sequence.

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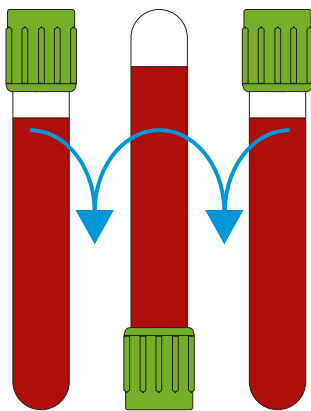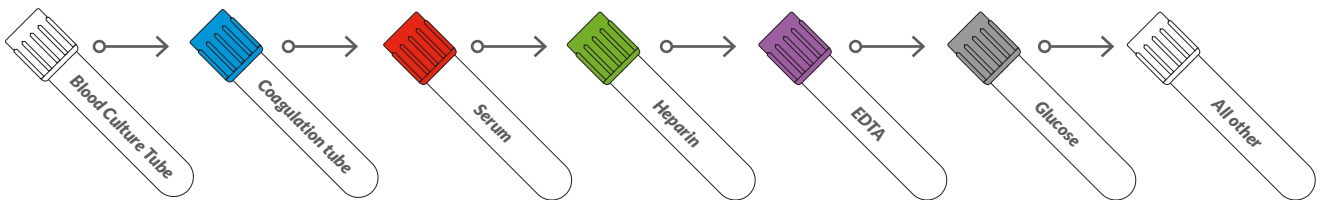= 1 inversion

Mixing and test tubes labelling: depending on the type of additive contained in the test tubes, all tubes must be gently reversed (without stirring, mixing vigorously can create foam or hemolysis) in order to get the right mix between blood and additive. The tubes containing EDTA and Heparin must be mixed in order to ensure that the sample does not coagulate. The tubes with the Clot Activator must be mixed otherwise the sample might not coagulate in stated time.

In order to ensure unambiguous identification of the sample, you must use a labelling system with barcode or restore the data into the appropriate fields of the label.

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Disposal: after collecting the last tube, remove the needle from the vein and dispose all devices (Holder, Luer adapter, multi-sample needle, butterfly needle, etc.) in special containers in accordance with the present regulations. No re-hooding needles, there is the risk of injury by puncturing and relative danger of infection. All sampling devices must be considered potentially infected after use.

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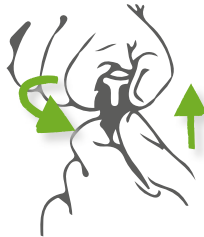
Centrifuge: make sure that the tubes are correctly inserted in the appropriate support for centrifugation. The serum tubes must be centrifuged after a period of time approximately of 30 minutes from the collection in order to avoid subsequent coagulations (fibrin formation) in the serum. This can cause false analysis results. With the centrifugation by swinging rotor you get a gel barrier slightly thicker and more stable. Centrifugation should be carried in a refrigerated centrifuge. High temperatures may have adverse effects on physical characteristics of gel.

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
**How to open?
Comment ouvrir?
Come aprire?**

Step 1: Double arms holding. One hand pinches the bottom of the cap, and tweak it, the other hand pushes the cap upwards the same time.

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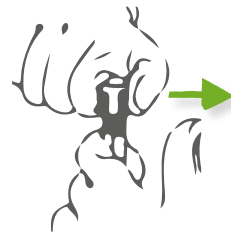
Step 2: Move the thumb before pull the cap out

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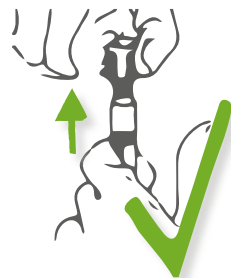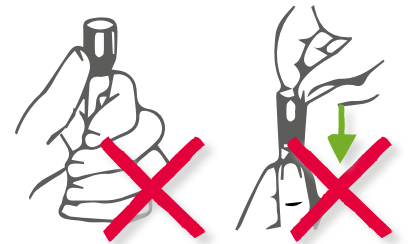
**How to close?
Comment fermer?
Come chiudere?**

Step 1: Make the cap right above the tube.

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Step 2: Tweak and meanwhile press the cap to the tube.

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TEST	Tubes Type	No Additive	Serum Clot Activator	Serum Gel+Clot Activator	Coagulation Sodium Citrate 3.2% and 3.8%	E.S.R - Sodium Citrate 3.8%	EDTA K ₂	EDTA K ₃	Lithium Heparin	Gel + Lithium Heparin	Sodium Fluoride/ Sodium Heparin	Urine test tube	Urine test tube 1% Boric Acid
URINE													
Urinary sediment analysis - cell counting, quantitative, qualitative													
Endogenous physiological substances - urea, creatinine, electrolytes, etc.													
Endogenous pathological substances - hormones with tumours, glucose, proteins with renal disorders, hemoglobin with hemolysis, bile pigments with liver disorder, etc.													
Exogenous substances - drugs and drugs of abuse or their metabolites, heavy metals or other toxins, bacteria, viruses or parasites with infections, etc.													
CLINICAL CHEMISTRY													
Metabolites - B-bilirubin conjugated -direct, -indirect, cholesterol, fructosans, uric acid, urea, HDL -cholesterol, creatinine, LDL-cholesterol, lipoprotein (a), triglyceride, etc.													
Electrolytes - chloride, sodium, potassium & magnesium (values higher in serum than in plasma), etc.													
Enzymes - ALAT, aldolase, AP, amylase, ASAT, cholinesterase, CK, CK-MB, lipase, etc.													
Proteins - albumin, α_1 -antitrypsin, α_1 -acid glycoprotein, α_2 -macroglobulin, microglobulin (α_1 - α_2), CD1, ceruloplasmin, CRP, ferritin, hemopexin, haptoglobin, IgA, IgE, IgG, IgM, myoglobin, neopterin, prealbumin, total protein, osteocalcin, procalcitonin, TBG, transferrin, troponin-I, troponin-T, etc.													
Thyroid function - MAK, TAK, TRAK, TBG, free T3, free T4, total T3 (serum is recommended), total T4, TSH, T3-uptake test													
Hormones - aldosterone, androstenedione, C-peptide, calcitonin, cortisol, DHEA, DHEAS, estradiol, FSH, HCG, parathormone (PTH), progesterone, prolactin, testosterone, etc.													
Tumor markers - AFP, CA125, CA15-3, CA19-9, CA50, CA72-4, CA549, calcitonin (HCT), CEA, CYFRA21-1, HCG, MCA, PSA, SCC, TG, TPA, etc.													
Cytokines - interferon, interleukin, interleukin-receptors, tumor necrosis factors, etc.													
Trace elements - aluminium, arsenic, lead, cadmium, chrome, iron, fluoride, cobalt, copper, lithium, manganese, mercury, molybdenum, selenium, thallium, zinc													
Vitamins - Folic acid													
Vitamins - A, B12, C, 1,25-dihydroxy-vitamin D3, 25-Hydroxyvitamin D3, E, folic acid													
Vitamins - B1, B2, B12, folic acid													
Vitamins - B1, B2, B6, B12, E, K													
TDM - anti-epileptics, bronchodilators, cardiac medication, antibiotics, immunosuppressives, antidepressants, psycho-tropic drugs, analgesics													
HEMATOLOGY													
Hemoglobin - free hemoglobin (qualitative & quantitative in plasma), carboxyhemoglobin (CO-Hb), methemoglobin, (metHb), hemoglobin F, hemoglobin A1c													
Complete blood count (CBC) - hemoglobin, hematocrit, erythrocytes, MCV, MCH, MCHC, leukocytes, thrombocytes, reticulocytes, differential blood count													
Differentiation of Lymphocytes - B-lymphocyte, T-lymphocyte, T-helper-cells (CD-4), T-suppressor cells (CD-8), natural killer cells													
COAGULATION													
Coagulation - quick test, aPTT, PTZ, antithrombin III, D-dimers, fibrinogen, fibrin degradation, coagulation factors, plasminogen, protein C, protein S													
IMMUNOLOGY													
Serology - ANA, anti-DNA, antistaphylococcal, antistreptolysin, C3/C4, CRP, protein-electrophoresis, rheumatoid factor, Rose-Waaler test, immune complex, immunoelectroph.													
Autoantibodies - AMA, ANA, ANCA													
Antibodies against exogenous antigens (allergy) - total IgE, allergy specific IgE													
Complement - CH50, C4, C1-INH, C3, C1q, factor B													
Immune hematology - blood group (ABO), Rh-factor, minor-major crossmatch, direct/indirect Coombs-test, cold agglutinin, cryoglobulins													
SPECIALTY													
Erythrocyte Sedimentation Rate (E.S.R)													
Glucose und Lactate													
Cerebrospinal fluid (CSF) - clinical chemistry, microbiological, cytological - examinations													
MICROBIOLOGICAL SEROLOGY													
Hepatitis serology (antibodies and antigens) - IgG- or IgM- antibodies e.g. anti-HAV, HBe, HBe, HDV, HEV and antigens e.g. HBe, HBe-antigen													
Antibodies against other viruses - IgG, IgM- antibodies against e.g.: CMV, EBV, HHV type 6, HSV, VZV, Adenovirus, Coxsackie virus, Dengue virus, ECHO virus, FSME, Hantavirus, HIV, HPV, Measles virus, Mumps													
Antibodies against parasites - Antibodies against e.g.: Amoeba, Echinococcus, Filarie, Malaria, Schistosoma, Toxoplasmosis													
Antibodies against bacteria and bacterial toxin - Antibodies against e.g.: Bordetella pertussis, Brucella, Chlamydia, Helicobacter pylori, cat scratch disease, Legionella, Listeria, Mycoplasma pneumoniae, Q fever, Rickettsia, Spirochaeta, Borrelia, Leptospira, Treponema													
Antibodies against fungi - Antibodies against e.g.: Aspergillus, Blastomycosis, Candidiasis, Coccidioidomycosis, Cryptococcosis													
Lues serology - TPHA													
Molecular biological identification of viruses, bacteria and parasites - (PCR, LCR, hybridization), e.g.: HBV-DNA, HCV-RNA, HIV-RNA													

The information contained in these tables are summary recommendations of the manufacturer. The user is obliged to verify the correctness of the data and the suitability of the tubes for the test that intends to carry out. Nuova Aptaca Srl is not responsible for any errors and/or omissions.

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Take note: the maximum tolerance of the vacuum is $\pm 10\%$. The blood quantity taken with vacuum tubes can vary with altitude, temperature, atmospheric pressure, the test tube production date, the pressure and venous sampling technique.

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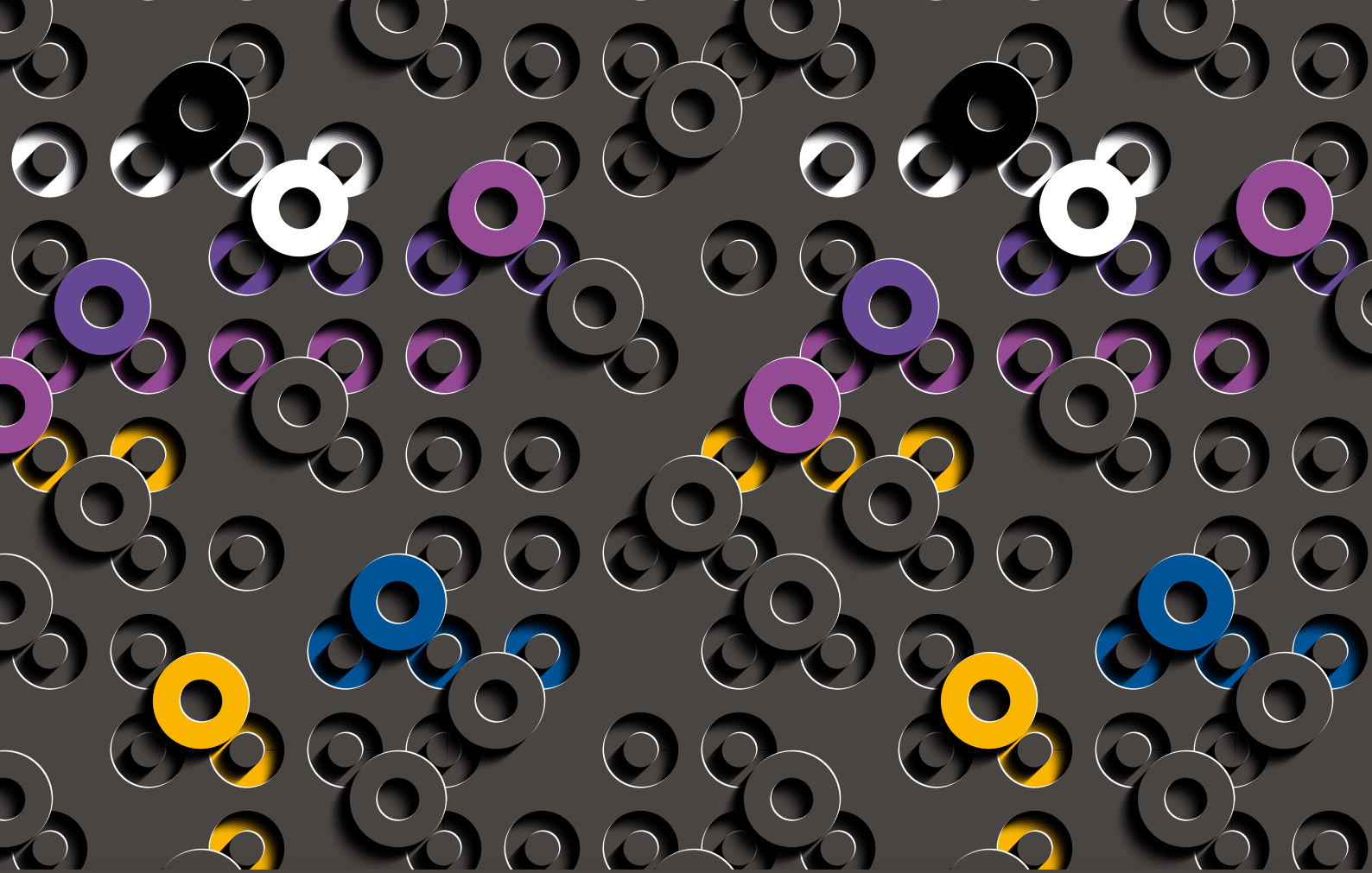
vacuoptaca

THE SAFETY AND QUALITY OF APTACA BRAND


NUOVA **APTACA** S.R.L.
Regione Monforte, 30
14053 CANELLI (ASTI) - ITALY
Tel. +39 0141 835075 / Fax +39 0141 835292
e-mail: info@aptaca.com
www.aptaca.com

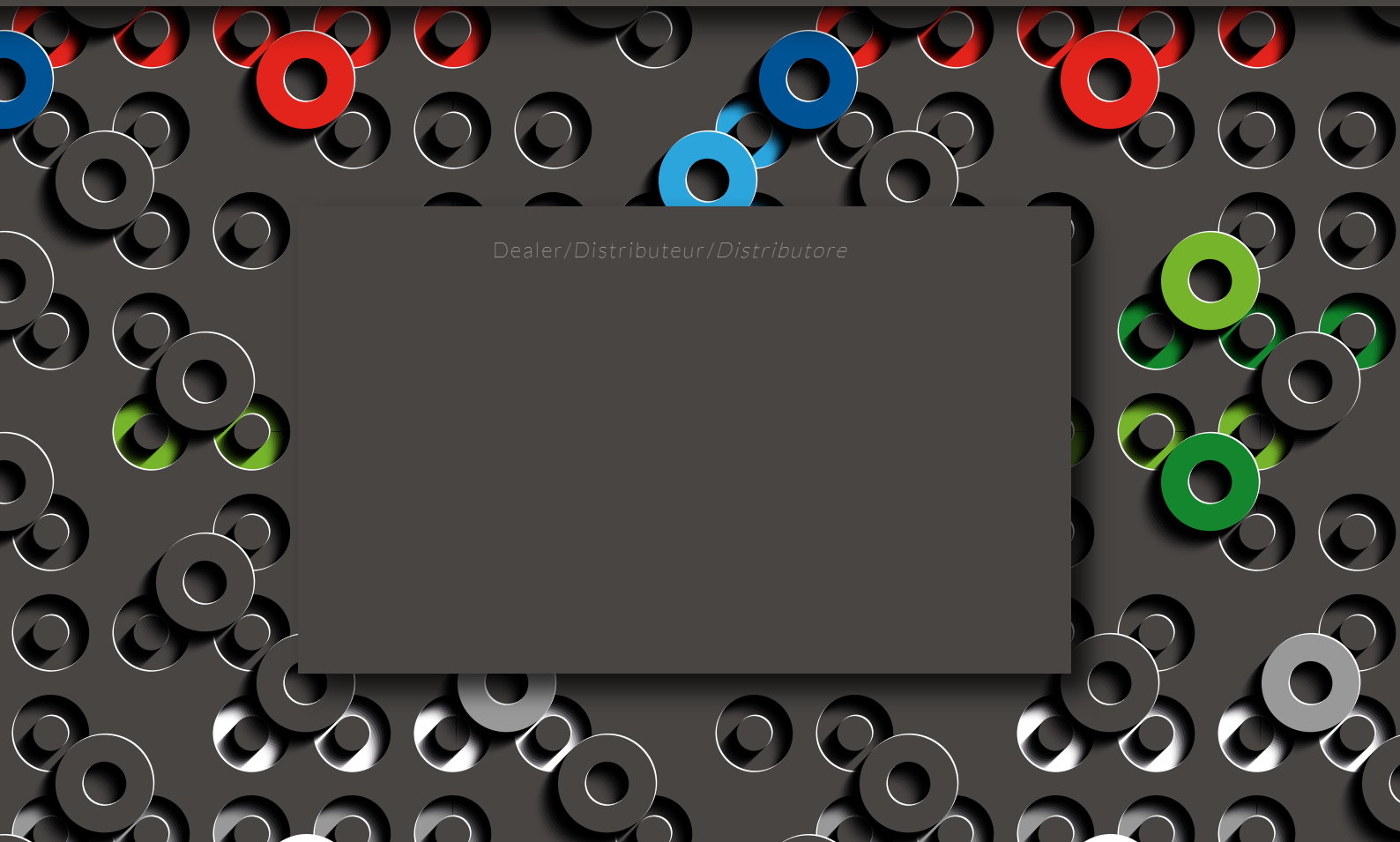www.vacuaptaca.it

Dealer/Distributeur/Distributore