iQ Lamina

REF 800-3236

TABLE OF CONTENTS

English	2
Français (FR)	4
Deutsch (DE)	6
Italiano (IT)	8
Español (ES)	10
Português Portugal (PT-PT)	12
Dansk (DA)	14
Svenska (SV)	16
Norsk (NO)	18
Ελληνικά (EL)	20
日本語 (JP)	22
中文 (ZH-CN)	24
Lietuviškai (LT)	26
Magyar (HU)	28
Polski (PL)	30
Čeština (CZ)	32
Slovenčina (SK)	34
한국어 (KO)	36
Türkçe (TR)	38
Русский (RU)	40
eesti keel (ET)	42
Hrvatski (HR)	44
Български (BG)	46
中文 (ZH-TW)	48
Română (RO)	50
Србија (SR)	52
Latviešu (LV)	54
Українська (UK)	56
Português Brasil (PT-BR)	58
Tiếng Việt (VI)	60

iQ Lamina

REF 800-3236

Rx Only in the U.S.A.

INTENDED USE

iQ Lamina is intended for *in vitro* diagnostic use with the iQ200 or DxU 840/850m Series of Automated Urine Microscopy Analyzers.

INTENDED USER

The device is intended to be used by a laboratory professional.

PRINCIPLE

iQ Lamina is used to hydrodynamically position the flow of specimen within the focal depth of the lens of the microscope and to ensure streamlined flow by rinsing and wetting all fluidic components.

WARNING AND PRECAUTIONS

- iQ Lamina has been determined to be nonhazardous under the guidelines issued by OSHA in 29 CFR 1910.1200(d).
- Appropriate precautions in the collection, handling, storage and disposal of specimens should be exercised accordingly.
- Product should present as a colorless to faint yellow, green, or blue transparent, odorless liquid. Product that differs from these attributes should be discarded.
- Wear appropriate personal protective equipment when handling.
- Avoid skin and eye contact. Refer to the Safety Data Sheet for first aid measures.
- Consult local or state authorities with regard to recommended method of disposal.

GHS HAZARD CLASSIFICATION

Not classified as hazardous

Safety Data Sheet is available at beckmancoulter.com/techdocs

NON-REACTIVE INGREDIENTS

- Water
- Stabilizers
- Preservatives

STORAGE AND STABILITY

- Store at 20°C to 28°C (68°F to 82.4°F).
- iQ Lamina is stable for eighteen months from date of manufacture.
- Open expiration: Unused solution that remains in the original capped container is stable until the expiration date.
- If frozen, bring back to room temperature and use.
- Do not store in direct sunlight.
- Do not use solution after the expiration date printed on the bottle label.

MATERIALS PROVIDED

- 2 x 7L iQ Lamina
- 1 x 5 um Filter

MATERIALS REQUIRED, BUT NOT PROVIDED

- iQ200 or DxU 840/850m Series of Automated Urine Microscopy Analyzers

- iQ Lamina Tubing Assembly
- Personal Protective Equipment

PROCEDURE

- Follow the directions in the iQ200 or DxU 840/850m Instructions for Use.

TECHNICAL SUPPORT

For additional information, or if damaged product is received, call Beckman Coulter Customer Service at 800-854-3633 (USA or Canada) or contact your local Beckman Coulter Representative.

A Glossary of Symbols (PN C27648) is available at beckmancoulter.com/techdocs.

NOTICE TO USER

For a patient/user/third party in the European Union and in countries with identical regulatory regime (*Regulation 2017/746/EU on In vitro Diagnostic Medical Devices*); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

TRADEMARKS

Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

May be covered by one or more pat. -see www.beckmancoulter.com/patents.

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Beckman Coulter Ireland Inc., Lismeehan, O'Callaghan's Mills, Co. Clare, Ireland +(353) (0) 65 683 1100