

DETRO PAA 2200 – USER MANUAL

- **PERACETIC ACID BASED INSTRUMENT AND ENDOSCOPE HIGH LEVEL DISINFECTANT**
- **READY TO USE**

Product Property

- Used for disinfection and cold sterilization of general surgical equipments, endoscopes and anesthesia equipments. Comprise from base and activator. After mixing base and activator, %0,22 peracetic acid occurs.

Usage Form

- Detrox PAA 2200 is used at manually or at endoscope washer machines for endoscopes disinfection.
- Before usage, pour activator into based drum, mix base and activator by shaking 10 min. to generate activated solution.
- At first usage, completely clean instruments, basins, endoscopes with enzymatic cleaner to prevent chemical reactions at instruments and endoscopes.
- Immerse instruments into activated solution and wait effect time (5 minutes)
- After effect time, take out instruments and rinse with sterile water.
- Control activity of solution by test strips.
- Solution can be used during 7 days.

Test strips user manual

- Test strips are use for understanding solution activity.
- Touch strip into solution and observe colour change.
- Compare colour with colour that exist strip tube and see if solution is
- You can change solution in the case of intensive organic contamination.



active

⇒ Warnings:

- Not wait instruments into solution more than effect time.
- You should record activation time and date.
- When activated, solution has 2200 ppm peracetic acid. Don't use solution when it decreases less than 900 ppm. You can control this by test strips.
- Do not use on aluminium, chrome, nickel covered instruments.

Disposal information

- Solution can be sent to sewage by diluting a little bit. Neutralization is unnecessary.

First aid Informations

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.