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
Pulse Oximeter
PM-30E (Model: AOJ-70A)

Instruction manual - Please read carefully!

23650 PM-30E 08-Aug-2023 Ver. 1.2
Original manual version: A0 / Software version: V1.0 /
Revision date: 2021-09 AOJ-WI-70A-0006

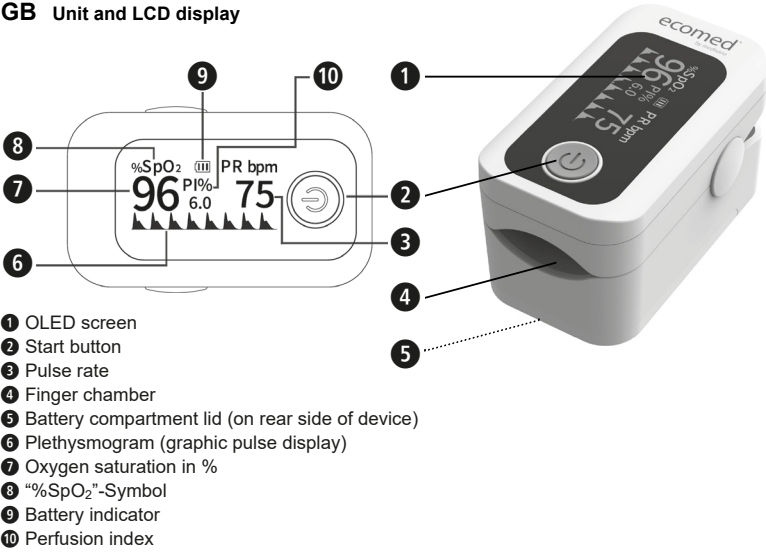GB Unit and LCD display


INTENDED USE

The pulse oximeter is a reusable device and intended for spot-checking of oxygen saturation and pulse rate for use with the finger of adult patients in healthcare environments. And it is not intended to be used under motion or low perfusion scenarios. Not suitable for medical purposes. Contraindications: None.

Explanation of symbols

	IMPORTANT Follow the instructions for use! Failure to follow these instructions may result in serious injury or damage to the unit.
	WARNING These warning notes must be observed to prevent any injury to the user.
	CAUTION These notes must be observed to prevent any damage to the device.
	NOTE These notes give you useful additional information on the installation or operation.
	Type BF equipment

	Oxygen saturation (in %)		The device has no Alarm system
	Degree of Protection Provided by Enclosures per IEC60529		Serial Number
	LOT number		Date of Manufacture
	Symbol for the marking of electrical and electronics devices according to Directive 200296/EC. The device, accessories and the packaging have to be disposed of waste correctly at the end of the usage. Please follow Local Ordinances or Regulations for disposal. Note: The oximeter is applied to this regulation.		CE sign
	Medical device		Temperature range
	Ambient pressure limit		Air humidity range
	Recycling symbols/codes: These are used to provide information about the material and its proper use and recycling.		Importer
	Authorised EU Representative		Distributor

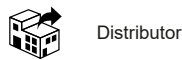IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!
Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

SAFETY INSTRUCTIONS

- Do not use the equipment near explosive or flammable substances (e.g. anaesthetic mixtures) - risk of explosion!
- This unit must not be used near magnetic resonance imaging (MRI) scanners or computerised tomography (CT) scanners.
- The pulse oximeter must not be operated alongside or in combination with other equipment.
- It is not recommended to use the device in high frequency environments (such as together with electrosurgical equipment).
- Further information on clinical restrictions and contraindications can be found by carefully studying the medical literature.
- Avoid direct sunlight, excessive infrared or ultraviolet radiation.
- The pulse oximeter is only an additional tool in the assessment of a patient situation. An assessment of the health situation is only possible if further clinical or professional examinations are carried out by a doctor.
- The device is not suitable for continuous monitoring.
- The device is NOT suitable for newborns and infants.
- The patient's finger thickness should be between 8 mm and 25.4 mm.
- Finger and device must be clean for proper measurement.
- If the measurement on one finger is not successful, use another finger.
- Be sure to keep the finger still during the measurement.
- Check the device for possible damage before use. Do not use the device if there is obvious damage or if in doubt.
- Discomfort or pain may occur if the device is used continuously, especially in patients with microcirculation disorders. It is recommended not to use the device on the same finger for more than 10 minutes.
- In patients who require more careful examination of the measurement site, the oximeter should not be used near oedema or other compromised sites.
- The device is only an aid to clinical diagnosis. The physiological data displayed on the device is for reference only and cannot be used directly for diagnostic interpretation.
- Never immerse the device in liquids.
- Swallowing small parts such as packaging material, battery, battery compartment cover, etc. can lead to choking. Children must therefore always be supervised when using this product.
- The device is used to measure the percentage of arterial oxygen saturation of functional haemoglobin. Factors that may affect the performance of the pulse oximeter or affect the accuracy of the measurement include: - Excessive light, such as sunlight or direct illumination, - Moisture in the device, - finger is outside the recommended size range, - flat pulse, - venous pulsation artefact, - anaemia or low haemoglobin concentration, - intravascular dyes, - carboxyhaemoglobin, - methaemoglobin, - dysfunctional haemoglobin, - artificial nails or fingernail polish, circulatory disorders.
- The infrared light emitted by the unit (invisible) is harmful to the eyes. Never look directly into the light.
- Always store the unit according to the storage conditions in this manual.
- In case of defects, the unit may only be repaired by qualified personnel. Do not disassemble the item and never attempt to repair it yourself. The unit does not contain any replaceable components. Contact the customer service.

BATTERY SAFETY INSTRUCTIONS

- Do not disassemble batteries!
- Immediately remove weak batteries from the battery compartment because they may leak and damage the unit!
- Increased risk of leakage, avoid contact with skin, eyes and mucous membranes!
- In case of contact with battery acid, rinse the affected areas immediately with plenty of clean water and consult a doctor immediately!
- If a battery has been swallowed, consult a doctor immediately!
- Insert the batteries correctly, observe the polarity!
- Keep batteries away from children!
- Do not recharge batteries! Do not short-circuit! Do not throw into fire! There is a risk of explosion!

Scope of delivery and packaging

Please first check whether the unit is complete and does not show any damage. If in doubt, do not operate the unit and contact your dealer or service centre.

The scope of delivery includes:

- 1 **ecomed** Pulse Oximeter **PM-30E**
- 2 Batteries (type AAA) 1.5V
- 1 Lanyard
- 1 Instruction manual & 2 inserts (EMC/Service)

If you notice any transport damage when unpacking, please contact your dealer immediately.

WARNING - Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

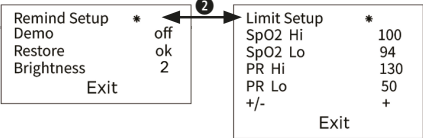
Insert / remove batteries

Remove: Before you can use your device, you must insert the enclosed batteries. To do this, open the battery compartment cover **➊** and insert the two 1.5V, AAA batteries. Pay attention to the polarity (as marked in the battery compartment). Close the battery compartment again.

Insert: Replace the batteries when the display becomes weak or the battery indicator **➎** flashes. If nothing appears on the display, the batteries are completely empty and must be replaced immediately. Remove the batteries if you are not going to use the unit for a long time.

Settings

After switching on the unit (briefly pressing the start button **➋**), press and hold the start button **➋** for approx. 1 to 2 seconds to access the unit settings. Press the start button **➋** once to switch between the settings, press and hold the start button **➋** again to switch the respective current settings on or off or to change further parameters. „Exit“ exits the settings mode.

Use

- Open the finger chamber **➊** by pressing the left-hand upper and lower parts of the unit together.
- Insert one of your fingers as far as possible into the finger chamber **➊** on the right-hand side of the unit and release the pressed-together parts of the unit.
- Press the start button **➋**. The OLED display switches on immediately.
- Keep your finger as still as possible during the measurement. After a short time, the pulse rate and the measured oxygen saturation appear on the display. By repeatedly pressing the start button **➋** you may switch between different display modes.

Notes for correct measurement results:

- The fingernail should point towards the upper chamber (where the sensor is located). Also, the finger should be fully inserted into the chamber. Otherwise, the measurement may produce inaccurate results.
- Before and after each measurement, you should clean the silicone inside the finger chamber **➋** and your finger with very little medical alcohol to achieve more accurate measurement results.

What does the displayed result mean?

The oxygen saturation (SpO₂) of the blood is a term referring to the concentration of oxygen attached to human hemoglobin. The normal value lies between 95 and 100 % SpO₂. A too low value may be an indication for existing diseases like e.g. cardiac defect, problems of the circulatory system, asthma or specific diseases of the lung. A too high value may be caused by a too fast and too deep breathing, what bears the danger of a too low blood carbon dioxide level. PI indicates the perfusion index, which is a measure for the pulse strength. The results lie in between 0.02% (very weak pulse amplitude) and 20% (very strong pulse amplitude). The value measured with this device is not suitable in any way to make or confirm a diagnosis - contact your doctor under all circumstances to get a correct diagnosis.

Cleaning and disinfection

Your appliance should be cleaned or disinfected regularly. Remove the batteries before cleaning or disinfecting the unit. Never use aggressive cleaning agents or strong brushes. Clean the unit with a soft cloth slightly moistened with a mild soap solution (diluted) or, for disinfection, with 70% ethanol. Always dilute according to the manufacturer's instructions or use the lowest possible concentration. Moisture must not penetrate the unit and you must not immerse the unit in liquids or pour liquids onto the unit. After cleaning or disinfecting, you can dry the unit with a soft, dry cloth. Then dry your unit in a ventilated and cool place and do not use it again until it is completely dry.

Notes:

- Never use abrasive materials (such as steel wool or silver polish) or corrosive cleaning agents (such as acetone or cleaning agents containing acetone).
- Do not disinfect the instrument with high temperature/high pressure disinfectant gas. Never use ETO (ethylene oxide) or formaldehyde for disinfection.
- If you spill liquid on the instrument or the instrument has become wet, contact customer service.

Error and remedy

Error: The device cannot be switched on.

Remedy: Remove the old batteries and insert 2 new ones. Press the START-button **➋**. If the device still cannot be switched on, contact the service centre.

Error: Display suddenly switches off.

Remedy: After approx. 15 seconds without further use, the unit switches off automatically. If this is not the cause, the OLED display may be defective or there is another technical fault. If the error display continues to appear even after changing the battery, contact the customer service.

Error: Measured values appear unusually high or low, are not displayed at all or differ greatly.

Remedy: The sensor may be dirty or defective or the user may have moved the finger a lot during the measurement. Clean and disinfect the device and perform a new measurement according to these instructions. Make sure that the finger is inserted as deeply as possible and that there are no disturbing light sources. If the pulse values vary greatly, please contact your doctor. If you are sure that the device is working properly and the values still appear unusual or are not displayed, please also contact your doctor and/or customer service.

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the batteries before disposing of the device. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

Directives / Norms

This device is certified in accordance with EC Guidelines and carries the CE symbol (conformity symbol). The specifications of MDR (EU) 2017/745 are met. The materials with which the user may come into contact are non-toxic and have no effect on the fabric and comply with ISO10993-1, ISO10993-5 and ISO10993-10 standards.

Electromagnetic compatibility: The device complies with the IEC 60601-1-2 standard for electromagnetic compatibility.

Technical specifications

Name and model:	ecomed Pulse Oximeter PM-30E (model:AOJ-70A)
Display system:	Digital display (OLED))
Measuring range:	SpO ₂ : 35-100%, Pulse: 25 - 250 beats / min., PI: 0%-50%
Display resolution:	SpO ₂ : 1%, Pulse: 1 beat/min.; PI: 0.1%
Accuracy:	SpO ₂ :± 2% (100%-70%), unspecified below Pulse: ±2 beats / min., Pulse intensity: Graphic display PI:0.1% (0%~1%); 1% (1%~20%); >20% no definition
Measurement performance in low perfusion condition:	0.3%
Power supply:	3 V $\overline{\text{---}}$, 2 batteries (type LR03, AAA) 1.5V
Power consumption:	30mA (normal) If the battery voltage is low (1.9V±0.2V), the unit is automatically switched off.
Battery life:	2xAAA 1.5V alkaline batteries (600mAh) could be continuously operated as long as 24 hours.
Classification:	Type of protection against electric shock: Internally powered equipment Degree of protection against electric shock: Type BF-Applied part (non-defibrillation proof)
Operating mode:	Spot checking
Degree of protection:	IP22
Application type:	Fingertip oximeter
Operating conditions:	+10°C - +40°C, 15% - 95% rel. humidity without condensation, atmospheric pressure 70 - 106 kPa -20°C - +50°C, 10% - 95% rel. humidity without condensation, atmospheric pressure 70 - 106 kPa
Storage/transport conditions:	approx. 63 x 39 x 36 mm approx. 38 g (without batteries)
Dimensions:	2 years
Weight:	Note: SpO2 and pulse rate accuracy are the essential requirements.
Lifespan:	Article number: 23650
Note: SpO2 and pulse rate accuracy are the essential requirements.	EAN number: 40 15588 23650 1

In the course of continuous product improvements, we reserve the right to make technical and design changes.

The current version of these instructions for use can be found at www.medisana.com

Warranty and repair terms

Your statutory warranty rights are not limited by our warranty set out below. Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

- The warranty period for **ecomed** products is 2 years from date of purchase.
In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Defects in material or workmanship will be removed free of charge within the warranty period.
- Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
- The following is excluded under the warranty:
 - All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - Accessories which are subject to normal wear and tear.
- Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

You will find the service address on the separate enclosed sheet.

	Share Info GmbH Address: Heerdt Lohweg 83, 40549 Düsseldorf, Germany Tel: 0049 179 5666 508; E-mail: EU-Rep@share-info.com
	ecomed® by medisana GmbH Carl-Schurz-Str. 2, 41460 NEUSS, Germany

Shenzhen AOJ Medical Technology Co., Ltd.
Room 301&4F, Block A, Building A, Jingfa Intelligent Manufacturing Park, Xiawei Yuan, Gushu Community, Xixiang Street, Bao'an District, 518126 Shenzhen, CHINA
Email: info@aojmedical.com
Website: <https://www.aojmedical.com>

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by this

CERTIFICATE

certifies that the Quality Management System of

PJSC "STEKLOPRIBOR"

18, Ozerna str., Zavodske, Lohvytsky district, Poltava region, 37240, Ukraine

has been established and duly implemented and company applies it in accordance with
the standard

ISO 9001:2015

provisions for the following areas:

Design, development, production and sales of technical thermometers and accessories thereof, hydrometers, hygrometers, vacuum manometers, domestic and glass thermometers, thermohygrometers and manometers, sand glasses, measuring and laboratory glassware and plastic components thereof and medical appliances (containers, vessels, medical thermometers, plastic overshoes, gynecological speculums and examination sets, otolaryngologic spatulae).

Certified location: 18, Ozerna str., Zavodske, Lohvytsky district, Poltava region, 37240, Ukraine

On the basis of the certification audit, protocol No. RE 037/19/47 it was proven that the management system meets the requirements of the above listed standard.

Certificate No.: Q-5497/22
Initial certification date: 01.12.2004
Date of issue: 04.11.2022
Expiry date: 03.05.2023

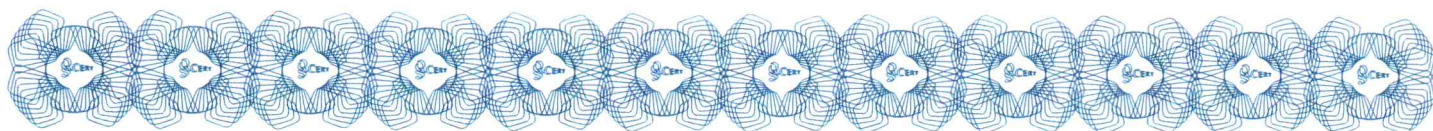
Certificate validity extended due to war actions in Ukraine.

By issuing this certificate, the validity of certificate No Q-5497/20 from 22.12.2020 is cancelled.

This certificate is valid only if it is published
among valid certificates on www.qscert.com

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Director adjunct

Radu STRATUTA

Digitally signed by Stratuța Radu
Date: 2024.06.21 19:58:51 EEST
Reason: MoldSign Signature
Location: Moldova

Pulse Oximeter EU Declaration of Conformity

Document No: CE-AOJ-70X-TD-07

Revision History:

Revision	Date	Change Description	Draft by	Reviewed by	Approved by
A0	2021-09-21	Initial release	Baiyu Xie	Jerry Gao	Jerry Gao
A1	2025-07-17	Update product information	Kexin Chen	Jerry Gao	Jerry Gao

Declaration of Conformity

This Declaration of Conformity is issued under the sole responsibility of the manufacturer. The device covered by the present Declaration is in conformity with all regulations or directives below, including compliance with related Essential Requirements and General Safety and Performance Requirements.

1. Object of the declaration:

Product Name	Pulse Oximeter
Model Number	AOJ-70A, AOJ-70B, AOJ-70C,AOJ-70D, AOJ-70E,OXI-70E
Product Type	Pulse Oximeter
Intended Purpose	The pulse oximeter is a reusable device and intended for spot-checking of oxygen saturation and pulse rate for use with the finger of adult patients in healthcare environments. And it is not intended to be used under motion or low perfusion scenarios.
Product Description	SpO ₂ stands for peripheral capillary oxygen saturation. Oxygen saturation is defined as the ratio of oxyhemoglobin (HbO ₂) to the total concentration of hemoglobin (i.e. Oxyhemoglobin + reduced hemoglobin) present in the blood. It is an important physiological parameter involved in respiration and circulation. The Pulse Oximeter feature herein is small, portable, non-invasive and easy to use. The user only needs to insert a finger into the chamber to measure his/her SpO ₂ and pulse rate. The oximeter is suitable for home healthcare environments.
Basic UDI-DI	697204011AOJ70X18B
Control Indicator	Lot number
Global Medical Device Nomenclature Code (GMDN) and Description or EMDN Code and Description	GMDN code: 45607 Pulse oximeter, battery-powered EMDN code: Z1203020408 PULSE OXIMETERS

The object of the Declaration described above is in conformity with the following regulations:

EU Regulation	Regulation (EU) 2017/745 of the European Parliament and of the Council of 05 April 2017 on medical devices (EU MDR)
Device Risk Classification	Class IIa based on Rule 10 in Annex VIII
Conformity Assessment Path	Annex IX Conformity assessment based on a quality management system and on assessment of technical documentation
Notified Body Name, Address, and ID	NB Name: TÜV SÜD Product Service GmbH Address: Ridlerstraße 65, 80339 MÜNCHEN, Germany NB Code: 0123
Certificate(s) issued	NO. G10 103703 0006

Standards	The products listed on this Declaration of Conformity have been assessed and/or tested in a typical configuration as described in the Manufacturer's accompanying documentation in accordance with the product standards listed below. EN 60601-1:2006/A2:2021, EN 60601-1-11: 2015/A1: 2021 , EN 60601-1-2: 2015/A1: 2021 , EN ISO 81060-2-61:2019,EN 14155:2020, EN ISO 14971: 2019/A11:2021 , IEC 60601-1-6:2010+AMD1:2013+AMD2:2020 , IEC 62304:2006/A1:2015, IEC 62366-1:2015+AMD1:2020, ISO 10993-1:2018, ISO 10993-10:2021, ISO 10993-5:2009, ISO 10993-23:2021, EN ISO 15223-1:2021
------------------	---

EU Directive	Directive 2011/65/EU of the European Parliament and of the Council of 08 June 2011 on the restrictions of the use of certain hazardous substances in electrical and electronic equipment, amended up to and inclusive of Directive (EU) 2017/2102 (RoHS)
Device Classification	Category 8, medical device, according to Annex I
Standards	The products listed on this Declaration of Conformity have been assessed and/or tested in a typical configuration as described in the Manufacturer's accompanying documentation in accordance with the product standards listed below. EN 62321-3-1:2014,EN 62321-7-1:2015,EN 62321-6:2015,EN 62321-8:2017

EU Directive	Regulation (EC) No 1907/2006 of the European Parliament and of the Council concerning the Registration, Evaluation, Authorisation and Restriction of Chemicals (REACH), as amended.
Conformity Assessment Path	Communication on Substances of Very High Concern (SVHCs) present above 0.1% (w/w) in the product.

EU Directive	Directive 2014/53/EU of the European Parliament and of the Council of 16 April 2014 on the harmonisation of the laws of the Member States relating to the making available on the market of radio equipment (RED)
Conformity Assessment Path	Annex II Module A
Standards	The radio equipment was tested to the following standards or technical specifications: ETSI EN 300 328 V2.2.2 (2019-07) ETSI EN 301 489-1 V2.2.3 (2019-11) ETSI EN 301 489-17 V3.2.4 (2020-09) EN 62368-1: 2014 + A11: 2017 EN 50663:2017 EN 62479:2010

2. Additional information:

Manufacturer	Name: Shenzhen AOJ Medical Technology Co., Ltd. Address: Room 301&4-5F, Block A, Building A, Jingfa Intelligent Manufacturing Park, Xiaweiyuan, Gushu Community, Xixiang Street, Bao'an District, 518126 Shenzhen, CHINA SRN: CN-MF-000018386
EU Authorized Representative	Name: Share Info GmbH Address: Am Schulzentrum 12,41564 Kaarst, Germany SRN: DE-AR-000005132
Quality Certificates Issued	The Manufacturer is certified by TUV to the following: EN ISO 13485:2016 , as evidenced by certificate number Q5 103703 0004

Signature (signed for and on behalf of Shenzhen AOJ Medical Technology Co., Ltd.): Date of Issue: 2025.12.09

Printed Name: Jerry Gao 

Place of Issue: Shenzhen

Title: Person Responsible for Regulatory Compliance

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