

**Evaluation Report of
HCV Rapid Test
(Whole Blood/Serum/Plasma)
by Hangzhou AllTest Biotech Co., Ltd.**

Contents

1. Background	3
2. Test principle	3
3. Clinical purpose	4
4. Study content	4
4.1 Specimen information	4
4.2 Product kit information	4
4.3 Study Method	4
4.4 Acceptable Criterion	5
5. Test Result	5
6. Result Analysis and conclusion	5

1. Background

Hepatitis C Virus (HCV) is a small, enveloped, positive-sense, singlestranded RNA Virus. HCV is now known to be the major cause of parenterally transmitted nonA, non-B hepatitis. Antibody to HCV is found in over 80% of patients with well-documented non-A, non-B hepatitis.

Conventional methods fail to isolate the virus in cell culture or visualize it by electron microscope. Cloning the viral genome has made it possible to develop serologic assays that use recombinant antigens. Compared to the first generation HCV EIAs using single recombinant antigen, multiple antigens using recombinant protein and/or synthetic peptides have been added in new serologic tests to avoid nonspecific cross-reactivity and to increase the sensitivity of the HCV antibody tests. The HCV Rapid Test Cassette (Whole Blood/Serum/Plasma) is a rapid test to qualitatively detect the presence of antibody to HCV in a whole blood, serum or plasma specimen. The test utilizes colloid gold conjugate and recombinant HCV proteins to selectively detect antibody to HCV in whole blood, serum or plasma. The recombinant HCV proteins used in the test kit are encoded by the genes for both structural (nucleocapsid) and non-structural proteins.

2. Test principle

The HCV Rapid Test Cassette (Whole Blood/Serum/Plasma) is a qualitative, membrane based immunoassay for the detection of antibody to HCV in whole blood, serum or plasma. The membrane is pre-coated with recombinant HCV antigen on the test line region of the cassette. During testing, the whole blood, serum or plasma specimen reacts with recombinant HCV antigen conjugated colloid gold. The mixture migrates upward on the membrane chromatographically by capillary action to react with recombinant HCV antigen on the membrane and generate a colored line. Presence of this colored line indicates a positive result, while its absence indicates a negative

result. To serve as a procedural control, a colored line will always appear at the control line region indicating that proper volume of specimen has been added and membrane wicking has occurred.

3. Clinical purpose

The purpose of this research is to determine the sensitivity of HCV Rapid Test Cassette(Whole Blood/Serum/Plasma) for detection of HCV positive and HCV negative specimen.

4. Study content

The parallel-grop test is adopted,and the CE marked product with the similar theory is taken as comparation device Alltest HCV Rapid Test Cassette(Whole Blood/Serum/Plasma) and the comparation device will do parallel test on each same specimen.All the result recorded respectively then evaluate the consistency between Alltest test device and the comparation device.

4.1 Specimen information

A total of 110 positives and 200 negatives individuals were subjected to testing procedures in order to diagnose hepatitis C infection.

4.2 Product kit information

4.2.1 Candidate Device

Alltest HCV Rapid Test Cassette(Whole Blood/Serum/Plasma)

Lot: ATHCV22060020-S

4.2.2 Predicate Device

Architect Anti-HCV (Abbott), REF. 6C37

4.3 Study Method

Alltest HCV Rapid Test Cassette(Whole Blood/Serum/Plasma) will be tested with specimens included in 4.1section.All the study should be operated by one person and reviewed by another person. Both AllTest test device and the comparation device will tested by each specimen according to the package insert provided in the kit. Any inconsistent result can be confirmed another reference method including:

1) by evaluation of the discrepant sample in further test systems.

- 2) by use of an alternative method or marker.
- 3) by a review of the clinical status and diagnosis of the patient, and by the testing of follow-up-samples.

4.4 Acceptable Criterion

AllTest HCV Rapid Test Cassette have same or better performance comparing to comparative device.

5. Test Result

Table 1- Clinical study result of IHC-402

Method		Predicate Device Testing		Total Results
Candidate Device	Results	Positive	Negative	
	Positive	110	1	111
	Negative	0	199	199
Total Results		110	200	310
Relative Sensitivity		100.0% (95%CI:*96.7%-100.0%)		
Relative Specificity		99.5% (95%CI:*97.2%-100.0%)		
Totally accuracy		99.68% (95%CI:*98.2%-100.0%)		
		*Confidence Intervals		

6. Result Analysis and conclusion

According to the testing result in table, the sensitivity of Alltest HCV Rapid Test Cassette(WholeBlood/Serum/Plasma) is 100.0%, and the specificity of Alltest HCV Rapid Test Cassette(WholeBlood/Serum/Plasma) is 99.5%.

Table 2- Data Sheet for Clinical Results

No.	No. Specimen No.	Sample collection time	Test Date Rapid Test	Type of specimen	Age	Gender	Result	
							Candidate Device	Predicate Device Testing
1	HCV0001	08/08/2022	09/08/2022	Serum	14	Female	Negative	Negative
2	HCV0002	08/08/2022	09/08/2022	Serum	63	Male	Negative	Negative
3	HCV0003	08/08/2022	09/08/2022	Serum	46	Male	Negative	Negative
4	HCV0004	08/08/2022	09/08/2022	Serum	80	Male	Negative	Negative
5	HCV0005	08/08/2022	09/08/2022	Serum	59	Female	Positive	Positive
6	HCV0006	08/08/2022	09/08/2022	Serum	27	Male	Negative	Negative
7	HCV0007	08/08/2022	09/08/2022	Serum	36	Male	Negative	Negative
8	HCV0008	08/08/2022	09/08/2022	Serum	72	Male	Negative	Negative
9	HCV0009	08/08/2022	09/08/2022	Serum	78	Male	Positive	Negative
10	HCV0010	08/08/2022	09/08/2022	Serum	53	Male	Negative	Negative
11	HCV0011	08/08/2022	09/08/2022	Serum	76	Male	Negative	Negative
12	HCV0012	08/08/2022	09/08/2022	Serum	62	Female	Positive	Positive
13	HCV0013	08/08/2022	09/08/2022	Serum	64	Male	Negative	Negative
14	HCV0014	08/08/2022	09/08/2022	Serum	21	Female	Negative	Negative
15	HCV0015	08/08/2022	09/08/2022	Serum	63	Male	Negative	Negative
16	HCV0016	08/08/2022	09/08/2022	Serum	61	Male	Negative	Negative
17	HCV0017	08/08/2022	09/08/2022	Serum	23	Male	Negative	Negative
18	HCV0018	08/08/2022	09/08/2022	Serum	32	Male	Positive	Positive
19	HCV0019	08/08/2022	09/08/2022	Serum	47	Male	Negative	Negative
20	HCV0020	08/08/2022	09/08/2022	Serum	38	Female	Positive	Positive
21	HCV0021	08/08/2022	09/08/2022	Serum	70	Male	Positive	Positive
22	HCV0022	08/08/2022	09/08/2022	Serum	40	Female	Positive	Positive
23	HCV0023	08/08/2022	09/08/2022	Serum	40	Female	Negative	Negative
24	HCV0024	08/08/2022	09/08/2022	Serum	52	Male	Negative	Negative
25	HCV0025	08/08/2022	09/08/2022	Serum	40	Male	Negative	Negative
26	HCV0026	08/08/2022	09/08/2022	Serum	30	Male	Negative	Negative
27	HCV0027	08/08/2022	09/08/2022	Serum	70	Female	Negative	Negative
28	HCV0028	08/08/2022	09/08/2022	Serum	42	Female	Negative	Negative
29	HCV0029	08/08/2022	09/08/2022	Serum	73	Male	Negative	Negative
30	HCV0030	09/08/2022	10/08/2022	Serum	26	Female	Negative	Negative
31	HCV0031	09/08/2022	10/08/2022	Serum	61	Female	Positive	Positive
32	HCV0032	09/08/2022	10/08/2022	Serum	57	Female	Positive	Positive
33	HCV0033	09/08/2022	10/08/2022	Serum	75	Male	Negative	Negative
34	HCV0034	09/08/2022	10/08/2022	Serum	53	Female	Negative	Negative
35	HCV0035	09/08/2022	10/08/2022	Serum	19	Male	Negative	Negative
36	HCV0036	09/08/2022	10/08/2022	Serum	58	Male	Negative	Negative



37	HCV0037	09/08/2022	10/08/2022	Serum	61	Female	Positive	Positive
38	HCV0038	09/08/2022	10/08/2022	Serum	27	Male	Negative	Negative
39	HCV0039	09/08/2022	10/08/2022	Serum	63	Male	Negative	Negative
40	HCV0040	09/08/2022	10/08/2022	Serum	76	Male	Negative	Negative
41	HCV0041	09/08/2022	10/08/2022	Serum	14	Male	Negative	Negative
42	HCV0042	09/08/2022	10/08/2022	Serum	68	Male	Negative	Negative
43	HCV0043	09/08/2022	10/08/2022	Serum	34	Male	Negative	Negative
44	HCV0044	09/08/2022	10/08/2022	Serum	40	Male	Positive	Positive
45	HCV0045	09/08/2022	10/08/2022	Serum	40	Male	Negative	Negative
46	HCV0046	09/08/2022	10/08/2022	Serum	74	Male	Negative	Negative
47	HCV0047	09/08/2022	10/08/2022	Serum	82	Female	Positive	Positive
48	HCV0048	09/08/2022	10/08/2022	Serum	49	Male	Negative	Negative
49	HCV0049	09/08/2022	10/08/2022	Serum	42	Male	Positive	Positive
50	HCV0050	09/08/2022	10/08/2022	Serum	79	Male	Positive	Positive
51	HCV0051	09/08/2022	10/08/2022	Serum	35	Female	Positive	Positive
52	HCV0052	09/08/2022	10/08/2022	Serum	55	Male	Negative	Negative
53	HCV0053	09/08/2022	10/08/2022	Serum	79	Male	Positive	Positive
54	HCV0054	09/08/2022	10/08/2022	Serum	63	Female	Negative	Negative
55	HCV0055	09/08/2022	10/08/2022	Serum	52	Male	Negative	Negative
56	HCV0056	09/08/2022	10/08/2022	Serum	84	Female	Positive	Positive
57	HCV0057	09/08/2022	10/08/2022	Serum	18	Male	Negative	Negative
58	HCV0058	09/08/2022	10/08/2022	Serum	25	Female	Positive	Positive
59	HCV0059	09/08/2022	10/08/2022	Serum	70	Female	Positive	Positive
60	HCV0060	09/08/2022	10/08/2022	Serum	54	Male	Negative	Negative
61	HCV0061	09/08/2022	10/08/2022	Serum	63	Male	Positive	Positive
62	HCV0062	09/08/2022	10/08/2022	Serum	44	Male	Positive	Positive
63	HCV0063	09/08/2022	10/08/2022	Serum	57	Male	Negative	Negative
64	HCV0064	09/08/2022	10/08/2022	Serum	53	Male	Negative	Negative
65	HCV0065	09/08/2022	10/08/2022	Serum	53	Male	Negative	Negative
66	HCV0066	09/08/2022	10/08/2022	Serum	38	Male	Positive	Positive
67	HCV0067	09/08/2022	10/08/2022	Serum	52	Male	Negative	Negative
68	HCV0068	09/08/2022	10/08/2022	Serum	28	Male	Negative	Negative
69	HCV0069	09/08/2022	10/08/2022	Serum	61	Male	Negative	Negative
70	HCV0070	09/08/2022	10/08/2022	Serum	10	Female	Positive	Positive
71	HCV0071	09/08/2022	10/08/2022	Serum	84	Male	Negative	Negative
72	HCV0072	09/08/2022	10/08/2022	Serum	79	Male	Positive	Positive
73	HCV0073	09/08/2022	10/08/2022	Serum	48	Female	Negative	Negative
74	HCV0074	09/08/2022	10/08/2022	Serum	29	Female	Positive	Positive
75	HCV0075	09/08/2022	10/08/2022	Serum	57	Male	Negative	Negative
76	HCV0076	10/08/2022	11/08/2022	Serum	39	Male	Positive	Positive
77	HCV0077	10/08/2022	11/08/2022	Serum	36	Male	Positive	Positive

Splošna
bolnišnica
Jesenice

78	HCV0078	10/08/2022	11/08/2022	Serum	75	Male	Negative	Negative
79	HCV0079	10/08/2022	11/08/2022	Serum	15	Male	Positive	Positive
80	HCV0080	10/08/2022	11/08/2022	Serum	31	Male	Negative	Negative
81	HCV0081	10/08/2022	11/08/2022	Serum	64	Male	Negative	Negative
82	HCV0082	10/08/2022	11/08/2022	Serum	56	Male	Negative	Negative
83	HCV0083	10/08/2022	11/08/2022	Serum	20	Male	Negative	Negative
84	HCV0084	10/08/2022	11/08/2022	Serum	71	Male	Negative	Negative
85	HCV0085	10/08/2022	11/08/2022	Serum	84	Male	Positive	Positive
86	HCV0086	10/08/2022	11/08/2022	Serum	40	Male	Negative	Negative
87	HCV0087	10/08/2022	11/08/2022	Serum	21	Male	Negative	Negative
88	HCV0088	10/08/2022	11/08/2022	Serum	26	Male	Negative	Negative
89	HCV0089	10/08/2022	11/08/2022	Serum	40	Male	Negative	Negative
90	HCV0090	10/08/2022	11/08/2022	Serum	21	Male	Negative	Negative
91	HCV0091	10/08/2022	11/08/2022	Serum	10	Male	Positive	Positive
92	HCV0092	10/08/2022	11/08/2022	Serum	23	Female	Negative	Negative
93	HCV0093	10/08/2022	11/08/2022	Serum	63	Female	Negative	Negative
94	HCV0094	10/08/2022	11/08/2022	Serum	60	Male	Positive	Positive
95	HCV0095	10/08/2022	11/08/2022	Serum	78	Male	Negative	Negative
96	HCV0096	10/08/2022	11/08/2022	Serum	43	Male	Positive	Positive
97	HCV0097	10/08/2022	11/08/2022	Serum	83	Male	Positive	Positive
98	HCV0098	10/08/2022	11/08/2022	Serum	22	Male	Negative	Negative
99	HCV0099	10/08/2022	11/08/2022	Serum	71	Male	Positive	Positive
100	HCV0100	10/08/2022	11/08/2022	Serum	39	Male	Negative	Negative
101	HCV0101	10/08/2022	11/08/2022	Serum	84	Female	Negative	Negative
102	HCV0102	10/08/2022	11/08/2022	Serum	80	Male	Positive	Positive
103	HCV0103	10/08/2022	11/08/2022	Serum	33	Female	Negative	Negative
104	HCV0104	10/08/2022	11/08/2022	Serum	73	Female	Positive	Positive
105	HCV0105	10/08/2022	11/08/2022	Serum	66	Male	Negative	Negative
106	HCV0106	10/08/2022	11/08/2022	Serum	46	Male	Positive	Positive
107	HCV0107	10/08/2022	11/08/2022	Serum	20	Male	Positive	Positive
108	HCV0108	10/08/2022	11/08/2022	Serum	72	Male	Negative	Negative
109	HCV0109	10/08/2022	11/08/2022	Serum	34	Male	Negative	Negative
110	HCV0110	10/08/2022	11/08/2022	Serum	83	Male	Negative	Negative
111	HCV0111	10/08/2022	11/08/2022	Serum	20	Female	Negative	Negative
112	HCV0112	10/08/2022	11/08/2022	Serum	23	Male	Negative	Negative
113	HCV0113	10/08/2022	11/08/2022	Serum	16	Female	Negative	Negative
114	HCV0114	10/08/2022	11/08/2022	Serum	37	Male	Negative	Negative
115	HCV0115	10/08/2022	11/08/2022	Serum	69	Female	Positive	Positive
116	HCV0116	10/08/2022	11/08/2022	Serum	50	Female	Negative	Negative
117	HCV0117	10/08/2022	11/08/2022	Serum	63	Male	Positive	Positive
118	HCV0118	10/08/2022	11/08/2022	Serum	42	Male	Positive	Positive

[Signature]

119	HCV0119	10/08/2022	11/08/2022	Serum	69	Male	Negative	Negative
120	HCV0120	10/08/2022	11/08/2022	Serum	83	Female	Negative	Negative
121	HCV0121	10/08/2022	11/08/2022	Serum	83	Male	Positive	Positive
122	HCV0122	10/08/2022	11/08/2022	Serum	33	Male	Negative	Negative
123	HCV0123	10/08/2022	11/08/2022	Plasma	56	Male	Negative	Negative
124	HCV0124	10/08/2022	11/08/2022	Plasma	41	Male	Negative	Negative
125	HCV0125	10/08/2022	11/08/2022	Plasma	43	Female	Negative	Negative
126	HCV0126	10/08/2022	11/08/2022	Plasma	58	Male	Negative	Negative
127	HCV0127	10/08/2022	11/08/2022	Plasma	60	Male	Negative	Negative
128	HCV0128	10/08/2022	11/08/2022	Plasma	22	Male	Negative	Negative
129	HCV0129	10/08/2022	11/08/2022	Plasma	25	Male	Negative	Negative
130	HCV0130	10/08/2022	11/08/2022	Plasma	15	Male	Negative	Negative
131	HCV0131	10/08/2022	11/08/2022	Plasma	80	Female	Positive	Positive
132	HCV0132	10/08/2022	11/08/2022	Plasma	18	Female	Negative	Negative
133	HCV0133	10/08/2022	11/08/2022	Plasma	63	Male	Positive	Positive
134	HCV0134	10/08/2022	11/08/2022	Plasma	37	Male	Negative	Negative
135	HCV0135	10/08/2022	11/08/2022	Plasma	73	Male	Positive	Positive
136	HCV0136	10/08/2022	11/08/2022	Plasma	23	Male	Positive	Positive
137	HCV0137	11/08/2022	12/08/2022	Plasma	45	Female	Negative	Negative
138	HCV0138	11/08/2022	12/08/2022	Plasma	41	Male	Positive	Positive
139	HCV0139	11/08/2022	12/08/2022	Plasma	73	Female	Negative	Negative
140	HCV0140	11/08/2022	12/08/2022	Plasma	10	Female	Negative	Negative
141	HCV0141	11/08/2022	12/08/2022	Plasma	28	Male	Positive	Positive
142	HCV0142	11/08/2022	12/08/2022	Plasma	78	Male	Negative	Negative
143	HCV0143	11/08/2022	12/08/2022	Plasma	66	Male	Negative	Negative
144	HCV0144	11/08/2022	12/08/2022	Plasma	78	Male	Negative	Negative
145	HCV0145	11/08/2022	12/08/2022	Plasma	54	Male	Positive	Positive
146	HCV0146	11/08/2022	12/08/2022	Plasma	85	Male	Positive	Positive
147	HCV0147	11/08/2022	12/08/2022	Plasma	37	Female	Negative	Negative
148	HCV0148	11/08/2022	12/08/2022	Plasma	47	Male	Negative	Negative
149	HCV0149	11/08/2022	12/08/2022	Plasma	34	Female	Negative	Negative
150	HCV0150	11/08/2022	12/08/2022	Plasma	52	Male	Negative	Negative
151	HCV0151	11/08/2022	12/08/2022	Plasma	49	Male	Negative	Negative
152	HCV0152	11/08/2022	12/08/2022	Plasma	38	Male	Negative	Negative
153	HCV0153	11/08/2022	12/08/2022	Plasma	55	Male	Negative	Negative
154	HCV0154	11/08/2022	12/08/2022	Plasma	12	Male	Negative	Negative
155	HCV0155	11/08/2022	12/08/2022	Plasma	27	Male	Positive	Positive
156	HCV0156	11/08/2022	12/08/2022	Plasma	28	Male	Negative	Negative
157	HCV0157	11/08/2022	12/08/2022	Plasma	22	Female	Negative	Negative
158	HCV0158	11/08/2022	12/08/2022	Plasma	67	Female	Negative	Negative
159	HCV0159	11/08/2022	12/08/2022	Plasma	45	Male	Positive	Positive

160	HCV0160	11/08/2022	12/08/2022	Plasma	39	Male	Negative	Negative
161	HCV0161	11/08/2022	12/08/2022	Plasma	29	Female	Negative	Negative
162	HCV0162	11/08/2022	12/08/2022	Plasma	36	Male	Positive	Positive
163	HCV0163	11/08/2022	12/08/2022	Plasma	55	Male	Negative	Negative
164	HCV0164	11/08/2022	12/08/2022	Plasma	82	Female	Positive	Positive
165	HCV0165	11/08/2022	12/08/2022	Plasma	76	Female	Negative	Negative
166	HCV0166	11/08/2022	12/08/2022	Plasma	71	Male	Negative	Negative
167	HCV0167	11/08/2022	12/08/2022	Plasma	53	Male	Negative	Negative
168	HCV0168	11/08/2022	12/08/2022	Plasma	70	Male	Negative	Negative
169	HCV0169	11/08/2022	12/08/2022	Plasma	57	Male	Positive	Positive
170	HCV0170	11/08/2022	12/08/2022	Plasma	81	Male	Negative	Negative
171	HCV0171	11/08/2022	12/08/2022	Plasma	77	Female	Negative	Negative
172	HCV0172	11/08/2022	12/08/2022	Plasma	19	Female	Negative	Negative
173	HCV0173	11/08/2022	12/08/2022	Plasma	19	Male	Negative	Negative
174	HCV0174	11/08/2022	12/08/2022	Plasma	40	Male	Negative	Negative
175	HCV0175	11/08/2022	12/08/2022	Plasma	48	Male	Negative	Negative
176	HCV0176	11/08/2022	12/08/2022	Plasma	70	Male	Positive	Positive
177	HCV0177	11/08/2022	12/08/2022	Plasma	67	Male	Negative	Negative
178	HCV0178	11/08/2022	12/08/2022	Plasma	31	Male	Negative	Negative
179	HCV0179	11/08/2022	12/08/2022	Plasma	18	Male	Negative	Negative
180	HCV0180	11/08/2022	12/08/2022	Plasma	47	Male	Negative	Negative
181	HCV0181	11/08/2022	12/08/2022	Plasma	72	Male	Positive	Positive
182	HCV0182	11/08/2022	12/08/2022	Plasma	29	Male	Negative	Negative
183	HCV0183	11/08/2022	12/08/2022	Plasma	53	Male	Negative	Negative
184	HCV0184	11/08/2022	12/08/2022	Plasma	35	Female	Positive	Positive
185	HCV0185	11/08/2022	12/08/2022	Plasma	45	Male	Positive	Positive
186	HCV0186	11/08/2022	12/08/2022	Plasma	68	Male	Negative	Negative
187	HCV0187	11/08/2022	12/08/2022	Plasma	11	Male	Positive	Positive
188	HCV0188	11/08/2022	12/08/2022	Plasma	52	Male	Negative	Negative
189	HCV0189	11/08/2022	12/08/2022	Plasma	18	Female	Negative	Negative
190	HCV0190	11/08/2022	12/08/2022	Plasma	47	Male	Positive	Positive
191	HCV0191	11/08/2022	12/08/2022	Plasma	43	Male	Negative	Negative
192	HCV0192	11/08/2022	12/08/2022	Plasma	24	Female	Negative	Negative
193	HCV0193	11/08/2022	12/08/2022	Plasma	15	Female	Negative	Negative
194	HCV0194	11/08/2022	12/08/2022	Plasma	48	Female	Negative	Negative
195	HCV0195	11/08/2022	12/08/2022	Plasma	82	Male	Positive	Positive
196	HCV0196	11/08/2022	12/08/2022	Plasma	78	Female	Positive	Positive
197	HCV0197	11/08/2022	12/08/2022	Plasma	11	Male	Positive	Positive
198	HCV0198	11/08/2022	12/08/2022	Plasma	59	Male	Negative	Negative
199	HCV0199	11/08/2022	12/08/2022	Plasma	80	Male	Negative	Negative
200	HCV0200	11/08/2022	12/08/2022	Plasma	18	Male	Negative	Negative

201	HCV0201	11/08/2022	12/08/2022	Plasma	67	Male	Negative	Negative
202	HCV0202	11/08/2022	12/08/2022	Plasma	49	Male	Negative	Negative
203	HCV0203	11/08/2022	12/08/2022	Plasma	31	Female	Negative	Negative
204	HCV0204	11/08/2022	12/08/2022	Plasma	18	Male	Positive	Positive
205	HCV0205	11/08/2022	12/08/2022	Plasma	62	Male	Positive	Positive
206	HCV0206	15/08/2022	16/08/2022	Plasma	69	Female	Negative	Negative
207	HCV0207	15/08/2022	16/08/2022	Plasma	72	Male	Positive	Positive
208	HCV0208	15/08/2022	16/08/2022	Plasma	41	Female	Positive	Positive
209	HCV0209	15/08/2022	16/08/2022	Plasma	54	Female	Negative	Negative
210	HCV0210	15/08/2022	16/08/2022	Plasma	65	Female	Negative	Negative
211	HCV0211	15/08/2022	16/08/2022	Plasma	48	Male	Negative	Negative
212	HCV0212	15/08/2022	16/08/2022	Plasma	58	Male	Negative	Negative
213	HCV0213	15/08/2022	16/08/2022	Plasma	58	Male	Positive	Positive
214	HCV0214	15/08/2022	16/08/2022	Plasma	43	Female	Negative	Negative
215	HCV0215	15/08/2022	16/08/2022	Plasma	77	Male	Positive	Positive
216	HCV0216	15/08/2022	16/08/2022	Plasma	38	Male	Negative	Negative
217	HCV0217	15/08/2022	16/08/2022	Plasma	13	Male	Positive	Positive
218	HCV0218	15/08/2022	16/08/2022	Plasma	13	Female	Positive	Positive
219	HCV0219	15/08/2022	16/08/2022	Plasma	20	Male	Positive	Positive
220	HCV0220	15/08/2022	16/08/2022	Plasma	45	Male	Negative	Negative
221	HCV0221	15/08/2022	16/08/2022	Plasma	25	Female	Positive	Positive
222	HCV0222	15/08/2022	16/08/2022	Plasma	64	Male	Positive	Positive
223	HCV0223	15/08/2022	16/08/2022	Plasma	71	Male	Negative	Negative
224	HCV0224	15/08/2022	16/08/2022	Plasma	84	Male	Positive	Positive
225	HCV0225	15/08/2022	16/08/2022	Plasma	40	Male	Negative	Negative
226	HCV0226	15/08/2022	16/08/2022	Plasma	34	Male	Positive	Positive
227	HCV0227	15/08/2022	16/08/2022	Plasma	73	Male	Positive	Positive
228	HCV0228	15/08/2022	16/08/2022	Plasma	70	Male	Positive	Positive
229	HCV0229	15/08/2022	16/08/2022	Plasma	39	Male	Negative	Negative
230	HCV0230	15/08/2022	16/08/2022	Plasma	26	Male	Negative	Negative
231	HCV0231	15/08/2022	16/08/2022	Plasma	45	Male	Negative	Negative
232	HCV0232	15/08/2022	16/08/2022	Plasma	20	Female	Negative	Negative
233	HCV0233	15/08/2022	16/08/2022	Plasma	18	Female	Negative	Negative
234	HCV0234	15/08/2022	16/08/2022	Plasma	76	Male	Negative	Negative
235	HCV0235	15/08/2022	16/08/2022	Plasma	72	Female	Positive	Positive
236	HCV0236	15/08/2022	16/08/2022	Plasma	23	Male	Negative	Negative
237	HCV0237	15/08/2022	16/08/2022	Plasma	34	Female	Positive	Positive
238	HCV0238	15/08/2022	16/08/2022	Plasma	49	Female	Positive	Positive
239	HCV0239	15/08/2022	16/08/2022	Plasma	19	Male	Negative	Negative
240	HCV0240	15/08/2022	16/08/2022	Plasma	37	Male	Negative	Negative
241	HCV0241	15/08/2022	16/08/2022	Plasma	48	Male	Negative	Negative

242	HCV0242	15/08/2022	16/08/2022	Plasma	52	Male	Negative	Negative
243	HCV0243	15/08/2022	16/08/2022	Plasma	59	Male	Negative	Negative
244	HCV0244	15/08/2022	16/08/2022	Plasma	46	Male	Negative	Negative
245	HCV0245	15/08/2022	16/08/2022	Plasma	61	Male	Positive	Positive
246	HCV0246	15/08/2022	16/08/2022	Plasma	49	Male	Negative	Negative
247	HCV0247	15/08/2022	16/08/2022	Plasma	18	Female	Positive	Positive
248	HCV0248	15/08/2022	16/08/2022	Plasma	24	Male	Positive	Positive
249	HCV0249	15/08/2022	16/08/2022	Plasma	42	Female	Positive	Positive
250	HCV0250	15/08/2022	16/08/2022	Plasma	38	Male	Negative	Negative
251	HCV0251	15/08/2022	16/08/2022	Plasma	77	Female	Positive	Positive
252	HCV0252	15/08/2022	16/08/2022	Plasma	65	Male	Negative	Negative
253	HCV0253	15/08/2022	16/08/2022	Plasma	56	Female	Negative	Negative
254	HCV0254	15/08/2022	16/08/2022	Plasma	11	Male	Negative	Negative
255	HCV0255	15/08/2022	16/08/2022	Plasma	71	Female	Negative	Negative
256	HCV0256	15/08/2022	16/08/2022	Plasma	44	Male	Positive	Positive
257	HCV0257	15/08/2022	16/08/2022	Plasma	49	Male	Negative	Negative
258	HCV0258	15/08/2022	16/08/2022	Plasma	68	Male	Positive	Positive
259	HCV0259	15/08/2022	16/08/2022	Plasma	58	Male	Negative	Negative
260	HCV0260	15/08/2022	16/08/2022	Plasma	26	Male	Negative	Negative
261	HCV0261	15/08/2022	16/08/2022	Plasma	35	Male	Negative	Negative
262	HCV0262	15/08/2022	16/08/2022	Plasma	58	Female	Positive	Positive
263	HCV0263	15/08/2022	16/08/2022	Plasma	54	Female	Negative	Negative
264	HCV0264	16/08/2022	17/08/2022	Plasma	50	Male	Negative	Negative
265	HCV0265	16/08/2022	17/08/2022	Plasma	79	Male	Positive	Positive
266	HCV0266	16/08/2022	17/08/2022	Plasma	43	Female	Negative	Negative
267	HCV0267	16/08/2022	17/08/2022	Plasma	73	Male	Negative	Negative
268	HCV0268	16/08/2022	17/08/2022	Plasma	10	Male	Negative	Negative
269	HCV0269	16/08/2022	17/08/2022	Plasma	22	Male	Positive	Positive
270	HCV0270	16/08/2022	17/08/2022	Plasma	62	Male	Negative	Negative
271	HCV0271	16/08/2022	17/08/2022	Plasma	50	Male	Negative	Negative
272	HCV0272	16/08/2022	17/08/2022	Plasma	18	Female	Positive	Positive
273	HCV0273	16/08/2022	17/08/2022	Plasma	67	Male	Negative	Negative
274	HCV0274	16/08/2022	17/08/2022	Plasma	41	Male	Negative	Negative
275	HCV0275	16/08/2022	17/08/2022	Plasma	55	Male	Negative	Negative
276	HCV0276	16/08/2022	17/08/2022	Plasma	81	Male	Positive	Positive
277	HCV0277	16/08/2022	17/08/2022	Plasma	62	Female	Negative	Negative
278	HCV0278	16/08/2022	17/08/2022	Plasma	56	Male	Positive	Positive
279	HCV0279	16/08/2022	17/08/2022	Plasma	49	Male	Negative	Negative
280	HCV0280	16/08/2022	17/08/2022	Plasma	50	Male	Negative	Negative
281	HCV0281	16/08/2022	17/08/2022	Plasma	56	Female	Negative	Negative
282	HCV0282	16/08/2022	17/08/2022	Plasma	10	Male	Positive	Positive

283	HCV0283	16/08/2022	17/08/2022	Plasma	17	Male	Negative	Negative
284	HCV0284	16/08/2022	17/08/2022	Plasma	66	Male	Positive	Positive
285	HCV0285	16/08/2022	17/08/2022	Plasma	49	Female	Positive	Positive
286	HCV0286	16/08/2022	17/08/2022	Plasma	60	Male	Positive	Positive
287	HCV0287	16/08/2022	17/08/2022	Plasma	23	Male	Negative	Negative
288	HCV0288	16/08/2022	17/08/2022	Plasma	61	Female	Negative	Negative
289	HCV0289	16/08/2022	17/08/2022	Plasma	13	Male	Positive	Positive
290	HCV0290	16/08/2022	17/08/2022	Plasma	71	Male	Negative	Negative
291	HCV0291	16/08/2022	17/08/2022	Plasma	42	Male	Positive	Positive
292	HCV0292	16/08/2022	17/08/2022	Plasma	75	Female	Negative	Negative
293	HCV0293	16/08/2022	17/08/2022	Plasma	57	Male	Positive	Positive
294	HCV0294	16/08/2022	17/08/2022	Plasma	34	Male	Positive	Positive
295	HCV0295	16/08/2022	17/08/2022	Plasma	67	Male	Negative	Negative
296	HCV0296	16/08/2022	17/08/2022	Plasma	38	Female	Positive	Positive
297	HCV0297	16/08/2022	17/08/2022	Plasma	53	Male	Negative	Negative
298	HCV0298	16/08/2022	17/08/2022	Plasma	38	Male	Positive	Positive
299	HCV0299	16/08/2022	17/08/2022	Plasma	80	Male	Positive	Positive
300	HCV0300	16/08/2022	17/08/2022	Plasma	65	Female	Positive	Positive
301	HCV0301	16/08/2022	17/08/2022	Plasma	68	Male	Negative	Negative
302	HCV0302	16/08/2022	17/08/2022	Plasma	42	Male	Positive	Positive
303	HCV0303	16/08/2022	17/08/2022	Plasma	74	Male	Positive	Positive
304	HCV0304	16/08/2022	17/08/2022	Plasma	50	Female	Positive	Positive
305	HCV0305	16/08/2022	17/08/2022	Plasma	35	Male	Negative	Negative
306	HCV0306	16/08/2022	17/08/2022	Plasma	76	Male	Negative	Negative
307	HCV0307	16/08/2022	17/08/2022	Plasma	16	Male	Negative	Negative
308	HCV0308	16/08/2022	17/08/2022	Plasma	51	Male	Positive	Positive
309	HCV0309	16/08/2022	17/08/2022	Plasma	20	Male	Negative	Negative
310	HCV0310	16/08/2022	17/08/2022	Plasma	32	Female	Negative	Negative

Doc. dr. Miha Skvarč, Ph.D. med., spec. of Clinical Microbiology, Head of the Microbiology Unit
at General Hospital , Jesenice, August 17, 2022